


Meril

ObturaTM

Vascular Closure Device

Assured Closure. Delivered !

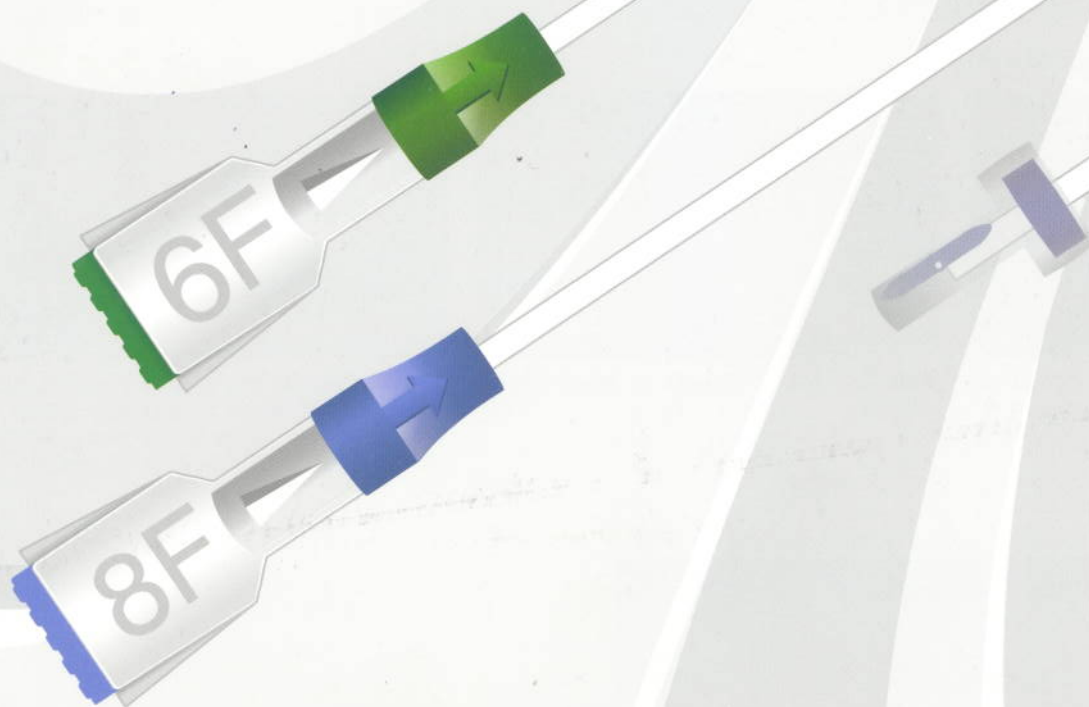


ObturaTM

Vascular Closure Device

Meril

The ObturaTM Vascular Closure Device is a mechanical closure system which provides an unassailable sealing of the arterial puncture without exchanging Introducer Sheath making it easy to use and providing rapid hemostasis.


- NO exchange of Introducer Sheath
- Less procedural time
- Rapid Hemostasis and less blood loss
- 90 days absorption
- Easy to use Mechanical technique

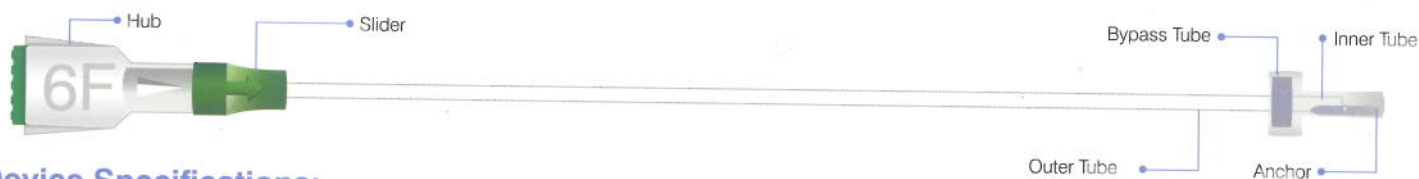

Device Specifications:

Parameter	Specification
Device Size (F)	6 and 8
Device Total Length (mm)	205
Device Effective Length(mm)	155
Complete Degradation Period* (days)	90
Device Size Compatibility	6F device compatible with 6F & 7F Introducer Sheath 8F device compatible with 8F & 9F Introducer Sheath
Introducer Sheath Compatibility	Medtronic: InPut Introducer Sheath Cordis Corporation: AVANTI TM Introducer Sheath Terumo Corporation: PINNACLE Introducer Sheath Oscor Inc: Adelante Meril Medical: Prelude
Compatible Introducer Sheath Length(cm)	12


Simple Three Step Procedure:

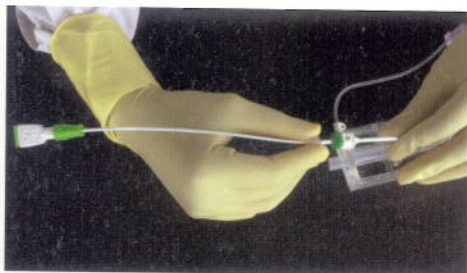

Step 1: Grasp the device at Bypass tube between two fingers like a pen such that size indicator 6F/8F on Hub and Arrow mark on Slider Hub appears upwards.



Step 2: Pull the Hub until the Black Marker appears.

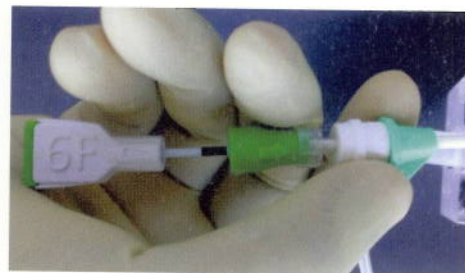

Step 3: Grasping the three components Hub, Slider Hub and Hub of the Introducer Sheath in palm, retract the entire device along with the introducer sheath. Keep the tension on Suture.

Ordering Information:

Reference Number	French Size
OBT6F	6F
OBT8F	8F


ObturaTM

Vascular Closure Device

Meril

The ObturaTM Vascular Closure Device is a mechanical closure system which provides an unassailable sealing of the arterial puncture without exchanging Introducer Sheath making it easy to use and providing rapid hemostasis.


- NO exchange of Introducer Sheath
- Less procedural time
- Rapid Hemostasis and less blood loss
- 90 days absorption
- Easy to use Mechanical technique

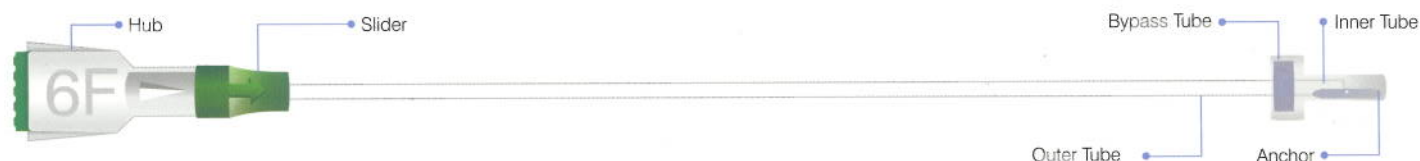


Device Specifications:

Parameter	Specification
Device Size (F)	6 and 8
Device Total Length (mm)	205
Device Effective Length(mm)	155
Complete Degradation Period (days)	90
Device Size Compatibility	6F device compatible with 6F & 7F Introducer Sheath 8F device compatible with 8F & 9F Introducer Sheath
Introducer Sheath Compatibility	Medtronic: InPut Introducer Sheath Cordis Corporation: AVANTI TM Introducer Sheath Terumo Corporation: PINNACLE Introducer Sheath Oscor Inc: Adelante Merit Medical: Prelude
Compatible Introducer Sheath Length(cm)	12

Simple Three Step Procedure:

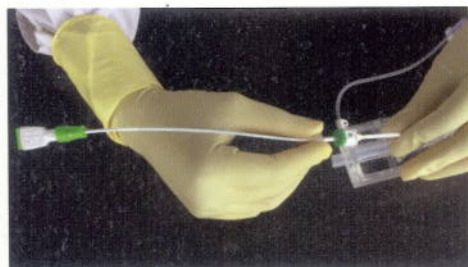

Step 1: Grasp the device at Bypass tube between two fingers like a pen such that size indicator 6F/8F on Hub and Arrow mark on Slider Hub appears upwards.

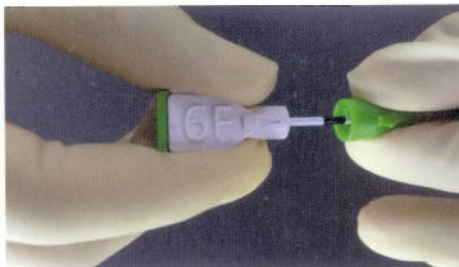

Step 2: Pull the Hub until the Black Marker appears.



Step 3: Grasping the three components Hub, Slider Hub and Hub of the Introducer Sheath in palm, retract the entire device along with the introducer sheath. Keep the tension on Suture.

Ordering Information:

Reference Number	French Size
OBT6F	6F
OBT8F	8F