

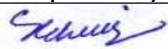
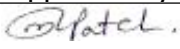


USER MANUAL [PRODUCT DESCRIPTION SHEET]

Issue No. : 1 Date : 01-09-2023

Rev. No. : 0 Date : 01-09-2023

Product Name		:	Angeltouch Extension Line																								
Product Code		:	IT-35B																								
CE Mark with No.		:	<div></div>																								
Manufacturer's Name		:	Angioplast Private Limited																								
Manufacturer's Address		:	Plot No. 4803, Phase IV, Vatva GIDC, Ahmedabad 382 445. [India]																								
EC REP Address		:	mdi Europa GmbH, Langenhagener Str. 71, 30855, Langenhagen, Germany																								
Intended Use		:	The sterile set is used for extending the infusion set.																								
Product Description		:	Set is made up of Protective Cap, Male Luer Lock, Tubing, Female Luer Lock, Protective Cap																								
Product Specification		:	<div></div>		<div><div>* Length : 150cm</div><div>* NON TOXIC * NON PYROGENIC</div><div>* Transparent.</div><div>* Pressure Resistance up to 7 bar.</div></div>																						
Material List& Make		:	<div><table><tr><th>Sr. No.</th><th>Name of Components</th><th>No.</th><th>Material Used</th></tr><tr><td>1</td><td>Male Luer Lock (Transparent)</td><td>1</td><td>Rigid PVC</td></tr><tr><td>2</td><td>Female Luer Lock (Transparent)</td><td>1</td><td>Rigid PVC</td></tr><tr><td>3</td><td>Tubing (Transparent)</td><td>1</td><td>DEHP Free Poly-vinyl chloride (PVC)</td></tr><tr><td>4</td><td>Protective cap for male fitting adaptor</td><td>2</td><td>High Density Polyethylene (HDPE)</td></tr><tr><td>5</td><td>Pouch</td><td>1</td><td>Paper Plastic Pouch</td></tr></table></div>	Sr. No.	Name of Components	No.	Material Used	1	Male Luer Lock (Transparent)	1	Rigid PVC	2	Female Luer Lock (Transparent)	1	Rigid PVC	3	Tubing (Transparent)	1	DEHP Free Poly-vinyl chloride (PVC)	4	Protective cap for male fitting adaptor	2	High Density Polyethylene (HDPE)	5	Pouch	1	Paper Plastic Pouch
Sr. No.	Name of Components	No.	Material Used																								
1	Male Luer Lock (Transparent)	1	Rigid PVC																								
2	Female Luer Lock (Transparent)	1	Rigid PVC																								
3	Tubing (Transparent)	1	DEHP Free Poly-vinyl chloride (PVC)																								
4	Protective cap for male fitting adaptor	2	High Density Polyethylene (HDPE)																								
5	Pouch	1	Paper Plastic Pouch																								

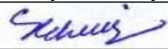
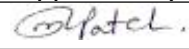
Prepared by	Page 1 of 2	Approved by
		
S. A Christian		M. M Patel

USER MANUAL [PRODUCT DESCRIPTION SHEET]

Issue No. : 1 Date : 01-09-2023

Rev. No. : 0 Date : 01-09-2023

Instruction For Use	:	1. Follow physician's instructions in opening package and use of contents. 2. Use aseptic technique. 3. Verify gastrostomy tube compatibility prior to use. 4. Attach inlet to outlet of administration set. 5. Attach extension set outlet to feeding tube. 6. Close clamp of IV set and plug until ready to use or when not in use.
Product Use	:	For Single Use Only (Shelf Life of 5 years from Date of Manufacturing)
Is product Sterilized	:	Yes
Sterilization Method	:	Ethylene Oxide Gas Sterilization
Storage Conditions	:	Keep away from getting wet. Store in cool & dry place.
Packing Detail		
Unit Pack [Pouch]	:	240mm x 130mm x Paper : 300G and Plastic : 200G
Gross Pack [Size]	:	260mm x 520mm x 150G
Gross Pack [Qty.]	:	50 Nos.
Shipper Pack [Size]	:	595mm x 375mm x 325mm
Shipper Pack [Qty.]	:	600 Nos.

Prepared by		Approved by
	Page 2 of 2	
S. A Christian		M. M Patel