M-6FD DYE

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**【MODEL】**  
M-6FD

**【PACKAGE SPECIFICATIONS】**  
48mL×1,12mL×4

**【PRINCIPLE】**  
This product is formulated to dye the white blood cells with the assist of M-6LD LYSE. It participates in the measurement of WBC-related parameters in the DIFF channel which is accomplished by using SF Cube cell analysis technology (3D analysis using information from scatter of laser light at two angles and fluorescence signals).

**【INTENDED USE】**  
The M-6FD DYE participates in WBC differentiation in the DIFF channel together with M-6LD LYSE.

**【ACTIVE INGREDIENTS】**  
Fluorochrome..... ≤0.01%  
Ethylene Glycol.....94-99 %

**【PRODUCT SPECIFICATIONS】**  
1. Appearance: Blue and transparent liquid without deposits, suspended grains or flocks.  
2. Blank count results: The blank count results of the reagent tested on the Mindray Auto Hematology Analyzer shall meet the requirements of Table 1.

| Table 1 Blank Count Requirements |                          |
|----------------------------------|--------------------------|
| Parameter                        | Blank Count Requirements |
| WBC-D                            | ≤0.2×10 <sup>9</sup> /L  |

**【STORAGE AND EXPIRATION DATE】**  
This product shall be stored in ventilated room free from corrosive gas and light at 2℃~30℃ with the humidity lower than 90%. The shelf life is 1 year.  
It shall be used at 15℃~32℃ and within 60 days after being opened.

**【APPLICABLE INSTRUMENT】**  
This product applies to BC-6000/BC-6000Plus/BC-6100/BC-6100Plus/BC-6200/BC-6600Plus/BC-6700Plus/BC-6800Plus Auto Hematology Analyzer produced by Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

**【INSTRUCTIONS FOR USE】**  
1) Restore the product to usage temperature;  
2) Open the external packing and insert the pickup tube into the reagent pack based on the matching between the reagent pack label and the inlet label of the analyzer cap assembly connector;  
3) Screw the cap assembly tight and replace the reagent according to the operator's manual of the analyzer;  
4) Run a blank count and check the results. If the results meet the blank count requirements defined in the operator's manual of the analyzer, the newly installed reagent can be used for sample analysis See the operator's manual of the analyzer for details.

**【WARNINGS】**  
Do not inhale and/or ingest. In case of inhalation or ingestion, induce vomiting and seek medical advice immediately.  
Avoid skin contact. In case of skin contact, flush the affected area with plenty of water immediately.  
Avoid eye contact. In case of eye contact, flush the affected area with plenty of water and seek medical advice immediately.

**【NOTICE】**  
1) This product shall only be used on matched Mindray Auto Hematology Analyzer. Read the package insert carefully before using this product. It shall be used before the expiration date and disposed of properly when expired.  
2) If the reagent is polluted or affected by other factors and becomes abnormal, stop using it and replace it with a normal one. Dispose of the waste, residual and polluted packing based on local regulations.  
3) After the product is opened, it shall be connected to the cap assemblies at once and shall not be left open in the air.  
4) Keep the product away from light and high temperature.  
5) The following factors can affect the sample analysis: expired or ineffective reagent; reagent polluted by dust in the air; improper disposal of the sample; mixed with or used with reagents produced by other company; mixed use of residual from the old container and the newly-opened; used in condition other than specified.  
6) This product must be operated by clinical professionals, physician or laboratorian trained by Mindray or Mindray authorized personnel.

**【INTELLECTUAL PROPERTY STATEMENT】**  
SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this Mindray product and this manual. This manual may refer to information protected by copyright or patents and does not convey any license under the patent rights or copyright of Mindray, or of others.  
Mindray intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.  
Release, amendment, reproduction, distribution, rental, adaptation, translation or any other derivative work of this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

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|                    |                                                    |
|--------------------|----------------------------------------------------|
| Manufacturer:      | Shenzhen Mindray Bio-Medical Electronics Co., Ltd. |
| E-mail Address:    | service@mindray.com                                |
| Tel:               | +86 755 81888998                                   |
| Fax:               | +86 755 26582680                                   |
| EC-Representative: | Shanghai International Holding Corp. GmbH(Europe)  |
| Address:           | Eiffestraße 80, 20537 Hamburg, Germany             |
| Tel:               | 0049-40-2513175                                    |
| Fax:               | 0049-40-255726                                     |

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