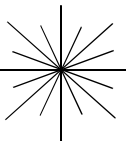
ABBOTT

CELL-DYN SYSTEM

RETICULOCYTE REAGENT

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PN 9155192A

NOTES

NAMECELL-DYN **RETICULOCYTE REAGENT****INTENDED USE****IVD** For in vitro diagnostic use.

CELL-DYN **RETICULOCYTE REAGENT** when used in combination with CELL-DYN Ruby CELL-DYN 3200, or CELL-DYN 3700 reticulocyte software, enables a whole blood specimen to be analyzed for reticulocytes.

SUMMARY AND EXPLANATION OF THE TEST

Reticulocytes are defined as transitional red cells between nucleated red cells and the so-called mature erythrocytes^[1]. The CELL-DYN reticulocyte assay enables determination of the percentage of reticulocytes using a whole blood specimen. If an RBC count is available, a reticulocyte absolute number is also derived. The method is based on light scatter measurements of stained cells.

BIOPHYSICAL AND BIOCHEMICAL PRINCIPLES OF THE PROCEDURE

In contrast to mature RBCs, reticulocytes contain ribosomal RNA. This RNA can be seen with certain supravital, cationic dyes that simultaneously stain and precipitate the polyanion to form a network or reticulum^[2]. The CELL-DYN Ruby, CELL-DYN 3200, and CELL-DYN 3700 reticulocyte method uses the thiazine dye New Methylene Blue N. Sample preparation is performed manually by diluting a small volume of blood into the pre-measured staining solution. At room temperature, staining of reticulum is complete within approximately 15 minutes. The stained sample is aspirated in the Open Mode and analyzed in the WBC Optical Count (WOC) channel. After the stained sample is aspirated, it is diluted with Reagent (WBC Lyse for CELL-DYN Ruby and CELL-DYN 3200 or Sheath for CELL-DYN 3700). Once diluted, the RBCs sphere due to the influence of the nonionic detergent incorporated into the staining solution. Sphering is necessary to eliminate optical orientational noise that would otherwise be introduced into the scatter measurements^[3]. The usual lytic action of the reagent (WBC Lyse for CELL-DYN Ruby and CELL-DYN 3200 or Sheath for CELL-DYN 3700) is prevented by electrolytes contained in the staining solution and the lack of the usual incubation period used in this channel during WBC analysis. Also the high New Methylene Blue concentration in the staining reagent exerts a stabilizing effect on RBCs.

During data acquisition, 0, 10, and 90 degree scatter are collected for up to 30,000 events. The 0 degree threshold is set high enough to exclude most platelets. Histogram data are used to differentiate reticulocytes, mature RBCs, platelet clumps and nucleated cells. Reticulocytes have similar 10 degree scatter to mature RBCs, but differ from them by exhibiting greater 90 degree scatter.

REAGENT

100 Tubes (3.7mL each) CELL-DYN Reticulocyte Reagent: Phosphate buffered saline solution containing New Methylene Blue, potassium oxalate and n-dodecyl β -D-maltoside. Preservative: Antimicrobial agent.

WARNINGS AND PRECAUTIONS**IVD** In Vitro Diagnostic Medical Device.**CAUTION:**

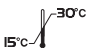
This product requires the handling of human specimens. It is recommended that all human sourced materials be considered potentially infectious and handled in accordance with the OSHA Standard on Bloodborne Pathogens^[6], Biosafety Level 2^[7] or other appropriate biosafety practices^{[8],[9]} should be used for materials that contain or are suspected of containing infectious agents.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

Handling Precautions**CAUTION:**

When using the reticulocyte reagent, avoid contact with skin and clothing. This reagent contains New Methylene Blue, which will stain skin, clothing, and many other surfaces.

Storage Instructions

 Do not place in the refrigerator. For long term storage protect from light.

PROCEDURE**Instrument Requirements**

The following software revisions must be installed on the CELL-DYN Ruby, CELL-DYN 3200, or CELL-DYN 3700 prior to performing the reticulocyte analysis:

CELL-DYN Ruby: Version 1.0 or higher.
CELL-DYN 3200: Version 2.0 or higher.
CELL-DYN 3700: Version 1.1 or higher.

Please contact your local Abbott representative if your CELL-DYN System is not using one of these versions or higher.

Materials Provided

REF 03H40-01 CELL-DYN Reticulocyte Reagent Kit, containing: 100 reticulocyte reagent tubes.

Materials Required but not provided

- CELL-DYN Reticulocyte Controls (Refer to the instrument specific CELL-DYN Systems Operator's Manual for list numbers of reticulocyte control products.)
- Pipettor Eppendorf type
- Pipettor tips
- Timer
- Tube rocker (optional)
- Tube rack
- Marker pen

Abbott Laboratories

CELL-DYN **RETICULOCYTE REAGENT**

Sample Collection

Use only whole blood specimens anticoagulated with EDTA. Do not run hemolyzed specimens, as hemolysis may cause inaccurate reticulocyte values.

Specimens may be run up to 8 hours after collection time without refrigeration.

Specimens older than eight (8) hours and up to 72 hours after collection may be processed if they have been stored in the refrigerator during that time. If refrigerated specimens are used, allowing them to come to room temperature before mixing will avoid damage to any fragile cells.

If a specimen is >8 hours old and the CBC was processed on the CELL-DYN System more than 8 hours ago, rerun the specimen to obtain the RBC value and to collect data that will be used during the flagging analysis. (The reticulocyte package will only select RBC values for specimens processed within the last 8 hours.)

For the Absolute Reticulocyte calculation, it is recommended that the RBC value used be selected from the same sample chosen for the Reticulocyte count and that it be run on the same analyzer.

See the appropriate System's Operator's Manual for complete instructions.

Staining Procedure

1. Label a tube of CELL-DYN Reticulocyte Reagent for each patient.
2. Verify that the whole blood specimen is warmed to room temperature and well mixed prior to sampling.
3. Pipette 20 µL of the whole blood specimen into each labeled tube of reticulocyte reagent.
4. Incubate the stained reticulocyte specimens on the tube rocker for 15 minutes or in a rack after fully inverting the stained specimens five (5) times. Incubation must be done at a room temperature of 15°–30°C.

NOTE: The stained reticulocyte specimens (specimen age <8 hours) may incubate for 15 minutes to 2 hours prior to processing on the CELL-DYN System. This timing will allow the reticulocyte specimens to be processed for STAT requests (15 minute incubation plus the one minute run time). Reticulocyte specimens can also be grouped and run in batches provided the 2-hour incubation time limit is not exceeded. For normal specimens, there is no clinically significant difference on samples up to 8 hours old that have been stained from 15 minutes to 2 hours.

The reticulocyte package is available for use only in the Open Sampler Mode.

Reagent Background Count Procedure

For background counts, a tube of reticulocyte reagent is run without an aliquot of whole blood, to check for particulate material in the reagent and system.

Quality Control Procedures

CELL-DYN reticulocyte control materials should be properly warmed and mixed according to the manufacturer's recommendations. Patient reticulocyte controls should be handled according to the laboratory's protocol. Refer to the reticulocyte quality control procedures provided in the appropriate CELL-DYN Operator's Manual.

RESULTS

CELL-DYN Reticulocyte results are acquired using information from the 0, 10, and 90 degree scatter channels. The reticulocytes are differentiated from mature red blood cells due to their scattering in the 90 degree channel. Reticulocytes are reported as a percentage of RBCs. If the RBC value is available, the Absolute Reticulocyte value is also reported.

FLAGS

Information pertaining to flagged results is displayed in the Flags field. For a description of the flags that may appear in this field, refer to the CELL-DYN System Operator's Manual, Reticulocyte Section. If ERL flagging is observed, re-run the sample and decrease incubation time to 15 minutes.

LIMITATIONS OF THE PROCEDURE

The CELL-DYN Ruby, CELL-DYN 3200, and CELL-DYN 3700 Reticulocyte method is a nucleic acid staining method. Therefore, other substances that contain nucleic acids could potentially be enumerated by the instrument as reticulocytes. If these interfering substances are present in sufficient numbers, they may interfere with the dynamic thresholds used to obtain the CELL-DYN reticulocyte count.

The information in the following table, based on NCCLS Document H44-A2^[1], indicates substances that are known or potential interferents. The CELL-DYN Ruby, CELL-DYN 3200, and CELL-DYN 3700 Reticulocyte method is designed to minimize some common interferents, including high WBC counts and NRBCs.

Potentially Interfering Substances

Cellular Elements	Cellular Inclusions	Miscellaneous
Platelet clumps	Howell-Jolly bodies	Abnormal red cells
Basophilic stippling	Heinz bodies	Paraproteins
Giant platelets	Pappenheimer bodies	Cold agglutinins
Leukocytes and leukocyte fragments	Parasites (malaria, babesia)	Platelet/erythrocyte coincidence
Nucleated erythrocytes		Hemolysis

EXPECTED VALUES

Bloods from healthy Abbott employees were tested on CELL-DYN Systems. Distributions of reticulocyte percentage results were calculated. The resultant means and standard deviations were used to determine the 95% normal distribution ranges (± 2 standard deviations). Across the CELL-DYN Systems these ranges were 0.88–2.37% for adults.^[2]

SPECIFIC PERFORMANCE CHARACTERISTICS

Refer to the CELL-DYN Ruby Operator's Manual for the instrument specific system performance results.

- RETC_Background (Count)
- Reproducibility (Precision)
- Analytical Measurement Range
- Comparability (Correlation)
- Carryover

For CELL-DYN 3700 and CELL-DYN 3200 Systems, refer to the performance specifications indicated below:

Background Counts

Daily background counts were run over an extended period using tubes from different lots of CELL-DYN Reticulocyte Reagent. In all cases, the total background count/count cycle did not exceed 100 during the study.

Reticulocyte Background Counts: ≤ 100 Counts

Precision

Replicate precision was quantified by running sets of normal specimens as $N=31$ and $N=20$ replicate runs. Paired precision was also run.

CV $\leq 15.0\%$.

Accuracy

Accuracy specifications are determined by correlation to a reference instrument using specimens from healthy donors and hospital patients. The CELL-DYN 3700 was correlated to the Becton Dickinson FACScan and CELL-DYN 4000 with specimens in the range of 0.1–40%. The CELL-DYN 3200 was correlated to a CELL-DYN 4000 with specimens in the range of 0.5–23%.

Each specimen was run twice ($N=2$) using 15-minute staining reagent incubation times. Multiple lots of CELL-DYN Reticulocyte Reagent were used during the specimen acquisition period. The methods described in the reference were used for the analysis^[4].

Correlation Coefficient ≥ 0.90 .

Linearity

CELL-DYN 3700: Linearity was quantified by running six (6) levels of reticulocyte-containing material prepared by using varying concentrations of two stock preparations. The general method described in the referenced NCCLS Standard^[5] was used. Specifically, six concentration levels were run in quadruplicate, and the method of least-squares regression was used for analysis of the reticulocyte percentage result.

CELL-DYN 3200: Linearity was determined by analyzing dilutions of a linearity material that contained no interfering substances and did not display any suspect parameter flags. Specifications were determined by taking multiple measurements on each dilution. The reticulocyte percentage limits were determined by regression analysis.

Linearity Range

For CELL-DYN 3700:

0–30% $\pm 7\%$ or 1.1 absolute

For CELL-DYN 3200:

0.50–23.00% $\pm 7\%$ or 1.1 absolute

Carryover

Carryover was determined by running samples with high proportions of reticulocytes or high RBC concentrations. Each sample was run in triplicate followed by three background cycles.

Refer to the appropriate CELL-DYN Operator's Manual for the formula to calculate % carryover.

Carryover $\leq 0.5\%$

BIBLIOGRAPHY

- [1] Clinical and Laboratory Standards Institute/NCCLS. *Methods for Reticulocyte Counting (Automated Blood Cell Counters, Flow Cytometry, and Supravital Dyes)*; Approved Guideline—Second Edition. CLSI/NCCLS document H44-A2 [ISBN 1-56238-527-5]. Clinical and Laboratory Standards Institute/NCCLS, 940 West Valley Road, Suite 1400, Wayne, PA, 2004.
- [2] Marshall, PN. *Reticulation, Polychromasia and Stippling of Erythrocytes - Review*. *Microscopica Acta* 1978; 81:89-106.
- [3] Kim, YR, Ornstein, L. *Isovolometric Sphering of Erythrocytes for More Accurate and Precise Cell Volume Measurement by Flow Cytometry*. *Cytometry* 1983; 3:419-427.
- [4] Clinical and Laboratory Standards Institute/NCCLS. *Method Comparison and Bias Estimation Using Patient Samples*; Approved Guideline—Second Edition. CLSI/NCCLS document EP9-A2 [ISBN 1-56238-472-4]. Clinical and Laboratory Standards Institute/NCCLS, 940 West Valley Road, Suite 1400, Wayne, PA, 2002.
- [5] Clinical and Laboratory Standards Institute/NCCLS. *Evaluation of the Linearity of Quantitative Measurement Procedures; a Statistical Approach*; Approved Guideline. CLSI/NCCLS document EP6-A [ISBN 1-56238-498-8]. Clinical and Laboratory Standards Institute/NCCLS, 940 West Valley Road, Suite 1400, Wayne, PA, 2003.
- [6] US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Bloodborne pathogens.
- [7] US Department of Health and Human Services. *Biosafety in Microbiological and Biomedical Laboratories*. 5th ed. Washington, DC: US Government Printing Office; December 2009.

[8] World Health Organization. *Laboratory Biosafety Manual*. 3rd ed. Geneva: World Health Organization; 2004.

[9] Clinical and Laboratory Standards Institute. *Protection of Laboratory Workers from Occupationally Acquired Infections: Approved Guideline—Third Edition*. CLSI Document M29-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2005.

Key to symbols used

Consult instructions for use.

Authorized Representative

In Vitro Diagnostic Medical Device

Manufacturer

Catalog Number

Batch Code

Reagent

Reticulocyte Reagent

Temperature Limitation

Exp

Use by

CELL-DYN *Enzymatic Cleaner Concentrate*

PRODUCT INFORMATION SHEET

ABBOTT

CELL-DYN SYSTEMS

INTENDED USE

CELL-DYN Enzymatic Cleaner Concentrate is a special cleaning agent for use on CELL-DYN Hematology Analyzers. Analyzers should be cleaned on the schedule recommended in the appropriate CELL-DYN Operator's Manual, or whenever a repeated clog or obstructed aperture is suspected. CELL-DYN Enzymatic Cleaner Concentrate was specifically developed for the removal of protein buildup and debris (clogs) in counting orifices, flow cells, and associated analyzer tubing. CELL-DYN Enzymatic Cleaner Concentrate is less harmful to delicate plastic and ceramic parts than is bleach (sodium hypochlorite).

INGREDIENTS

Subtilisin	<5.0%	Anti-Microbial Agent	<1.0%
Buffer	<25.0%	Stabilizer	<3.0%
Polyoxyethylene Ether	<10.0%		

PRECAUTIONS

IVD For *in vitro* diagnostic use.

The following warnings and precautions apply to the CELL-DYN Enzymatic Cleaner Concentrate:

DANGER: Contains subtilisin and tris hydroxymethyl aminomethane.

H334 May cause allergy or asthma symptoms or breathing difficulty if inhaled.

H318 Causes serious eye damage.

H315 Causes skin irritation.

H335 May cause respiratory irritation.

Prevention

P261 Avoid breathing mist / vapours / spray.

P280 Wear protective gloves / protective clothing / eye protection.

Safety Data Sheets are available at www.abbottdiagnostics.com or contact your local representative.

STORAGE INSTRUCTIONS

8°C Protect from freezing or excessive heat. Cloudiness from cold storage or color change from lot to lot (yellow–dark brown) will not affect the effectiveness of CELL-DYN Enzymatic Cleaner Concentrate.

2°C

PROCEDURES

Bring cleaner to room temperature before use.

Refer to CELL-DYN Operator's manual for specific instructions on how to clean your automated CELL-DYN analyzer.

Instruments with Exposed Transducers - Place 10 drops (0.5 mL) of enzymatic cleaner concentrate in a sample cup, add 10 mL of lukewarm (not hot) water and mix. Place cup on sample stage and press RUN button. After cycle completes, allow cleaner to work for 5–10 minutes. Remove vial, rinse outside of transducer with the appropriate diluent and place a new cup containing 20 mL of fresh diluent (only) on sample stage. Cycle instrument a minimum of 3 times.

Instruments with Internal Transducers - Consult the appropriate CELL-DYN Operator's Manual.

LIMITATIONS OF THE PROCEDURE

- This product is intended to be used as a cleaner for CELL-DYN hematology analyzers and should not be considered to have disinfectant properties.
- Diluted concentrate should be discarded after six hours.
- CELL-DYN Enzymatic Cleaner Concentrate solution must be thoroughly removed from the analyzer before running specimens.**
- Ensure the Wash Block and/or the Probe are free from Enzymatic Cleaner after completion of the Auto Clean and Extended Auto Clean. Consult Operator's Manual for cleaning instructions, if appropriate for your system.**
- Do not use product beyond expiration date.

ORDERING INFORMATION

REF 99644-01 CELL-DYN Enzymatic Cleaner Concentrate 2 x 50 mL quantity.

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CELL-DYN is a trademark of ABBOTT LABORATORIES in various jurisdictions. CELL-DYN ist eine Marke von ABBOTT LABORATORIES in verschiedenen Ländern. CELL-DYN est une marque commerciale d'ABBOTT LABORATORIES dans plusieurs pays. CELL-DYN es una marca comercial registrada de ABBOTT LABORATORIES en varios países. CELL-DYN è un marchio commerciale di ABBOTT LABORATORIES in diverse giurisdizioni.
Abbott GmbH & Co. KG