Citric Acid 50 %

Product Information

Liquid concentrate for the cleaning, decalcification and heat-disinfection of hemodialysis machines

Composition and quality

1000 ml contain 500 g anhydrous citric acid (acc. to Ph. Eur) in aqua purificata.

Properties

Citric Acid 50%

- is the agent to use for optimum decalcifying with cleaning effect
- moreover Citric Acid 50% has excellent disinfecting properties: bactericidal, fungicidal, tuberculocidal, virucidal (incl. HBV, HCV, HIV) at 83 °C, exposure time 15 minutes
- the active substance is listed in the DGHM¹⁾ list
- is economical because only a low application concentration is required
- requires only little time per application
- is safe to use and neutral to the environment

¹⁾ German Society for Hygiene and Microbiology

Characteristics

The efficacy of citric acid is extremely well supported by expert opinions:

- Expert report on sporicidal effect, Prof. Dr. Ohgke / Prof. Dr. Solbach, Lübeck, 1998 (Dialog dialysis machine)
- Expert report on virucidal effect, Priv.-Doz. Dr. Thraenhart, Eurovir Hygiene Institute, Essen, 2003

This expert report was prepared on the basis of comprehensive tests with the extremely resistant Parvovirus which is mentioned as a test virus in EN 14476 for the testing of chemo-thermal processes at temperatures exceeding 30 °C. Result: Citric Acid 50% acts effectively against Parvovirus, enteroviruses, hepatitis B virus, hepatitis C virus and HIV virus.

Field of application

Draw the concentrate in through the disinfectant nozzle. The working concentration is 3%.

Please note the manufacturer's instructions for the dialysis machine.

Notes

Mild irritation of skin and mucous membranes. In case of contact with the eyes, rinse thoroughly with water and consult your doctor. If swallowed drink plenty of water. In case of accidental contact with skin, wash with plenty of water immediately. During work, wear safety gloves and goggles. Do not mix Citric Acid 50% with chlorinated products. Keep container closed when not in use.

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B. Braun Avitum AG