-  Optional Sensor
-  Optional Third Overflow Safety Bottle

# LIFETIME SA01HT

Product Page

Designed to evacuate the fluid accumulated in the operated areas of the patients during the surgical operations.

### LED Electronic User Panel

It allows all settings to be made electronically on a single panel.

### Electronic Jar Replacement

It is now much easier to replace the jar electronically with its special design.

### Automatic Shutoff Valve

When the jar is full, the device automatically shuts itself off and prevents liquid overflows.

### Electronic Foot Switch

It provides a more comfortable use to the user without touching the device during the operation.

### Autoclavable Jar

Thanks to its autoclavable polycarbonate jars at 121 degrees, it provides hygienic use for many times.

### Low Noise Level

With a sound level of 42 dB, it is one of the lowest among competitors.

### Technical Specifications

Compressor	Oilless Piston Pump
Voltage	230 VAC $\pm$ 10% , 50 Hz
Max. Free Air Flow	90lt or 60lt/min $\pm$ 10%
Max. Vacuum	-700 mmHg $\pm$ 5%
Power Consumption	Max. 500W
Vacuum Setting	Yes
Collection Jar	2x (2-3-5 lt)
Float System	Yes
Hydrophobic Filter	Yes
Standard	ISO 10079 – 1 / CE
Device Dimensions	38x30x34 cm
Weight of Device	11 kg
Box Dimensions	44x38x40 cm
Gross Weight (Device)	13 kg
Box Dimensions (Stand)	50x48x72 cm
Weight (Stand)	12 kg
Gross Weight (Stand)	13 kg
Stock Code	HT01B000
Warranty Period	2 years

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Please read this manual thoroughly before use of the Device

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### SUBJECTS

### PAGE NO

1. INTENDED USE	5
2. CAUTIONS	5
3. DEVICE DESCRIPTION	6
4. ACCESORIES	8
5. TECHNICAL FEATURES	10
6. SETUP AND CONTROLS	12
7. APPLICATION	15
7.1. APPLICATION OF OPTIONAL FOOT PEDAL	15
7.2.SETTING VACUUM LEVELS	15
7.3.EVACUATION OF THE JARS	15
8. SAFETY SYSTEMS	17
9. REQUISITS DURING APPLICATION	17
10. MAINTANENCE AND CLEANING	19
11. CHANGING THE FUSE	19
12. TROUBLESHOOTING	21
13. ELECTRICAL CIRCUIT DIAGRAM	21
14. HANDLING AND STORAGE	23
15. WARRANTY	23
16. COPYRIGHT	23
17. CERTIFICATES	23
18. MANUFACTURER COMPANY INFORMATION	23

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SERİ NO

**SN**

CAUTION  
CONSULT  
ACCOMPANYING  
DOCUMENTS

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## 1.INTENDED USE

LIFETIME Surgical Aspirator Devices are used for picking up the fluids and particles in surgical stage to a separate collection jar by the negative pressure generated with the vacuum pump.

LIFETIME Surgical Aspirator Devices are used in; operating theaters, ICUs, emergency rooms, polyclinics and clinics.

## 2.CAUTIONS

Please make sure that you have read and understand this user manual along with its cautions and descriptions before setup.

The person or the establishment buying this device is responsible for the content of this manual and instructions of the relevant personnel.

Do not use the device except for intended use.

The device should be operated by trained personnel.

The device should be plugged into a grounded outlet with the norms of TS3166 electrical wiring.

High Vacuum and High Flow generating device can be used in continuous or discontinuous modes.

The device's lifetime is 10 years. At the end of this time the device should be considered "contaminated medical waste".

Perform periodic maintenance of the device on regular basis.

Replace the filter after septic cases.

If hydrophobic filter is used in the device, fluid contact will stop the aspiration system. In this case, please change the filter.

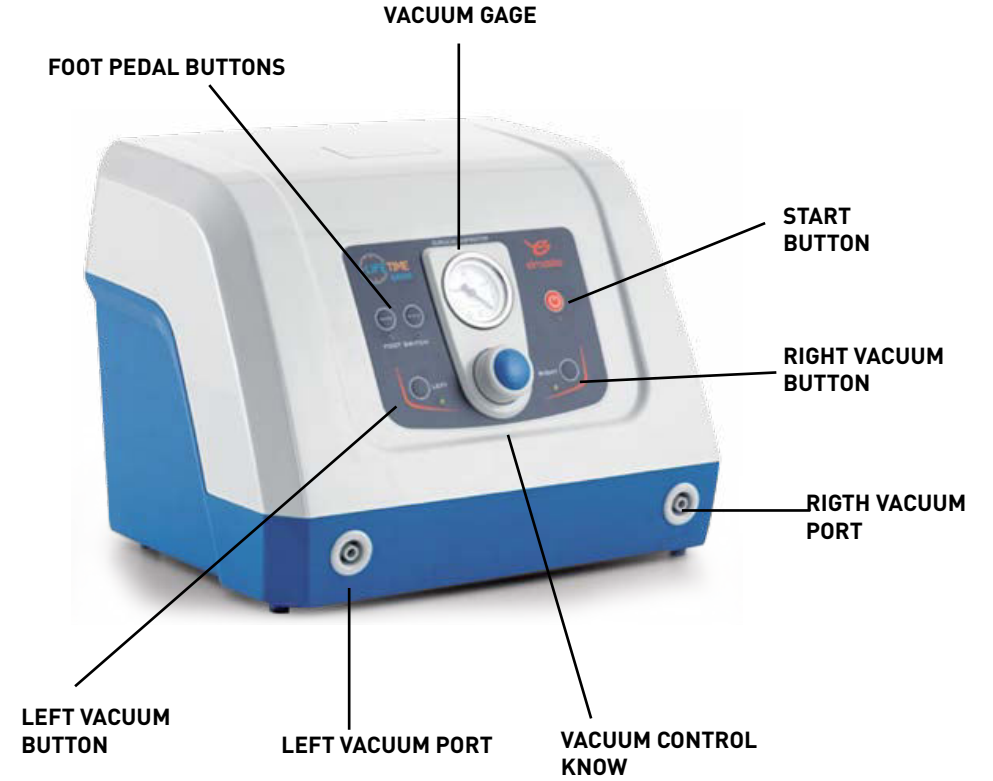
Precautions should be taken at the power input against sudden voltage drops. Voltage regulator is recommended.

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
### 3.DEVICE DESCRIPTION

- START BUTTON** : It is used to start the vacuum pump.
- RIGHT VACUUM BUTTON** : To activate the right vacuum port.
- LEFT VACUUM BUTTON** : To activate the left vacuum port.
- RIGHT VACUUM PORT** : Used for vacuum
- LEFT VACUUM PORT** : Used for vacuum
- VACUUM CONTROL KNOB** : It is used to set the vacuum level.
- VACUUM GAGE** : It is used to show the vacuum flow when the device is working.
- FOOT PEDAL BUTTONS** : It is used for control of the device with foot. There are two modes; continuous and start – stop.

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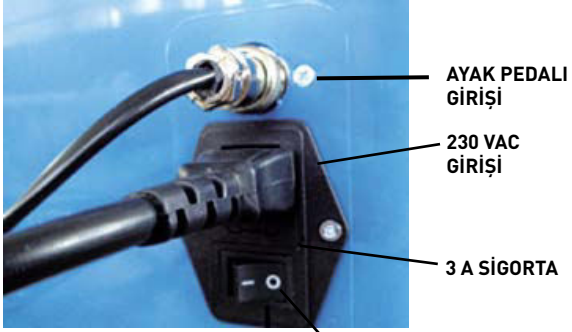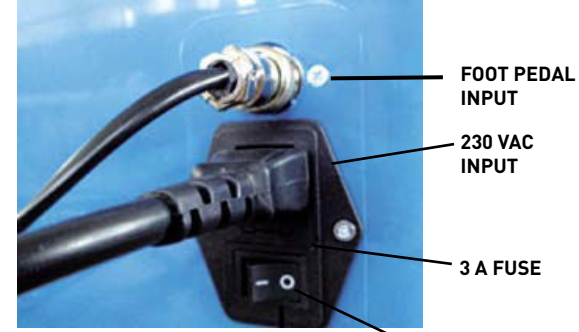
**FOOT PEDAL** : For the use of the device with foot.

**ON-OFF BUTTON** : It is placed in the back of the device and it opens or closes the electricity coming to device.

**3 A FUSE** : For device protection against electrical hazards.

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## 6.SETUP AND CONTROLS

Open the box and take out the device and other accessories from the box.

Check for any damages that can be occurred during transport. Please refer to the related sales point.

If your device is w/ trolley model, assemble the trolley with the screws provided.

Please mount the jars to the device with the holding apparatus. Be sure to make the hose connections.

Please check the following controls in the first setup and before each use ;

- Check to see if there is any cracks on the jar lid that will lead to an air leakage.
- Check the jar lid and overflow protection system.
- Make sure the jar lid seal is connected to the jar lid and jar lid is correctly connected to the jar.
- Make sure the jars are correctly mounted to the device.
- Check if there is any wear, opening etc. On the aspiration hose.
- Check for any damages on the electrical cable. Do not use damaged electrical cables.
- Turn on the ON/OFF button in the back of the device and make sure the right or left vacuum buttons illuminate.
- Press Start button to run the vacuum pump.
- Check the vacuum gauge by blocking the suction port.

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## 7. APPLICATION

Plug-in the electric cord of the device.  
Turn on the OFF/ON button of the device located at the back.  
Switching on the device, either the right or the left vacuum buttons of the device illuminates.  
Decide which vacuum port right or left one you want to use.  
After the selection of right or left port for suction press on START button, the device starts running.  
You can change the suction port by pressing either right or left vacuum button when the device is running. To stop the vacuum please press START button once again.

### 7.1. Application of optional Foot Pedal feature.

Press the foot pedal when the device is in stop position to activate this feature. Then press START button to start vacuum pump.

The Foot Pedal has two alternative applications, staccato and continuous. One can select the application as desire.

### 7.2. Setting the vacuum levels.

The vacuum adjustment system is available in the device. This helps you to use the device in the desired vacuum value. To make the adjustment please run the device by pressing START button and the vacuum pump will be activated. Please take the reading of vacuum gauge after closing the tip of the suction port. Adjust the setting to desired value using the adjusting nob.

### 7.3. Evacuation of Collection Jars

During the use of the device please control the waste accumulated in the collection jar in a regular interval. The collected waste level should not exceed the "Maximum" level mark on the jar. To evacuate the collection jar;  
Turn off the device and remove the input-output hoses from the collection jar lid. Take off the collection jar from the accessory rail by holding it up. Open the lid of the collection jar taking it outside the clinic. Consider the collected waste liquid inside the collection jar as "contaminated medical waste" and always evacuate carefully outside clinical zone. Please subject the discharged collection jar disinfection treatment. Cover the lid and be sure that it has been closed tightly otherwise, leakage will occur between the collection jar and its lid causing loss of device's vacuum performance.  
This instruction should strictly be followed to avoid overflow of the waste liquid and elope into the pump during evacuation process. Otherwise, as the performance of the device will reduce but this will cause malfunctioning the pump which will shorten the life of the device.

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## 8. SAFETY SYSTEMS

**1. Float System:** it's located under the collection jar lid. If the accumulated waste exceeds the maximum level in the jar and the evacuation does not take place then the float system actuated and protect the vacuum pump from waste. When this system is activated, the device stops working and the jar do not collect excess waste.

**2. Bacteria Filter:** mounted on the hose between the device and the collection jar. Filter the air vacuumed from collection jar towards the device and helps to dispose clean air to the environment. At the end of its life the filter is blocked and the devices can't perform suction. Filter should be replaced with a new one.

**3. Hydrophobic Filter:** This is an optional system mounted on the hose in between the device and the collection jar. It filters the air vacuumed from collection jar toward the device and helps to dispose clear air to the environment. At the end of its life the filter is blocked and the devices can't perform suction. Filter should be replaced with a new one. It also prevents to escape of waste to the device from the collection jar.

## 9. REQUISITES DURING APPLICATION

To avoid misuse and applications which may harm the patient, user or device guidelines should be strictly followed.

Do not damage the power cord.

Please control robustness of the power cord before use each time.

Do not use damaged power cords. Use the device by plugging definitely in a grounded power socket.

Do not use too long (more than 2 meters) aspiration tube. Increase of aspiration tube length may cause to low vacuum level.

Prevent implementation of any weight or force on the aspiration hose which may cause clogging of the hose.

Please use clean aspiration hose and collection jar each time. In case of emergency please keep enough collection jars and aspiration hose in tow. At the end of the use of the device the OFF/ON switch located at the back of the device should be turned off.

Do not use the device near explosives or in Oxygen-rich environment. Otherwise, it may elicit the danger of explosion or fire.

Perform the periodic maintenance of the device regularly by authorized personnel.

Do not run the device once the waste level reaches to the maximum before discharge the collection jar.

Do not use the device for the purpose other than described in this manual.

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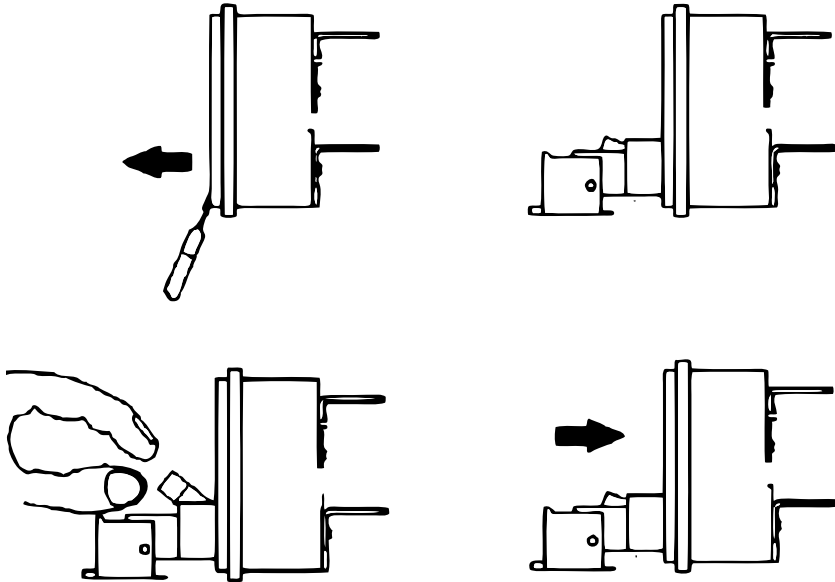
## 10. MAINTENANCE AND CLEANING

Please be sure that the device has been switched off and the power cord has been unplugged before proceed for maintenance and cleaning of the device. Wear protective gloves during this procedure.

### 10.1 CLEANING OF THE DEVICE

Each time before using the device disinfect the outer surface of the device, collection jars and lids using medical disinfectant. The disinfectant should be used as per its own operating instructions.

### 10.2. STERILIZATION

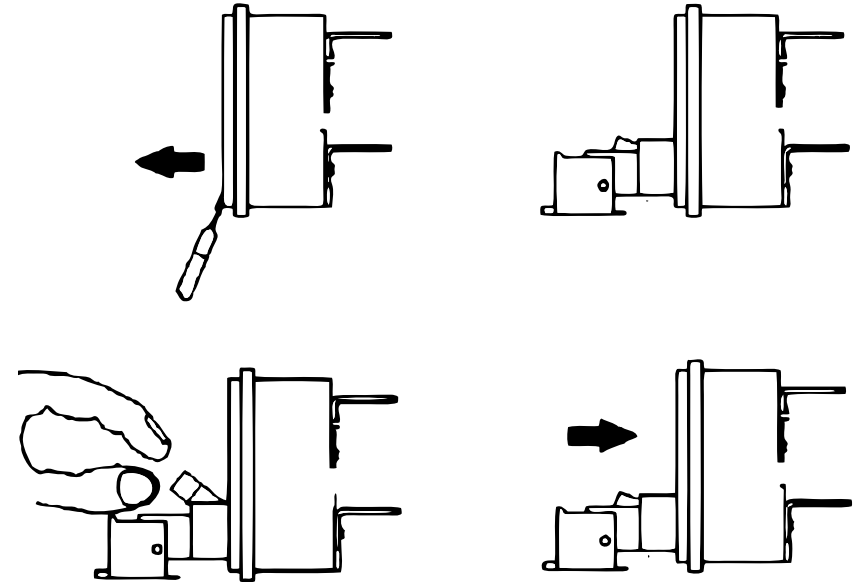
The collection jars of the device can be autoclaved at 134°C after discharging the collected waste and cleaning the jars.

### 10.3. CHANGE OF FILTER

Replace the bacteria filters and the hydrophobic filters used in the device with new ones when soiled or obstructing the vacuum process.

## 11. CHANGING THE FUSE

Unplug the power cord of the device from the plug socket and pull off the fuse cover located below the input power socket using a screwdriver. Remove the defective fuse and replace with a new one. Cover the fuse by pushing it off.

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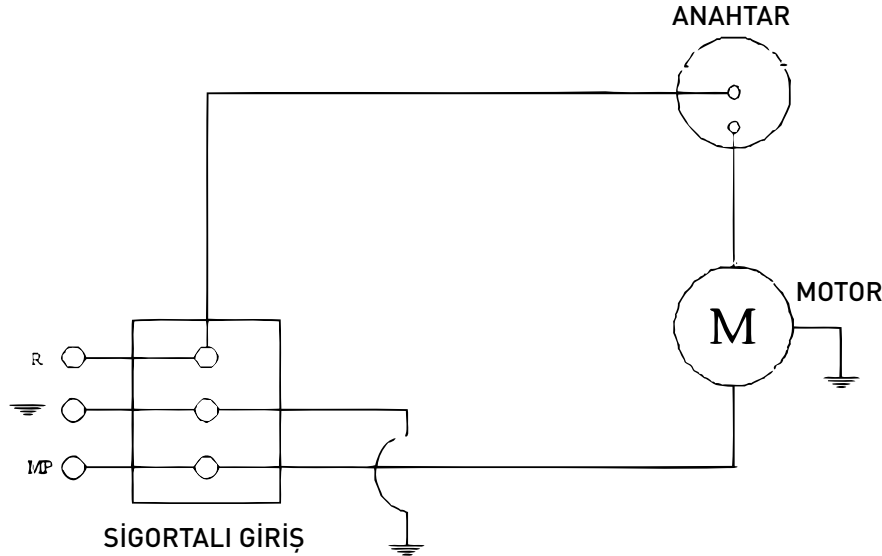
## 12. TROUBLESHOOTING


When there is malfunctioning in the device use the table below. If you encounter with a breakdown not mentioned in the table please contact an authorized technic service.

FAULT	CAUSE	SOLUTION
Motor is not running	No power in the motor Power cord is disconnected Motor is faulty	Check the fuse of the device check the power cord and the plug contact an authorized technic service
Motor running, signal lamps are not lighting	Faulty signal lamps	Contact an authorized technic service
Motor running, low vacuum value	Dirty Filters Filters has been clogged Collection jar filled up Lids of the jars not placed properly Faulty vacuum gauge	Replace the filters Discharge the waste from the jar Check the lids Contact an authorized technic service
Abnormal sound from the device	Defective motor Worn-out motor vibration dampers Waste entered into the motor	Contact authorized technic service
The device doesn't move freely	Defective castors	Contact Authorized technic service

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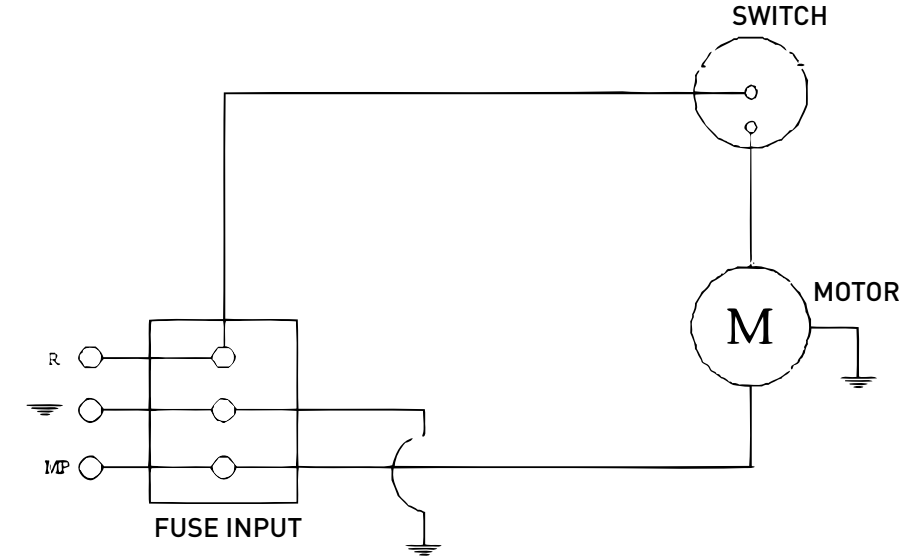ELMASLAR A.Ş.  
ASO 1.OSB BABÜRŞAH CD NO : 17 SİNCAN ANKARA

## 14. HANDLING AND STORAGE

The device and accessories must be transported in its original box. If you have no original box please make a suitable package using Styrofoam and carton box to prevent damage during transport of the device. The device should not be flipped during transport. Please act in accordance with the instructions contained on the box of the device during transportation. Store the device at normal room condition.

## 15. WARRANTY TERMS

1. The warranty period starts from the date of delivery of the goods to the customers.
2. If the device malfunction within warranty period, the time spent in repairing the device added in the warranty period.
3. In the event of complaint about the device, please send the device to your device provider or send to our company with prepaid courier along with its original packaging and warranty certificate inside.
4. The warranty doesn't cover the faults occurred due to improper use of the device.
5. Opening the device by a non-authorized person will cause invalidity of the warranty.
6. The faults resulting from the use of the device contrary to the clauses mentioned in the user manual, warranty doesn't cover.
7. This warranty given to the devices implies, once all these terms mentioned here are accepted.
8. Consumable parts are not covered under warranty

## 16. COPYRIGHT

This document has been prepared by ELMASLAR TIBBI CIHAZ İMALAT A.S. and reserves the right to make changes to product specifications. No part of this publication may be reproduced or copied without written permission of the company.

## 17. CERTIFICATES

ISO 9001 , ISO 13485 , CE

## 18. MANUFACTURER COMPANY INFORMATION

ELMASLAR A.S.  
ASO 1.OSB BABURSAH CD. NO: 17 SINCAN / ANKARA / TURKEY

## **ELMASLAR A.S.**

ASO 1.OSB BABURSAH CD. NO: 17 SINCAN / ANKARA / TURKEY

Tel: +90 312 394 80 01 Fax: +90 312 394 80 04

info@elmaslarmedikal.com.tr • elmaslarmedical@gmail.com

[www.elmaslarmedikal.com.tr](http://www.elmaslarmedikal.com.tr)

SURGICAL ASPIRATOR

LIFETIME

elmaslar

FOOT SWITCH

LEFT

RIGHT