



ALLMED MEDICAL PRODUCTS CO.,LTD.

No.18 Qixing Road, Majiadian Town,443200
Zhijiang City, Hubei province, (China)

GAUZE BANDAGE

TECHNICAL PROPERTIES

Content: 100% bleached cotton yarn;
Weft-Warp String number: 40Sx40S/30x30-124cm, $\geq 32\text{g/m}^2$;
Width: 10cm, 14cm, other sizes available;
Length: Can be cut as required;
Shelf life: 5 years


For and on behalf of
ALLMED MEDICAL PRODUCTS CO., LIMITED

Susan Liao

.....
Authorized Signature(s)

Non-Sterile Hydrophilic Gauze Bandage

REF Code	SIZE	BOX in (Roll)	Box Size (cm)
AM-GB10500	10cm x 5m	540	45 x 36.5 x 38 cm
AM-GB14700	14cm x 7m	360	51 x 48.5 x 31 cm

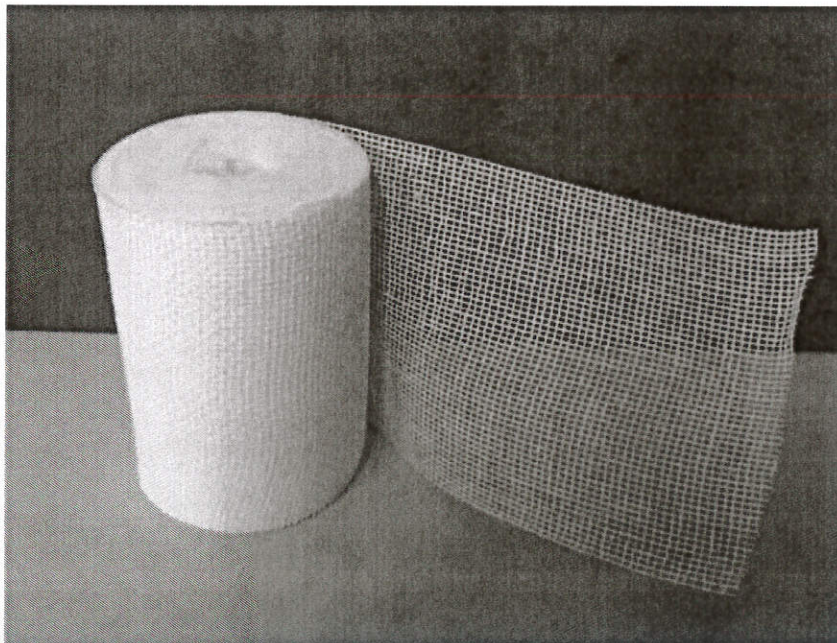

Intended use:

This device is intended to be used for fixation of dressings or tampon on the wound, with protection and barrier function (mechanical) of the injured skin. It provides mild compression and support. MD Class I according to Regulation (EU) 2017/745 of the European Parliament and Council.

Product standards: In compliance with EN 14079.

Certificates: CE, ISO13485.

Labeling: in accordance with Chapter III (23) of Annex I of Regulation 2017/745 of the European Commission.



ALLMED MEDICAL PRODUCTS CO.,LTD.

No.18 Qixing Road, Majiadian Town,443200

Zhijiang City, Hubei province, (China)

Cautions:

- Read instructions before use.
- Before usage check the packing completeness, the LOT number and expiry date.
- Use only if package is undamaged and validity not expired;
- For single use.
- Destroy after use.

Properties:

- Non-toxic;
- Hypoallergenic;
- Non-inflammable;
- PHT and DEHP free;
- Latex free;
- Soft and comfortable;
- Low linting;
- High absorbance capacity;
- Easy to cut;
- Free from loose wires;
- Non-Sterile unit packages.

For and on behalf of

ALLMED MEDICAL PRODUCTS CO., LIMITED

Susan Liao

.....
Authorized Signature(s)

Instructions for use:

1. Select the appropriate Gauze Bandage size for your procedure.
2. Examine individual package for integrity of packaging prior to use. Do not use if packaging is broken.
3. Wash hands. (Wear gloves where necessary).
4. Before usage, open the unit package.
5. Apply the bandage in accordance with your facility's protocol.
6. Support the injured area while bandaging (if necessary).
7. Immediately dispose of after single use.

Packing:

- each bandage is rolled and individually packed
- then is packed in strong export carton

Storage:

Protect from sunlight, extreme humidity and heat sources. Not store with organic or chemical solvents. Not store with corrosive gases. Store in dry, cool, good ventilated and clean room, where temperatures between 10°C and 40°C and relative humidity between 30% and 60% are recommended



ALLMED MEDICAL PRODUCTS CO.,LTD.

Address: No.18 Qixing Road, Majiadian Town,443200, Zhijiang City, Hubei province, China.