

# MYS-1050N

PATIENT TRANSFER CHAIR



## USER MANUAL



**MEYOSİS HASTANE EKİPMANLARI SANAYİ VE TİCARET A. Ş.**

**Address:** Küçükbakkalköy Mh. Dudullu Cd No:23-25/B Brandium Rezidans R2 Blok Kat:23 D:217  
Ataşehir – İstanbul/ TURKEY

**Tel:** (+90) 216 574 67 38 **Fax:** (+90) 216 574 67 39

**http://** [www.meyosis.com](http://www.meyosis.com) **E-mail:** [info@meyosis.com](mailto:info@meyosis.com)

**PRODUCT:**

**TRADE MARK** : MEYOSIS  
**MODEL** : MYS-1050N  
**NAME** : PATIENT TRANSFER CHAIR

**1. USER MANUAL INSTRUCTION:**

**Brake Mechanism:**

- a) The wheels are locked by default.
- b) **To release the central brake:** Pull the bar with foot to up position.
- c) **To activate the central brake:** Press the bar to down position mechanism.



*Figure 1*



*Figure 2*



*Figure 3*

**Movements:**

- a) The chair comes in the position that is seen on Figure 1 by default. are locked by default.
- b) Back rest and leg rest move synchronously by gas spring.
- c) To set up Figure 2 & Figure 3 positions, pull the red handles located on the left and the right sides of the chair and push the back rest downwards at the same time. Go on pushing the back rest downwards until the required angle is achieved.
- d) Pull the foot board that is located underneath of the chair mechanically by hand.

### **3. CARE INSTRUCTIONS:**

- Any kind of cleaning materials can be used for cleaning the product. The recommendation for Aluminum and Stainless steel components is to only wipe them with a soft and humid cloth and not to use any chemical agents to clean them.
- Please make sure that the cleaning cloths used to clean the product are lint-free.
- The cloth should not be pressed onto the product in such a way as to cause damage during the wiping action. Failure to implement one or more of these commands cause the proper product use to be prevented and the product to become outside the warranty coverage.

### **4. SAFETY RULES:**



- This product has not been manufactured for carrying loads. Therefore, please do not place or carry any loads on it.
- Please store the product in a dry and non-humid environment.

### **5. SPARE PARTS LIST:**

- Brake Handle
- Armrest Part,
- Wheels,
- Footrest Part

### **6. MANUFACTURER**

#### **MEYOSİS HASTANE EKİPMANLARI SANAYİ VE TİCARET A. Ş.**

**Address:** Küçükbakkalköy Mh. Dudullu Cd No:23-25/B Brandium Rezidans R2 Blok Kat:23 D:217 Ataşehir – İstanbul/ TURKEY

**Tel:** (+90) 216 574 67 38 **Fax:** (+90) 216 574 67 39

**http://** www.meyosis.com **E-mail:** info@meyosis.com

### **7. AUTHORIZED SERVICE**

#### **MEYOSİS HASTANE EKİPMANLARI SANAYİ VE TİCARET A. Ş.**

**Address:** Küçükbakkalköy Mh. Dudullu Cd No:23-25/B Brandium Rezidans R2 Blok Kat:23 D:217 Ataşehir – İstanbul/ TURKEY

**Tel:** (+90) 216 574 67 38 **Fax:** (+90) 216 574 67 39

**http://** www.meyosis.com **E-mail:** info@meyosis.com

