



Logistic details			
Capacity	Pcs / Box	No. of boxes / Pallet	Pcs / Europallet
Box	12	80	960
Refill	12	112	1344

Velox® Wipes NA

Alcohol-free wipes for cleaning and disinfecting sensitive surfaces of non-invasive medical equipment

Product features:

- ▶ Active in 1 min. against Bacteria, Fungi, enveloped viruses and in 5 min. against Tuberculosis.
- ▶ For surfaces sensitive to alcohol.
- ▶ Destined for disinfection of various medical equipment surfaces made of glass, porcelain, steel, rubber, plastic and acrylic glass, operating chairs, rehabilitation equipment, incubators.
- ▶ Recommended for ultrasound lamps and heads.
- ▶ 2 Non-woven wipes 13 x 20 cm, 23 g / m².

Directions for use:

Place the refill in the box. Take a tissue out of the box and wipe a surface to be cleaned and disinfected. Close the lid. After opening the package, use the tissues within 30 days. Do not use if a tissue is dry. The surface does not need to be rinsed with water. Before applying on particularly sensitive surfaces, test the wipes on a non-visible area. For professional use only.

Application:

Velox® Wipes NA are recommended for disinfecting surfaces of non-invasive medical equipment made of plastic, acrylic glass, stainless steel, metal, aluminium, rubber and porcelain. Suitable for disinfection of medical appliances, rehabilitation equipment, treatment chairs, incubators, the heads of ultrasound machines and lamps in intensive care units and operating theatres. Free from alcohol, aldehydes and phosphates, the wipes do not discolour the surfaces. The product has a pleasant scent. The product is suitable for medical equipment surface having contact with food. Tested and recommended by Famed - a leading European producer of top quality hospital equipment.

Wipes

Active in 1 min. against B, F, V

Non-invasive medical devices

0% Alcohol-free product

Ingredients:	Effectiveness	EN 14885	Conditions	
			clean	dirty
100 pieces of tissues contain: 173 g of saturated formulation 100 g contains: 0.15 g N- (3-aminopropyl) -N-dodecylpropane-1,3-diamine, 0.14 g Poli (oxy-1,2-ethanedyl), alpha - [2-(didecylmethylamino) ethyl] - omega. hydroxy, propanoate (salt)	Bacteria (including MRSA)	EN 16615	60 sec.	3 min.
		EN 13727	60 sec.	60 sec.
	Fungi (C. albicans)	EN 13624	60 sec.	60 sec.
		EN 16615	60 sec.	60 sec.
	Mycobacteria tuberculosis (M. terrae)	EN 14348	5 min.	10 min.
Symbol	Enveloped viruses (HIV, HBV, HCV, Ebola, Influenza, Vaccinia, BVDV, Herpes simplex, SARS-CoV-2)	EN 14476	60 sec.	60 sec.
SSE-GOT-ML993 → box SSE-GOT-ML986 → refill				

Packaging units: 100 pieces box, 100 pieces refill
Non-woven wipes: 13 x 20 cm, 23 g/m²