

Intervention time

2 hours

Periodicity

Once a year for less than 50 cycles per day

Twice a year for more than 50 cycles per day

Specific kit or consumables

Maintenance kit: 1300033061



Disposable gloves, eye protections and a lab coat must be worn by the operator.
Local or national regulations must be applied in all the operations.

1. Maintenance Kit Overview

The maintenance kit consists of the following elements:

Reference	Designation	Quantity
FAA040A (1202401040)	O'RING,5DIFF SYRINGE D=12,1	2
XDA622A (1209131622)	O'RING,REAGENT SYR. + WASHER	1
FAA053A (1202401053)	O'RING,SAMPL. NEEDLE MIC/YUMI	1
FAA055A (1202401055)	O'RING,MICROS SAMPLING SYRINGE	2
FAA067A (1202401067)	O'RING,5DIFF SYRINGE D=2,4	4
FAA017A (1202401017)	O'RING,TANK WASTE MIC/YUMI	1
1300020486	O RING, APERTURE FLAT D4.1 BLACK	2
FAA046A (1202401046)	O'RING,COAXIAL CABLE	1
DAJ007A (1201941007)	OPTICAL,LAMP FOR BENCH	1
XEC297A (1209171297)	KIT,WASTE LINE TUBINGS YUMI	1
XEC298A (1209171298)	KIT,TRANSFERT LINE TUBINGS YUM	1
XEC299A (1209171299)	KIT,REAGENT LINE TUBINGS YUMI	1
FAA054A (1202401054): CT only	O'RING,SAMPLE NEEDLE CT/C+/P80	2
FAA036A (1202401036): CT only	O'RING,FLOW CELL+LYSE DISP.MIC	2