OPHTHALMIC CANNULAS


Instructions for Use

INTENDED USE

Ophthalmic cannulas are micro-tubes ending in tips which have a plethora of shapes and sizes. They assist surgeons to perform certain tasks (e.g. for the delivery or removal of fluids) in a variety of ophthalmic surgical procedures, such as cataract, refractive, vitreoretinal and glaucoma operations. They are based on a hub which can be attached on top of a sharp pointed instrument in order to be inserted through precise incisions into cavities of the eye or of the surrounding area.

FURTHER INFORMATION

1. Construction

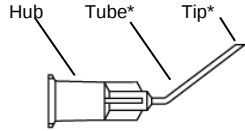(* Shapes vary per product description)

2. Materials

Hub : Polypropylene (colour-coded per Gauze according to ISO 6009)
Tube : Stainless Steel

3. Packaging

Each ophthalmic cannula is individually packed in a pouch to ensure sterility; sold batched in box.

WARNINGS / PRECAUTIONS

- Read, follow and keep these Instructions for Use.
- Refer to the intended use and use the product only in accordance with it.
- Use is allowed only by skilled medical personnel.
- Store any new products in a dry, clean and safe place.
- Avoid high temperature, high humidity, direct sunlight, soaking and preserve at room temperature.
- Do not preserve at a place with chemical products and gas occurrence.
- Preserve package material at a non-break and non-pinhole status
- Confirm the use period on each package and box and dispose those whose use period is expired.
- Prior to each use, inspect the product for its integrity.
- If product is damaged or defective, do not use it and set it aside.

CONTRAINDICATIONS - PROHIBITIONS

- Re-use is prohibited.
- Re-sterilization is prohibited.
- Do not use it after expiry date.
- Do not use it if product's aseptic condition is considered spoiled by broken package.

SYMBOLS AND EXPLANATIONS

	Caution! Refer to accompanying document
	Do not re-use
	Do not use if the package is damaged
	Keep away from sunlight
	Sterilization by ethylene oxide
	Do not re-sterilize
	Manufactured by
	Lot Number
	Manufacture date
	Expiry date
	CE marking and notified-body No.
	Contribution to recycling system

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