---

## Air Leak Tester **LT-7F** Ver.2

---

---

Version / Версия / Verze / Έκδοση / Versión / Versioon / Versio / Leagan / Verzija / Verzió / Útgáfa / Versione / Versija / Veržjoni / Versie / Versjon / Wersja / Versão / Versiunea / Verzia / Različica / Versiyon / 版本	4.0	2022-03-17
--	-----	------------

---

en	bg	cs	da	de	el	es	et	fi	fr	ga	hr	hu	is	it
lt	lv	mt	nl	no	pl	pt	ro	sk	sl	sv	tr	zh		

## 1. Important Safety Information

### For the USA Market - CAUTION:

Federal law restricts this device to sale by or on the order of a physician.

### <Intended Use>

This product is designed to apply air pressure to the inside of a FUJIFILM endoscope in order to check for air leaks from the endoscope. Do not use this product for any other purpose.

### <Precautions in Using This Product>

This product is intended for use by medical professionals who have received proper training. Only medical professionals who have received proper training are allowed to use this product.

Wear appropriate personal protective equipment when cleaning and disinfecting (or sterilizing) the endoscope including leak testing. Before using this product, carefully read this manual as well as the Operation Manual (Cleaning, Disinfection and Storage) or Reprocessing Manual of the endoscope in use.

### <Preparation and Inspection Before Use>

Prior to using this product, prepare a spare one to avoid unexpected accidents such as equipment failure.

Check this product before use according to the procedures provided in this manual, to avoid unexpected accidents and to take full advantage of this product's capabilities. If inspection shows any abnormality, do not use the product.

### <Disposal>

#### WARNING

- When disposing of this product, comply with local laws and regulations in your area. Determine whether or not the product is to be treated as a biohazard, then handle and dispose of accordingly.
- Before disposing of this product, perform cleaning and disinfection (or sterilization) according to the procedure described in this manual. There is a risk of being a source of infection.

When disposing of this product, handle it in a manner which will prevent the spread of contamination from the reprocessing area, and follow all applicable national and local laws regarding disposal.

### <Incident Reporting>

Regulation (EU) 2017/745 requires that any serious incident that has occurred in relation to the device should be reported to the manufacturer or European Authorized Representative, and the competent authority of your country. In case you become aware of any serious incident, please report immediately to your local FUJIFILM dealer and the competent authority of your country.

### <Conventions Used in This Manual>

This manual uses the following conventions for easier understanding.

Convention	Description
<b>WARNING</b>	Explains dangerous situations that may cause death or serious injury if not avoided.
<b>CAUTION</b>	Explains situations that may cause injury if not avoided. Explains situations that may cause damage to equipment if not avoided.
(1), (2), (3), ...	Indicates consecutive numbers in operating procedures for the order in which successive steps in the procedure should be taken.
[Note]	Indicates a comment or supplementary information.

## <Symbols>

The following explains the safety signs used in this product.

Symbol	Description
	Lot number
	Date of manufacture
	Manufacturer
	Authorised representative in the European Community
	Authorised representative domiciled in Switzerland
	Consult instructions for use
	Temperature limitation
	Keep dry
	Non-sterile
	CE marking
	Humidity limitation
	Atmospheric pressure limitation
	Quantity
MADE IN JAPAN	Made in Japan

## 2. Method of Use

### WARNING


- Do not store this product together with the endoscope and accessories that have been reprocessed. The endoscopes and accessories may be re-contaminated, causing infection.
- Store this product in a place out of reach of persons who have not received proper training. If the air leak tester is touched carelessly, it may cause infection.

### CAUTION

- Do not perform leak testing with an air leak tester suspected to be in an abnormal condition. The endoscope may malfunction.
- When attaching this product to the endoscope, securely fix the connector of this product. If it is not fully and properly attached, the interior of the endoscope will not be properly pressurized and accurate leak testing will be impossible.
- When attaching this product to the endoscope, ensure that the connector of this product and the ventilation connector of the endoscope are thoroughly dry. Do not attach or remove this product while the endoscope is immersed in water or chemical solution. If moisture enters this product, do not attach it to the endoscope. If water droplets enter the connector or tube of this product, dry it out before use. Water on the surface of this product or inside this product may enter the endoscope, causing endoscope failure.
- When supplying air into the endoscope with this product, apply pressure until this product reaches the testing pressure (20±1.5 kPa). If the pressure is below the specified range, the inside of the endoscope is not pressurized properly and accurate leak testing will be impossible.
- When supplying air into the endoscope with this product, apply pressure until this product reaches the testing pressure (20±1.5 kPa). If the pressure exceeds the specified range, it may cause endoscope failure.
- When detaching this product from the endoscope, release air until the indicator falls and returns to its starting position (0 kPa). If this product is detached from the endoscope before releasing air properly, the air pressure inside the endoscope may not vent properly, causing endoscope failure.
- Do not perform manual cleaning of the endoscope with this product attached. The endoscope may malfunction.

Prepare equipment and materials used for leak testing.

## 2.1. Nomenclature and Functions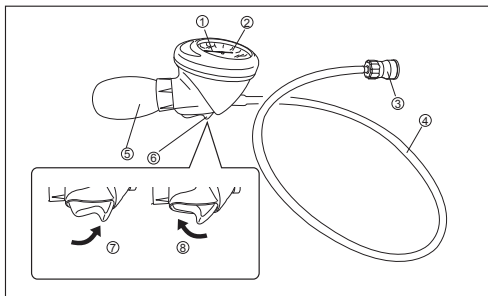

No.	Name	Function
1	Indicator	-
2	Meter	The indicator shows the current air pressure.
3	Connector	Connected to the ventilation connector of the endoscope.
4	Tube	-
5	Blower	Squeeze the blower to pressurize the endoscope.
6	Trigger	Press the pressure side to maintain air pressure. Press the release side to release air pressure.
7	Pressure side	-
8	Release side	-

## 2.2. Inspection Prior to Use

- Visually check the tube leading to the connector for abnormalities such as folds, bends or flaws.
- Check the O ring inside the connector for abnormalities such as scratches or missing.
- Press the trigger of this product to the pressure side. Squeeze the blower of this product to supply air until this product indicates 20 kPa. Wait for 30 seconds and check for any change in the indicator needle. Attach the connector of this product to the ventilation connector of the endoscope and make sure that the indicator needle falls down.

[Note]


- If the change in the indication is 5 kPa or more (10 gradations or more), the air tightness of this product is insufficient.
- Make sure that no moisture remains in the tube of this product.
  - Press the trigger of this product to the release side to discharge air in the tube and make sure that the indicator needle falls to 0 kPa.

## 2.3. Procedure


- Fill the clean, large basin with clean water.

[Note]


- The temperature of clean water must be between +10 to +40°C (50 to 104°F).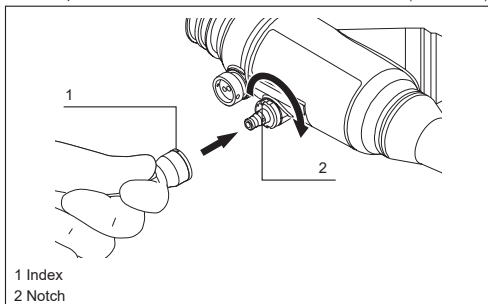

- Connect this product to the ventilation connector of the endoscope. Align the notch of the ventilation connector of the endoscope with the index of the connector of this product and then push in the connector of this product and rotate it clockwise to attach it to the ventilation connector.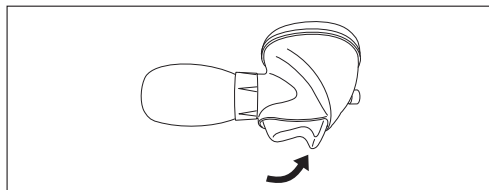

- Securely set the trigger of this product to the pressure side.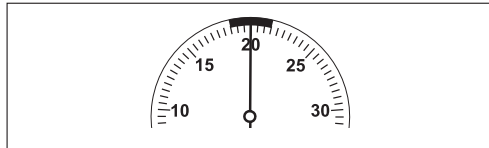


- Squeeze the blower of this product to supply air until it indicates 20 kPa.

[Note]

- Release your finger from the trigger so as not to inadvertently release the pressure.
- Allow this product to stand for about 30 seconds and check the change in the indication.

[Note]


- If the change in the indication is 5 kPa or more (10 gradations or more), the air tightness of the endoscope is insufficient. If the air tightness is insufficient, stop the air leak test and contact your local FUJIFILM dealer immediately.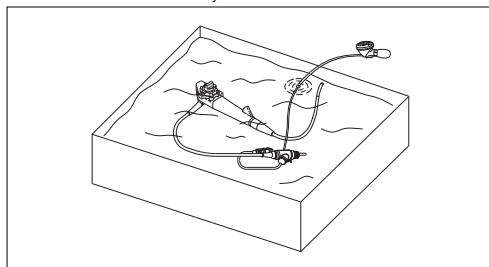

- Squeeze the blower of this product to pump air into the endoscope until the indication reaches 20 kPa and then immerse the endoscope in clean water.


[Note]

- Completely immerse the endoscope.
  - Do not immerse the meter and blower of this product. Immerse the endoscope to the extent that the connector and tube of this product is immersed.
  - When the indicator falls down slowly, squeeze the blower to supply air until the indicator reaches 20 kPa.
  - Keep the endoscope immersed for not less than one minute.
- Wipe off bubbles of recessed areas or irregularly shaped surfaces such as the valve cylinders, channel ports, up/down and left/right angulation locks and up/down and left/right angulation knobs of the endoscope where bubbles are more likely to remain.
  - Operate the up/down and left/right angulation locks and up/down and left/right angulation knobs of the endoscope to move the insertion portion and bending section in various directions and ensure that a steady stream of air bubbles (indicative of a leak) is not found.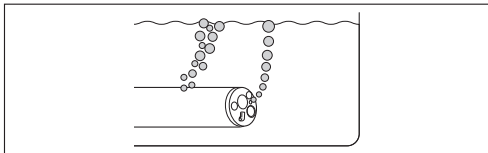


[Note]

- If a stream of air bubbles comes out from the endoscope while it is immersed in clean water for at least one minute, the endoscope has a leak. If the air tightness of the instrument channel or suction channel is insufficient, a stream of air bubbles comes out from the openings of the endoscope channels (such as the distal end, suction valve cylinder, instrument channel inlet and suction connector).
- (9) Remove the endoscope from clean water, wipe all the surfaces of the ventilation connector and the connector of this product with dry gauze, and then dry them out.
- (10) Pull the trigger of this product to the release side to release air pressure from the endoscope.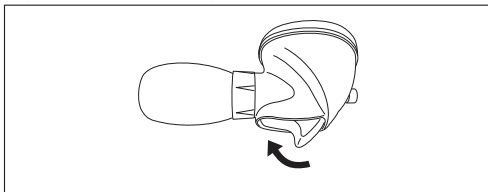

[Note]

- Release air properly until the indicator falls and then stops at the bottom of the gauge.

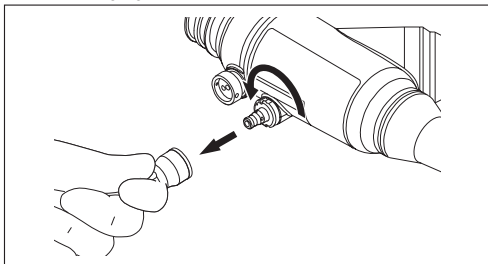- (11) Remove the connector of this product by pressing and then rotating it counterclockwise.

## 2.4. Maintenance

This product can be cleaned by wiping gently with gauze moistened with a mild, neutral pH detergent prepared per manufacturer's recommendations. If needed, this product can be wiped with gauze moistened with alcohol.

## 3. Main Specification

### <Operating Environment>

Temperature : +10 to +45°C (50 to 113°F)  
 Humidity : 30 to 85%RH (no dew condensation)  
 Pressure : 70 to 106 kPa (10.2 to 15.4 psia or 525 to 795 mmHg) (within range of atmospheric pressure)

### <Storage Method>

Store this product in a location that satisfies the following conditions.

### <Transport and Storage Conditions>

Temperature : -10 to +45°C (14 to 113°F)  
 Humidity : 30 to 95%RH (no dew condensation)  
 Pressure : 70 to 106 kPa (10.2 to 15.4 psia or 525 to 795 mmHg) (within range of atmospheric pressure)

### <Term of Validity/Period for Use (Durability)>

This product is a consumable supply. If any abnormality is found, replace with a new one.

### <Medical Device Regulation>

This product complies with the requirements of REGULATION (EU) 2017/745.  
 Classification : Class I

### <Specifications of Main Unit>

Operating pressure : 18.5 to 21.5 kPa (2.7 to 3.1 psi or 140 to 160 mmHg)  
 Maximum limit : 40 kPa (5.8 psi or 300 mmHg)

## 4. Compatible Endoscopes

This product should be used in combination with FUJIFILM endoscopes having a ventilation connector.

## 5. Packaging Content

Air Leak Tester: 1  
 Operation manual: 1

## 6. Warranty and After-Sales Service

If this product does not work properly and if determined to be covered by manufacturer's warranty, FUJIFILM will perform an exchange with a new one or equivalent product.

The warranty period is one year from the date of purchase.

Note that the warranty is not applicable in the following cases:

- Damage caused by fire or natural disaster such as storms or floods.
- Problems caused by careless handling or misuse of the product on the part of the user.
- Product modified by the user or non-authorized FUJIFILM personnel.

## 7. Service Center

Contact our regional representative below or the distributor from which you purchased the product.

<Europe>

FUJIFILM Europe GmbH

<http://www.fujifilm.eu/eu/>

See our website to locate our representative in your country.

If you are not a resident of the regions above, contact the distributor from which you purchased the product.

The manuals for the regions that comply with Regulation (EU) 2017/745 are on the following website.

**FUJIFILM Medical Documents Library**

URL: <https://med-lib.fujifilm.com/>

When referring to the manuals of the product you purchased, please access "Endoscopy-Therapeutic device & Accessory" pages with User name and Password below.

**User name: Endoscopy-med-lib**

**Password: Etda1311-fmyt**

If you have any questions, please contact your local FUJIFILM dealer.

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FUJIFILM Healthcare Americas Corporation
FUJIFILM Australia Pty Ltd
FUJIFILM Asia Pacific Pte. Ltd.
**FUJIFILM Corporation**