# DefiMonitor XD

Service instructions  
English / German

Detailed descriptions of the work steps are provided for trained service engineers only.

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02/2023

© by METRAX GmbH

This service manual or any part thereof shall not be reproduced, transmitted, saved on electronic media or translated into any human or computer language without our approval. Any violations of this restriction not only infringe our copyright, but also limit our ability to provide accurate and up-to-date information to the user and the operator of this device.

All rights to make changes to this service manual are reserved.

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## 1.1 Foreword / Vorwort

Dear Technician,

You are about to service or repair a PRIMEDIC Defibrillator.

These service instructions are intended to assist you in your crucial work.

Before you start work, we as the manufacturers of this unit urgently recommend that you refer to the unit's operating instructions to inform yourself of how the unit that you are about to service or repair functions and is operated!

Please read the manual carefully!

Follow the instructions when working!

It is essential that you follow the safety instructions!

We would be pleased to assist you if you have any questions about the unit or other METRAX products.

You can find the relevant phone number in the imprint at the beginning of this service manual.

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## 1.2 Validity / Gültigkeit

The descriptions in this service manual refer to various extension stages of the defibrillator

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## PRIMEDIC DefiMonitor XD

of the company METRAX GmbH.

Please remember that depending on the unit's configuration, it may not display all the properties described in the operating instructions and that repairs may only be carried out by trained professionals.

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### 1.3 PRIMEDIC DefiMonitor XD types / Bauarten des PRIMEDIC DefiMonitor XD

Note for the new generation:

There are no more different variants available (XD vs. XDxe)

In order to be able to recognize which functions are available the functions were added to the identification plate.

New versions will have **new article numbers**.

Model 3rd edition	Product designations so far 2nd edition	Basic device	AED option	PACER option	SPO2 option
DefiMonitor XD	DefiMonitor XD1 DefiMonitor XD1xe	X			
DefiMonitor XD SPO2	DefiMonitor XD3 DefiMonitor XD3xe	X			X
DefiMonitor XD PACER	DefiMonitor XD10 DefiMonitor XD10xe	X		X	
DefiMonitor XD PACER, SPO2	DefiMonitor XD30 DefiMonitor XD30xe	X		X	X
DefiMonitor XD AED	DefiMonitor XD100 DefiMonitor XD100xe	X	X		
DefiMonitor XD AED, PACER	DefiMonitor XD110 DefiMonitor XD110xe	X	X	X	
DefiMonitor XD AED, SPO2	DefiMonitor XD300 DefiMonitor XD300xe	X	X		X
DefiMonitor XD AED, PACER, SPO2	DefiMonitor XD330 DefiMonitor XD330xe	X	X	X	X

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On these units, it is necessary to perform a periodical technical safety check.

(→ Operating instructions "technical safety checks")

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The PRIMEDIC units described in this manual conform - both individually and in conjunction with the accessories and optional extras - to the currently valid safety standards and meet the requirements of the Medical Products Directive.

The units and their accessories are safe if used correctly and when the descriptions and notices in the operating instructions are observed.

If serviced or repaired incorrectly, the units and their accessories can pose hazards to you, to the user or to patients!

We therefore emphatically point out that all persons who service or repair these units

- must possess good, specific professional electrical and mechanical knowledge!
- must have received instruction and authorization from the manufacturer!
- must read and obey this manual and the operating instructions relevant to the unit concerned, particularly the safety and hazard notices listed therein!

Carry out all service work

- in accordance with the descriptions in this manual.
- correctly and with the greatest possible degree of precision

observing the relevant safety and accident prevention regulations

After service work has been carried out, the service technician must always check that the unit is working perfectly and is operationally safe!

The service technician must certify all work carried out.

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### Do not test the unit

- in the presence of flammable agents (e.g. petroleum ether or similar)
- in an atmosphere enriched with oxygen or with flammable gases / vapours
- if it is defective (e.g. paddle cable damaged)

When carrying out service work on a unit that is ready for operation, there is a risk of potentially lethal electric shocks and burn injury.

Remove any electrically conductive jewellery such as chains, bracelets or rings before carrying out any work!

If necessary, inform your staff that service work is being carried out and point out any resulting hazards!

Perform all work extremely conscientiously!

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Carry out all work as described in this service manual!

Do not approve the unit for operation unless it is undamaged and fully functional, all technical safety checks can be performed and have been performed, and you have completely filled in and signed the inspection protocol!

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**Applying a defibrillation shock to yourself or to someone else can result in cardiac fibrillation and therefore death!**

Therefore, never use the unit "just for fun"!

Explain the hazards to colleagues who have not been trained in handling the unit and who therefore may not be aware of its potential hazards!

Never leave unsupervised units in operation!

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**Maintenance and repair work on the units may only be undertaken by persons trained and authorised by METRAX.**

The units have no parts that can be repaired – defective parts and modules must be replaced!

The units may only be fitted and operated with original accessories from PRIMEDIC!

Observe the rules for ESD protection!

When carrying out service work on units which are ready for operation, there is a risk of unit defects if you bridge contacts with measuring devices or tools, disconnect connectors or put in electrically conductive things.

Never repair damaged modules or individual parts, instead replace them with the manufacturer's original spare parts!

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The following requirements must be fulfilled to carry out the described repair work on the **PRIMEDIC units**:

You must have successfully completed the manufacturer's training and must be authorised to carry out these repairs.

You are responsible for suitable tools and equipment to disassemble and assemble the unit.

For checking the charge of the HV capacitor you need a high-resistance multimeter (input resistance > 10 M Ohm, min. measuring range 4.500 V DC) and high voltage-resistant measuring lines and test prods.

To discharge the HV capacitor you need a discharge resistor with high voltage (2.500 V) shock hazard protection and min. 2.2 KOhm / 50 W.

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One of the following identification plates must be attached to the rear of the unit (similar to illustration, depending on version):

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## Attention:

All serial numbers from replaced circuit boards or modules must be added to the Metrax GmbH database.

Check the following things before you start to repair the device (turned off):

Check the device for external damages (e.g., case, contact sockets, display windows, gaskets etc.).

Check the identification plate.

Check whether the warranty seal is intact.

Inform the customer that his unit is ready to be repaired. If applicable, inform the customer that the warranty seal is damaged and that the repair is not a warranty matter.

Remove the old warranty seal.

### Valid warranty seals of Metrax GmbH:

Green Warranty seal only for service partner, part no. 55307

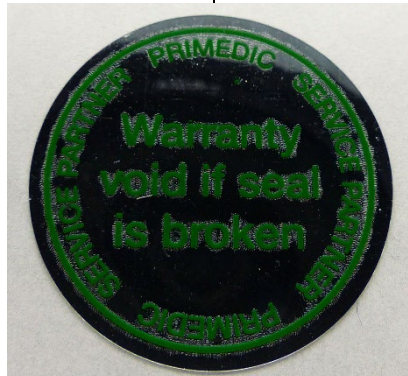
Figure 3

Figure 4

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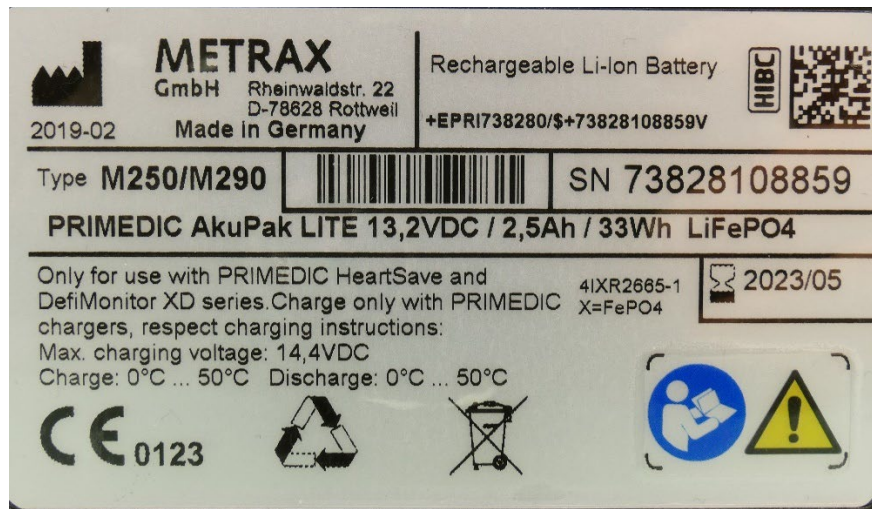
One of the following identification plates must be fitted to the energy module (similar to illustration, depending on version):

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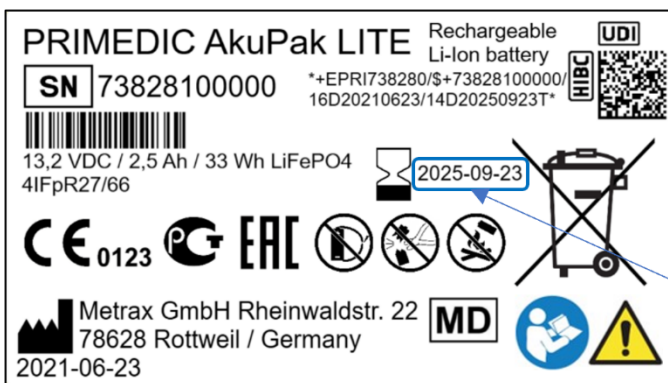
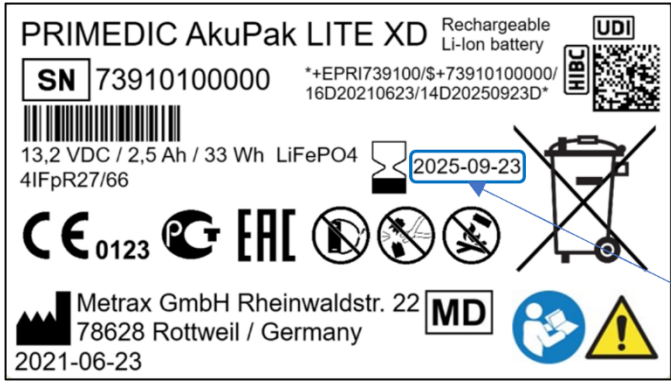
### How to check the PRIMEDIC AkuPak:

- Connect the programming adapter with your PC
- Put the AkuPak into the programming adapter
- Click double on "ReadBQ.bat"
- Now you get the data of the AkkuPack on your PC like shown in this example:

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```

Current date/time:
Tue 04 Nov 2014 09:20
BQ2019.EXE V1.14 was used to write the data below on:
Thu 04 Sep 2014 01:00
TYPE           = Lithium-Ion accumulator      (0x50)
MODEL          = 2 .0                          (0x50)
ACCU SW-VERSION = 2.07
ACCU SW-CHECKSUM = 4F56:4EFD
ACCU CHARGE STATE = IDLE
SERIAL NUMBER   = 00002516                    (0x000009D4)
DATE           = 09/04/2014 00:00:00        (0x5407AB80)
MIN CHARGE     = 300                        mAh (0x0125)
MAX CHARGE     = 2500                       mAh (0x098B)
MIN VOLTAGE    = 11600                      mV  (0x2D50)
MAX VOLTAGE    = 15400                      mV  (0x3C28)
RSR            = 0.003                      Ohm (0x0116)
DATE END LIFE  = 12/04/2018 00:00:00        (0x5C05C380)
MIN WORK TEMPERATURE = -10                  C   (0xF6)
MAX WORK TEMPERATURE = 60                   C   (0x3C)
DC RESISTANCE  = 0                          mOhm (0x0000)
SELF DISCHARGE RATE = 0.00                  %/year (0x0000)
MULTIPLICATOR1 = 100                        %   (0x64)
MULTIPLICATOR2 = 100                        %   (0x64)
MULTIPLICATOR3 = 100                        %   (0x64)
    
```

CHARGE EFFICIENCY = 100 % (0x64)  
 AGING CONSTANT = 0 10ppm/Ah (0x0000)  
 DISCHARGE END = 12000 mV (0x2EE0)  
 LIMIT CHARGING VOLTAGE = 14400 mV (0x3840)  
 MIN CHARGING TEMPERATURE = -5 C (0xFB)  
 MAX CHARGING TEMPERATURE = 50 C (0x32)  
 CHARGE CURRENT = 5120 mA (0x1400)  
 NUMBER CHARGE = 0 cycles (0x0000)  
 NUMBER CALIBRATION = 0 cycles (0x0000)

TYPE LAST DEFINITION = 00000005 (0x00000005)  
 SN LAST DEFINITION = 00006231 (0x00001857)  
 DATE FIRST USE = 10/03/2014 17:24:16 (0x542EDBC0)  
 DATE LAST USE = 10/27/2014 13:49:23 (0x544E4D63)  
 INTEGRAL DISCHARGE MANUFACTURE = 0.0 Ah (0x00000000)  
 DATE CALIBRATION = 09/04/2014 00:00:00 (0x5407AB80)  
 INTEGRAL SELF-DISCHARGE = 0 mAh (0x000000)  
 INTEGRAL SLEEP TIME = 0 h (0x0000)  
 INTEGRAL DISCHARGE MAINTENANCE = 0.0 Ah (0x0000)  
 LAST MAINTENANCE = 0 Hours ago  
 ORIGINAL CAPACITY = 2499 mAh  
 Qmax = 2500 mAh  
 Qmin = 300 mAh  
 QCCR = 0 mAh  
 QCEFF = 0 mAh  
 QDCR = 621 mAh  
 QSELF = 0 mAh  
 QSLEEP = 0 mAh  
 QAGE = 0 mAh  
 QAV = 1879 mAh  
**CURRENT CAPACITY = 1579 mAh (72 %)**

Reading from BQ2019...Success

RAM/FLASH PAGE0 0x00 05 00 00 00 57 18 00 00

0x08 C0 DB 2E 54 63 4D 4E 54

0x10 00 00 00 00 80 AB 07 54

0x18 00 00 00 00 00 00 00 01

FLASH PAGE1 0x20 1E 50 D4 09 00 00 80 AB

0x28 07 54 25 01 8B 09 50 2D

0x30 28 3C 16 01 00 00 8B 09

0x38 56 4F FD 4E 07 02 00 D4

FLASH PAGE2 0x40 80 C3 05 5C F6 3C 00 00

0x48 00 00 64 64 64 00 00

0x50 E0 2E 40 38 FB 32 00 14

0x58 00 00 00 00 00 00 00 48

TEMP 0x60 26 01

FCMD 0x62 00

CLR 0x63 00

MODE/WOE 0x64 4E

CTC 0x65 00 00

DTC 0x67 00 00

SCR 0x69 00 00

CCR 0x6B 00 00

DCR 0x6D 5F 02

FPD 0x6F 00

FPA 0x70 00

RESERVED 0x71 00 06 F9 00  
OFFCT 0x75 00 00  
CAL/OFFCTH 0x77 80  
IDROM 0x78 78 01 16 00 00 00 20 19

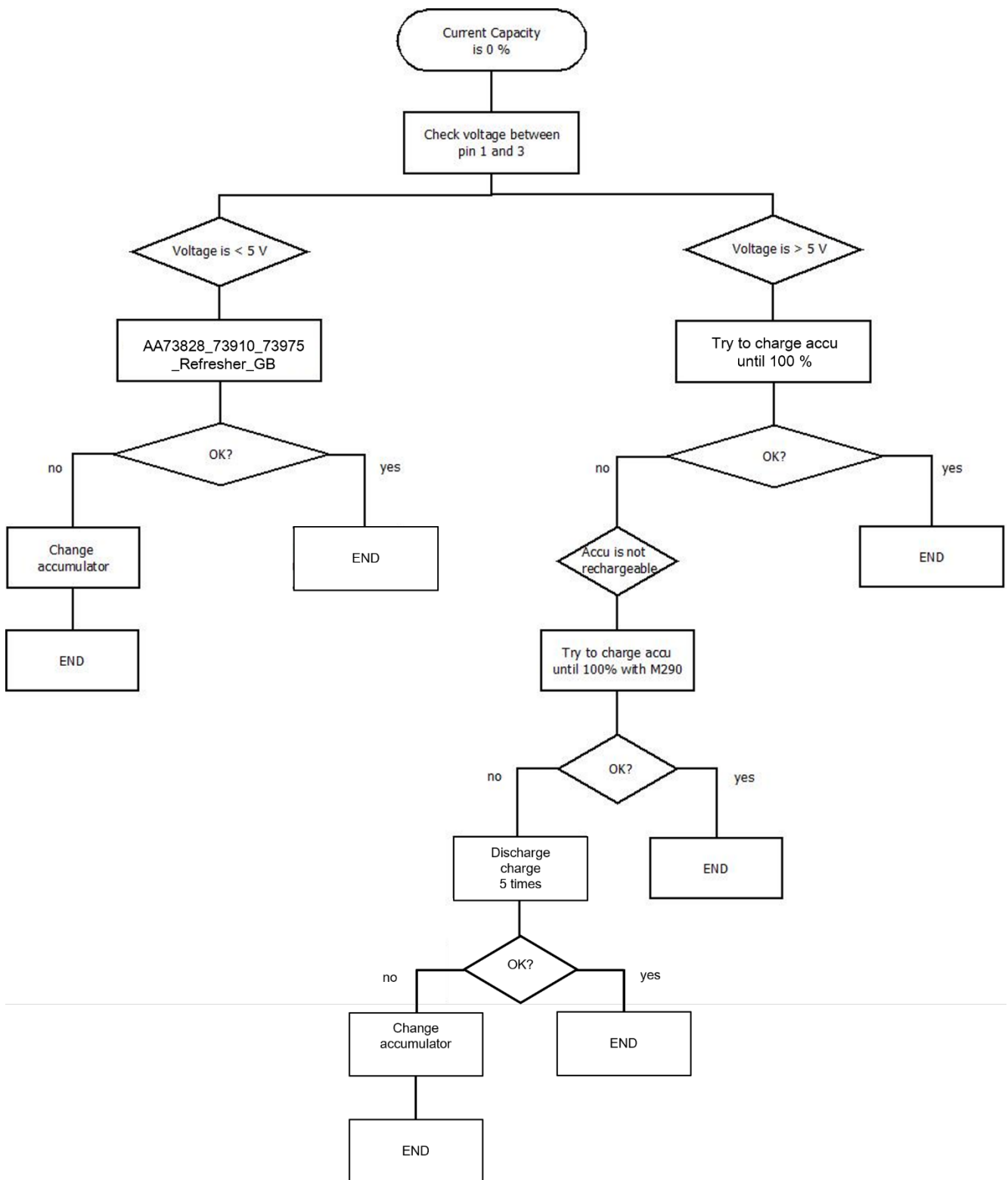
Check the **current capacity** of the accumulator:

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Remove the CF card as described in → Operating instructions "5.2.2 Changing the SaveCard".

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The CF card used in the DefiMonitor XD must be formatted with the FAT16 file system. Since it can easily be the case with a Microsoft® Windows XP operating system that the CF card has been incorrectly formatted with the FAT32 or RAW file system, or a CF card from another unit is used, you should ensure correct formatting as follows:

### How to format the CF card with Microsoft® Windows XP and Microsoft® Windows 2000 + 8 and 8.1:

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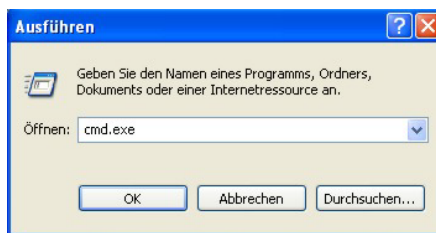
Figure 9

Place the CF card into the card reader on your computer.

Start a command line window on your computer with: "Start" → "Run" and enter "cmd.exe" in the input box. (Figure 10)

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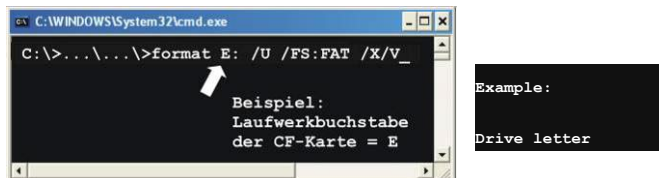
Figure 10

The command line window then opens. Now enter the following:

format @: /Q /FS:FAT /X/V

whereby "@" serves as a space-saver for the drive letter of your CF card. Adapt the drive letter accordingly.

Conform your entry and all subsequent messages with the Enter key.

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### 6.1 without SaveCard / ohne SaveCard

Switch on the unit by inserting the PRIMEDIC AkuPak without the SaveCard.

The unit switches itself on automatically after the PRIMEDIC AkuPak is inserted and perform a self-test.

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### 6.2 with SaveCard / mit SaveCard

Ensure that you use the correct SaveCard and that it has been formatted with the correct format (➔ 6.2 Formatting the CF card).

Insert an empty SaveCard as described in the operating instructions.

#### Perform another function check

Remove the CF card.

Then read out SaveCard

Read out the "syserr.txt" message with the Windows Editor or Windows WordPad and analyze the error code based on the error code list.

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```

2022/10/05 08:44:59 0000003396 System started
2022/10/05 08:44:59 0000003396 Model: M290 XD1
2022/10/05 08:44:59 0000003396 MSP event 10 at 2020/11/16 23:20:44 Restarting of MSP program by external watchdog
2022/10/05 08:44:59 0000003396 MSP event 10 at 2020/11/16 23:20:44 Restarting of MSP program by external watchdog
2022/10/05 08:44:59 0000003396 MSP event 15 at 2020/11/16 23:20:52 Disconnection of external source
2022/10/05 08:44:59 0000003396 MSP event 14 at 2020/11/16 23:40:33 Connection of external source
2022/10/05 08:44:59 0000003396 MSP event 12 at 2020/11/16 23:40:37 Received command from ARM about hardware error
2022/10/05 08:44:59 0000003396 MSP event 15 at 2022/10/04 15:29:11 Disconnection of external source
2022/10/05 08:44:59 0000003396 MSP event 14 at 2022/10/04 15:30:34 Connection of external source
2022/10/05 08:45:00 0000003396 System test
2022/10/05 08:45:00 0000003396 Battery test success
2022/10/05 08:45:01 0000003396 ARM: 5BE6545F | MSP: 39AA | DSP: 2719 | ULF: EDDD7C2D | EXT MSP: C8FC
2022/10/05 08:45:01 0000003396 ARM: 1.21 | MSP: 2.07 | DSP: 2.14 | EXT MSP: 2.11
2022/10/05 08:45:01 0000003396 BQ: Typ: 1 | Ver: 2.0 | SN: 13468 | CAP: 2500 | PD: 2022/01/11 00:00:00
2022/10/05 14:08:09 0000003396 Battery capacity low
2022/10/05 14:10:41 0000003396 Data file closed (2022-10-05 12.43.22_B.AED)
2022/10/05 14:10:41 0000003396 0000 Success, no errors
2022/10/05 14:10:41 0000003396 Next Selftest: 20:00:00
2022/10/05 14:10:41 0000003396 ARM: 5BE6545F | MSP: 39AA | DSP: 2719 | ULF: EDDD7C2D | EXT MSP: C8FC
2022/10/05 14:10:41 0000003396 ARM: 1.21 | MSP: 2.07 | DSP: 2.14 | EXT MSP: 2.11
2022/10/05 14:10:41 0000003396 BQ: Typ: 1 | Ver: 2.0 | SN: 13468 | CAP: 0 | PD: 2022/01/11 00:00:00
2022/10/05 14:10:41 0000003396 System shut down

```

Figure 11

## 6.3 Troubleshooting / Fehlersuche

If there is a problem with the device, follow the troubleshooting instructions step by step.

Look at the description of the syserr.txt

Read out the CF-card for an syserr.txt code (description look at 6.2) and follow the troubleshooting list for error codes. If there is no error code on the CF-card, follow the troubleshooting list without error codes.

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There are two lists with errors and following steps to repair the device and one list with the instruction to repair the device.

If the troubleshooting list doesn't help, call the Metrax Technical Service for help.

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#### Example for working with the Troubleshooting list:

1. Search the error in the troubleshooting list, for example search error "0084 HV, PRWIN fail > 100 ms"

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Error code	Description of failure	Repairing steps
79	HV, Error in calculation of charge voltage	1
84	HV, PRWIN fail > 100 ms	29/28/3
87	Program update error	6/1

2. Follow each step at the end of the troubleshooting list.

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Error code	Description of failure	Repairing steps
79	HV, Error in calculation of charge voltage	1
84	HV, PRWIN fail > 100 ms	29/28/3
87	Program update error	6/1

3. First look at repairing instruction list for step 29

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Step	Instruction	Note	Part number
28	check the battery-/rechargeable battery-contacts and check the spring contacts in the battery compartment	-	-
29	put the battery/rechargeable battery in the battery compartment and wait until the selftest is finished.	-	-

If the first step doesn't help do the next step 28, and so on.

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Step	Instruction	Note	Part number
28	check the battery-/rechargeable battery-contacts and check the spring contacts in the battery compartment	-	-
29	put the battery/rechargeable battery in the battery compartment and wait until the selftest is finished.	-	-

If all repairing steps doesn't help call Metrax Technical Service.

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Error code	Description of failure	Repairing steps
54 or 105	HV, +24V failed	1
56	HV, HVPWR voltage failed	3/10
57	HV, RLPWR voltage failed	1
58	HV, Current setup DAC failed	1
59	HV, HVPWR voltage failed	3
63	HV, Disarm cannot be performed	12/8
64	HV, HV capacitor overvoltage was detected	10/1
65	HV, HV charger error was detected	1
66	HV, error of HV measurement circuit	1
67	HV, HV charge time is too big	1
68	HV, Too big current was detected	21/8/1
69	HV, Too low current at shock was detected	8/1
76	HV, Residual voltage after disarm error	1
77	HV, Capacitor voltage does not match energy level	1
78	HV, Capacitor voltage is not constantly in-creasing	1
79	HV, Error in calculation of charge voltage	1
84	HV, PRWIN fail > 100 ms	29/28/3
87	Program update error	6/1
92	Critical key error	20/13/1
109	Battery Drop	18
110	Temperature exceed	30/31
121	DSP timeout error	19
122	Previous error detected need full test	19
124	Keyboard error	19/20/13
125	ECG calibration error	1
126	Audio output test error	19/20/11/9
130	SPI communication error	1
133	Switch ECG source error	7
134.8	XtBoard err - power source	32
135 (.8)	XtBoard err - paddle logic	2
136 (.15)	XtBoard err - key logic	11/2
140	Impedance measurement error	2

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Description of failure	Repairing steps
the device say "check electrodes"	22/1
there is no ECG with electrodes or paddles and the device show no impedanz	22/1
paddlekeys do not work	14/2
device does not start	3/32/1
disturbances and noise are to be heard over the loudspeaker	1
shock value approx. 15% too small or second half wave is missing	1
power supply and fan have no function	15
release key LED's doesn't shine	11/9/1
no reaction when the release key is pushed	6/1
the device can't detect the CF card	1
charge LED doesn't shine	25
printer starts to print, however no hard copy comes	26
printer starts to print, no paper feed motion	19/17
strips in the display	27/16

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Step	procedure	Note	Part number
1	change main board		73455/ 74004
2	extension board		73695
3	there is a problem with the Battery/rechargeable Battery	chapter "6.2 Inspecting the PRIMEDIC AkuPak"	Akku: 97196/ 97311
4	formatting the CF card	chapter "6.5 Formatting the CF card (SaveCard)"	
5	remove battery + main cable wait 2 min restart the unit	if error persist, change the XT-board (2) with a new	
6	patient coil is not correct plugged in or the coil is defect	do an visual check, measure the resistance it should be a small value and check the mounting direction of the coil	21443
7	there is a problem with the release key		73562
8	capacitor cable is not correct plugged in or is defect	measure the resistance it should be a small value	21866
9	check FFC flat cable's, if these are on the correct place and correct plugged in or if these are defect (contacts mechanically damaged)		360mm: 21788 / 500mm: 21789

Step	procedure	Note	Part number
10	the cable to the discharge resistors are not correct plugged in or defect	measure the resistance it should be a small value	21780
11	test the membrane keypad	look for damages	without Pacer: 21170 / with Pacer: 21169 / with arrow key without Pacer: 22325 / with arrow key with Pacer: 22324 for new generation without Pacer: 23948/ 23936 and with Pacer: 23935/ 23947
12	check the paddle-jack or check the paddles		paddle: 73496 / paddle jack: 21821
13	check the cable with spring contacts		21790
14	change the display		21152
15	change the printer and the controller board of the printer		printer: 21186 / printer controller: 21187
16	change the back-up battery (BT1) on the main board		20612
17	start the device with the on/off button and wait for the end of the selftest	if error persist than change the main board with a new one (1.)	73455/ 74004
18	do the keypad-test or the MMI-test	if MMI-test not passed than change the membrane keyboard with a new one	xx
19	check the mounting direction of the coil (lettering must be towards inside the device)		
20	check the electrode-cable and the electrodes		
21	rechargeable battery capacity > 80% →rechargeable battery will not be charged		
22	check the power module		21181/ 23954
23	thermal printing paper is to old, change the printing paper		18122
24	turn the paper feed roll for a quarter, if this doesn't work, change the paper feed roll		
25	check the connector		
26	check the battery-/rechargeable battery-contacts and check the spring contacts in the battery compartment		
27	put the battery/rechargeable battery in the battery compartment and wait until the selftest is finished.		
28	Check temperature		

Step	procedure	Note	Part number
29	Read out accu or battery if all values "0" change it		
30	change the hallsensor board		

## 7 Disassembling the DefiMonitor XD / DefiMonitor XD zerlegen

### Precautions

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Switch off the unit before you open it.

Disconnect the power supply from the main and remove the PRIMEDIC AkuPak from the unit!

Do not reconnect the unit to the main until you have finished the repair work and the housing is closed!

Ensure that you do not touch any active parts!

Do not leave switched-on units unattended!

The disassembling and assembling descriptions below inform you how to remove and reinstalling circuit boards and modules of the unit. The sequence of descriptions is not specific and is selected corresponding to a repair down to the final circuit board.

Before starting any repairs consider which sequence is most appropriate for disassembly and assembly.

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Details of this step are available only for trained service engineers

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### Attention:

For every order you need the serial number of the DefiMonitor XD or DefiMonitor XDxe

### Warning:

An exchange main board is not prepared for defibrillation. After changing the main board the software of the device must be installed. The exchange main board is delivered with no functional software.

For programming the main board see "Setup Manual".

It is not permitted to replace one of the components on the circuit board!

The circuit board must be removed, sent to the manufacturer and replaced by a new circuit board!

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