





HIV 1+2 / Syphilis Combo Card Test

Anti HIV 1+2 / Syphilis antibody detection card test (Whole blood / Serum / Plasma)

FEATURES

- Clear background.
- Self contained test procedure.
- Specifically engineered for whole blood specimen, as well as serum and plasma.
- ≥ 100% sensitivity and 100% specificity.
- Clear differentiation between HIV & Syphilis infections.
- Detects antibodies to gp41 & gp36 of HIV.
- Detects antibodies to P47, P45, P17 & P15 of Syphilis.
- High-tech membrane strip locks in visible test results at ambient conditions even months later
- Storage temperature 4°C -30 °C & long shelf life up to 24 months.

BENEFITS

- Unmistakable results at 15 minutes.
- Simple and easy to use, does not require advanced medical training.
- Cost effective, can be used effectively in resourcelimited conditions.
- Accurate and reliable.
- ▶ Helps to detects separately for HIV and Syphilis.
- Sensitive and accurate with most known subgroups and early sero conversion samples.
- Does not require cold chain & convenient for small volume users.

KIT CONTENTS

- · Test device with desiccant
- · Assay buffer bottle
- Specimen transfer device
- Sterile lancet
- · Alcohol swab
- · Instructions for use


EXTERNAL LABORATORY EVALUATION REPORTS									
Place of Evaluation	Year of Testing	Sensitivity		Specificity					
Flace of Evaluation		HIV	Syphilis	HIV	Syphilis				
WHO Evaluation Report	2018	100%	99%	99.5%	100%				
Ghana	2018	100%	100%	100%	100%				
Zimbabwe	2019	100%	100%	100%	100%				


TEST PROCEDURE Add 20µl of Whole blood/ Serum/ Plasma (Filled up to the marking) to the specimen well (S). Result at 15 min. Do not interpret after 25 min.

OPDERING INFORMAT

RESULTS INTERPRETATION

Negative	HIV Positive		e Sy _l	philis Posi	tive HIV	HIV & Syphilis Positive		
FIRST RESPONSE HIV 1+2/Syphilis		FIRST RESPONSE HIV 1+2/Syphilis		FIRST RESPONSE HIV 1+2/Syphilis		FIRST RESPONSE HIV 1+2/Syphilis		
c syp HiV		c syp		C SYP HIV		C SYP HIV		
S		S		S S		S S		

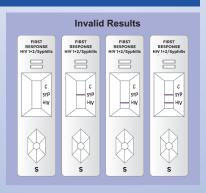

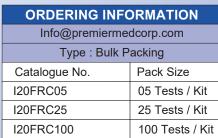



ISO 13485 & EN ISO 13485 Certified Company

Part No.: I20-BRO-001, Rev. AD, Date: 2021-04-05



Premier Medical Corporation Private Limited

Mfg. Unit: A1-302, GIDC, Sarigam 396155. Dist. Valsad, Gujarat, INDIA

• Customer support email : info@premiermedcorp.com • Tel.: +91 260 2780112/113,

• www.premiermedcorp.com