EC Declaration of Conformity

according to Directive 98/79/EC, on in vitro diagnostic medical devices

Ref. No.:20220513-A05

Manufacturer
(Name, Address)

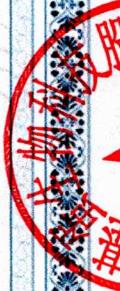
Getein Biotech, Inc.
No. 9 Bofu Road, Luhe District, Nanjing, 211505, China

Authorized Representative
(Name, Address)

CMC Medical Devices & Drugs S.L.
Add: C/ Horacio Lengo N° 18, CP 29006, Málaga, Spain

Medical device

No.	Product Name
1	Getein 1100 Immunofluorescence Quantitative Analyzer
2	Cardiac Troponin I Fast Test Kit (Immunofluorescence Assay)
3	NT-proBNP Fast Test Kit (Immunofluorescence Assay)
4	hs-CRP+CRP Fast Test Kit (Immunofluorescence Assay)
5	NT-proBNP/cTnI Fast Test Kit (Immunofluorescence Assay)
6	CK-MB/cTnI/Myo Fast Test Kit (Immunofluorescence Assay)
7	D-Dimer Fast Test Kit (Immunofluorescence Assay)
8	PCT Fast Test Kit (Immunofluorescence Assay)
9	CysC Fast Test Kit (Immunofluorescence Assay)
10	mAlb Fast Test Kit (Immunofluorescence Assay)
11	NGAL Fast Test Kit (Immunofluorescence Assay)
12	β 2-MG Fast Test Kit (Immunofluorescence Assay)
13	CK-MB/cTnI Fast Test Kit (Immunofluorescence Assay)
14	HCG+ β Fast Test Kit (Immunofluorescence Assay)
15	H-FABP Fast Test Kit (Immunofluorescence Assay)
16	PCT/CRP Fast Test Kit (Immunofluorescence Assay)
17	CK-MB/cTnI/H-FABP Fast Test Kit (Immunofluorescence Assay)
18	HbA1c Fast Test Kit (Immunofluorescence Assay)
19	NT-proBNP/NGAL Fast Test Kit (Immunofluorescence Assay)
20	CK-MB Fast Test Kit (Immunofluorescence Assay)
21	hs-cTnI Fast Test Kit (Immunofluorescence Assay)
22	T3 Fast Test Kit (Immunofluorescence Assay)
23	T4 Fast Test Kit (Immunofluorescence Assay)
24	TSH Fast Test Kit (Immunofluorescence Assay)
25	Scr Fast Test Kit (Immunofluorescence Assay)
26	PLGF Fast Test Kit (Immunofluorescence Assay)

- 27 HCY Fast Test Kit (Immunofluorescence Assay)
- 28 Anti-CCP Fast Test Kit (Immunofluorescence Assay)
- 29 25-OH-VD Fast Test Kit (Immunofluorescence Assay)
- 30 Lp-PLA2 Fast Test Kit (Immunofluorescence Assay)
- 31 FOB Fast Test Kit (Immunofluorescence Assay)
- 32 SAA Fast Test Kit (Immunofluorescence Assay)
- 33 H. pylori Fast Test Kit (Immunofluorescence Assay)
- 34 PRL Fast Test Kit (Immunofluorescence Assay)
- 35 Transferrin Fast Test Kit (Immunofluorescence Assay)
- 36 Insulin Fast Test Kit (Immunofluorescence Assay)
- 37 PG I /PG II Fast Test Kit (Immunofluorescence Assay)
- 38 LH Fast Test Kit (Immunofluorescence Assay)
- 39 FSH Fast Test Kit (Immunofluorescence Assay)
- 40 Anti-TP Fast Test Kit (Immunofluorescence Assay)
- 41 AFP/CEA Fast Test Kit (Immunofluorescence Assay)
- 42 AMH Fast Test Kit (Immunofluorescence Assay)
- 43 fT3 Fast Test Kit (Immunofluorescence Assay)
- 44 fT4 Fast Test Kit (Immunofluorescence Assay)
- 45 Total IgE Fast Test Kit (Immunofluorescence Assay)
- 46 Vit-B12 Fast Test Kit (Immunofluorescence Assay)
- 47 Prog Fast Test Kit (Immunofluorescence Assay)
- 48 Testosterone Fast Test Kit (Immunofluorescence Assay)
- 49 E2 Fast Test Kit (Immunofluorescence Assay)
- 50 RF Fast Test Kit (Immunofluorescence Assay)
- 51 ASO Fast Test Kit (Immunofluorescence Assay)
- 52 Ferritin Fast Test Kit (Immunofluorescence Assay)
- 53 ST2 Fast Test Kit (Immunofluorescence Assay)
- 54 CA125 Fast Test Kit (Immunofluorescence Assay)
- 55 CA19-9 Fast Test Kit (Immunofluorescence Assay)
- 56 CA15-3 Fast Test Kit (Immunofluorescence Assay)
- 57 RSV/Influenza A/B Fast Test Kit (Immunofluorescence Assay)
- 58 Influenza A/B Fast Test Kit (Immunofluorescence Assay)
- 59 RSV Fast Test Kit (Immunofluorescence Assay)
- 60 IL-6 Fast Test Kit (Immunofluorescence Assay)
- 61 BNP Fast Test Kit (Immunofluorescence Assay)
- 62 SAA/CRP Fast Test Kit (Immunofluorescence Assay)
- 63 Folate acid Fast Test Kit (Immunofluorescence Assay)
- 64 hs-CRP Fast Test Kit (Immunofluorescence Assay)
- 65 TnT Fast Test Kit (Immunofluorescence Assay)
- 66 PCT/IL-6 Fast Test Kit (Immunofluorescence Assay)

- 67 HBP Fast Test Kit (Immunofluorescence Assay)
- 68 S100-β Fast Test Kit (Immunofluorescence Assay)
- 69 CK-MB/hs-cTnI/Myo Fast Test Kit (Immunofluorescence Assay)
- 70 Cortisol Fast Test Kit (Immunofluorescence Assay)
- 71 CEA Fast Test Kit (Immunofluorescence Assay)
- 72 AFP/CEA Fast Test Kit (Immunofluorescence Assay)

Classification Other device (according to Annex II of the directive 98/79/EC)

Conformity assessment route Annex III of the 98/79/EC

Applicable	EN 13612:2002	EN ISO 14971:2019	EN ISO15223-1:2016
coordination	EN ISO 18113-1:2011	EN ISO 18113-2:2011	EN ISO 18113-3:2011
standards	EN ISO 23640:2015	EN ISO 13485:2016	ISO 780:2015
	EN 61326-2-6:2006	IEC 61326-1:2013	
	EN 61010-2-101:2002	IEC 61010-1:2010	

Signatory representative declares herein the above-mentioned device meets the basic requirements of the European Parliament and the Council's in vitro diagnostic medical devices directive: 98/79/EC Annex I.

This declaration of conformity is based on European Parliament and the Council's 98/79/EC directive Annex III. The compiled technical file and quality system document according to 98/79/EC directive Annex III are testified and the quality system certificate has issued by BSI Group The Netherlands B. V. The manufacturer is exclusively responsible for the declaration of conformity.

General Manager Enben Su

Nanjing
13th, May, 2022
 (place and date of issue)

 (name and signature or equivalent marking of authorized person)

CE

Certificate of Registration

QUALITY MANAGEMENT SYSTEM - ISO 13485:2016

This is to certify that: **Getein Biotech, Inc.**
No.9 Bofu Road
Luhe District
Nanjing
Jiangsu
211505
China

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Holds Certificate No: **MD 728432**

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 for the following scope:

Design & Development, Manufacture and Distribution of Chemiluminescence Immunoassay, Biochemistry Assay, Point of Care Assay (including Colloidal Gold Assay, Immunofluorescence Assay, Dry Chemistry Assay). Design & Development, Manufacture and Distribution of Analyzers in use of Chemiluminescence Immunoassay, Biochemistry Assay, Point of Care Assay (including Colloidal Gold Assay, Immunofluorescence Assay, Dry Chemistry Assay).

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For and on behalf of BSI:

Gary E Slack, Senior Vice President - Medical Devices

Original Registration Date: 2020-05-29

Latest Revision Date: 2020-07-22

Effective Date: 2020-07-26

Expiry Date: 2023-07-25

Page: 1 of 1

...making excellence a habit.™

Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China
Tel: 86-25-68568508 Email: overseas@geteincom.cn Web: www.bio-GP.com.cn

Document No.: GP-GMSQ-2022-110

Letter of Authorization

To whom it may concern,

We, Getein Biotech, Inc. (No.9 BoFu Road, Luhe District, Nanjing, 211505, China), hereby authorize Sanmedico SRL. as our official distributor for registering, promoting, selling, distributing, taking part in tenders, maintaining & after sale technical services of under-mentioned product in the territory of Moldova:

Sanmedico SRL will comply with the laws and regulations of the countries and regions where they are located in and where they are selling mentioned product to, otherwise, the risks and losses arising therefrom shall be undertaken by Sanmedico SRL

This authorization starts from Jan 1, 2022 and will be valid to December 31 2023

Getein Biotech, Inc. has the right to terminate the authorization before validity and will inform Sanmedico SRL with 10 days in advance.

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A handwritten signature in black ink that reads 'Steven Zhou'.

CK-MB/cTnI/Myo Fast Test Kit

(Immunofluorescence Assay)

User Manual

Getein1100: Cat.# IF1005
Getein1600: Cat.# IF2005

INTENDED USE

CK-MB/cTnI/Myo Fast Test Kit (Immunofluorescence Assay) is intended for *in vitro* quantitative determination of CK-MB/cTnI/Myo in serum, plasma or whole blood. This test is used as an aid in the clinical diagnosis, prognosis and evaluation of myocardial injury such as Acute Myocardial Infarction (AMI), Unstable Angina, Acute Myocarditis and Acute Coronary Syndrome (ACS).

SUMMARY

Creatine kinases are dimer isozymes composed of two monomer subunits, CK-M (for skeletal muscle derived) and CK-B (for brain derived), which can form all three combinations of monomers: CK-BB, CK-MM, and CK-MB. BB is found primarily in the brain. Skeletal muscles primarily contain the MM isoform, with trace amount of MB (around 1-4% of total CK activity). Cardiac muscles also contain the MM isoform, but higher amount of MB, typically around 20% of total CK activity. CK-MB is a more sensitive marker of myocardial injury than total CK activity, because it has a lower basal level and a much narrower normal range. Medical literatures commonly state that CK-MB levels are elevated in 4 to 6 hours, peak at 10 to 24 hours, and return to normal within 3 to 4 days after an acute myocardial infarction. Classically, an increase of the myocardial-specific enzyme CK-MB is considered as the hallmark of acute myocardial infarction, and increased levels are frequently interpreted by the clinician as objective evidence of myocardial cell damage.

Troponin complex consists of three regulatory proteins: T, which connects the troponin complex and tropomyosin (another

cardiac muscle regulatory protein); I, which prevents muscle contraction in the absence of calcium; and C, which binds calcium. Cardiac troponin I (MW 22.5 kDa) and the two skeletal muscle isoforms of troponin I have considerable amino acid sequence homology, but cTnI contains an additional N-terminal sequence and is highly specific for myocardia.

Clinical studies have demonstrated the release of cTnI into the blood stream within hours following acute myocardial infarctions (AMI) or ischemic damage. Elevated levels of cTnI are detectable in blood within 4 to 6 hours after the onset of chest pain, reaching peak concentrations in approximately 8 to 28 hours, and remain elevated for 3 to 10 days following AMI. Due to the high myocardial specificity and the long duration of elevation, cTnI has become an important marker in the diagnosis and evaluation of patients suspected of having an AMI.

Myoglobin is a small monomeric protein which serves as an intracellular oxygen storage site. It is found in abundance in the muscle and can get through into the blood circulation directly when myocardial cell is damaged mildly. Therefore, myoglobin has been advocated as a sensitive marker for early acute myocardial injury by American College of Cardiology Committee.

PRINCIPLE

Mixed monoclonal antibodies against human CK-MB, cTnI and Myo are conjugated with fluorescence latex and another set of anti-human CK-MB/cTnI/Myo monoclonal antibodies were coated on different test lines respectively. After the sample has been applied to the test strip, the fluorescence latex-labelled anti-human CK-MB, cTnI and Myo monoclonal antibodies will bind with the CK-MB, cTnI and Myo in sample respectively and form marked antigen-antibody complexes. These complexes move to the test card detection zone by capillary action. Then marked antigen-antibody complexes will be captured on different test lines by another set of monoclonal antibodies against human CK-MB, cTnI or Myo respectively resulting in the accumulation of fluorescence particles on the test lines. The fluorescence intensity of each test line increases in proportion to the amount of CK-MB, cTnI or Myo in sample.

Then insert test card into Getein1100 Immunofluorescence Quantitative Analyzer/Getein1600 Immunofluorescence Quantitative Analyzer (hereinafter referred to as Getein1100 and Getein1600), the concentration of CK-MB, cTnI and Myo

in sample will be measured and displayed on the screen. The value will be stored in Getein1100/Getein1600 and available for downloading. The result can be easily transmitted to LIS and HIS.

CONTENTS

- A kit for Getein1100 contains:
 - Getein CK-MB/cTnI/Myo test card in a sealed pouch with desiccant 25
 - Disposable pipet 25
 - Whole blood buffer 1
 - SD card 1
 - User manual 1
- A kit for Getein1600 contains:
 - Sealed cartridge with 24/48 Getein CK-MB/cTnI/Myo test cards 2
 - User manual 1
 - Package specifications:
 - 2x24 tests/kit, 2x48 tests/kit
 - Materials required for Getein1600:
 - Sample diluent 1
 - Box with pipette tips 1
 - Mixing plate 1
- Sample diluent/Whole blood buffer composition:
 - Phosphate buffered saline, proteins, detergent, preservative, stabilizer.
- A test card consists of:
 - A plastic shell and a reagent strip which is composed of a sample pad, nitrocellulose membrane (one end of the membrane is coated with fluorescence latex-labelled anti-human CK-MB, cTnI and Myo monoclonal antibodies, these three lines are coated with another anti-human CK-MB, another anti-human cTnI and another anti-human Myo monoclonal antibody, respectively, and the control line C is coated with rabbit anti-mouse IgG antibody), absorbent paper and liner.

Note: Do not mix or interchange different batches of kits.

APPLICABLE DEVICE

Getein1100 Immunofluorescence Quantitative Analyzer

Getein1600 Immunofluorescence Quantitative Analyzer

STORAGE AND STABILITY

Store the test card at 4-30°C with a valid period of 24 months. Use the test card for Getein1100 within 1 hour once the foil pouch is opened.

For test card of Getein1600: if the cartridge is opened, it could be stable within 24 hours once exposed to air. If the test cards can't be used up at a time, please put the cartridge back to the foil pouch and reseal along the entire edge of zip-seal. The remaining test cards should be used up within 7 days.

Store the sample diluent/whole blood buffer at 0-30°C with a valid period of 24 months.

Store the sample diluent/whole blood buffer at 2-8°C for better results.

PRECAUTIONS

- For *in vitro* diagnostic use only.
- For professional use only.
- Do not use the kit beyond the expiration date.
- Do not use the test card if the foil pouch or the cartridge is damaged.
- Do not open pouches or the cartridge until ready to perform the test.
- Do not reuse the test card.
- Do not reuse the pipet.
- Handle all specimens as potentially infectious. Proper handling and disposal methods should be followed in accordance with local regulations.
- Carefully read and follow user manual to ensure proper test performance.

SPECIMEN COLLECTION AND PREPARATION

- This test can be used for *serum, plasma and whole blood samples*. *Heparin and sodium citrate* should be used as the anticoagulant for plasma and whole blood. Samples should be free of hemolysis.
- Suggest using serum or plasma for better results.
- Serum or plasma can be used directly. For whole blood sample, one drop of whole blood buffer must be added before testing.
- If testing will be delayed, serum and plasma samples may

be stored up to 7 days at 2~8°C or stored at -20°C for 6 months before testing (whole blood sample may be stored up to 3 days at 2~8°C).

- Refrigerated or frozen sample should reach room temperature and be homogeneous before testing. Avoid multiple freeze-thaw cycles.
- Do not use heat-inactivated samples.
- SAMPLE VOLUME (for Getein1100): 100 µl.**

TEST PROCEDURE

- Collect specimens according to user manual.
- Test card, sample and reagent should be brought to room temperature before testing.

For Getein1100:

- Confirm SD card Lot No. in accordance with test kit Lot No.. Perform "SD Card Calib" calibration when necessary (Details refer to 8.5.2 of Getein1100 User Manual).
- On the main interface of Getein1100, press "ENT" button to enter testing interface.
- Remove the test card from the sealed pouch immediately before use. Label the test card with patient or control identification.
- Put the test card on a clean table, horizontally placed.
- Using sample transfer pipette, deliver **100 µl** of sample (or 3~4 drops of sample when using disposable pipet) into the sample port on the test card (for whole blood sample, one drop of whole blood buffer must be added after loading 100 µl sample on the test card).
- Reaction time: 15 minutes.** Insert the test card into Getein1100 and press "ENT" button after reaction time is elapsed. The result will be shown on the screen and printed automatically.

For Getein1600:

- Each cartridge for Getein1600 contains a specific RFID card which can calibrate automatically.
- Place samples in the designed area of the sample holder, insert the holder and select the right test item, Getein1600 will do the testing and print the result automatically.

Notes:

- It is required to perform "SD Card Calib" calibration when using a new batch of kits.
- It is suggested to calibrate once for one batch of kits for Getein1100.

- Make sure the test card and the sample insertion is correct and complete.

TEST RESULTS

Getein1100/Getein1600 can scan the test card automatically and display the result on the screen. For additional information, please refer to the user manual of Getein1100/Getein1600.

EXPECTED VALUE

The expected normal value for CK-MB was determined by testing samples from 500 apparently healthy individuals. The 99th percentile of the concentration for CK-MB is 5.0 ng/ml. (The probability that value of a normal person below 5.0 ng/ml is 99%.)

The expected normal value for cTnI was determined by testing samples from 500 apparently healthy individuals. The 99th percentile of the concentration for cTnI is 0.1 ng/ml. (The probability that value of a normal person below 0.1 ng/ml is 99%.)

The expected normal value for Myo was determined by testing samples from 500 apparently healthy individuals. The 95th percentile of the concentration for Myo is 50 ng/ml. The 97.5th percentile of the concentration for Myo is 70 ng/ml. (According to different Statistics method, the probability that value of a normal person below 50 ng/ml is 95% or below 70 ng/ml is 97.5%.)

It is recommended that each laboratory establish its own expected values for the population it serves.

PERFORMANCE CHARACTERISTICS

	CK-MB	cTnI	Myo
Measuring Range	2.5~80,0 ng/ml	0.1~50,0 ng/ml	30,0~600,0 ng/ml
Lower Detection Limit	≤ 2.5 ng/ml	≤ 0.1 ng/ml	≤ 30.0 ng/ml
Within-Run Precision	≤10%		
Between-Run Precision	≤15%		

Method Comparison:

The assay was compared with HITACHI 7600/OLYMPUS AU5400 and its matching CK-MB test kits, SIEMENS IMMULITE 1000/2000 and its matching cTnI and Myo test kits with 200

serum samples (60 positive samples and 140 negative samples). The correlation coefficient (r) for CK-MB is 0.928, the correlation coefficient (r) for cTnI is 0.952, the correlation coefficient (r) for Myo is 0.938.

LIMITATIONS

- As with all diagnostic tests, a definitive clinical diagnosis should not be made based on the result of a single test. The test results should be interpreted considering all other test results and clinical information such as clinical signs and symptoms.
- Samples containing interferences may influence the results. The table below listed the maximum allowance of these potential interferences.

Interferent	Hemoglobin	Triglyceride	Bilirubin
Concentration (Max)	5 g/L	10 g/L	0.2 g/L

REFERENCES

- Mauro Pantaghini; Undefined International Federation of Clinical Chemistry and Laboratory Medicine (IFCC). Scientific Division Committee on Standardization of Markers of Cardiac Damage. Clin Chem Lab Med, 1998, 36:887~893.
- Antman EM, Anbe DT, Armstrong PW, et al. ACC/AHA guidelines for the management of patients with ST-elevation myocardial infarction: a report of the American College of Cardiology/American Heart Association Task Force on Practice Guidelines (Committee to Revise the 1999 Guidelines for the Manage 2004).
- EN ISO 18113-1:2009 *In vitro* diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 1: Terms, definitions and general requirements.
- EN ISO 18113-2:2009 *In vitro* diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 2: *In vitro* diagnostic reagents for professional use (ISO 18113-2:2009).

DESCRIPTION OF SYMBOLS USED

The following graphical symbols used in or found on CK-MB/cTnI/Myo Fast Test Kit (Immunofluorescence Assay) are the

most common ones appearing on medical devices and their packaging. They are explained in more details in the European Standard EN 980:2008 and International Standard ISO 15223-1:2007.

Key to symbols used			
	Manufacturer		Expiration date
	Do not reuse		Date of manufacture
	Consult instructions for use		Batch code
	Temperature limitation		<i>In vitro</i> diagnostic medical device
	Sufficient for		Authorized representative in the European Community
	CE mark		Do not use if package is damaged

Thank you for purchasing CK-MB/cTnI/Myo Fast Test Kit (Immunofluorescence Assay). Please read this user manual carefully before operating to ensure proper use.

Version: WIF09-S-02

Getein Biotech, Inc.
Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China
Tel: +86-25-68568508
Fax: +86-25-68568500
E-mail: tech@getein.com.cn
overseas@getein.com.cn
Website: www.bio-GP.com.cn

Cardiac Troponin I

Fast Test Kit

(Immunofluorescence Assay)

Getein1100: Cat.# IF1001

Getein1600: Cat.# IF2001

User Manual

INTENDED USE

Cardiac Troponin I Fast Test Kit (Immunofluorescence Assay) is intended for *in vitro* quantitative determination of Cardiac Troponin I (cTnI) in serum, plasma or whole blood. This test is used as an aid in the diagnosis of myocardial injury such as Acute Myocardial Infarction (AMI), Unstable Angina, Acute Myocarditis and Acute Coronary Syndrome (ACS).

SUMMARY

Troponin, a molecular complex that is bound to the thin filament (actin) of striated muscle fibers, acts with intracellular calcium to control the interaction of the thin filament with the thick filament (myosin), thus regulating muscle contraction. Troponin consists of three regulatory proteins: T, which connects the troponin complex and tropomyosin (another cardiac muscle regulatory protein); I, which prevents muscle contraction in the absence of calcium; C, which binds calcium. Cardiac Troponin I (MW 22.5 kDa) and the two skeletal muscle isoforms of Troponin I have considerable amino acid sequence homology, but cTnI contains an additional N-terminal sequence and is highly specific for myocardium.

Clinical studies have demonstrated the release of cTnI into the blood stream within hours following acute myocardial infarction (AMI) or ischemic damage. Elevated levels of cTnI are detectable in blood within 4 to 6 hours after the onset of chest pain, reaching peak concentrations in approximately 8 to 28 hours, and remain elevated for 3 to 10 days following AMI. Due to the high myocardial specificity and the long duration of elevation, cTnI has become an important marker in the diagnosis and evaluation of patients suspected of having an AMI.

The current guideline of The Joint European Society of Cardiology/ American College of Cardiology Committee support the use of cTnI as a preferred marker of myocardial injury. Several major studies have shown that cTnI is also a predictor of cardiac risk in patients with unstable angina. The American College of Cardiology and the American Heart Association's current

guidelines recommend using troponin results when making treatment decisions regarding unstable angina and non-ST segment elevation MI (NSTEMI).

PRINCIPLE

The test uses an anti-human cTnI monoclonal antibody conjugated with fluorescence latex and another anti-human cTnI monoclonal antibody coated on the test line. After the sample has been applied to the test strip, the fluorescence latex-labelled anti-human cTnI monoclonal antibody binds with the cTnI in sample and forms a marked antigen-antibody complex. This complex moves to the test card detection zone by capillary action. Then marked antigen-antibody complex is captured on the test line by the anti-human cTnI monoclonal antibody. The fluorescence intensity of the test line increases in proportion to the amount of cTnI in sample.

Then insert test card into Getein1100 Immunofluorescence Quantitative Analyzer/Getein1600 Immunofluorescence Quantitative Analyzer (hereinafter referred to as Getein1100 and Getein1600), the concentration of cTnI in sample will be measured and displayed on the screen. The value will be stored in Getein1100/Getein1600 and available for downloading. The result can be easily transmitted to the laboratory or hospital information system.

CONTENTS

- A kit for Getein1100 contains:
 - Getein cTnI test card in a sealed pouch with desiccant 25
 - Disposable pipet 25
 - Whole blood buffer 1
 - SD card 1
 - User manual 1
- A kit for Getein1600 contains:
 - Sealed cartridge with 24/48 Getein cTnI test cards 2
 - User manual 1
 - Package specifications:
 - 2x24 tests/kit, 2x48 tests/kit
 - Materials required for Getein1600:
 - Sample diluent 1
 - Box with pipette tips 1
 - Mixing plate 1
- Sample diluent/Whole blood buffer composition:
 - Phosphate buffered saline, proteins, detergent, preservative, stabilizer.
- A test card consists of:
 - A plastic shell and a reagent strip which is composed of a sample pad, nitrocellulose membrane (one end of the membrane is coated with a fluorescence latex-labelled anti-

human cTnI monoclonal antibody, the test line is coated with another anti-human cTnI monoclonal antibody, and the control line is coated with rabbit anti-mouse IgG antibody), absorbent paper and liner.

Note: Do not mix or interchange different batches of kits.

APPLICABLE DEVICE

Getein1100 Immunofluorescence Quantitative Analyzer
Getein1600 Immunofluorescence Quantitative Analyzer

STORAGE AND STABILITY

Store the test card at 4~30°C with a valid period of 24 months. Use the test card for Getein1100 within 1 hour once the foil pouch is opened.

For test card of Getein1600: if the cartridge is opened, it could be stable within 24 hours once exposed to air. If the test cards can't be used up at a time, please put the cartridge back to the foil pouch and reseal along the entire edge of zip-seal. The remaining test cards should be used up within 7 days.

Store the sample diluent/whole blood buffer at 0~30°C with a valid period of 24 months.

Store the sample diluent/whole blood buffer at 2~8°C for better results.

PRECAUTIONS

- For *in vitro* diagnostic use only.
- For professional use only.
- Do not use the kit beyond the expiration date.
- Do not use the test card if the foil pouch or the cartridge is damaged.
- Do not open pouches or the cartridge until ready to perform the test.
- Do not reuse the test card.
- Do not reuse the pipet.
- Handle all specimens as potentially infectious. Proper handling and disposal methods should be followed in accordance with local regulations.
- Carefully read and follow user manual to ensure proper test performance.

SPECIMEN COLLECTION AND PREPARATION

- This test can be used for *serum, plasma and whole blood samples*. *Heparin and sodium citrate* should be used as the anticoagulant for plasma and whole blood. Samples should be free of hemolysis.

- Suggest using serum or plasma for better results.
- Serum or plasma can be used directly. For whole blood sample, one drop of whole blood buffer must be added before testing.
- If testing will be delayed, serum and plasma samples may be stored up to 7 days at 2–8°C or stored at -20°C for 6 months before testing (whole blood sample may be stored up to 3 days at 2–8°C).
- Refrigerated or frozen sample should reach room temperature and be homogeneous before testing. Avoid multiple freeze-thaw cycles.
- Do not use heat-inactivated samples.
- SAMPLE VOLUME (for Getein1100): 100 µl.

TEST PROCEDURE

- Collect specimens according to user manual.
- Test card, sample and reagent should be brought to room temperature before testing.

For Getein1100:

- Confirm SD card lot No. in accordance with test kit lot No.. Perform "SD Card Calib" calibration when necessary (Details refer to 8.5.2 of Getein1100 User Manual).
- On the main interface of Getein1100, press "ENT" button to enter testing interface.
- Remove the test card from the sealed pouch immediately before use. Label the test card with patient or control identification.
- Put the test card on a clean table, horizontally placed.
- Using sample transfer pipette, deliver 100 µl of sample (or 3–4 drops of sample when using disposable pipet) into the sample port on the test card (for whole blood sample, one drop of whole blood buffer must be added after loading 100 µl sample on the test card).
- Reaction time: 10 minutes.** Insert the test card into Getein1100 and press "ENT" button after reaction time is elapsed. The result will be shown on the screen and printed automatically.

For Getein1600:

- Each cartridge for Getein1600 contains a specific RFID card which can calibrate automatically.
- Place samples in the designed area of the sample holder, insert the holder and select the right test item, Getein1600 will do the testing and print the result automatically.

Notes:

- It is required to perform "SD Card Calib" calibration when using a new batch of kits.
- It is suggested to calibrate once for one batch of kits for Getein1100.
- Make sure the test card and the sample insertion is correct and complete.

TEST RESULTS

Getein1100/Getein1600 can scan the test card automatically and display the result on the screen. For additional information, please refer to the user manual of Getein1100/Getein1600.

EXPECTED VALUE

The expected normal value for Troponin I was determined by testing samples from 500 apparently healthy individuals. The 99th percentile of the concentration for cTnI is 0.1 ng/ml. (The probability that value of a normal person below 0.1 ng/ml is 99%.)

It is recommended that each laboratory establish its own expected values for the population it serves.

PERFORMANCE CHARACTERISTICS

Measuring Range	0.1–50 ng/ml
Lower Detection Limit	≤ 0.1 ng/ml
Within-Run Precision	≤10%
Between-Run Precision	≤15%

Method Comparison:

The assay was compared with SIEMENS IMMULITE 2000 and its matching cTnI test kits with 200 serum samples (60 positive samples and 140 negative samples). The correlation coefficient (r) for cTnI is 0.952.

LIMITATIONS

- As with all diagnostic tests, a definitive clinical diagnosis should not be made based on the result of a single test. The test results should be interpreted considering all other test results and clinical information such as clinical signs and symptoms.
- Samples containing interferents may influence the results. The table below listed the maximum allowance of these potential interferents.

Interferent	Hemoglobin	Triglyceride	Bilirubin
Concentration (Max)	5 g/L	10 g/L	0.2 g/L

REFERENCES

- Mauro Pantaghini. Undefined International Federation of Clinical Chemistry and Laboratory Medicine (IFCC). Scientific Division Committee on Standardization of Markers of Cardiac Damage. Clin Chem Lab Med, 1998, 36:887–893.
- Antman EM, Anbe DT, Armstrong PW, et al. ACC/AHA guidelines for the management of patients with ST-elevation myocardial infarction: a report of the American College of Cardiology/American Heart Association Task Force on Practice

Guidelines (Committee to Revise the 1999 Guidelines for the Management 2004).

- EN ISO 18113-1:2009 *In vitro* diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 1: Terms, definitions and general requirements.
- EN ISO 18113-2:2009 *In vitro* diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 2: *In vitro* diagnostic reagents for professional use (ISO 18113-2:2009).

DESCRIPTION OF SYMBOLS USED

The following graphical symbols used in or found on Cardiac Troponin I Fast Test Kit (Immunofluorescence Assay) are the most common ones appearing on medical devices and their packaging. They are explained in more details in the European Standard EN 980:2008 and International Standard ISO 15223-1:2007.

Key to symbols used			
	Manufacturer		Expiration date
	Do not reuse		Date of manufacture
	Consult instructions for use		Batch code
	Temperature limitation		<i>In vitro</i> diagnostic medical device
	Sufficient for		Authorized representative in the European Community
	CE mark		Do not use if package is damaged

Thank you for purchasing Cardiac Troponin I Fast Test Kit (Immunofluorescence Assay). Please read this user manual carefully before operating to ensure proper use.

Version: WIF02-S-02

Getein Biotech, Inc.
 Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China
 Tel: +86-25-68568508
 Fax: +86-25-68568500
 E-mail: tech@getein.com.cn
 overseas@getein.com.cn
 Website: www.bio-GP.com.cn

D-Dimer Fast Test Kit

(Immunofluorescence Assay)

Getein1100: Cat.# IF1006

Getein1600: Cat.# IF2006

User Manual

INTENDED USE

D-Dimer Fast Test Kit (Immunofluorescence Assay) is intended for *in vitro* quantitative determination of D-Dimer in plasma or whole blood. The test is used as an aid in the assessment and evaluation of patients suspected of deep-vein thrombosis or pulmonary embolism.

SUMMARY

Deep-vein thrombosis is a common condition, with a lifetime cumulative incidence of 2 to 5 percent. Untreated deep-vein thrombosis can result in pulmonary embolism, a potentially fatal outcome. Anticoagulant therapy reduces both morbidity and mortality from venous thromboembolism, and early diagnosis is therefore important. Accurate diagnosis of deep-vein thrombosis minimizes the risk of thromboembolic complications and averts the exposure of patients without thrombosis to the risks of anticoagulant therapy.

D-Dimer is a marker of endogenous fibrinolysis and should therefore be detectable in patients with deep-vein thrombosis. In recent years, an increasing number of studies have shown the D-Dimer assay has a high negative predictive value and D-Dimer is a sensitive but nonspecific marker of deep-vein thrombosis. Negative D-Dimer can exclude deep-vein thrombosis and pulmonary embolism.

PRINCIPLE

The test uses an anti-human D-Dimer monoclonal antibody conjugated with fluorescence latex and another anti-human D-Dimer monoclonal antibody coated on the test line. After the sample has been applied to the test strip, the fluorescence latex-labelled anti-human D-Dimer monoclonal antibody binds with the D-Dimer in sample and forms a marked antigen-antibody

complex. This complex moves to the test card detection zone by capillary action. Then marked antigen-antibody complex is captured on the test line by another anti-human D-Dimer monoclonal antibody. The fluorescence intensity of the test line increases in proportion to the amount of D-Dimer in sample. Then insert test card into Getein1100 Immunofluorescence Quantitative Analyzer/Getein1600 Immunofluorescence Quantitative Analyzer (hereinafter referred to as Getein1100 and Getein1600), the concentration of D-Dimer in sample will be measured and displayed on the screen. The value will be stored in Getein1100/Getein1600 and available for downloading. The result can be easily transmitted to the laboratory or hospital information system.

CONTENTS

- A kit for Getein1100 contains:
 - Getein D-Dimer test card in a sealed pouch with desiccant 25
 - Disposable pipet 25
 - Sample diluent 25
 - SD card 1
 - User manual 1
- A kit for Getein1600 contains:
 - Sealed cartridge with 24/48 Getein D-Dimer test cards 2
 - User manual 1
 - Package specifications:
 - 2×24 tests/kit, 2×48 tests/kit
 - Materials required for Getein1600:
 - Sample diluent 1
 - Box with pipette tips 1
 - Mixing plate 1
- Sample diluent composition:
 - Phosphate buffered saline, proteins, detergent, preservative, stabilizer.
- A test card consists of:
 - A plastic shell and a reagent strip which is composed of a sample pad, nitrocellulose membrane (one end of the membrane is coated with a fluorescence latex-labelled anti-human D-Dimer monoclonal antibody, the test line is coated with another anti-human D-Dimer monoclonal antibody and the control line is coated with rabbit anti-mouse IgG antibody), absorbent paper and liner.

Note: Do not mix or interchange different batches of kits.

APPLICABLE DEVICE

Getein1100 Immunofluorescence Quantitative Analyzer
Getein1600 Immunofluorescence Quantitative Analyzer

STORAGE AND STABILITY

Store the test card at 4~30°C with a valid period of 24 months. Use the test card for Getein1100 within 1 hour once the foil pouch is opened.

For test card of Getein1600: if the cartridge is opened, it could be stable within 24 hours once exposed to air. If the test cards can't be used up at a time, please put the cartridge back to the foil pouch and reseal along the entire edge of zip-seal. The remaining test cards should be used up within 7 days.

Store the sample diluent/whole blood buffer at 0~30°C with a valid period of 24 months.

Store the sample diluent/whole blood buffer at 2~8°C for better results.

PRECAUTIONS

- For *in vitro* diagnostic use only.
- For professional use only.
- Do not use the kit beyond the expiration date.
- Do not use the test card if the foil pouch or the cartridge is damaged.
- Do not open pouches or the cartridge until ready to perform the test.
- Do not reuse the test card.
- Do not reuse the pipet.
- Handle all specimens as potentially infectious. Proper handling and disposal methods should be followed in accordance with local regulations.
- Carefully read and follow user manual to ensure proper test performance.

SPECIMEN COLLECTION AND PREPARATION

- This test can be used for *plasma and whole blood samples*. *Sodium citrate* can be used as the anticoagulant for plasma and whole blood. Samples should be free of hemolysis.
- Suggest using plasma for better results.
- If testing will be delayed, plasma sample may be stored up to 3 days at 2~8°C or stored at -20°C for 1 month before testing (whole blood sample may be stored up to 3 days at 2~8°C).
- Refrigerated or frozen sample should reach room temperature

and be homogeneous before testing. Avoid multiple freeze-thaw cycles.

5. Do not use heat-inactivated samples.
6. SAMPLE VOLUME (for *Getein1100*): 100 μ L.

TEST PROCEDURE

1. Collect specimens according to user manual.
2. Test card, sample and reagent should be brought to room temperature before testing.

For *Getein1100*:

3. Confirm SD card lot No. in accordance with test kit lot No.. Perform "SD Card Calib" calibration when necessary (Details refer to 8.5.2 of *Getein1100* User Manual).
4. On the main interface of *Getein1100*, press "ENT" button to enter testing interface.
5. Remove the test card from the sealed pouch immediately before use. Label the test card with patient or control identification.
6. Put the test card on a clean table, horizontally placed.
7. Using sample transfer pipette, deliver 100 μ L of sample into one tube of sample diluent, mix gently and thoroughly. Then drop 100 μ L of sample mixture (or 3-4 drops of sample when using disposable pipet) into the sample port on the test card.
8. **Reaction time: 10 minutes.** Insert the test card into *Getein1100* and press "ENT" button after reaction time is elapsed. The result will be shown on the screen and printed automatically.

For *Getein1600*:

9. Each cartridge for *Getein1600* contains a specific RFID card which can calibrate automatically.
10. Place samples in the designed area of the sample holder, insert the holder and select the right test item, *Getein1600* will do the testing and print the result automatically.

Notes:

1. It is required to perform "SD Card Calib" calibration when using a new batch of kits.
2. It is suggested to calibrate once for one batch of kits for *Getein1100*.
3. Make sure the test card and the sample insertion is correct and complete.

TEST RESULTS

Getein1100/Getein1600 can scan the test card automatically and display the result on the screen. For additional information, please refer to the user manual of *Getein1100/Getein1600*.

EXPECTED VALUE

The expected normal value for D-Dimer was determined by testing samples from 500 apparently healthy individuals. The 95th percentile of the concentration for D-Dimer is 0.5 mg/L. (The probability that value of a normal person below 0.5 mg/L is 95%.)

It is recommended that each laboratory establish its own expected values for the population it serves.

PERFORMANCE CHARACTERISTICS

Measuring Range	0.1~10.0 mg/L
Lower Detection Limit	\leq 0.1 mg/L
Within-Run Precision	\leq 10%
Between-Run Precision	\leq 15%

Method Comparison:

The assay was compared with SIEMENS CA-7000 and its matching D-Dimer test kits with 200 plasma samples (60 positive samples and 140 negative samples). The correlation coefficient (r) for D-Dimer is 0.978.

LIMITATIONS

1. As with all diagnostic tests, a definitive clinical diagnosis should not be made based on the result of a single test. The test results should be interpreted considering all other test results and clinical information such as clinical signs and symptoms.
2. Samples containing interferences such as rheumatoid factor, human anti-mouse antibody and heterophile antibody may influence the results. In this case, results of this test should be used in conjunction with clinical findings and other tests. The table below listed the maximum allowance of these potential interferences.

Interferent	Hemoglobin	Triglyceride	Bilirubin
Concentration (Max)	5 g/L	25 g/L	0.1 g/L

REFERENCES

1. Sarig G, Kil-Drori AJ, Chap-Marshak D, Brenner B, Drugan A. Activation of coagulation in amniotic fluid during normal human pregnancy. *Thromb Res.* 2011 Apr 18.
2. Roldán V, Marín F, Muiña B, Torregrosa JM, Hernández-Romero D, Valdés M, Vicente V, Lip GY. Plasma von Willebrand Factor Levels Are an Independent Risk Factor for Adverse Events Including Mortality and Major Bleeding in Anticoagulated

- Atrial Fibrillation Patients. *J Am Coll Cardiol.* 2011 Apr 11.
3. Sakamoto K, Yamamoto Y, Okamatsu H, Okabe M. D-dimer is helpful for differentiating acute aortic dissection and acute pulmonary embolism from acute myocardial infarction. *Hellenic J Cardiol.* 2011 Mar-Apr; 52(2):123-127.
4. EN ISO 18113-1:2009 *In vitro* diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 1: Terms, definitions and general requirements.
5. EN ISO 18113-2:2009 *In vitro* diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 2: *In vitro* diagnostic reagents for professional use (ISO 18113-2:2009).

DESCRIPTION OF SYMBOLS USED

The following graphical symbols used in or found on D-Dimer Fast Test Kit (Immunofluorescence Assay) are the most common ones appearing on medical devices and their packaging. They are explained in more details in the European Standard EN 980:2008 and International Standard ISO 15223-1:2007.

Key to symbols used			
	Manufacturer		Expiration date
	Do not reuse		Date of manufacture
	Consult instructions for use		Batch code
	Temperature limitation		<i>In vitro</i> diagnostic medical device
	Sufficient for		Authorized representative in the European Community
	CE mark		Do not use if package is damaged

Thank you for purchasing D-Dimer Fast Test Kit (Immunofluorescence Assay). Please read this user manual carefully before operating to ensure proper use.

Version: WIF05-S-02

Getein Biotech, Inc.
Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China
Tel: +86-25-68568508
Fax: +86-25-68568500
E-mail: tech@getein.com.cn
overseas@getein.com.cn
Website: www.bio-GP.com.cn

Cardiac
Markers

Coagulation
Markers

Diabetes
Mellitus

Inflammation

Thyroid
Function

Metabolic
Marker

Renal
Function

Tumor
Markers

Reproduction
/Fertility

Infectious
Disease

Getein
Biotech, Inc.

Stock Code: 603387

OPTIMIZED POINT-OF-CARE SOLUTION

MAKING TEST EASY

Getein 1100

Immunofluorescence Quantitative Analyzer

Getein 1100

Immunofluorescence Quantitative Analyzer

HIGHLY EFFICIENT & ACCURATE

Advanced fluorescence immunoassay

Multiple quality control

REAL-TIME AND RAPID TEST

One-step test

3-15 min/test

5 sec/test for multiple tests

OPERATION MODES

Inside Mode (single sample rapid test mode)

Sample Transfer

Test Card Insert

Click "Start" Icon

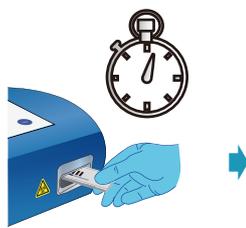
Result Show and Print

Quick mode (mass samples rapid test mode)

Sample Transfer

Timing the Reaction Manually

Click "Start" Icon

Result Show and Print

CONVENIENT OPERATION

RFID card calibration

Keyboard and mouse connectivity through USB port

Handwriting input available

Continuous test for 3 hours with optional lithium battery

USER-FRIENDLY INTERFACE

Android system

7-inch touch screen

1 7-inch Touch Screen

3 Test Card Slot

5 USB Slot

2 SD Card Recognition Zone

4 SD Card Slot

6 Built-in Thermal Printer

PORTABLE DESIGN

Small in size: 261 × 241 × 115 mm

Light in weight: 2.0 kg

LARGE MEMORY

Up to 10,000 results storage capacity

TECHNICAL PARAMETERS

Methodology	Immunofluorescence	Screen	7-inch touch screen
Result	Quantitative	Power Supply	100-240 V~50 Hz/60 Hz, 60 VA
Sample Type	WB, Plasma, Serum, Urine, Stool, Nasal swab, Saliva, Capillary blood	Working Environment	Temperature: 10-35°C Relative humidity ≤ 70% Air pressure 70.0~106.0 kpa
Storage Capacity	10000 data	Dimensions	261 mm×241 mm×115 mm (D×W×H)
Language	English/Chinese/Spanish/Portuguese	Weight	2.0 kg

TEST ITEMS

Cat. #	TEST ITEMS	DISEASES	CUT-OFF VALUE	SAMPLE TYPES	MEASURING RANGE	SAMPLE VOLUME	REACTION TIME	QUALIFICATION
Cardiac Markers								
IF1001	cTnI	Myocardial infarction	0.10 ng/mL	S/P/WB	0.10-50.00 ng/mL	100 µL	10 min	NMPA CE
NEW IF1098	TnT	Myocardial infarction	14.0 pg/mL	S/P/WB	10.0-10000.0 pg/mL	100 µL	15 min	NMPA CE
IF1089	BNP	Heart failure	100.0 pg/mL	P/WB	5.0-5000.0 pg/mL	100 µL	10 min	NMPA CE
IF1002	NT-proBNP	Heart failure	300 pg/mL	S/P/WB	100-35000 pg/mL	100 µL	10 min	NMPA CE
IF1005	CK-MB/cTnI/Myo	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL Myo: 70.0 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL 30.0-600.0 ng/mL	100 µL	10 min	NMPA CE
IF1012	CK-MB/cTnI	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL	100 µL	10 min	CE
IF1014	H-FABP	Myocardial damage	6.36 ng/mL	S/P/WB	1.00-120.00 ng/mL	100 µL	3 min	NMPA CE
IF1016	CK-MB/cTnI/H-FABP	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL H-FABP: 6.36 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL 2.00-100.00 ng/mL	100 µL	10 min	NMPA CE
IF1018	CK-MB	Myocardial injury	5.00 ng/mL	S/P/WB	2.50-80.00 ng/mL	100 µL	10 min	CE
Coagulation Marker								
IF1006	D-Dimer	Venous thromboembolism	0.50 mg/L	P/WB	0.10-10.00 mg/L	100 µL	10 min	NMPA CE
Inflammation								
IF1003	hs-CRP+CRP	Cardiovascular inflammation /normal inflammation	3.0 mg/L 10.0 mg/L	S/P/WB/ Fingertip blood	0.5-200.0 mg/L	10 µL	3 min	NMPA CE
IF1007	PCT	Sepsis, bacterial infection	0.10 ng/mL	S/P/WB	0.05-50.00 ng/mL	100 µL	15 min	NMPA CE
IF1015	PCT/CRP	Sepsis, bacterial infection	PCT: 0.10 ng/mL CRP: 3.0 mg/L	S/P/WB	0.10-50.00 ng/mL 0.5-200.0 mg/L	100 µL	15 min	NMPA CE
IF1044	SAA	Bacterial/Virus infection	10.0 mg/L	S/P/WB/ Fingertip blood	5.0-200.0 mg/L	10 µL	5 min	NMPA CE
IF1090	SAA/CRP	Neonatal sepsis, Bacterial/virus infection	SAA: 10.0 mg/L CRP: 10.0 mg/L	S/P/WB/ Peripheral blood	5.0-200.0 mg/L 0.5-200.0 mg/L	10 µL	5 min	NMPA CE
IF1088	IL-6	Acute inflammation	7.0 pg/mL	S/P/WB/ Peripheral blood	1.5-4000.0 pg/mL	100 µL	15 min	NMPA CE
Renal Function								
IF1008	CysC	Acute and chronic renal diseases	0.51-1.09 mg/L	S/P/WB	0.50-10.00 mg/L	10 µL	3 min	NMPA CE
IF1009	mAlb	Diabetic nephropathy, hypertensive nephropathy	20.0 mg/L	Urine	10.0-200.0 mg/L	100 µL	3 min	NMPA CE
IF1010	NGAL	Acute kidney injury	Serum: 200.0 ng/mL Urine: 100.0 ng/mL	S/Urine	50.0-5000.0 ng/mL	10 µL	10 min	NMPA CE
IF1011	β ₂ -MG	Acute and chronic kidney diseases/tumours	0.80-3.00 mg/L	S/P/WB	0.50-20.00 mg/L	10 µL	3 min	NMPA CE
Diabetes Mellitus								
IF1017	HbA1c	Diabetes mellitus	3.80%-5.80%	WB	2.00%-14.00%	10 µL	5 min	NGSP NMPA IFCC CE
Metabolic Marker								
IF1031	25-OH-VD	Osteomalacia, osteoporosis	30.00-50.00 ng/mL	S/P	8.00-70.00 ng/mL	40 µL	15 min	NMPA CE
Thyroid Function								
IF1024	TSH	Thyroid malfunction	0.27-4.20 µIU/mL	S/P	0.10-50.00 µIU/mL	100 µL	15 min	NMPA CE
IF1022	T3	Hyperthyroidism, hypothyroidism	1.30-3.10 nmol/L	S/P	0.30-10.00 nmol/L	40 µL	15 min	NMPA CE
IF1023	T4	Hyperthyroidism, hypothyroidism	59.00-154.00 nmol/L	S/P	5.40-320.00 nmol/L	100 µL	15 min	NMPA CE
IF1067	fT3	Hyperthyroidism, hypothyroidism	3.10-6.80 pmol/L	S/P/WB	0.60-50.00 pmol/L	100 µL	15 min	CE
IF1068	fT4	Hyperthyroidism, hypothyroidism	12.00-22.00 pmol/L	S/P/WB	0.30-100.00 pmol/L	100 µL	15 min	CE

Cat. #	TEST ITEMS	DISEASES	CUT-OFF VALUE	SAMPLE TYPES	MEASURING RANGE	SAMPLE VOLUME	REACTION TIME	QUALIFICATION
Reproduction/Fertility								
IF1013	HCG+β	Fertility	5.1 mIU/mL	S/P	5.0-100000.0 mIU/mL	100 μL	10 min	NMPA CE
IF1055	LH	PCOS, infertility evaluation	Refer to User Manual	S/P	0.20-150.00 mIU/mL	100 μL	15 min	NMPA CE
IF1056	FSH	PCOS, infertility evaluation and pituitary disorders	Refer to User Manual	S/P	0.20-150.00 mIU/mL	100 μL	15 min	NMPA CE
IF1066	AMH	Fertility, PCOS, gonadal function, precocious/late puberty	Refer to User Manual	S/P	0.10-20.00 ng/mL	200 μL	15 min	CE
IF1048	PRL	Infertility, gonadal disorders	Refer to User Manual	S/P	0.50-200.00 ng/mL	100 μL	15 min	NMPA CE
IF1071	Prog	Infertility, evaluation of ovulation	Refer to User Manual	S/P	0.10-40.00 ng/mL	100 μL	15 min	CE
NEW IF1073	Testosterone	Female polycystic ovary syndrome, male testosterone insufficiency	Male: 1.75-7.81 ng/mL Female: 0.10-0.75 ng/mL	S/P	0.10-16.00 ng/mL	100 μL	15 min	CE
NEW IF1074	E2	Ovarian function	Refer to User Manual	S/P	40.0-4800.0 pg/mL	100 μL	15 min	CE
Tumor Markers								
IF1053	tPSA	Prostate cancer	4.00 ng/mL	S/P	0.50-100.00 ng/mL	100 μL	15 min	NMPA
IF1072	fPSA	Prostate cancer	1.00 ng/mL	S/P	0.05-30.00 ng/mL	100 μL	10 min	NMPA
IF1050	AFP	Liver cancer, cancer of ovaries or testicles, etc.	7.0 ng/mL	S/P	2.0-500.0 ng/mL	100 μL	15 min	CE
IF1051	CEA	Cancer marker: colon cancer etc.	4.7 ng/mL	S/P	2.0-500.0 ng/mL	100 μL	15 min	CE
Infectious Disease								
IF1057	Anti-HCV	Hepatitis C	1.00 S/CO	S/P	1.00-20.00 S/CO	100 μL	15 min	
IF1058	Anti-TP	Syphilis	1.00 S/CO	S/P	1.00-50.00 S/CO	100 μL	15 min	CE
IF1059	Anti-HIV	AIDS	1.00 S/CO	S/P	1.00-1000.00 S/CO	100 μL	15 min	
IF1064	HBsAg	Hepatitis B	1.00 IU/mL	S/P	1.00-100.00 IU/mL	100 μL	15 min	
NEW IF1063	Anti-HBs	Hepatitis B	10.00 mIU/mL	S/P/WB	10.00-1000.00 mIU/mL	100 μL	15 min	
IF1084	2019-nCoV IgM/IgG	COVID-19	1.00 COI	S/P/WB		100 μL	10 min	CE
NEW IF1091	SARS-CoV-2 Antigen	COVID-19	1.00 COI	Nasal swab/Saliva		100 μL	15 min	CE
NEW IF1095	SARS-CoV-2 Neutralizing Antibody	COVID-19	Refer to User Manual	S/P/WB/ Fingertip blood		40 μL	15 min	CE
IF1047	<i>H. pylori</i>	<i>H. pylori</i> infection	5.0 ng/mL	Stool	1.0-200.0 ng/mL	3 drops (about 100 μL)	10 min	CE
NEW IF1086	Influenza A/B	Respiratory viral infection	1.00 S/CO	Nasal swab		100 μL	15 min	CE
NEW IF1136	Dengue NS1 Ag	Dengue virus infection	1.00 S/CO	S/P/WB	0.50-50.00 S/CO	100 μL	15 min	CE
Specific Protein and Rheumatism								
NEW IF1075	RF	Rheumatoid arthritis	15.9 IU/mL	S/P/WB	10.0-640.0 IU/mL	10 μL	10 min	CE
NEW IF1076	ASO	Rheumatic fever, acute glomerulonephritis, group A streptococcal infection	408.0 IU/mL	S/P/WB	60.0-1370.0 IU/mL	10 μL	10 min	CE
NEW IF1029	Anti-CCP	Rheumatoid arthritis	25.0 U/mL	S/P/WB	10.0-400.0 U/mL	10 μL	15 min	CE
Others								
NEW IF1077	Ferritin	Anemia/tumors	Male: 30.00-400.00 ng/mL Female: 13.00-150.00 ng/mL	S/P	0.50-1000.00 ng/mL	10 μL	15 min	CE
NEW IF1069	Total IgE	Allergic disorders	Refer to User Manual	S/P/WB	1.00-2000.00 IU/mL	100 μL	15 min	CE
NEW IF1052	PG I/PG II	Atrophic gastritis, stomach cancer	PG I < 70.0 ng/mL PG I/PG II < 3.0 ng/mL	S/P	PG I: 1.0-200.0 ng/mL PG II: 1.0-100.0 ng/mL	100 μL	15 min	

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EC Declaration of Conformity

In accordance with Directive 98/79/EC

Legal Manufacturer: *Zhejiang Orient Gene Biotech Co., Ltd*

Legal Manufacturer Address: *3787#, East Yangguang Avenue, Dipu Street,
Anji 313300, Huzhou, Zhejiang, China*

Declares, that the products
Product Name and Model(s)

Fecal Occult Blood Rapid Test Strip (Feces)	GEFOB-601b
Fecal Occult Blood Rapid Test Cassette (Feces)	GEFOB-602b

Classification: *Other*
Conformity assessment route: *Annex III (EC DECLARATION OF CONFORMITY)*

We, the Manufacturer, herewith declare with sole responsibility that our product/s mentioned above meet/s the provisions of the Directive 98/79/EC of the European Parliament and of the Council on In-Vitro Diagnostic Medical Devices.

We hereby explicitly appoint

EC Representative's Name: *Shanghai International Holding Corp. GmbH (Europe)*

EC Representative's Address: *Eiffestrasse 80, 20537 Hamburg, Germany*

to act as our European Authorized Representative as defined in the aforementioned Directive.

I, the undersigned, hereby declare that the medical devices specified above conform with the directive 98/79/EC on in vitro diagnostic medical devices and pertinent essential requirements

Date Signed: November 28, 2017

Name of authorized signatory: *Joyce Pang*
Position held in the company: *Vice-President*

Certificate

No. Q5 092305 0001 Rev. 01

Holder of Certificate: **Zhejiang Orient Gene Biotech Co., Ltd.**
3787#, East Yangguang Avenue, Dipu Street Anji
313300 Huzhou, Zhejiang
PEOPLE'S REPUBLIC OF CHINA

Certification Mark:

Scope of Certificate: **Design and Development, Production and Distribution of In Vitro Diagnostic Reagent and Instrument for the Detection of Drugs of Abuse, Fertility, Infectious Diseases, Oncology, Biochemistry, Cardiac Diseases, Allergic Disease based on Rapid Test, PCR and Liquid Biochip Method.**

The Certification Body of TÜV SÜD Product Service GmbH certifies that the company mentioned above has established and is maintaining a quality management system, which meets the requirements of the listed standard(s). All applicable requirements of the testing and certification regulation of TÜV SÜD Group have to be complied with. For details and certificate validity see: [www.tuvsud.com/ps-cert?q=cert:Q5 092305 0001 Rev. 01](http://www.tuvsud.com/ps-cert?q=cert:Q5_092305_0001_Rev.01)

Report No.: SH2198802

Valid from: 2022-04-11

Valid until: 2024-03-16

Date, 2022-04-11

Christoph Dicks

Head of Certification/Notified Body

Product Service

Certificate

No. Q5 092305 0001 Rev. 01

Applied Standard(s): EN ISO 13485:2016
Medical devices - Quality management systems -
Requirements for regulatory purposes
(ISO 13485:2016)
DIN EN ISO 13485:2016

Facility(ies): Zhejiang Orient Gene Biotech Co., Ltd.
3787#, East Yangguang Avenue, Dipu Street Anji, 313300
Huzhou, Zhejiang, PEOPLE'S REPUBLIC OF CHINA

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STATEMENT

We, Zhejiang Orient Gene Biotech Co., Ltd , having a registered office at 3787#, East Yangguang Avenue, Dipu Street Anji 313300, Huzhou, Zhejiang, China assign SRL SANMEDICO having a registered office at A. Corobceanu street 7A, apt. 9, Chişinău MD-2012, Moldova, as non-exclusive authorized representative for Orient Gene Brand product in correspondence with the conditions of directive 98/79/EEC.

We declare that the company mentioned above is authorized to register, notify, renew or modify the registration of medical devices on the territory of the Republic of Moldova.

This Statement letter will be valid from Feb.21th,2023 to Feb.20th, 2024.

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*Urine container, faeces container, universal
container, Pasteur pipette, Petri dishes, Sterile
loops, Sterile loops open "L", Eppendorf tips yellow
and blue, cuvettes for spectrophotometer, samples
cups, Rod to detach clot, disposable forceps,
Disposable plastic tubes, winged stoppers for tubes
diam. 12mm & 16mm, Test tube with granules and clot
activator, vacuum test tube, SEDIPLAST system,
micro test tubes, Slides Mailer, "TESTSIMPLETS" slide,
rack for test tubes, rack for micro test tubes,
Bottles for urine collection.*

SYNTESYS

ISO9001:2008
Cert. N. 6574/0

SYNTESYS S.A.S. DI RINALDO R. & C.
VIA G. GALILEI, 10/3
35037 Z.I. SELVE DI TEOLO (PD)
TEL. +39 049 9903866 R.A. FAX +39 049 9903867
COD.FISCALE P.IVA N.REG.IMP. PADOVA 03573950288
E-MAIL INFO@SYNTESYS.IT - WEB WWW.SYNTESYS.IT

Materiale/ Material

**Polipropilene, Polistirolo, Polietilene e
Polimetilmetacrilato**

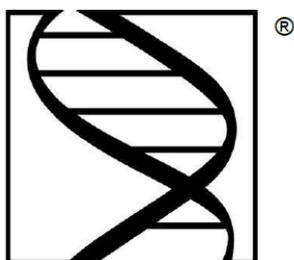
***Polypropylene, Polystyrene, Polyethylene and
Polymethylmetacrylate***

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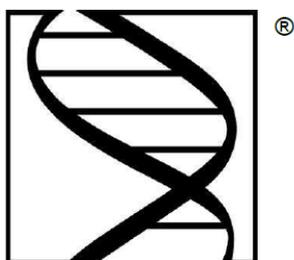
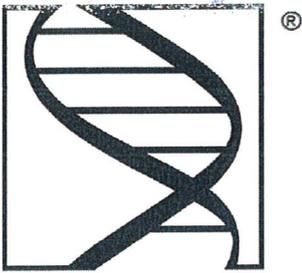
AUTHORIZATION LETTER

We, **Syntesys S.R.L.** having a registered office at Via G. Galilei 10/3, 35037 Selve di Teolo - PD - Italy, assign **Sanmedico SRL** having a registered office at A.Corobceanu str., apt. 9, Chişinău MD-2012, Moldova, as authorized representative.

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Rinaldo Ruggero
CEO and Legal Representative
SYNTESYS S.R.L.

Certificate

CISQ/ICIM S.P.A. has issued an IQNet recognized certificate that the organization:

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Quality Management System

for the following scope:

Trading of products for laboratory analysis. Manufacturing of products for laboratory analysis and sanitary products. Design and production management of sterile swabs for the collection and the preservation of biological samples, also for surgical application, with or without transport medium.

which fulfils the requirements of the following standard:

ISO 9001:2015

Issued on: **2022-06-05**

First issued on: **2013-06-05**

Expires on: **2025-06-04**

This attestation is directly linked to the IQNet Partner's original certificate and shall not be used as a stand-alone document.

Registration Number: **IT-83562**

Alex Stoichitoiu
President of IQNET

Mario Romersi
President of CISQ

This attestation is directly linked to the IQNET Member's original certificate and shall not be used as a stand-alone document.

IQNET Members*:

AENOR Spain **AFNOR Certification** France **APCER** Portugal **CCC** Cyprus **CISQ** Italy **CQC** China **CQM** China **CQS** Czech Republic **Cro Cert** Croatia **DQS Holding GmbH** Germany **EAGLE Certification Group** USA **FCAV** Brazil **FONDONORMA** Venezuela **ICONTEC** Colombia **ICS** Bosnia and Herzegovina **Inspecta Sertifointi Oy** Finland **INTECO** Costa Rica **IRAM** Argentina **JQA** Japan **KFQ** Korea **LSQA** Uruguay **MIRTEC** Greece **MSZT** Hungary **Nemko AS** Norway **NSAI** Ireland **NYCE-SIGE** México **PCBC** Poland **Quality Austria** Austria **SII** Israel **SIQ** Slovenia **SIRIM QAS International** Malaysia **SQS** Switzerland **SRAC** Romania **TSE** Turkey **YUQS** Serbia

* The list of IQNET Members is valid at the time of issue of this certificate. Updated information is available under www.iqnet-certification.com

CISQ is a member of

The International Certification Network
www.iqnet-certification.com

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ICIM S.p.A.