

## STERILISATION REELS USER GUIDE

\*Reels are 5, 7.5, 10, 15, 20, 25, 30, 35 and 40 cm(+/- 1 cm) width and 200m length (+/- 1mt)

\*The reels are cut with 5-7cm part from top and bottom.

\*After cutting process , the product is placed into the reel.

\*Top and bottom part will be opened. It is closed with suitable heat and sealing machine.

\*If necessary the indicator can be put into the reel. After that it will be ready for sterilization process.

\*There are 2 or 3 indicator on the side of reels. Indicators are for steam, ethylene oxide and formaldehyde.

\*After sterilization process color change as below;

(STEAM) Blue to Green      (EO) Pink to Orange      (FO) Purple to Green

\*The sterilization process is made at 134°C for 3,5min or 7 min 121°C for 15min

### ATTENTION

\*Please pay attention to the deformation and tear on the packs.

\*Do not overload the packs.

\*Please pay attention to the seal strength of both ends.

\*Do not use damaged products

\*Keep products at room temperature 25°C