

HOLINX

Changzhou Holinx Industries Co., Ltd.

Add.: No. 1 Dongtang Road, Zhenglu Town, Tianning District, Changzhou City, Jiangsu, China

<https://www.holinx.com/>

Document No.	HLX/ZS-20ml-002	Revision No.	A/0	Date of Revision	2022.02.01
--------------	-----------------	--------------	-----	------------------	------------

TECHNICAL DATASHEET OF STERILE HYPODERMIC SYRINGES FOR SINGLE USE



PRODUCT DETAILS

Description: Disposable 3-Part Syringe 20ml, with needle 21GX1 1/2"

Type: Luer Slip

Graduation: 1ml

Shelf life: 5 years from date of manufacturing

REF No: H20LS21/112

Composition: Disposable 3-Part Syringe with needle is a medical device composed of:

-Barrel;	-Luer Slip precision ensures a secure connection of the needle; -Transparent cylinder allows easier and precise dosing, observation and elimination of air bubbles; -Clear and precise scale allows accurate and simple dosing; -Equipped with stopping design, which prevents the piston from accidentally leaves.
-Piston;	-Anatomically designed piston, which facilitates handling of the syringe. -Easy aspiration up to maximum volume.
-Gasket;	-Latex-free, with double ring; -Ensures good seal and high precision; -Allows a smooth and regular piston progress when in contact with high quality lubricant.
-Needle;	-Provides sharp and smooth puncture.
-Protective cap.	-Tight closure ensures high level of safety against accidental needle stick injury when handling the syringe.

Materials: PP, Rubber (Latex Free), Stainless Steel, Lubricant

HOLINX

Changzhou Holinx Industries Co., Ltd.

Add.: No. 1 Dongtang Road, Zhenglu Town, Tianning District, Changzhou City, Jiangsu, China

<https://www.holinx.com/>

Document No.	HLX/ZS-20ml-002	Revision No.	A/0	Date of Revision	2022.02.01
--------------	-----------------	--------------	-----	------------------	------------

Sterilization: EO (Ethylene Oxide)**Product standards:** ISO 7886-1:2017, ISO 7864-2016**Certificates:** CE, ISO13485**Packaging:** Single Blister packaging needle at side, 50pcs per box, 18boxes per carton**Intended Use:**

Its purpose is to suck and introduce liquids. Not suitable for high pressures or cancer medicines.

Cautions:

- Read instructions before use.
- Before usage confirm the sterility validity, packing completeness, the LOT number and expiry date.
- Use only if package is undamaged and validity not expired;
- The syringes should be used immediately after unit package opened.
- To be used by qualified doctor or paramedic only.
- Check the tightness of needle before the use.
- For single use. The re-sterilization and/or use by more than one patient may cause a cross-infection.
- Destroy after use.

Instructions for use:

1. Select the appropriate syringe with hypodermic needle for your procedure.
2. Examine individual package for integrity of packaging prior to use. Do not use if packaging has been torn.
3. Before usage, open the unit package using aseptic technique.
4. Remove protective needle cap.
5. Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.
6. Apply protective cap when procedure is finished.
7. Any used syringe is considered contaminated and should be disposed of in an approved container immediately after use.
8. To avoid risk of infection, immediately dispose of after single use.

Features:

- Non-toxic;
- Non-pyrogenic;
- PVC, PHT and DEHP free;
- Latex free;
- Sterile unit packages.



HOLINX

Changzhou Holinx Industries Co., Ltd.

Add.: No. 1 Dongtang Road, Zhenglu Town, Tianning District, Changzhou City, Jiangsu, China

<https://www.holinx.com/>

Document No.	HLX/ZS-20ml-002	Revision No.	A/0	Date of Revision	2022.02.01
--------------	-----------------	--------------	-----	------------------	------------

Storage:

Keep away from sunlight. Protect from extreme humidity and heat sources. Not store with organic or chemical solvents. Not store with corrosive gases. Store in dry, cool, good ventilated and clean room, where the relative humidity is less than 80%.


Sample blister packaging design:



Seringă de unică folosință

Precauție:

- Nu resterilizați
- Steril dacă ambalajul nu a fost deschis sau deteriorat
- Nu utilizați dacă ambalajul este deteriorat sau a fost deschis anterior
- Aruncați după o singură utilizare
- Nu depozitați la lumina directă a soarelui, la temperaturi extreme sau la umiditate ridicată



STERILE ISO 13485

Shanghai International Holding Corp. GmbH(Europe)
Add: Eiffestrasse 80, 20537 Hamburg, Germany

Changzhou Holinx Industries Co., Ltd.
No. 1 Dongtang Rd., Zhenglu Town, Tianning District, Changzhou City, Jiangsu, China
Repr. Aut. in Republica Moldova: "HOLINX INDUSTRIES S.R.L."
Adr.: No 42, Gh. Asachi str., Chișinău, Republica Moldova, 2002

20 ML
21GX1 1/2"

REF: HZ00521-112
LOT: 20220816
PROD: 2022.08.16
EXP: 2027.08.15

PROJUS-11-CHINA



Editor:

Audit:

Approve:

