BioGló™ Fluorescein Sodium Ophthalmic Strips USP – fluoresceinovými testními lentmi

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Instructions for use

1. Intended use Designed for use when fitting hard contact lenses. **2. Product description** The BioGló™ diagnostic strips are supplied in packs of 100 or 300. Each diagnostic strip is individually packaged in a sealed and uncoated medical package and in film to ensure product sterility. The rounded part of the strip (orange) is impregnated with 1 mg of fluorescein sodium (I.P) and the white part is for holding. **3. Indications** As recommended by the ophthalmologist. **4. Contraindications** Do not use with soft contact lenses due to the risk of permanent staining of the lens. **5. Users** The product is designed exclusively for professional users. **6. Medical device class** An Is class medical device, bearing the CE mark, has an EU declaration of conformity. **7. Warnings** Discontinue use in the event of persistent irritation or burning. **8. Use** Hold the exposed strip by grasping the white holding paper. Moisten the impregnated area of the strip with sterile saline or tear fluid from the lower conjunctival fold. Move the moistened tip along the conjunctiva of the eyeball with the patient looking up. For best results, the patient should blink several times. **9. Disposal** Because of the possibility of contamination, dispose of only with medical waste. The product is not recyclable.

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Madhu Instruments Pvt. Ltd.