

## Detro<sup>®</sup> Matic A

Alkaline Cleaner for Medical Instrument Washing Machines



### USAGE AREAS

Used for alkaline cleaning of medical equipment made of metal, glass, ceramic, porcelain, plastic materials (sabo, slider, etc.) in automatic washing machines.

► In order for the product to wash effectively at the correct dosage, the device manufacturer's recommendations must be followed. Detro Matic NF or NS is recommended for neutralization of alkali residues after using Detro matic A.

### PRODUCT PROPERTIES

- Automatically cleans the organic soil at all levels of water hardness.
- Cleans and chemothermically disinfects the heat-resistant instruments at 93°C.
- Provides cleaning of heat resistant materials at 40-60°C.
- Contains corrosion inhibitors.
- Compatible with all types of washing machines.

### USAGE

• Used at %0,1 - 0,2 concentration (10 - 20 ml product into 10 L water) in automatic washing machines.

► When using an automatic washing machine, use it together with Detro Matic A (Alkaline Solution) and Detro Matic NS / Detro Matic NF to get better results.



### COMPOSITION

Potassium hydroxide, surfactants, silicates, complexing agents, corrosion inhibitors, auxiliary materials.

### PACKAGE

QUANTITY	PACKING TYPE	BARCODE	CATALOG CODE
5L	3 x CANISTER	8680097100375	S-M.DET.3.0006



## Detro<sup>®</sup> Matic NF-NS

Neutralization Solution for Medical Instrument Washing Machines

### USAGE AREAS

Used in the neutralization process after alkaline cleaning (main washing process) of instruments made of metal, glass, ceramic, porcelain, plastic materials in automatic washing machines and for the removal of lime deposits which may occur in the machine.

► In order for the product to wash effectively at the correct dosage, the device manufacturer's recommendations must be followed. For better results, use with Detro Matic A (Alkaline solution).

### PRODUCT PROPERTIES

- Removes alkali remnants and ensures clear drying of instruments.
- Contains corrosion inhibitors.
- Compatible with all types of washing machines.
- Low usage concentration, economic usage.

### USAGE

• Used at %0,05 - 0,1 concentration (5-10 ml product into per 10 lt water) in automatic washing machines.

• Added to the first rinse water after use of the alkaline cleaner.

► When using an automatic washing machine, use it together with Detro Matic A (Alkaline Solution) and Detro Matic NS / Detro Matic NF to get better results.



### PACKAGE

QUANTITY	PACKING TYPE	BARCODE	CATALOG CODE
(NF) 5L	3 x CANISTER	8680097100382	S-M.DET.3.0008
(NS) 5L	3 x CANISTER	8680097100597	S-M.DET.3.0007

### COMPOSITION

Phosphoric acid or citric acid, surfactant, corrosion inhibitors.