
# STERISEPT WIPES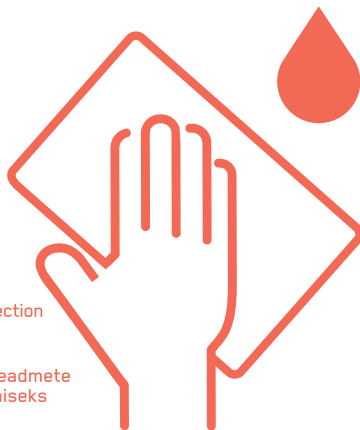

**EN**  
Alcohol-free wipes for rapid disinfection and cleaning of medical devices

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**100 pcs**

LOT

**EN** Alcohol-free disinfection and cleaning wipes for non-invasive medical devices and alcohol sensitive surfaces. Sterisept wipes have a broad spectrum of bactericidal (incl. MRSA, VRE), mycobactericidal and fungicidal activity. The product is also active against viruses (incl. HBV, HIV, HCV, Vaccinia and Rota) and spores (incl. Clostridium difficile).

**USAGE:** Sterisept wipes are ready for use. Thoroughly wipe the surface with the impregnated wipe, ensuring complete coverage. Contact time is at least 60 seconds or depending on required efficacy (see table). After use, close the lid immediately to prevent wipes from drying out.

**INGREDIENTS:** 100 g of product contains active ingredients: Didecyl-Dimethyl-Ammonium Chloride (DDAC) 0,45g ; N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine 0,45g

**STANDARDS:** EN13624, EN13697, EN13704, EN13727, EN14348, EN14476, EN14561, EN16615.

**PRECAUTIONS.** Harmful to aquatic life with long lasting effects. Dispose of contents and / or containers in accordance with regulations on hazardous waste or packaging and packaging waste respectively.

Efficacy/ Mikroorganism	Contact time/ Toimeaeg
Bactericidal/ Bakterid (incl./sh. MRSA, VRE)	1 min
Yeasticidal/ Pärmseened	1 min
Bactericidal and yeasticidal/ Bakterid ja pärmseened	1 min
Mycobactericidal/ Müikobakter	5 min
All enveloped viruses/ Kõik ümbrisega viirused	0,5 min
Rotavirus/ Rotaviirus	5 min
Fungicidal/ Hallitusseened	10 min
Clostridium difficile	15 min

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