


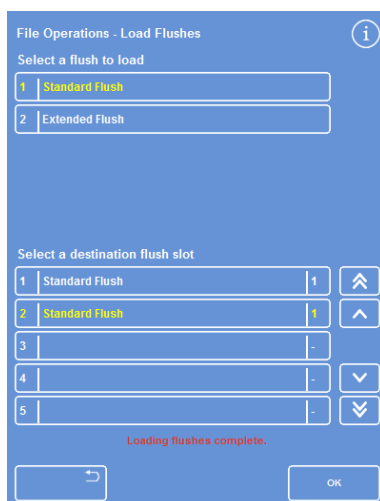

Press OK to confirm

Loading and Saving Setup

Instrument setup information can be saved to a USB memory stick and then loaded from the stick to other instruments. The following setup information will be saved:

- Reagent names
- Storage temperatures
- Rotation management settings
- Reagent use limits
- Access code settings

Setup data is saved to the instrument's source folder on the USB stick. Source folders are labelled with the instrument serial number.

Note

Before saving or loading instrument setup, ensure that a USB memory stick is in place in the instrument. If a USB memory stick is not in place, the options will appear shaded.

To save instrument setup:

- Select **Options > File Operations > Save Setup**.

The setup data is saved to the instrument's source folder on the USB memory stick.

Note

Any existing setup data in the source folder will be overwritten. To save more than one set of setup data, use a separate USB memory stick.

To load instrument setup:

- Select **Options > File Operations > Load Setup**.

The File Operations – Load Setup Screen appears. The current source folder is displayed within the Select a source folder button.

- To change the source folder, press the **Select a source folder** button and select the required folder from the list and press **OK**.

The Options – File Operations screen reappears.

- To return to the Main Screen, press **OK** twice.