



DETROCID ENZYM - USER MANUAL

DETROCID ENZYM ENZYMATIC PRECLEANING AND DISINFECTANT

- ENZYMATIC BASED CLEANING AND DISINFECTION
- SUIT FOR MANUAL, ULTRASONIC AND ENDOSCOPE WASHERS

Product Properties

- Effects in 15 min at %0,5 concentration
- Economic due to very low usage concentration
- Remove contaminations thanks to enzymes
- Has corrosion inhibitors

Usage Areas

- It is used for enzymatic pre-cleaning and disinfection of general surgical equipments, endoscopes and anesthesia equipments etc.

Usage form

- Add 5 ml product into 1 lt water.
- Immerse instruments and wait during 15 min. If possible, wash a little bit instruments before immersing.
- After effect time, take them out and rinse with plenty of water.

Effect time: 15 min.

Preparation of Product Concentration

WATER AMOUNT	%0,5 SOLUTION	EFFECT TIME
1 L	5 ml	15 min.
5 L	25 ml	15 min.
10 L	50 ml	15 min.
15 L	75 ml	15 min.
20 L	100 ml	15 min.

Ingredients

Quarternary ammonium compounds, Amine groups, enzymes (lipase, amylase, protease), auxiliary substances

First aid informations:

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr