

7.4 Image Adjustment Tool Buttons

7.4.1 Control Tools

Icon	Name	Description
 Select	Select	The default status of a mouse cursor. You can select an image or annotations.
 Select All	Select All	Selects all images in the study.
 Zoom	Zoom	Enlarges or reduce an image.
 Pan	Pan	Moves the selected image to the desired position.
 W/L	W/L	Adjusts the brightness and contrast of an image.
 ROI W/L	ROI W/L	Adjusts the brightness and contrast of image in the selected region of interest.
 Glass	Glass	Magnifies a specific region of interest by twice.
 Rect	Crop	Crops an image into a desired shape and size.
 Ellipse	Ellipse	Crops an image into an ellipse.
 Rect	Rectangle	Crops an image into a rectangle.
 Polygon	Polygon	Crops an image into a polygon.
 Free	Free	Crop an image into a desired size of area.
 CW	CW	Rotates an image by 90° clockwise.
 CCW	CCW	Rotates an image by 90° counterclockwise.
 Line	Line	Rotates an image freely by the reference line.
 Free	Free	Rotates an image to a desired degree.
 Fit	Fit	Fits an image to the main image window size. If the image is cropped, the cropped area is fit to the window size.
 Inv.	Invert	Inverts black and white of an image.
 Flip	Flip	Flips an image vertically.