

Dose-control Syringe

INDICATIONS FOR USE

Dose-control syringe is intended to facilitate manual contrast media injection during angiographic procedure.

CONTRAINDICATIONS

None known.

STORAGE AND TRANSPORT

1. Store in a dry, clean, well ventilated place and protect from corrosive gas.
2. Keep away from sunlight, rain, snow and water. Avoid bearing heavy weight during transport.

PRECAUTIONS AND WARNINGS

READ INSTRUCTIONS PRIOR TO USE.

DO NOT USE IF THE UNIT PACKAGING OR THE PRODUCT HAVE BEEN DAMAGED OR SOILED.

1. Ensure that you are making secure connections when using this device to prevent the introduction of air into the system.
2. All connections should be finger tightened.
3. Over tightening can cause cracks and leaks to occur.
4. Examine product carefully for entrapped air and fully bubble prior to injection.
5. Contents supplied STERILE using an ethylene oxide (EO) process.
6. Do not use if sterile barrier is damaged.
7. Inspect prior to use to verify that no damage has occurred during shipping.
8. For single patient use only. Do not reuse, reprocess or re-sterilize. Reuse, reprocessing or re-sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another.
9. Contamination of the device may lead to injury, illness or death of the patient.
10. This device can not be connected with active devices.

Note: After use, the dose-control syringe may be a potential bio hazard. Handle and dispose of in accordance with accepted hospital procedures.

MAINTENANCE

1. To keep the clear of dose-control syringe.
2. The operation should according to the principle of aseptic strictly.
3. To observe whether there is abnormal situation or not.

GRAPHIC SYMBOLS CONTAINED IN DEVICE LABELING



Keep dry



Do not use if
package is
damaged



Consult operating
instructions



Keep away
from sunlight



Date of
Manufacture



Sterilized using
ethylene oxide



Lot number



Do not reuse



Do not resterilize



Attention, see
instructions for use



Catalog number



Use by



CE Mark



SCW Medicath Ltd.

NO.4 Baolong 6th Road, Baolong Industrial Town, Longgang District, Shenzhen, 518116 Guangdong, P.R.China

TEL: (86)755-89312160

FAX: (86)755-89312239



Obelis s.a.

Bd. Général Wahis 53, 1030 Brussels, Belgium