


PAGINA 0

**CREATA PENTRU A
PUTEA SEMNA
ELECTRONIC
DOCUMENTUL**

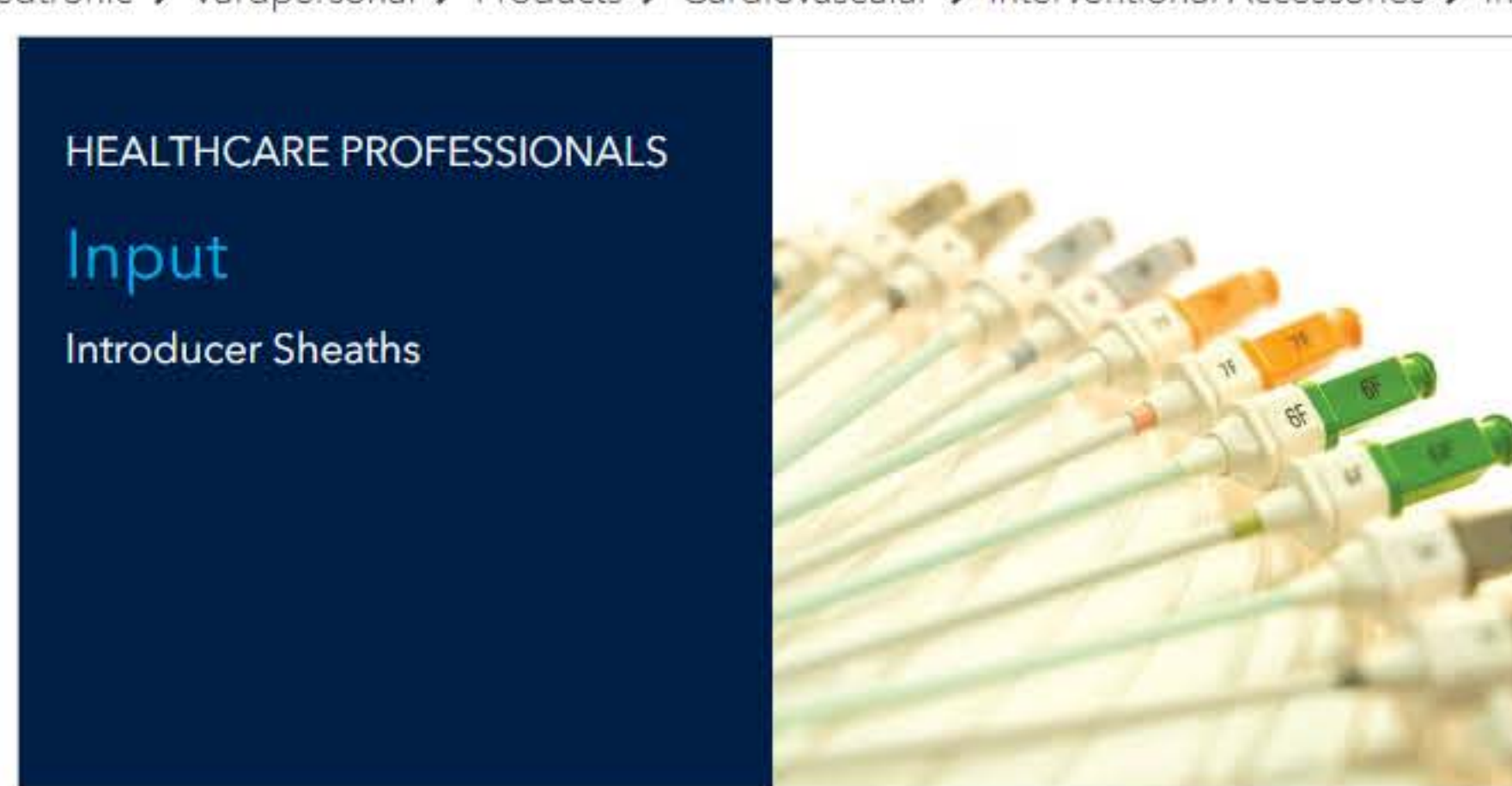

▲ Indications, Safety, and Warnings

INPUT INTRODUCER SHEATHS

Designed for easy insertion and for patient comfort, Input® Introducer Sheaths are offered in two brands – Input TS and Input PS.

HOW IT WORKS

HOW THE INPUT INTRODUCER SHEATH WORKS




Advancing the guidewire

The sheath is introduced with the accompanying dilator.



Introducing the sheath

The dilator is removed. The sheath is left behind in the blood vessel holding the tract open and protecting the vessel from trauma.

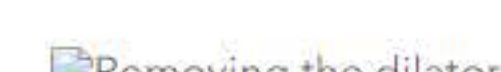

Removing the dilator



Input Introducer Sheaths

Medtronic offers a comprehensive range of access products for the diagnostic angiogram. Designed for easy insertion and for patient comfort, Input® Introducer Sheaths are offered in two brands – Input TS and Input PS.

Input TS is Medtronic's FEP sheath with silicone coating to enhance the low drag coefficient of the sheath and dilator system. The polyethylene dilator with silicone coating and gradual taper reduces force during insertion.

Input PS is based on the Input TS platform. The Input PS sheath includes a proprietary HydroPel coating that reduces insertion force. The Hydrophilic coating activates on contact with fluids to produce a smooth, slick surface. The Pebax material shaft adds the right balance of strength and flexibility for atraumatic insertion into tortuous anatomies.

Product features of Input Introducer Sheaths include:

- Sit-Up Capability: Kink-resistant system provides 60 degree sit-up comfort for patients with an obturator in place.
- Locking Hub: Threaded dilator-to-hub interlock prevents dilator backout during insertion, and secure attachment of the sterile sleeve.
- Enhanced Tip Geometry: Advanced radiofrequency shaping technology creates a long, continuously smooth taper, for minimum resistance during insertions.
- Large Internal Geometry: Input TS's exceptionally large internal geometries facilitate hemodynamic monitoring and fluid administration.

Collapse All

SPECIFICATIONS

SPECIFICATIONS

Input Introducer Sheaths		
	Input TS	Input PS
Sheath Material	FEP	Pebax
Coating	Silicone	HydroPel
Sheath Length (cm)	7, 11, 23	11, 23
Fr Sizes	5, 6, 7, 8, 9, 10, 11	5, 6, 7, 8, 9

ORDERING INFORMATION

INPUT TS ORDERING INFORMATION

7 cm sheath	Product Number	French Size	Obturator Included	Sheath Length (cm)	Wire Style
	040028A	5	No	7	0.038DDJ
	040031A	6	No	7	0.038DDJ
	040035A	7	No	7	0.038DDJ

11 cm sheath	Product Number	French Size	Obturator Included	Sheath Length (cm)	Wire Style
	051101A	5	No	11	0.038DDJ
	061101A	6	No	11	0.038DDJ
	071101A	7	No	11	0.038DDJ
	081101A	8	No	11	0.038DDJ
	091101A	9	No	11	0.038DDJ
	010101A	10	No	11	0.038DDJ
	011101A	11	No	11	0.038DDJ

23 cm sheath	Product Number	French Size	Obturator Included	Sheath Length (cm)	Wire Style
	060037A	6	Yes	23	N/A
	070037A	7	Yes	23	N/A
	080037A	8	Yes	23	N/A
	090037A	9	Yes	23	N/A

SHEATH WITH NEEDLE AND GUIDEWIRE

11 cm sheath	Product Number	French Size	Obturator Included	Needle Length (cm)	Needle Gauge
	051102A	5	11	7	18
	061102A	6	11	7	18
	071102A	7	11	7	18
	081102A	8	11	7	18
	091102A	9	11	7	18

INPUT PS ORDERING INFORMATION

11 cm sheath	Product Number	French Size	Obturator Included	Sheath Length (cm)	Wire Style
	050011	5	No	11	0.038DDJ
	060011	6	No	11	0.038DDJ
	070011	7	No	11	0.038DDJ
	080011	8	No	11	0.038DDJ
	090011	9	No	11	0.038DDJ
	853311	5	No	11	N/A
	863311	6	No	11	N/A
	873311	7	No	11	N/A
	883311	8	No	11	N/A
	893311	9	No	11	N/A

23 cm sheath	Product Number	French Size	Obturator Included	Sheath Length (cm)	Wire Style
	060023	6	Yes	23	N/A
	070023	7	Yes	23	N/A
	080023	8	Yes	23	N/A
	090023	9	Yes	23	N/A

INTRODUCER SHEATH STERILE SLEEVES

	Product Number	Sheath Length
	026000	26
	060000	60

PATIENTER OCH VÅRDGIVARE

Översikt

VÅRDPERSONAL

- Behandlingar
- Översikt
- Behandlingar och procedurer
- Produkter
- Utbildning
- Produktbeställningar
- Produktretur
- Kliniskt register
- MRI information

VÅRT FÖRETAG

- Vilka vi är
- Målsättning
- Företagsinformation
- Ledarskap
- Karriär
- Medtronic Sverige
- Kontor

VÅR PÅVERKAN

Vår påverkan