Sterile Airway Needle/Subculture Units

REF 233766

43-03311
527-03-83

For use with blood culture bottles.

100 Units/20 Gauge

For Laboratory Use.

Disposable: Use once and destroy.

Sterility assured unless individual seal is broken.

Preliminary comments and precautions

1. Remove culture bottles from instrument as soon as possible after being flagged positive.
2. Place positive culture bottles under a safety hood and wear protective equipment.
3. When venting, general caution should be taken as positive culture bottles could have been overfilled or contain high gas-producing organisms. **Positive culture bottle contents may be under increased internal pressure.**
4. If overfilling or increased gas pressure is suspected (e.g. by bulging bottle septum) place the bottle at room temperature for at least 10 minutes before continuing.

Directions for Venting Procedure


1. Disinfect stopper of positive culture bottle.
2. Break seal of sterile airway/subculture unit. Remove white cap from unit, exposing short beveled needle. Do not disturb the clear plastic sheath.
3. Always direct the positive culture bottle to be vented away from the user to minimize the risk of exposure should liquid under pressure escape from the bottle.
4. Insert beveled needle through center of disinfected stopper. To ensure adequate venting, partially lift the plastic sheath off the venting needle base (approximately 0.5 cm). Hold for two or three seconds before reseating the sheath.
5. Excess gas will escape from the positive culture bottle.

Directions for Subculture Procedure

1. If dispersion of the bottle contents is required, keep the previously vented bottle in an upright position and gently swirl the bottle in a circular motion.
2. Aseptically remove the clear plastic sheath, exposing the longer, blunt-ended subculture needle.
3. Tip bottle and place 1-2 drops of broth onto surface of plated medium and 1 drop onto a glass slide for staining.
4. After subculture, carefully remove airway/subculture unit and dispose into sharps container. Please follow your institution's guidelines for proper disposal of sharps. **DO NOT REUSE AIRWAY/SUBCULTURE UNIT.**

For technical assistance in the USA, contact bioMérieux Customer Service at 1-800-682-2666. Outside the USA, contact your local bioMérieux Representative.

Dist. by
bioMérieux, Inc.
Box 15969
Durham, North Carolina 27704-0969

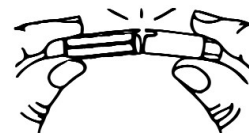To open Airway/
Subculture Unit,
break seal with a
twisting motion.

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© BIOMÉRIEUX 2004, 2006