



Optochin Discs

DD009

Intended use

Recommended for identification and differentiation of *Streptococcus pneumoniae* and viridans streptococci.

Directions

Prepare Soyabean Casein Digest Agar (M290) w/blood or Blood Agar Base (M073) w/blood plates and streak pure culture of organism to be tested across one half of the plate. Streak a known *Pneumococcus* (for eg. *Streptococcus pneumoniae*) culture across the other half of the plate as positive control. Immediately place Optochin discs in the centre of the two halves of the plate and incubate at 35-37°C for 18-24 hours in 5% CO₂. Following incubation observe for zone of inhibition around the discs.

Principle And Interpretation

Alpha haemolytic (viridans) streptococci and *Pneumococcus* (*Streptococcus pneumoniae*) cannot be easily distinguished on Blood Agar plates as pneumococci strain shows partial clearing of blood and greenish discolouration (α -haemolysis). Optochin is inhibitory for pneumococcal growth whereas other streptococci strains show good growth or a very small zone of inhibition. Bowers and Jeffries have shown a correlation between bile solubility and full Optochin susceptibility for the differentiation of *Streptococcus pneumoniae* from other streptococci (1). Hence optochin test is a useful diagnostic aid for identification /differentiation of pneumococci and viridans streptococci. Optochin discs are filter paper discs impregnated with optochin. The test is based on the property of viridans streptococci to grow in the presence of Optochin (ethyl hydrocuprein hydrochloride) which inhibits pneumococci. This test is performed for the diagnosis of pneumococcal infections. Specimens of sputum, lung aspirate or urine are first examined by Gram's stain, cultured and the isolates are then subjected to Optochin Sensitivity Test.

Type of specimen

Clinical samples - pure isolate

Specimen Collection and Handling:

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (2,3).

After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions :

In Vitro diagnostic use only. For professional use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling clinical specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations :

1. Other biochemical and serological tests must be performed for confirmation.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Filter paper discs of 6 mm diameter bearing letters "Op" in continuous printing style

Cultural Response

DD009 : Cultural response observed after an incubation at 35-37°C for 18-24 hours in 5% CO₂ on seeded Soyabean Casein Digest Agar (M290) with added sterile defibrinated blood using Optochin discs.

Organism	Diameter of Zone of Inhibition & Interpretation
<i>Streptococcus pneumoniae</i> ATCC 6303	More than or equal to 15 mm, Sensitive
<i>Streptococcus pyogenes</i> ATCC 19615	No zone of inhibition or a minimal zone, Resistant

Storage and Shelf Life

Store between 2-8°C in a tightly closed container. Use before expiry date on the label. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (2,3).

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
Reference

1. Bowers. E.F. and Jeffries L.R., 1995, J. Clin. Path., 8:58.
2. Isenberg, H.D. Clinical Microbiology Procedures Handbook 2nd Edition.
3. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

Revision : 02/2022



HiMedia Laboratories Pvt.
Limited, Plot No.C-40, Road
No.21Y, MIDC, Wagle Industrial
Area, Thane (W) -400604, MS,
India




CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



**In vitro diagnostic
medical device**



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CE Marking



Storage temperature



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.



Xylose-Lysine Deoxycholate Agar (XLD Agar)

M031

Intended use

Recommended for the isolation and enumeration of *Salmonella* Typhi and other *Salmonella* species from clinical and non-clinical samples.

Composition**

Ingredients	g / L
Yeast extract	3.000
L-Lysine	5.000
Lactose	7.500
Sucrose	7.500
Xylose	3.500
Sodium chloride	5.000
Sodium deoxycholate	2.500
Sodium thiosulphate	6.800
Ferric ammonium citrate	0.800
Phenol red	0.080
Agar	15.000
Final pH (at 25°C)	7.4±0.2

**Formula adjusted, standardized to suit performance parameters

Directions

Suspend 56.68 grams in 1000 ml purified / distilled water. Heat with frequent agitation until the medium boils. **DO NOT AUTOCLAVE OR OVERHEAT.** Transfer immediately to a water bath at 50°C. After cooling, pour into sterile Petri plates. It is advisable not to prepare large volumes that will require prolonged heating, thereby producing precipitate. **Note :** Slight precipitation in the medium may occur, which is inheritant property of the medium, and does not affect the performance of the medium.

Principle And Interpretation

XLD Agar has been recommended for the identification of *Enterobacteriaceae* (1) and for the microbiological testing. XLD Agar was formulated by Taylor (2-6) for the isolation and differentiation of enteric pathogens including *Salmonella* Typhi from other *Salmonella* species of foods, water and dairy products (7-11). XLD Agar exhibits increased selectivity and sensitivity as compared to other plating media e.g. SS Agar (M108), EMB Agar (M022) and Bismuth Sulphite Agar (M027) (3,5,7,12-15). The media formulation does not allow the overgrowth of other organisms over *Salmonella* and *Shigella* (16). Samples suspected of containing enteric pathogens, along with other mixed flora, are initially enriched in Modified Semisolid RV Medium Base (M1482) (17). It is also recommended by FDA (18).

The medium contains yeast extract, which provides nitrogen and vitamins required for growth. Though the sugars xylose, lactose and sucrose provide sources of fermentable carbohydrates, xylose is mainly incorporated into the medium since it is not fermented by *Shigellae* but practically by all enterics. This helps in the differentiation of *Shigella* species. Sodium chloride maintains the osmotic balance of the medium. Lysine is included to differentiate the *Salmonella* group from the non-pathogens. *Salmonellae* rapidly ferment xylose and exhaust the supply. Subsequently lysine is decarboxylate by the enzyme lysine decarboxylase to form amines with reversion to an alkaline pH that mimics the *Shigella* reaction. However, to prevent this reaction by lysine-positive coliforms, lactose and sucrose are added to produce acid in excess. Degradation of xylose, lactose and sucrose to acid causes phenol red indicator to change its colour to yellow. Bacteria that decarboxylate lysine to cadaverine can be recognized by the appearance of a red colouration around the colonies due to an increase in pH. These reactions can proceed simultaneously or successively, and this may cause the pH indicator to exhibit various shades of colour or it may change its colour from yellow to red on prolonged incubation. To add to the differentiating ability of the formulation, an H₂S indicator system, consisting of sodium thiosulphate and ferric ammonium citrate, is included for the visualization of hydrogen sulphide produced, resulting in the formation of colonies with black centers. The non-pathogenic H₂S producers do not decarboxylase lysine; therefore, the acid reaction produced by them prevents the blackening of the colonies (2).

XLD Agar is both selective and differential medium. It utilizes sodium deoxycholate as the selective agent and therefore it is inhibitory to gram-positive microorganisms.

Type of specimen

Clinical samples - Faeces; Food and dairy samples; Water samples.

Specimen Collection and Handling:

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (19,20). For food and dairy samples, follow appropriate techniques for sample collection and processing as per guidelines (9,10). For water samples, follow appropriate techniques for sample collection, processing as per guidelines and local standards (8). After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions :

In Vitro diagnostic Use. For professional use only. Read the label before opening the container. Wear protective gloves/ protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations :

1. Slight precipitation in the medium may occur, which is inheritant property of the medium, and does not affect the performance of the medium.
2. This medium is general purpose medium and may not support the growth of fastidious organisms.
3. Some *Proteus* strains may give red to yellow colouration with most colonies developing black centers, giving rise to false positive reactions.
4. Non-enterics like *Pseudomonas* and *Providencia* may exhibit red colonies.
5. *S. Paratyphi A*, *S. Choleraesuis*, *S. Pullorum* and *S. Gallinarum* may form red colonies without H₂S, thus resembling *Shigella* species.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Light yellow to pink homogeneous free flowing powder

Gelling

Firm, comparable with 1.5% Agar gel

Colour and Clarity of prepared medium

Red coloured clear to slightly opalescent gel forms in Petri plates

Reaction

Reaction of 5.67% w/v aqueous solution at 25°C . pH : 7.4±0.2

pH

7.20-7.60

Cultural Response

Cultural response was observed after an incubation at 35-37°C for specified time. Recovery rate is considered as 100% for bacteria growth on Soyabean Casein Digest Agar.

Organism	Inoculum (CFU)	Growth	Observed Lot value (CFU)	Recovery	Colour of Colony	Incubation period
<i>Salmonella</i> Typhimurium ATCC 14028 (00031*)	50 -100	luxuriant	25 -100	≥50 %	red with black centres	18 -72 hrs
<i>Salmonella</i> Abony NCTC 6017 (00029*)	50 -100	good-luxuriant	25 -100	≥50 %	red with black centres	18 -72 hrs
<i>Escherichia coli</i> ATCC 8739 (00012*)	50 -100	fair	10 -30	20 -30 %	yellow	18 -72 hrs
<i>Escherichia coli</i> ATCC 25922 (00013*)	50 -100	fair	10 -30	20 -30 %	yellow	18 -72 hrs

Please refer disclaimer Overleaf.

<i>\$ Proteus hauseri</i> ATCC 13315	50 -100	good-luxuriant	25 -100	≥ 50 %	grey with black centres	18 -72 hrs
<i>Salmonella</i> Paratyphi A ATCC 9150	50 -100	good-luxuriant	25 -100	≥ 50 %	red	18 -72 hrs
<i>Salmonella</i> Paratyphi B ATCC 8759	50 -100	good-luxuriant	25 -100	≥ 50 %	red with black centres	18 -72 hrs
<i>Salmonella</i> Enteritidis ATCC 13076 (00030*)	50 -100	good-luxuriant	25 -100	≥ 50 %	red with black centres	18 -72 hrs
<i>Salmonella</i> Typhi ATCC 6539	50 -100	good-luxuriant	25 -100	≥ 50 %	red with black centres	18 -72 hrs
<i>Shigella dysenteriae</i> ATCC 13313	50 -100	good-luxuriant	25 -100	≥ 50 %	red	18 -72 hrs
<i>Shigella flexneri</i> ATCC 12022 (00126*)	50 -100	fair-good	15 -40	30 -40 %	red	18 -72 hrs
<i>Shigella sonnei</i> ATCC 25931	50 -100	fair-good	15 -40	30 -40 %	red	18 -72 hrs
# <i>Klebsiella aerogenes</i> ATCC 13048 (00175*)	50 -100	fair	10 -30	20 -30 %	yellow	18 -72 hrs
<i>Enterobacter cloacae</i> ATCC 13047 (00083*)	50 -100	fair	10 -30	20 -30 %	yellow	18 -72 hrs
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	$\geq 10^4$	inhibited	0	0%		≥ 72 hrs
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 6538 (00032*)	$\geq 10^4$	inhibited	0	0%		≥ 72 hrs
<i>Enterococcus faecalis</i> ATCC 29212 (00087*)	$\geq 10^4$	inhibited	0	0%		≥ 72 hrs

Key : *Corresponding WDCM numbers.

(#) Formerly known as *Enterobacter aerogenes* \$ Formerly known as *Proteus vulgaris*

Storage and Shelf Life

Store between 10-30°C in a tightly closed container and the prepared medium at 20-30°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use.

Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (19,20).

Reference

1. Chadwick P., Delisle G. H and Byer M., 1974, Can. J. Microbiol., 20, 1653-1664.
2. Taylor W. L., 1965, Am. J. Clin. Pathol., 44:471-475.
3. Taylor W. L. and Harris B., 1965, Am. J. Clin. Pathol., 44:476.
4. Taylor W. L. and Harris B., 1967, Am. J. Clin. Pathol., 48:350.
5. Taylor W. L. and Schelhart B., 1967, Am. J. Clin. Pathol., 48:356.
6. Taylor W. L. and Schelhart B., 1968, Am. J. Clin. Pathol., 16:1387.
7. Taylor W. L. and Schelhart B., 1969, Appl. Microbiol., 18:393-395.
8. Lipps WC, Braun-Howland EB, Baxter TE, eds. Standard methods for the Examination of Water and Wastewater, 24th ed. Washington DC:APHA Press; 2023.

Please refer disclaimer Overleaf.

9. Salfinger Y., and Tortorello M.L. Fifth (Ed.), 2015, Compendium of Methods for the Microbiological Examination of Foods, 5th Ed., American Public Health Association, Washington, D.C.
10. Wehr H. M. and Frank J. H., 2004, Standard Methods for the Microbiological Examination of Dairy Products, 17th Ed., APHA Inc., Washington, D.C.
11. Williams H., (Ed.), 2005, Official Methods of Analysis of the Association of Official Analytical Chemists, 19th Ed., AOAC, Washington, D.C.
12. Dunn C. and Martin W. J., 1971, Appl. Microbiol., 22, 17-22.
- 14 MacCarthy M. D., 1966, N. Z. J. Med. Lab. Technol., 20, 127-131.
15. Rollender M. A., Beckford O., Belsky R. D and Kostroff B. 1969, Am. J. Clin. Pathol., 51, 284-286.
16. Taylor W. L. and Schelhart B., 1969, Appl. Micro. 18, 1387-1392.
17. Isenberg H. D., Kominos S., and Sigel M., 1969, Appl Microbiol., 18, 656-659.
18. FDA Bacteriological Analytical Manual, 2005, 18th Ed., AOAC, Washington, D.C.
19. Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
20. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock, D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

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
Revision: 06/2024




HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



**In vitro diagnostic
medical device**



Storage temperature

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

CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.



Kligler Iron Agar

M078

Intended Use:

Recommended for differential identification of gram-negative enteric bacilli from clinical and non-clinical samples on the basis of the fermentation of glucose (dextrose), lactose and hydrogen sulphide production.

Composition**

Ingredients	g / L
Peptone	15.000
HM Peptone B #	3.000
Yeast extract	3.000
Proteose peptone	5.000
Lactose	10.000
Dextrose	1.000
Ferrous sulphate	0.200
Sodium chloride	5.000
Sodium thiosulphate	0.300
Phenol red	0.024
Agar	15.000
Final pH (at 25°C)	7.4±0.2

**Formula adjusted, standardized to suit performance parameters

- Equivalent to Beef extract

Directions

Suspend 57.52 grams in 1000 ml purified/distilled water. Heat to boiling to dissolve the medium completely. Mix well and distribute into tubes. Sterilize by autoclaving at 15 lbs pressure (121°C) for 15 minutes. Allow the tubes to cool in slanted position to form slopes with about 1 inch butts. Best reactions are obtained on freshly prepared medium. Do not use screw capped tubes or bottles.

Note: Avoid overheating otherwise it may produce precipitate in the medium.

Principle And Interpretation

Kligler Iron Agar is a combination of the lead acetate medium described by Kligler (1,2) and Russels Double Sugar Agar (3) and is used as a differentiation medium for typhoid, dysentery and allied bacilli (4). Bailey and Lacey substituted phenol red for Andrade indicator previously used as pH indicator (4). Kligler Iron Agar differentiates lactose fermenters from the non-fermenters. It differentiates *Salmonella* Typhi from other *Salmonellae* and also *Salmonella* Paratyphi A from *Salmonella* Scottmuelleri and *Salmonella* Enteritidis (5). Fermentation of dextrose results in production of acid, which turns the indicator from red to yellow. Since there is little sugar i.e. dextrose, acid production is very limited and therefore a reoxidation of the indicator is produced on the surface of the medium, and the indicator remains red. However, when lactose is fermented, the large amount of acid produced, avoids reoxidation and therefore the entire medium turns yellow. Kligler Iron Agar, in addition to Peptone, HM peptone B and yeast extract, contains lactose and glucose (dextrose), which enables the differentiation of species of enteric bacilli. Phenol red is the pH indicator, which exhibits a colour change in response to acid produced during the fermentation of sugars. The combination of ferrous sulphate and sodium thiosulphate enables the detection of hydrogen sulphide production, which is evidenced by a black color either throughout the butt, or in a ring formation near the top of the butt. Lactose non-fermenters (e.g., *Salmonella* and *Shigella*) initially produce a yellow slant due to acid produced by the fermentation of the small amount of glucose (dextrose). When glucose (dextrose) supply is exhausted in the aerobic environment of the slant, the reaction reverts to alkaline (red slant) due to oxidation of the acids produced. The reversion does not occur in the anaerobic environment of the butt, which therefore remains acidic (yellow butt). Lactose fermenters produce yellow slants and butts because of lactose fermentation. The high amount of acids thus produced helps to maintain an acidic pH under aerobic conditions. Tubes showing original colour of the medium indicates the fermentation of neither glucose (dextrose) nor lactose. Gas production (aerogenic reaction) is detected as individual bubbles or by splitting or displacement of the agar by the formation of cracks in the butt of the medium.

Pure cultures of suspected organisms from plating media such as MacConkey Agar (M081), Bismuth Sulphite Agar (M027) or Deoxycholate Citrate Agar (M065), SS Agar (M108) etc. are inoculated on Kligler Iron Agar for identification.

Type of specimen

Isolated microorganism from clinical, food, dairy and water samples.

Specimen Collection and Handling

For water samples, follow appropriate techniques for sample collection, processing as per guidelines and local standards (6). For food and dairy samples, follow appropriate techniques for sample collection and processing as per guidelines (7,8,9). For clinical samples follow appropriate techniques for handling specimens as per established guidelines (10,11). After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions

In Vitro diagnostic use. For professional use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations

1. Results should be noted after 18-24 hours to avoid erroneous results.
2. Straight wire loop should be used for inoculation.
3. Pure isolates should be used to avoid erroneous results.
4. Other biochemical and serological tests must be performed for complete identification

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Light yellow to pink homogeneous free flowing powder

Gelling

Firm, comparable with 1.5% Agar gel

Colour and Clarity of prepared medium

Red coloured, clear to slightly opalescent gel forms in tubes as slants

Reaction

Reaction of 5.75% w/v aqueous solution at 25°C. pH : 7.4±0.2

pH

7.20-7.60

Cultural Response

Cultural characteristics observed after an incubation at 35-37°C for 18 - 48 hours.

Organism	Growth	Gas	H ₂ S	Slant	Butt
<i>Escherichia coli</i> ATCC 25922 (00013*)	luxuriant	positive reaction	negative reaction, no blackening of medium	acidic reaction, yellowing of the medium	acidic reaction, yellowing of the medium
<i>Klebsiella aerogenes</i> ATCC 13048 (00175*)	luxuriant	positive reaction	negative reaction, no blackening of medium	acidic reaction, yellowing of the medium	acidic reaction, yellowing of the medium
<i>Citrobacter freundii</i> ATCC 8090	luxuriant	positive reaction	positive reaction, blackening of medium	acidic reaction, yellowing of the medium	acidic reaction, yellowing of the medium
<i>Proteus hauseri</i> ATCC 13315	luxuriant	negative reaction	positive reaction, blackening of medium	alkaline reaction, red colour of the medium	acidic reaction, yellowing of the medium
<i>Klebsiella pneumoniae</i> ATCC 13883 (00087*)	luxuriant	positive reaction	negative reaction, no blackening of medium	acidic reaction, yellowing of the medium	acidic reaction, yellowing of the medium
<i>Salmonella Paratyphi A</i> ATCC 9150	luxuriant	positive reaction	negative reaction, no blackening of medium	alkaline reaction, red colour of the medium	acidic reaction, yellowing of the medium

<i>Salmonella</i> Schottmuelleri ATCC 10719	luxuriant	positive reaction	positive reaction, blackening of medium	alkaline reaction, red colour of the medium	acidic reaction, yellowing of the medium
<i>Salmonella</i> Typhi ATCC 6539	luxuriant	negative reaction	positive reaction, blackening of medium	alkaline reaction, red colour of the medium	acidic reaction, yellowing of the medium
<i>Salmonella</i> Enteritidis ATCC 13076 (00030*)	luxuriant	positive reaction	positive reaction, blackening of medium	alkaline reaction, red colour of the medium	acidic reaction, yellowing of the medium
<i>Shigella flexneri</i> ATCC 12022 (00126*)	luxuriant	negative reaction	negative reaction,no blackening of medium	alkaline reaction, red colour of the medium	acidic reaction, yellowing of the medium
<i>Pseudomonas aeruginosa</i> ATCC 27853 (00025*)	luxuriant	negative reaction	negative reaction, blackening of medium	alkaline reaction, red colour of the medium	alkaline reaction,red colour of the medium
<i>Yersinia enterocolitica</i> ATCC 27729	luxuriant	variable reaction	negative reaction,no blackening of medium	alkaline reaction,red colour of the medium	acidic reaction, yellowing of the medium
<i>Enterobacter cloacae</i> ATCC 13047 (00083*)	luxuriant	positive reaction	negative reaction,no blackening of medium	acidic reaction, yellowing of the medium	acidic reaction, yellowing of the medium

Key : * Corresponding WDCM numbers

(#) Formerly known as *Enterobacter aerogenes*

Formerly known as *Proteus vulgaris*

Storage and Shelf Life

Store between 10-30°C in a tightly closed container and the prepared medium at 2-8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (10,11).

Reference

- 1.Kligler I. J., 1917, Am. J. Publ. Health, 7:1041.
- 2.Kligler I. J., 1918, J. Exp. Med., 28:319.
- 3.Russell F. F., 1911, J. Med. Res., 25:217.
- 4.Bailey S. F. and Lacey G. R., 1927, J. Bacteriol., 13:183.
- 5.Ewing, 1986, Edwards and Ewings Identification of the Enterobacteriaceae, 4th Ed., Elsevier Science Publishing Co., Inc., N.Y.
- 6.Lipps WC, Braun-Howland EB, Baxter TE, eds. Standard methods for the Examination of Water and Wastewater, 24th ed. Washington DC:APHA Press; 2023.
- 7.American Public Health Association, Standard Methods for the Examination of Dairy Products, 1978, 14th Ed., Washington D.C.
- 8.Salfinger Y., and Tortorello M.L., 2015, Compendium of Methods for the Microbiological Examination of Foods, 5th Ed., American Public Health Association, Washington, D.C.
- 9.Wehr H. M. and Frank J. H., 2004, Standard Methods for the Microbiological Examination of Dairy Products, 17th Ed., APHA Inc., Washington, D.C.
- 10.Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
- 11.Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock, D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1

Revision : 05/2024



HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



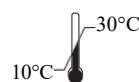
CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



**In vitro diagnostic
medical device**



CE Marking



Storage temperature



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.



Columbia Blood Agar Base

M144

Intended Use:

For preparation of blood agar, chocolate agar and for preparation of various selective and identification media and isolation of organisms from clinical and non clinical samples.

Composition**

Ingredients	g / L
Peptone, special	23.000
Corn starch	1.000
Sodium chloride	5.000
Agar	15.000
Final pH (at 25°C)	7.3±0.2

**Formula adjusted, standardized to suit performance parameters

Directions

Suspend 44.0 grams of in 1000 ml purified/ distilled water. Heat to boiling to dissolve the medium completely. Sterilize by autoclaving at 15 lbs pressure (121°C) for 15 minutes. Cool to 45-50°C before adding heat sensitive compounds.

For Blood Agar: Add 5% v/v sterile defibrinated sheep blood to sterile cool base.

For Chocolate Agar: Add 10% v/v sterile defibrinated sheep blood to sterile cool base. Heat to 80°C for 10 minutes with constant agitation.

The medium can be made selective by adding different antimicrobials to sterile base.

For *Brucella* species: Add rehydrated contents of 1 vial of NPBCVN Selective Supplement (FD005) to 500 ml sterile molten base.

For *Campylobacter* species: Add rehydrated contents of 1 vial of Blaser-Wang Selective Supplement (FD006) or Butzler Selective Supplement (FD007) or Skirrow Selective Supplement (FD008) or VTCA Selective Supplement (FD090) or Butzler VI Selective Supplement (FD106) to 500 ml sterile molten base along with rehydrated contents of 1 vial of Minerals Growth Supplement (FD009) and 5-7% v/v horse or sheep blood.

For *Gardnerella* species: Add rehydrated contents of 1 vial of GNA Selective Supplement (FD056) to 500 ml sterile molten base.

For Cocci: Add rehydrated contents of 1 vial of NC Selective Supplement (FD030) or NNP Selective Supplement (FD031) or CO Selective Supplement (FD119) to 500 ml sterile molten base.

Principle And Interpretation

Columbia Blood Agar Base was devised by Ellner et al (1). This medium contains special peptone which supports rapid and luxuriant growth of fastidious and non-fastidious organisms. Also, this medium promotes typical colonial morphology; better pigment production and more sharply defined haemolytic reactions. Fildes found that Nutrient Agar supplemented with a digest of sheep blood supplied both of these factors and the medium would support the growth of *H. influenzae* (2,3). The inclusion of bacitracin makes the enriched Columbia Agar Medium selective for the isolation of *Haemophilus* species from clinical specimens, especially from upper respiratory tract (4). Columbia Agar Base is used as the base for the media containing blood and for selective media formulations in which different combinations of antimicrobial agents are used as additives.

Corn starch serves as an energy source and also neutralizes toxic metabolites. Sheep blood permits the detection of haemolysis and also provides heme (X factor) which is required for the growth of many bacteria. However it is devoid of V factor (Nicotinamide adenine dinucleotide) and hence *Haemophilus influenzae* which needs both the X and V factors, will not grow on this medium.

Columbia Agar Base with added sterile serum provides an efficient medium for *Corynebacterium diphtheriae* virulence test medium. After following the established technique for *C. diphtheriae*, lines of toxin-antitoxin precipitation are clearly visible in 48 hours. Many pathogens require carbon dioxide; therefore, plates may be incubated in an atmosphere containing approximately 3-10% CO₂.

Precaution: Brucella cultures are highly infective and must be handled carefully; incubate in 5-10% CO₂. Campylobacter species are best grown at 42°C in a micro aerophilic atmosphere. Plates with Gardnerella supplements plates should be incubated at 35°C for 48 hours containing 7% CO₂ (2).

Type of specimen

Clinical samples : throat swabs, pus.

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (5,6).
After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions

In Vitro diagnostic use only. For professional use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations

1. Certain fastidious organisms like *Haemophilus influenzae* may not grow on the medium, blood supplementation may be required.
2. As this medium have a relatively high carbohydrate content, beta-hemolytic *Streptococci* may exhibit a greenish hemolytic reaction which may be mistaken for the alpha haemolysis.
3. Biochemical characterization is required on colonies of pure culture for complete identification.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Cream to yellow homogeneous free flowing powder.

Gelling

Firm, comparable with 1.5% Agar gel.

Colour and Clarity of prepared medium

Basal medium: Light amber coloured clear to slightly opalescent gel.

After addition of 5%w/v sterile defibrinated blood : Cherry red coloured opaque gel forms in Petri plates.

Reaction

Reaction of 4.4% w/v aqueous solution at 25°C. pH : 7.3±0.2

pH

7.10-7.50

Cultural Response

Cultural characteristics observed with added 5% w/v sterile defibrinated blood, after an incubation at 35-37°C for 24-48 hours.

Organism	Inoculum (CFU)	Growth	Recovery	Haemolysis
<i>Neisseria meningitidis</i> ATCC 13090	50-100	luxuriant	≥70%	none
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	50-100	luxuriant	≥70%	beta / gamma
<i>Staphylococcus epidermidis</i> ATCC 12228 (00036*)	50-100	luxuriant	≥70%	gamma
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 6538 (00032*)	50-100	luxuriant	≥70%	beta / gamma
<i>Streptococcus pneumoniae</i> ATCC 6303	50-100	luxuriant	≥70%	alpha
<i>Streptococcus pyogenes</i> ATCC 19615	50-100	luxuriant	≥70%	beta
<i>Clostridium sporogenes</i> ATCC 19404 (00008*)	50-100	luxuriant	≥50 %	
<i>Clostridium sporogenes</i> ATCC 11437	50-100	luxuriant	≥50 %	
<i>Clostridium perfringens</i> ATCC 13124 (00007*)	50-100	luxuriant	≥50 %	
<i>Clostridium perfringens</i> ATCC 12934	50-100	luxuriant	≥50 %	

Key : (*) Corresponding WDCM numbers.

Storage and Shelf Life

Store between 10-30°C in a tightly closed container and the prepared medium at 2-8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (5,6).

Reference

1. Ellner P. P., Stoessel C. J., Drakeford E. and Vasi F., 1966, Am. J. Clin. Pathol., 45:502.
2. Fildes P., 1920, Br. J. Exp. Pathol., 1:129.
3. Fildes P., 1921, Br. J. Exp. Pathol., 2:16.
4. Chapin K. C. and Doern G. V., 1983, J. Clin. Microbiol., 17:1163.
5. Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
6. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.

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
Revision : 05/2024



HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



**In vitro diagnostic
medical device**



Storage temperature



CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.



Mueller Hinton Agar

M173

Intended Use:

Recommended for determination of susceptibility of microorganisms to antimicrobial agents isolated from clinical samples.

Composition**

Ingredients	g / L
HM infusion solids B # (from 300g)	2.000
Acicase ##	17.500
Starch	1.500
Agar	17.000
Final pH (at 25°C)	7.3±0.1

**Formula adjusted, standardized to suit performance parameters

- Equivalent to Beef heart infusion

- Equivalent to Casein acid hydrolysate

Directions

Suspend 38.0 grams in 1000 ml purified/ distilled water. Heat to boiling to dissolve the medium completely. Sterilize by autoclaving at 15 lbs pressure (121°C) for 15 minutes. Cool to 45-50°C. Mix well and pour into sterile Petri plates. Note: The performance of this batch has been tested and standardised as per the current CLSI (formerly, NCCLS) document M6-protocols for Evaluating Dehydrated Mueller Hinton Agar.

Principle And Interpretation

The Mueller Hinton formulation was originally developed as a simple, transparent agar medium for the cultivation of pathogenic *Neisseria* species (1). Other media were subsequently developed that replaced the use of Mueller Hinton Agar for the cultivation of pathogenic *Neisseria* species, but it became widely used in the determination of sulfonamide resistance of gonococci and other organisms. Mueller Hinton Agar is now used as a test medium for antimicrobial susceptibility testing (2). Mueller Hinton Agar is recommended for the diffusion of antimicrobial agents impregnated on paper disc through an agar gel as described in CLSI Approved Standard (3). Mueller Hinton Agar has been selected by the CLSI for several reasons:

- It demonstrates good batch-to-batch reproducibility for susceptible testing.
- It is low in sulfonamide, trimethoprim and tetracycline inhibitors.
- It supports the growth of most non-fastidious bacterial pathogens and
- Many data and much experience regarding its performance have been recorded (4).

Kirby-Bauer et al recommended this medium for performing antibiotic susceptibility tests using a single disc of high concentration (5). WHO Committee on Standardization of Susceptibility Testing has accepted Mueller Hinton Agar for determining the susceptibility of microorganisms because of its reproducibility (6). Mueller Hinton Agar with 5% sheep blood and Mueller Hinton Agar with Hemoglobin have been recommended for antimicrobial susceptibility testing of *Streptococcus pneumoniae* and *Haemophilus influenzae*.

HM infusion B from and acicase provide nitrogenous compounds, carbon, sulphur and other essential nutrients. Starch acts as a protective colloid against toxic substances present in the medium. Starch hydrolysis yields dextrose, which serves as a source of energy. These ingredients are selected for low thymine and thymidine content as determined by MIC values for *Enterococcus faecalis* with sulfamethoxazole trimethoprim (SXT).

The Kirby-Bauer procedure is based on agar diffusion of antimicrobial substances impregnated on paper discs. This method employs disc with a single concentration of antimicrobial agent and the zone diameters observed are correlated with minimum inhibitory concentration (MIC) values (7,1,2). A standardized suspension of the organism is swabbed over the entire surface of the medium.

Paper discs impregnated with specific amounts of antimicrobial agents are then placed on the surface of the medium, incubated and zones of inhibition around each disc are measured. The susceptibility is determined by comparing with CLSI standards (4). The various factors, which influence disc diffusion susceptibility tests, are agar depth, disc potency, inoculum concentration, pH of the medium and beta-lactamase production by test organisms (4,8).

Mueller Hinton Agar is not appropriate for assay by disc diffusion method with slow growing organisms, anaerobes and

capnophiles. With slow growing organisms, increased incubation may cause deterioration of diffusing antibiotic and produce unprecise readings (9). Mueller Hinton Agar is recommended for the diffusion of antimicrobial agents impregnated on paper disc through an agar gel as described in NCCLS (National Committee for Clinical Laboratory Standards), now CLSI (Clinical and Laboratory Standards Institute) Approved Standard (10).

Type of specimen

Clinical samples : Pure cultures isolated from urine , stool, blood etc.

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (2,10-13).

After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions

In Vitro diagnostic use only. For professional use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations

1. This medium is recommended for susceptibility testing of pure cultures only.
2. Inoculum density may affect the zone size. Heavy inoculum may result in smaller zones or too less inoculum may result in bigger zones.
3. Fastidious organisms may not grow on this medium and may require supplementation of blood.
4. Fastidious anaerobes may not grow on this medium.
5. As antimicrobial susceptibility is carried with antibiotic disc, proper storage of the disc is desired which may affect the potency of the disc.
6. Under certain circumstances, the in vitro results of antibiotic susceptibility may not show the same in vivo.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Cream to yellow homogeneous free flowing powder

Gelling

Firm, comparable with 1.7% agar gel.

Colour and Clarity of prepared medium

Light amber coloured clear to slightly opalescent gel forms in Petri plates

Reaction

Reaction of 3.8% w/v aqueous solution at 25°C. pH : 7.3±0.1

pH

7.20-7.40

Cultural Response

Antibiotic susceptibility tests are performed in accordance with, and meet the acceptance limits of the current ISO/TS 16782 (15). Performance of the medium is checked in accordance with the CLSI/ EUCAST guidelines.

For testing *S. pneumoniae* : The medium was supplemented with 5% Horse blood and 20 mg/l NAD , incubated at 34-36°C for 18-20 hours in 5% CO₂ .

For testing *H. influenzae* : The medium was supplemented with 5% Horse blood and 20 mg/l -NAD, incubated at 34-36°C for 18-20 hours in 5% CO₂ .

Antibiotic Sensitivity test

Various discs were tested for standard ATCC strains and zone of inhibition were measured after an incubation 30-35°C for 18 hours. (As per the latest CLSI Protocol M6 & Standards as per the current CLSI M100).

Thymine/Thymidine Content

The zones for these discs are indicative of the Thymine/Thymidine content of the medium.

Divalent Cation Content

\$ The zones for these discs are indicative of the Divalent Cation content of the medium

Organism	Growth	Standard Zone	Incubation temperature	Incubation period
<i>Escherichia coli</i> ATCC 25922 (00013*)	luxuriant		34-36°C	16-20 hours
Cephalothin CEP 30mcg		15-21 mm		
Ampicillin AMP 10mcg		15-22 mm		
Chloramphenicol C 30 mcg		21-27 mm		
Gentamicin GEN 10mcg		19-26 mm		
Co-Trimoxazole (Sulpha/Trimethoprim) (COT) 25 mcg		23-29 mm		
Sulphafurazole SF 300 mcg		15-23 mm		
Cefotaxime CTX 5 mcg		25-31 mm		
Tigecycline TGC 15mcg		20-27 mm		
Tetracycline TE 30 mcg		18-25 mm		
Amoxicillin- clavulanate AMC 30 mcg		18-24 mm		
Ciprofloxacin CIP 5mcg		29-38 mm		
<i>Escherichia coli</i> ATCC 35218	luxuriant		34-36°C	16-20 hours
Amoxicillin- clavulanate AMC 30 mcg		17-22 mm		
Piperacillin/Tazobactam PIT 100/10 mcg		24-30 mm		
Ticarcillin TI 75 mcg		6 mm		
Ticarcillin/Clavulanic acid TCC 75/10mcg		21-25mm		
Ampicillin AMP 10 mcg		6 mm		
Ampicillin/Sulbactam A/S 10/10 mcg		13-19 mm		
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	luxuriant		34-36°C	16-20 hours
Erythromycin E 15 mcg		22-30 mm		
Linezolid LZ 30 mcg		24-30 mm		
Tetracycline TE 30 mcg		24-30 mm		
Ciprofloxacin CIP 5mcg		22-30 mm		
Amoxyclav(Amoxicillin/Clavulanic acid) AMC 30 mcg		28-36 mm		
Co-Trimoxazole COT 25 mcg		24-32 mm		
Cefoxitin CX 30 mcg		23-29 mm		
Oxacillin OX 1mcg		18-24 mm		
Pristinomycin RP 15 mcg		21-28 mm		
Gentamicin GEN 10 mcg		19-27 mm		
Penicillin-G 10 units		26-37 mm		
Ampicillin/Sulbactam A/S 10/10 mcg		29-37 mm		
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 29213 (00131*)	luxuriant		34-36°C	16-20 hours
Penicillin-G P 1 unit		12-18 mm		
Cefoxitin CX 30 mcg		24-30 mm		
Erythromycin E 15 mcg		23-29 mm		
Linezolid LZ 10 mcg		21-27 mm		
Gentamicin GEN 10 mcg		19-25 mm		
Tetracycline TE 30 mcg \$		23-31 mm		
Ciprofloxacin CIP 5mcg		21-27 mm		

<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 43300 (MRSA) (00211*)	luxuriant	34-36°C	24 hours
Oxacillin OX 1 mcg	Very Hazy to No Zone		
Cefoxitin CX 30 mcg	<=21 mm		
<i>Pseudomonas aeruginosa</i> ATCC 27853 (00025*)	luxuriant	34-36°C	16-20 hours
Ceftazidime CAZ 30 mcg	22-29 mm		
Ciprofloxacin CIP 5mcg	25-33 mm		
Tobramycin TOB 10 mcg \$	20-26 mm		
Amikacin AK 30 mcg \$	20-26 mm		
Aztreonam AT 3mcg	23-29 mm		
Cephotaxime CTX 30 mcg	18-22 mm		
Gentamicin GEN 10 mcg \$	17-23 mm		
Imipenem IPM 10 mcg	20-28 mm		
Piperacillin PI 100 mcg	25-33 mm		
Piperacillin Tazobactam PIT 30/6 mcg	23-29 mm		
<i>Enterococcus faecalis</i> ATCC 29212 (00087*)	luxuriant	34-36°C	16-20 hours
Trimethoprim TR 5 mcg #	24-32 mm		
Ampicillin AMP 2 mcg	15-21 mm		
Imipenem IPM 10 mcg	24-30 mm		
Linezolid LZ 10 mcg	19-25 mm		
Nitrofurantoin NIT 100 mcg	18-24 mm		
Co-Trimoxazole (Sulpha/ Trimethoprim) (COT) 25 mcg	26-34 mm		
Vancomycin VA 5 mcg	10-16 mm		
<i>Enterococcus faecalis</i> ATCC33186 (00210*)	luxuriant	34-36°C	16-20 hours
Co-Trimoxazole (Sulpha/ Trimethoprim) (COT) 25 mcg	<=20 mm		
<i>Streptococcus pneumoniae</i> ATCC 49619	luxuriant	34-36°C	18-20 hours
Vancomycin VA 5 mcg	17-23 mm		
<i>Haemophilus influenzae</i> ATCC 49247	luxuriant	34-36°C	18-20 hours
Ampicillin AMP 2 mcg	6-12 mm		
<i>Haemophilus influenzae</i> ATCC 49766	luxuriant	34-36°C	18-20 hours
Cefixime CFM 5 mcg	29-35 mm		

Key : *Corresponding WDCM numbers.

Storage and Shelf Life

Store between 10-30°C in a tightly closed container and the prepared medium at 20-30°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (3,5).

References

- 1.Murray P. R., Baron J. H., Pfaller M. A., Jorgensen J. H. and Tenover F. C., (Ed.), 2003, Manual of Clinical Microbiology, 8th Ed., American Society for Microbiology, Washington, D.C.
- 2.National Committee for Clinical Laboratory Standards, 2000, Approved Standard: M7-A5. Methods for Dilution Antimicrobial Susceptibility Tests for Bacteria that grow aerobically, 5th Ed., NCCLS, Wayne, Pa.
- 3.Isenberg, H.D. Clinical Microbiology Procedures Handbook. 2nd Edition.
- 4.MacFaddin J. F., 1985, Media for Isolation-Cultivation-Identification-Maintenance of Medical Bacteria, Vol. 1, Williams and Wilkins, Baltimore
- 5.Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
- 6.Present Status and Future Work, WHO Sponsored collaborative study, Chicago, Oct. 1967.
- 7.Ericsson H. M. and Sherris J. L., 1971, Acta Pathol. Microbiol., Scand. Sect B Suppl., 217:1.
- 8.NCCLS Approved Standard: ASM-2, 1979, Performance Standards for Antimicrobial disc Susceptibility Tests, 2nd Ed.,National Committee for Clin. Lab. Standards.
- 9.Mueller J. H. and Hinton J., 1941, Proc. Soc. Exp. Biol. Med., 48:330.
10. Performance Standards of Antimicrobial Susceptibility Testing; 34th Edition. M100-Ed34, Vol.44, No.5, Jan-2024.
- 11.ISO/TS 16782:2016, Confirmed in 2021 Clinical laboratory testing - Criteria for acceptable lots of dehydrated Mueller-Hinton agar and broth for antimicrobial susceptibility testing
- 12.European Committee on Antimicrobial Susceptibility Testing Breakpoint tables for interpretation of MICs and zone diameters Version 14.0, valid from 2024-01-01.
- 13.European Committee on Antimicrobial Susceptibility Testing Routine and extended internal quality control as recommended by EUCAST Version 14.0, valid from 2024-01-01.

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
Revision : 06/2024



HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



**In vitro diagnostic
medical device**



Storage temperature



CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



**Do not use if
package is damaged**

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.



HiCrome™ UTI Agar

M1353

Intended use

Recommended for presumptive identification and confirmation of microorganisms mainly causing urinary tract infections, can also be used for testing water, food, environmental and other clinical samples.

Composition**

Ingredients	g / L
Peptone, special	15.000
Chromogenic mixture	2.450
Agar	15.000
Final pH (at 25°C)	6.8±0.2

**Formula adjusted, standardized to suit performance parameters

Directions

Suspend 32.45 gram in 1000 ml purified /distilled water. Heat to boiling to dissolve the medium completely. Sterilize by autoclaving at 15 lbs pressure (121°C) for 15 minutes. Cool to 45-50°C. Mix well and pour into sterile Petri plates.

Principle And Interpretation

Urinary tract infections are bacterial infections affecting parts of urinary tract. The common symptoms of urinary tract infection are urgency and frequency of micturition, with associated discomfort or pain. The common condition is cystitis, due to infection of the bladder with a uropathogenic bacterium, which most frequently is *Escherichia coli*, but sometimes *Staphylococcus saprophyticus* or especially in hospital-acquired infections, *Klebsiella* species, *Proteus mirabilis*, other coliforms, *Pseudomonas aeruginosa* or *Enterococcus faecalis* (1). HiCrome™ UTI Agar is formulated on basis of work carried out by Pezzlo (2) Wilkie et al (3), Friedman et al (4), Murray et al (5), Soriano and Ponte (6) and Merlino et al (7). These media are recommended for the detection of urinary tract pathogens where HiCrome™ UTI Agar has broader application as a general nutrient agar for isolation of various microorganisms. It facilitates and expedites the identification of some gram-negative bacteria and some gram-positive bacteria on the basis of different contrasted colony colours produced by reactions of genus or species specific enzymes with two chromogenic substrates. The chromogenic substrates are specifically cleaved by enzymes produced by *Enterococcus* species, *E.coli* and coliforms. Presence of amino acids like phenylalanine and tryptophan from peptones helps for detection of tryptophan deaminase activity, indicating the presence of *Proteus* species, *Morganella* species and *Providencia* species.

One of the chromogenic substrate is cleaved by β -glucosidase possessed by Enterococci resulting in formation of blue colonies. *E.coli* produce pink colonies due to the enzyme β -D-galactosidase that cleaves the other chromogenic substrate. Further confirmation of *E.coli* can be done by performing the indole test. Coliforms produce purple coloured colonies due to cleavage of both the chromogenic substrate. Colonies of *Proteus*, *Morganella* and *Providencia* species appear brown because of tryptophan deaminase activity. Peptone special provides nitrogenous, carbonaceous compounds, long chain amino acids, vitamins and other essential growth nutrients. This medium can be made selective by supplementation with antibiotics for detecting microorganisms associated with hospital borne infections.

Type of specimen

Clinical samples : urine, faeces, etc.; Food samples, Water samples.

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (8,9).

For food and dairy samples, follow appropriate techniques for sample collection and processing as per guidelines (10,11). For water samples, follow appropriate techniques for sample collection, processing as per guidelines and local standards (12).

After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions

In Vitro diagnostic use. For professional use only. Read the label before opening the container. Wear protective gloves/ protective clothing/eye protection/face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets.

Limitations

1. Since it is an enzyme-substrate based reaction, the intensity of colour may vary with isolates.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Cream to yellow homogeneous free flowing powder

Gelling

Firm, comparable with 1.5% Agar gel

Colour and Clarity of prepared medium

Light amber coloured, clear to slightly opalescent gel forms in Petri plates

Reaction

Reaction of 3.24% w/v aqueous solution at 25°C. pH : 6.8±0.2

pH

6.60-7.20

Cultural Response

Cultural characteristics observed after an incubation at 35-37°C for 16-24 hours.

Organism	Inoculum (CFU)	Growth	Recovery	Colour of Colony
<i>Escherichia coli</i> ATCC 25922 (00013*)	50-100	luxuriant	≥70%	Purple to magenta
<i>Enterococcus faecalis</i> ATCC 29212 (00087*)	50-100	luxuriant	≥70%	blue-green (small)
<i>Klebsiella pneumoniae</i> ATCC 13883 (00097*)	50-100	luxuriant	≥70%	blue to purple, mucoid
<i>Proteus mirabilis</i> ATCC 12453	50-100	luxuriant	≥70%	light brown
<i>Pseudomonas aeruginosa</i> ATCC 27853 (00025*)	50-100	luxuriant	≥70%	colourless (greenish pigment may be observed)
<i>Staphylococcus aureus</i> subsp. <i>aureus</i> ATCC 25923 (00034*)	50-100	luxuriant	≥70%	golden yellow

Key : *Corresponding WDCM numbers.

Storage and Shelf Life

Store between 15-25°C in a tightly closed container and the prepared medium at 2-8°C. Use before expiry date on the label. On opening, product should be properly stored dry, after tightly capping the bottle in order to prevent lump formation due to the hygroscopic nature of the product. Improper storage of the product may lead to lump formation. Store in dry ventilated area protected from extremes of temperature and sources of ignition Seal the container tightly after use. Product performance is best if used within stated expiry period.

Disposal

User must ensure safe disposal by autoclaving and/or incineration of used or unusable preparations of this product. Follow established laboratory procedures in disposing of infectious materials and material that comes into contact with clinical sample must be decontaminated and disposed of in accordance with current laboratory techniques (8,9).

Reference

- Collee J. G., Fraser A. G., Marmion B. P., Simmons A., (Eds.), Mackie and McCartney, Practical Medical Microbiology, 1996, 14th Edition, Churchill Livingstone.
- Pezzlo M., 1998, Clin. Microbiol. Rev., 1:268-280.
- Wilkie M. E., Almond M. K., Marsh F. P., 1992, British Medical Journal 305:1137-1141.
- Friedman M. P. et al, 1991, J. Clin. Microbiol., 29:2385-2389.

5. Murray P., Tenenbaum P. Hopson D., 1992, J. Clin. Microbiol., 30:1600- 1601.
6. Soriano F., Ponte C., 1992, J. Clin. Microbiol., 30:3033-3034.
7. Merlino et al, 1995, Abstr. Austr. Microbiol. 16(4):17-3.
8. Isenberg, H.D. Clinical Microbiology Procedures Handbook 2nd Edition.
9. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., Funke, G., Landry, M.L., Richter, S.S and Warnock., D.W. (2015) Manual of Clinical Microbiology, 11th Edition. Vol. 1.
10. Salfinger Y., and Tortorello M.L., 2015, Compendium of Methods for the Microbiological Examination of Foods, 5th Ed., American Public Health Association, Washington, D.C.
11. Wehr H. M. and Frank J. H., 2004, Standard Methods for the Microbiological Examination of Dairy Products, 17th Ed., APHA Inc., Washington, D.C.
12. Lipps WC, Braun-Howland EB, Baxter TE, eds. Standard methods for the Examination of Water and Wastewater, 24th ed. Washington DC:APHA Press; 2023.

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
Revision : 07/2024



HiMedia Laboratories Pvt. Limited,
Plot No.C-40, Road No.21Y,
MIDC, Wagle Industrial Area,
Thane (W) -400604, MS, India



In vitro diagnostic
medical device



Storage temperature



CEpartner4U, Esdoornlaan 13,
3951DB Maarn, NL
www.cepartner4u.eu



CE Marking



Do not use if
package is damaged

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.

Cefoxitin (Cephoxitin)**CX****30 mcg****SD041**

Cefoxitin (Cephoxitin) CX 30 mcg discs are used for antimicrobial susceptibility testing of bacterial cultures as per Kirby- Bauer Method

Composition***Ingredients**

	Concentration
Cefoxitin (Cephoxitin)	30 mcg/disc

Susceptibility Test Procedure:

1. Prepare plates with Mueller Hinton Agar (M173/M1084) for rapidly growing aerobic organisms as per Kirby- Bauer Method. The medium in the plates should be sterile and should have a depth of about 4 mm.
2. Inoculate 4-5 similar colonies with a wire, needle or loop to 5 ml Tryptone Soya Broth (M011) and incubate at 35-37°C for 2-8 hours until light to moderate turbidity develops. Compare the inoculum turbidity with that of standard 0.5 McFarland (prepared by mixing 0.5 ml of 1.175% barium chloride and 99.5 ml of 0.36N sulfuric acid). Dilute the inoculum or incubate further as necessary to attain comparative turbidity. Alternatively, the inoculum can be standardized by other appropriate optical method (0.08 - 0.13 OD turbid suspension at 625 nm)
3. Dip a sterile non-toxic cotton swab on a wooden applicator into the standardized inoculum and rotate the soaked swab firmly against the upper inside wall of the tube to express excess fluid. Streak the entire agar surface of the plate with the swab three times, turning the plate at 60° angle between each streaking. Allow the inoculum to dry for 5 - 15 minutes with lid in place.
4. Apply the discs using aseptic technique. When using cartridges, the discs can be applied using the specially designed applicator. When the vials are used, apply the discs using sterile forceps.
5. Deposit the discs with centers at least 24 mm apart. For fastidious organisms and for Penicillins and Cephalosporins, the discs should preferably be deposited with centers 30 mm apart.
6. Incubate immediately at 35 ± 2°C and examine after 16-18 hours or longer, if necessary. For fastidious organisms incubate at appropriate temperature and time.
7. Measure the zones showing complete inhibition and record the diameters of the zones to the nearest millimeter using a calibrated instrument like zone scales (PW096/PW297)

Principle:

Antimicrobial susceptibility testing (AST) of bacterial and fungal isolates is a common and important technique in most clinical laboratories. The results of these tests are used for selection of the most appropriate antimicrobial agent(s) for treatment against the infectious organisms. Till the 1950s, laboratories were lacking in the methodologies and equipments for the accurate determination of in vitro responses of organisms to antimicrobial agents. Bauer et al (1) began the development of standardized methods for antimicrobial susceptibility testing, using disc diffusion system. However the susceptibility results may not always correlate with the patient's response to therapy. The response of an infected patient to antimicrobial agent(s) is a complex interrelationship of host responses, drug dynamics and microbial activity. Antimicrobial susceptibility tests are either quantitative or qualitative. Disc diffusion test is a qualitative test method. The National Committee for Clinical Laboratory Standards (NCCLS), now known as Clinical Laboratory Standards Institute (CLSI) has published comprehensive documents regarding the disc diffusion systems. The agar disc diffusion test is the most convenient and widely used method for routine antimicrobial susceptibility testing. In subsequent and current practice, antimicrobial impregnated paper discs are applied onto the agar surface. Based on the Kirby- Bauer Method, standardized reference procedures for the disc systems were published by WHO and FDA and are periodically updated by the CLSI (formerly NCCLS)(2). For any antimicrobial testing, Quality control or clinical testing, the method to be followed is the same as mentioned above.

However few precautions are to be maintained while handling of the Sensitivity discs,

- On receipt the discs are to be immediately stored at the recommended temperature.
- Medium preparation, Inoculum preparation and incubation to be done as specified.

Interpretation:

Use following interpretive criteria for susceptibility categorization*

Antimicrobial agent	Interpretative criteria for	Sensitive	Intermediate	Resistant
		mm or more	mm	mm or less
Cefoxitin (Cephoxitin) 30 mcg	<i>Enterobacterales</i>	16	13-15	14
	<i>For S.aureus & S.lugdunensis</i>	22	-	21
	<i>For Coagulase- negative Staphylococci except S.lugdunensis & S.pseudintermedius</i>	25	-	24
	<i>Neisseria gonorrhoeae</i>	28	24-27	23

Quality Control:

Appearance: Filter paper discs of 6mm diameter with printed "CX 30" on centre of each side of the disc.

Cultural response: Average diameter of zone of inhibition observed on Mueller Hinton Agar (M173) after 18 hours incubation at 35-37°C for standard cultures.

Organisms (ATCC)	Std. zone of diameter (mm)*
<i>E.coli</i> (25922)	23-29
<i>S.aureus</i> (25923)	23-29

* = Interpretive criteria & QC ranges as per CLSI & EUCAST standards.

Storage and Shelf-life:

On receipt discs should always be stored at -20°C under dry conditions, along with the dessicator provided in individual pack. Use before expiry date on the label.

References:

1. Bauer, Kirby, Sherris and Turck, 1966, Am. J. Clin. Path., 45: 493
2. Performance standards of Antimicrobial Disc Susceptibility Tests, M100S, 32nd Ed., CLSI Vol. 42 No.2, Feb-2022.
3. EUCAST, Breakpoint tables for interpretation of MIC's & zone diameters, version 12.0, valid from 01.01.2022.

Note :

Use following media to carry out susceptibility test

For rapidly growing aerobic organisms : Mueller Hinton Agar (M173/M1084)

For *Haemophilus* spps : Haemophilus Test Agar (M1259 + FD117)

For *S.pneumoniae* : Muller Hinton Agar supplemented with 5% Sheep Blood

For *Neisseria* spps : G.C.Agar +1% defined growth supplement (M434 + FD025)

*** Not for Medicinal Use**

In vitro diagnostic medical device



CE Marking



Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
C-40, Road No.21Y, MIDC, Wagle
Industrial Area, Thane (W)
-400604, MS, India



CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Revision : 03 / 2022

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.

Ceftazidime**CAZ 30 mcg****SD062**

Ceftazidime CAZ 30 mcg discs are used for antimicrobial susceptibility testing of bacterial cultures as per Bauer-Kirby Method

Composition

*Ingredients	Concentration
Ceftazidime	30 mcg/disc

Susceptibility Test Procedure:

1. Prepare plates with Mueller Hinton Agar (M173/M1084) for rapidly growing aerobic organisms as per Bauer-Kirby Method. The medium in the plates should be sterile and should have a depth of about 4 mm.
2. Inoculate 4-5 similar colonies with a wire, needle or loop to 5 ml Tryptone Soya Broth (M011) and incubate at 35-37°C for 2-8 hours until light to moderate turbidity develops. Compare the inoculum turbidity with that of standard 0.5 McFarland (prepared by mixing 0.5 ml of 1.175% barium chloride and 99.5 ml of 0.36N sulfuric acid). Dilute the inoculum or incubate further as necessary to attain comparative turbidity. Alternatively, the inoculum can be standardized by other appropriate optical method (0.08 - 0.13 OD turbid suspension at 625 nm)
3. Dip a sterile non-toxic cotton swab on a wooden applicator into the standardized inoculum and rotate the soaked swab firmly against the upper inside wall of the tube to express excess fluid. Streak the entire agar surface of the plate with the swab three times, turning the plate at 60° angle between each streaking. Allow the inoculum to dry for 5 - 15 minutes with lid in place.
4. Apply the discs using aseptic technique. When using cartridges, the discs can be applied using the specially designed applicator. When the vials are used, apply the discs using sterile forceps.
5. Deposit the discs with centers at least 24 mm apart. For fastidious organisms and for Penicillins and Cephalosporins, the discs should preferably be deposited with centers 30 mm apart.
6. Incubate immediately at 35 ± 2°C and examine after 16-18 hours or longer, if necessary. For fastidious organisms incubate at appropriate temperature and time.
7. Measure the zones showing complete inhibition and record the diameters of the zones to the nearest millimeter using a calibrated instrument like zone scales (PW096/PW297)

Principle:

Antimicrobial susceptibility testing (AST) of bacterial and fungal isolates is a common and important technique in most clinical laboratories. The results of these tests are used for selection of the most appropriate antimicrobial agent(s) for treatment against the infectious organisms. Till the 1950s, laboratories were lacking in the methodologies and equipments for the accurate determination of in vitro responses of organisms to antimicrobial agents. Bauer et al (1) began the development of standardized methods for antimicrobial susceptibility testing, using disc diffusion system. However the susceptibility results may not always correlate with the patient's response to therapy. The response of an infected patient to antimicrobial agent(s) is a complex interrelationship of host responses, drug dynamics and microbial activity. Antimicrobial susceptibility tests are either quantitative or qualitative. Disc diffusion test is a qualitative test method. The National Committee for Clinical Laboratory Standards (NCCLS), now known as Clinical Laboratory Standards Institute (CLSI) has published comprehensive documents regarding the disc diffusion systems. The agar disc diffusion test is the most convenient and widely used method for routine antimicrobial susceptibility testing. In subsequent and current practice, antimicrobial impregnated paper discs are applied onto the agar surface. Based on the Bauer-Kirby Method, standardized reference procedures for the disc systems were published by WHO and FDA and are periodically updated by the CLSI (formerly NCCLS)(2). For any antimicrobial testing, Quality control or clinical testing, the method to be followed is the same as mentioned above.

However few precautions are to be maintained while handling of the Sensitivity discs,

- On receipt the discs are to be immediately stored at the recommended temperature.
- Medium preparation, Inoculum preparation and incubation to be done as specified.

Interpretation:

Use following interpretive criteria for susceptibility categorization*

Antimicrobial agent	Interpretative criteria for	Sensitive	Intermediate	Resistant
		mm or more	mm	mm or less
Ceftazidime 30 mcg	<i>Enterobacteriaceae, B.cepacia</i>	21	18-20	17
	<i>P.aeruginosa, Acinetobacter & Staphylococcus</i>	18	15-17	14
	<i>Haemophilus influenzae & Haemophilus parainfluenzae</i>	26	-	-
	<i>Neisseria gonorrhoeae</i>	31	-	-

Quality Control:

Appearance: Filter paper discs of 6mm diameter with printed "CAZ 30" on centre of each side of the disc.

Cultural response: Average diameter of zone of inhibition observed on Mueller Hinton Agar (M173) after 18 hours incubation at 35-37°C for standard cultures.

Organisms (ATCC)	Std. zone of diameter (mm)*
<i>E.coli</i> (25922)	25-32
<i>S.aureus</i> (25923)	16-20
<i>P.aeruginosa</i> (27853)	22-29
<i>K.pneumoniae</i> (700603)	10-18

* = Interpretive criteria & QC ranges as per CLSI standards.

Storage and Shelf-life:

On receipt discs should always be stored at -20°C under dry conditions, along with the dessicator provided in individual pack. Use before expiry date on the label.

References:

1. Bauer, Kirby, Sherris and Turck, 1966, Am. J. Clin. Path., 45: 493
2. Performance standards of Antimicrobial Disc Susceptibility Tests, M100S, 32nd Ed., CLSI Vol. 42 No.2, Feb-2022.
3. EUCAST, Breakpoint tables for interpretation of MIC's & zone diameters, version 12.0, valid from 01.01.2022.

Note :

Use following media to carry out susceptibility test

For rapidly growing aerobic organisms : Mueller Hinton Agar (M173/M1084)

For *Haemophilus* spps : Haemophilus Test Agar (M1259 + FD117)

For *S.pneumoniae* : Muller Hinton Agar supplemented with 5% Sheep Blood

For *Neisseria* spps : G.C.Agar +1% defined growth supplement (M434 + FD025)

*** Not for Medicinal Use**

IVD

In vitro diagnostic medical
device

CE Marking

On receipt store at 

Storage temperature

Do not use if package is
damagedHiMedia Laboratories Pvt. Limited,
C-40, Road No.21Y, MIDC, Wagle
Industrial Area, Thane (W)
-400604, MS, India

EC REP

CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu**Disclaimer :**

Revision : 03 / 2022

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.

Amoxyclav AMC 30 mcg (Amoxycillin/Clavulanic acid) (20/10mcg)

SD063

Amoxyclav (Amoxycillin/Clavulanic acid) AMC 30mcg (20/10mcg) discs are used for antimicrobial susceptibility testing of bacterial cultures as per Bauer-Kirby Method

Composition

*Ingredients	Concentration
Amoxyclav (Amoxycillin/Clavulanic acid)	30mcg/disc

Susceptibility Test Procedure:

1. Prepare plates with Mueller Hinton Agar (M173/M1084) for rapidly growing aerobic organisms as per Bauer-Kirby Method. The medium in the plates should be sterile and should have a depth of about 4 mm.
2. Inoculate 4-5 similar colonies with a wire, needle or loop to 5 ml Tryptone Soya Broth (M011) and incubate at 35-37°C for 2-8 hours until light to moderate turbidity develops. Compare the inoculum turbidity with that of standard 0.5 McFarland (prepared by mixing 0.5 ml of 1.175% barium chloride and 99.5 ml of 0.36N sulfuric acid). Dilute the inoculum or incubate further as necessary to attain comparative turbidity. Alternatively, the inoculum can be standardized by other appropriate optical method (0.08 - 0.13 OD turbid suspension at 625 nm)
3. Dip a sterile non-toxic cotton swab on a wooden applicator into the standardized inoculum and rotate the soaked swab firmly against the upper inside wall of the tube to express excess fluid. Streak the entire agar surface of the plate with the swab three times, turning the plate at 60° angle between each streaking. Allow the inoculum to dry for 5 - 15 minutes with lid in place.
4. Apply the discs using aseptic technique. When using cartridges, the discs can be applied using the specially designed applicator. When the vials are used, apply the discs using sterile forceps.
5. Deposit the discs with centers at least 24 mm apart. For fastidious organisms and for Penicillins and Cephalosporins, the discs should preferably be deposited with centers 30 mm apart.
6. Incubate immediately at 35 ± 2°C and examine after 16-18 hours or longer, if necessary. For fastidious organisms incubate at appropriate temperature and time.
7. Measure the zones showing complete inhibition and record the diameters of the zones to the nearest millimeter using a calibrated instrument like zone scales (PW096/PW297)

Principle:

Antimicrobial susceptibility testing (AST) of bacterial and fungal isolates is a common and important technique in most clinical laboratories. The results of these tests are used for selection of the most appropriate antimicrobial agent(s) for treatment against the infectious organisms. Till the 1950s, laboratories were lacking in the methodologies and equipments for the accurate determination of in vitro responses of organisms to antimicrobial agents. Bauer et al (1) began the development of standardized methods for antimicrobial susceptibility testing, using disc diffusion system. However the susceptibility results may not always correlate with the patient's response to therapy. The response of an infected patient to antimicrobial agent(s) is a complex interrelationship of host responses, drug dynamics and microbial activity. Antimicrobial susceptibility tests are either quantitative or qualitative. Disc diffusion test is a qualitative test method. The National Committee for Clinical Laboratory Standards (NCCLS), now known as Clinical Laboratory Standards Institute (CLSI) has published comprehensive documents regarding the disc diffusion systems. The agar disc diffusion test is the most convenient and widely used method for routine antimicrobial susceptibility testing. In subsequent and current practice, antimicrobial impregnated paper discs are applied onto the agar surface. Based on the Bauer-Kirby Method, standardized reference procedures for the disc systems were published by WHO and FDA and are periodically updated by the CLSI (formerly NCCLS)(2). For any antimicrobial testing, Quality control or clinical testing, the method to be followed is the same as mentioned above.

However few precautions are to be maintained while handling of the Sensitivity discs,

- On receipt the discs are to be immediately stored at the recommended temperature.
- Medium preparation, Inoculum preparation and incubation to be done as specified.

Interpretation:

Use following interpretive criteria for susceptibility categorization*

Antimicrobial agent	Interpretative criteria for	Sensitive mm or more	Intermediate mm	Resistant mm or less
Amoxyclav (Amoxycillin/Clavulanic acid) 30mcg (20/10mcg)	<i>Enterobacteriaceae</i>	18	14-17	13
	<i>Haemophilus influenzae</i> & <i>Haemophilus parainfluenzae</i>	20	-	19

Quality Control:

Appearance: Filter paper discs of 6mm diameter with printed "AMC 30" on centre of each side of the disc.

Cultural response: Average diameter of zone of inhibition observed on Mueller Hinton Agar (M173) after 18 hours incubation at 35-37°C for standard cultures.

Organisms (ATCC)	Std. zone of diameter (mm)*
<i>E. coli</i> (25922)	18-24
<i>S. aureus</i> (25923)	28-36
<i>E. coli</i> (35218)	17-22

* = Interpretive criteria & QC ranges as per CLSI standards.

Storage and Shelf-life:

On receipt discs should always be stored at -20°C under dry conditions, along with the dessicator provided in individual pack. Use before expiry date on the label.

References:

1. Bauer, Kirby, Sherris and Turck, 1966, Am. J. Clin. Path., 45: 493
2. Performance standards of Antimicrobial Disc Susceptibility Tests, M100S, 32nd Ed., CLSI Vol. 42 No.2, Feb-2022.
3. EUCAST, Breakpoint tables for interpretation of MIC's & zone diameters, version 12.0, valid from 01.01.2022.

Note :

Use following media to carry out susceptibility test

For rapidly growing aerobic organisms : Mueller Hinton Agar (M173/M1084)

For *Haemophilus* spps : Haemophilus Test Agar (M1259 + FD117)

For *S. pneumoniae* : Muller Hinton Agar supplemented with 5% Sheep Blood

For *Neisseria* spps : G.C. Agar +1% defined growth supplement (M434 + FD025)

*** Not for Medicinal Use****IVD**

In vitro diagnostic medical device



CE Marking

On receipt store at 

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
C-40, Road No.21Y, MIDC, Wagle
Industrial Area, Thane (W)
-400604, MS, India

EC REP

CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

Revision : 03 / 2022

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.

Ceftriaxone**CTR 30 mcg****SD065**

Ceftriaxone CTR 30 mcg discs are used for antimicrobial susceptibility testing of bacterial cultures as per Bauer-Kirby Method

Composition

*Ingredients	Concentration
Ceftriaxone	30 mcg/disc

Susceptibility Test Procedure:

1. Prepare plates with Mueller Hinton Agar (M173/M1084) for rapidly growing aerobic organisms as per Bauer-Kirby Method. The medium in the plates should be sterile and should have a depth of about 4 mm.
2. Inoculate 4-5 similar colonies with a wire, needle or loop to 5 ml Tryptone Soya Broth (M011) and incubate at 35-37°C for 2-8 hours until light to moderate turbidity develops. Compare the inoculum turbidity with that of standard 0.5 McFarland (prepared by mixing 0.5 ml of 1.175% barium chloride and 99.5 ml of 0.36N sulfuric acid). Dilute the inoculum or incubate further as necessary to attain comparative turbidity. Alternatively, the inoculum can be standardized by other appropriate optical method (0.08 - 0.13 OD turbid suspension at 625 nm)
3. Dip a sterile non-toxic cotton swab on a wooden applicator into the standardized inoculum and rotate the soaked swab firmly against the upper inside wall of the tube to express excess fluid. Streak the entire agar surface of the plate with the swab three times, turning the plate at 60° angle between each streaking. Allow the inoculum to dry for 5 - 15 minutes with lid in place.
4. Apply the discs using aseptic technique. When using cartridges, the discs can be applied using the specially designed applicator. When the vials are used, apply the discs using sterile forceps.
5. Deposit the discs with centers at least 24 mm apart. For fastidious organisms and for Penicillins and Cephalosporins, the discs should preferably be deposited with centers 30 mm apart.
6. Incubate immediately at $35 \pm 2^\circ\text{C}$ and examine after 16-18 hours or longer, if necessary. For fastidious organisms incubate at appropriate temperature and time.
7. Measure the zones showing complete inhibition and record the diameters of the zones to the nearest millimeter using a calibrated instrument like zone scales (PW096/PW297)

Principle:

Antimicrobial susceptibility testing (AST) of bacterial and fungal isolates is a common and important technique in most clinical laboratories. The results of these tests are used for selection of the most appropriate antimicrobial agent(s) for treatment against the infectious organisms. Till the 1950s, laboratories were lacking in the methodologies and equipments for the accurate determination of in vitro responses of organisms to antimicrobial agents. Bauer et al (1) began the development of standardized methods for antimicrobial susceptibility testing, using disc diffusion system. However the susceptibility results may not always correlate with the patient's response to therapy. The response of an infected patient to antimicrobial agent(s) is a complex interrelationship of host responses, drug dynamics and microbial activity. Antimicrobial susceptibility tests are either quantitative or qualitative. Disc diffusion test is a qualitative test method. The National Committee for Clinical Laboratory Standards (NCCLS), now known as Clinical Laboratory Standards Institute (CLSI) has published comprehensive documents regarding the disc diffusion systems. The agar disc diffusion test is the most convenient and widely used method for routine antimicrobial susceptibility testing. In subsequent and current practice, antimicrobial impregnated paper discs are applied onto the agar surface. Based on the Bauer-Kirby Method, standardized reference procedures for the disc systems were published by WHO and FDA and are periodically updated by the CLSI (formerly NCCLS)(2). For any antimicrobial testing, Quality control or clinical testing, the method to be followed is the same as mentioned above.

However few precautions are to be maintained while handling of the Sensitivity discs,

- On receipt the discs are to be immediately stored at the recommended temperature.
- Medium preparation, Inoculum preparation and incubation to be done as specified.

Interpretation:

Use following interpretive criteria for susceptibility categorization*

Antimicrobial agent	Interpretative criteria for	Sensitive	Intermediate	Resistant
		mm or more	mm	mm or less
Ceftriaxone 30 mcg	<i>Enterobacteriaceae</i>	23	20-22	19
	<i>P.aeruginosa, Acinetobacter & Staphylococcus</i>	21	14-20	13
	<i>Haemophilus influenzae & Haemophilus parainfluenzae</i>	26	-	-
	<i>Neisseria meningitidis</i>	34	-	-
	<i>Neisseria gonorrhoeae</i>	35	-	-
	<i>Streptococcus spp. Viridians group</i>	27	25-26	24
	<i>Streptococcus spp. beta haemolytic group</i>	24	-	-

Quality Control:

Appearance: Filter paper discs of 6mm diameter with printed "CTR 30" on centre of each side of the disc.

Cultural response: Average diameter of zone of inhibition observed on Mueller Hinton Agar (M173) after 18 hours incubation at 35-37°C for standard cultures.

Organisms (ATCC)	Std. zone of diameter (mm)*
<i>E.coli</i> (25922)	29-35
<i>S.aureus</i> (25923)	22-28
<i>P.aeruginosa</i> (27853)	17-23
<i>K.pneumoniae</i> (700603)	16-24

* = Interpretive criteria & QC ranges as per CLSI standards.

Storage and Shelf-life:

On receipt discs should always be stored at -20°C under dry conditions, along with the dessicator provided in individual pack. Use before expiry date on the label.

References:

1. Bauer, Kirby, Sherris and Turck, 1966, Am. J. Clin. Path., 45: 493
2. Performance standards of Antimicrobial Disc Susceptibility Tests, M100S, 32nd Ed., CLSI Vol. 42 No.2, Feb-2022.
3. EUCAST, Breakpoint tables for interpretation of MIC's & zone diameters, version 12.0, valid from 01.01.2022.

Note :

Use following media to carry out susceptibility test

For rapidly growing aerobic organisms : Mueller Hinton Agar (M173/M1084)

For *Haemophilus* spp : Haemophilus Test Agar (M1259 + FD117)

For *S.pneumoniae* : Muller Hinton Agar supplemented with 5% Sheep Blood

For *Neisseria* spp : G.C.Agar +1% defined growth supplement (M434 + FD025)

* Not for Medicinal Use



In vitro diagnostic medical device



CE Marking



Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
C-40, Road No.21Y, MIDC, Wagle
Industrial Area, Thane (W)
-400604, MS, India



CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

Revision : 03 / 2022

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.

Pefloxacin PF 5 mcg**SD070**

Pefloxacin PF 5 mcg discs are used for antimicrobial susceptibility testing of bacterial cultures as per Bauer-Kirby Method

Composition

*Ingredients	Concentration
Pefloxacin	5 mcg/disc

Susceptibility Test Procedure:

1. Prepare plates with Mueller Hinton Agar (M173/M1084) for rapidly growing aerobic organisms as per Bauer-Kirby Method. The medium in the plates should be sterile and should have a depth of about 4 mm.
2. Inoculate 4-5 similar colonies with a wire, needle or loop to 5 ml Tryptone Soya Broth (M011) and incubate at 35-37°C for 2-8 hours until light to moderate turbidity develops. Compare the inoculum turbidity with that of standard 0.5 McFarland (prepared by mixing 0.5 ml of 1.175% barium chloride and 99.5 ml of 0.36N sulfuric acid). Dilute the inoculum or incubate further as necessary to attain comparative turbidity. Alternatively, the inoculum can be standardized by other appropriate optical method (0.08 - 0.13 OD turbid suspension at 625 nm)
3. Dip a sterile non-toxic cotton swab on a wooden applicator into the standardized inoculum and rotate the soaked swab firmly against the upper inside wall of the tube to express excess fluid. Streak the entire agar surface of the plate with the swab three times, turning the plate at 60° angle between each streaking. Allow the inoculum to dry for 5 - 15 minutes with lid in place.
4. Apply the discs using aseptic technique. When using cartridges, the discs can be applied using the specially designed applicator. When the vials are used, apply the discs using sterile forceps.
5. Deposit the discs with centers at least 24 mm apart. For fastidious organisms and for Penicillins and Cephalosporins, the discs should preferably be deposited with centers 30 mm apart.
6. Incubate immediately at 35 ± 2°C and examine after 16-18 hours or longer, if necessary. For fastidious organisms incubate at appropriate temperature and time.
7. Measure the zones showing complete inhibition and record the diameters of the zones to the nearest millimeter using a calibrated instrument like zone scales (PW096/PW297)

Principle:

Antimicrobial susceptibility testing (AST) of bacterial and fungal isolates is a common and important technique in most clinical laboratories. The results of these tests are used for selection of the most appropriate antimicrobial agent(s) for treatment against the infectious organisms. Till the 1950s, laboratories were lacking in the methodologies and equipments for the accurate determination of in vitro responses of organisms to antimicrobial agents. Bauer et al (1) began the development of standardized methods for antimicrobial susceptibility testing, using disc diffusion system. However the susceptibility results may not always correlate with the patient's response to therapy. The response of an infected patient to antimicrobial agent(s) is a complex interrelationship of host responses, drug dynamics and microbial activity. Antimicrobial susceptibility tests are either quantitative or qualitative. Disc diffusion test is a qualitative test method. The National Committee for Clinical Laboratory Standards (NCCLS), now known as Clinical Laboratory Standards Institute (CLSI) has published comprehensive documents regarding the disc diffusion systems. The agar disc diffusion test is the most convenient and widely used method for routine antimicrobial susceptibility testing. In subsequent and current practice, antimicrobial impregnated paper discs are applied onto the agar surface. Based on the Bauer-Kirby Method, standardized reference procedures for the disc systems were published by WHO and FDA and are periodically updated by the CLSI (formerly NCCLS)(2). For any antimicrobial testing, Quality control or clinical testing, the method to be followed is the same as mentioned above.

However few precautions are to be maintained while handling of the Sensitivity discs,

- On receipt the discs are to be immediately stored at the recommended temperature.
- Medium preparation, Inoculum preparation and incubation to be done as specified.

Quality Control:

Appearance: Filter paper discs of 6mm diameter with printed "PF 5" on centre of each side of the disc.

Cultural response: Average diameter of zone of inhibition observed on Mueller Hinton Agar (M173) after 18 hours incubation at 35-37°C for standard cultures.

Organisms (ATCC)	Std. zone of diameter (mm)
<i>E. coli</i> (25922)	29-33
<i>S.aureus</i> (25923)	24-28
<i>P.aeruginosa</i> (27853)	17-21

Storage and Shelf-life:

Discs should always be stored at -20°C to +8°C under dry conditions, along with the dessicator provided in individual pack. Use before expiry date on the label.

References:

1. Bauer, Kirby, Sherris and Turck, 1966, Am. J. Clin. Path., 45: 493
2. Performance standards of Antimicrobial Disc Susceptibility Tests, M100S, 32nd Ed., CLSI Vol. 42 No.2, Feb-2022.
3. EUCAST, Breakpoint tables for interpretation of MIC's & zone diameters, version 12.0, valid from 01.01.2022.

Note :

Use following media to carry out susceptibility test


For rapidly growing aerobic organisms : Mueller Hinton Agar (M173/M1084)

For *Haemophilus* spps : Haemophilus Test Agar (M1259 + FD117)


For *S.pneumoniae* : Muller Hinton Agar supplemented with 5% Sheep Blood

For Neisseria spps : G.C.Agar +1% defined growth supplement (M434 + FD025)


* Not for Medicinal Use



In vitro diagnostic medical device




CE Marking




On receipt store at -20°C

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
C-40, Road No.21Y, MIDC, Wagle
Industrial Area, Thane (W)
-400604, MS, India

EC

REP

CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Revision : 03 / 2022

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.