PROBE CLEANSER (50mLx1)

【PRODUCT NAME】
Probe Cleanser
【INTENDED USE】
Probe Cleanser applies to Mindray BC-20, BC-21, BC-10, BC-11, BC-6800, BC-6600, BC-5000, BC-5150, BC-5390 CRP, BC-5180 CRP, BC-30s, BC-31s, BC-30e, BC-20s, BC-21s, BC-5120, BC-5130, BC-5310 CRP, BC-5100 CRP, BC-5140, BC-30, BC-31, BC-6000, BC-6000Plus, BC-6100, BC-6100Plus, BC-6200, BC-6600Plus, BC-6700Plus, BC-6800Plus, BC-10e , BC-7500[B] CRP , BC-7500[N] CRP , BC-7500[R] CRP , BC-7500[NR] CRP, BC-700[B], BC-700[R], BC-720[R], BC-760[B], BC-760[R], BC-780[R], BC-700[B] CS, BC-700[R] CS, BC-760[B] CS, BC-760[R] CS Auto Hematology Analyzer, BC-5000Vet Veterinary Auto Hematology Analyzer, SC-120 Auto Slide Maker & Stainer and CRP-M100 Specific Protein Analyzer, BP 200n, BP 260n Auto Specific Protein and Erythrocyte Sedimentation Rate Analyzer. It is used to clean the instrument regularly.

【ACTIVE INGREDIENTS】
Surfactant.....≤0.2%
Sodium hypochlorous.....≤12%
Sodium hydroxide.....≤5%

【WARNINGS】
Probe Cleanser is corrosive and irritating. Do not inhale and/or ingest. In case of inhaling or ingestion, seek medical advice immediately. Avoid contact with skin. In case of contact with skin, rinse with plenty of water and seek medical advice immediately. Avoid contact with eyes. In case of contact with eyes, rinse with plenty of water and seek medical advice immediately.

【NOTICE】
Do not use expired products. Dispose of waste product, unused product and contaminated packaging in accordance with governmental requirements. This product must be operated by clinical professionals, physician or laboratorian trained by Mindray or Mindray authorized personnel.

【STORAGE AND STABILITY】
It is important to store in dark place and keep container closed when not in use. The product will be stable for 1 year when stored at 2℃ to 30℃ in a dark place. The working temperature range of the product is consistent with that of its applicable instruments. The open-vial validity is 60 days.

【INTELLECTUAL PROPERTY STATEMENT】
SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this Mindray product and this manual. This manual may refer to information protected by copyright or patents and does not convey any license under the patent rights or copyright of Mindray, or of others. Mindray intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

【COMPANY CONTACT】
Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Address: Mindray Building, Keji 12th Road South, High-tech industrial park, Nanshan, Shenzhen 518057, P.R.China
E-mail Address: service@mindray.com
Tel: +86 755 81888998
Fax: +86 755 26582680
EC-Representative: Shanghai International Holding Corp. GmbH (Europe)
Address: Eiffestraße 80, 20537 Hamburg, Germany
Tel: 0049-40-2513175
Fax: 0049-40-255726

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Üretici: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
AB-Temsilcisi: Shanghai International Holding Corp. GmbH(Europe)