

SOP Number: <b>ITCSOP-000354</b>	Version: <b>10.0; Most-Recent; Effective; CURRENT</b>	Effective Date: <b>13-Jun-2020</b>
Title:	<b>Product Service Maintenance Plan</b>	

GTS Management / ITC T/S Management	<p><b>Preventative Maintenance</b></p> <p>7. Establishes and defines the Preventative Maintenance in the applicable Service Test Procedure for each product based upon R&amp;D input.</p> <ol style="list-style-type: none"> <li>a. The Preventative Maintenance cycle is typically a minimum of once every twelve (12) months, but can vary based upon the system configuration, product use, or environmental conditions.</li> <li>b. Customers are not typically billed for preventative maintenance during the system warranty period.</li> <li>c. The number of preventative maintenance cycles per year, is shown in Table 1 below and the parts to be replaced are defined in the applicable Service Test Procedure.</li> </ol>																																																	
	<p style="text-align: center;"><b>TABLE 1</b></p> <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="text-align: left;">PRODUCT</th> <th style="text-align: center;">PREVENTIVE MAINTENANCE</th> </tr> </thead> <tbody> <tr><td>3000LE™</td><td style="text-align: center;">1</td></tr> <tr><td>ARGOS MOVU</td><td style="text-align: center;">1*</td></tr> <tr><td>Centurion®</td><td style="text-align: center;">1</td></tr> <tr><td>CONSTELLATION®</td><td style="text-align: center;">2</td></tr> <tr><td>INFINITI®</td><td style="text-align: center;">1</td></tr> <tr><td>LAUREATE™</td><td style="text-align: center;">1</td></tr> <tr><td>LenSx®</td><td style="text-align: center;">2</td></tr> <tr><td>Luxor® / QVue® Microscope</td><td style="text-align: center;">1</td></tr> <tr><td>LX3 Microscope</td><td style="text-align: center;">1</td></tr> <tr><td>Ngenuity</td><td style="text-align: center;">1</td></tr> <tr><td>OCUSCAN® RxP</td><td style="text-align: center;">1</td></tr> <tr><td>ALL ORA VerifEye, VerifEye+™ and VLynk</td><td style="text-align: center;">1</td></tr> <tr><td>PurePoint™</td><td style="text-align: center;">1</td></tr> <tr><td>SL1000 Slit Lamp</td><td style="text-align: center;">1</td></tr> <tr><td>WaveLight® Allegretto Wave® Eye-Q (1010)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® Allegretto Wave® (1007/1008/1009)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® Allegro Analyzer® I &amp; II (1070/1071/1073)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® Allegro® OB820 (BioGraph) (1088)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® Allegro Oculyzer® I &amp; II (1074/1077)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® Allegro Topolyzer® (1075/1076)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® Allegro Topolyzer Vario® (1029)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® EX500® (1016)</td><td style="text-align: center;">2</td></tr> <tr><td>WaveLight® FS200® (1025)</td><td style="text-align: center;">2</td></tr> <tr><td>Verion™</td><td style="text-align: center;">1**</td></tr> </tbody> </table> <p>* Per ARGOS Service Manual</p> <p>→ d. Periodic maintenance of the instrument is not required by the user. Maintenance of the machine is required when the machine does not pass its function check (Daily Calibration process). When the function check is not successful (Figure 179), the user is prompted to contact technical service or the distributor.</p> <p><b>NOTE:</b> PM's are optional and can be used for sites with contracts including annual</p> <p>** Per Verion STP, under "Purpose and Scope"</p> <p>→ e. All functional tests "described by this instruction" and "documented by the checklist" includes the requirements for Preventive Maintenance. No separate activities are defined or required to perform a PM. Using this functional test for PM purposes is optional and can be used for sites with contracts including annual PM.</p>	PRODUCT	PREVENTIVE MAINTENANCE	3000LE™	1	ARGOS MOVU	1*	Centurion®	1	CONSTELLATION®	2	INFINITI®	1	LAUREATE™	1	LenSx®	2	Luxor® / QVue® Microscope	1	LX3 Microscope	1	Ngenuity	1	OCUSCAN® RxP	1	ALL ORA VerifEye, VerifEye+™ and VLynk	1	PurePoint™	1	SL1000 Slit Lamp	1	WaveLight® Allegretto Wave® Eye-Q (1010)	2	WaveLight® Allegretto Wave® (1007/1008/1009)	2	WaveLight® Allegro Analyzer® I & II (1070/1071/1073)	2	WaveLight® Allegro® OB820 (BioGraph) (1088)	2	WaveLight® Allegro Oculyzer® I & II (1074/1077)	2	WaveLight® Allegro Topolyzer® (1075/1076)	2	WaveLight® Allegro Topolyzer Vario® (1029)	2	WaveLight® EX500® (1016)	2	WaveLight® FS200® (1025)	2	Verion™
PRODUCT	PREVENTIVE MAINTENANCE																																																	
3000LE™	1																																																	
ARGOS MOVU	1*																																																	
Centurion®	1																																																	
CONSTELLATION®	2																																																	
INFINITI®	1																																																	
LAUREATE™	1																																																	
LenSx®	2																																																	
Luxor® / QVue® Microscope	1																																																	
LX3 Microscope	1																																																	
Ngenuity	1																																																	
OCUSCAN® RxP	1																																																	
ALL ORA VerifEye, VerifEye+™ and VLynk	1																																																	
PurePoint™	1																																																	
SL1000 Slit Lamp	1																																																	
WaveLight® Allegretto Wave® Eye-Q (1010)	2																																																	
WaveLight® Allegretto Wave® (1007/1008/1009)	2																																																	
WaveLight® Allegro Analyzer® I & II (1070/1071/1073)	2																																																	
WaveLight® Allegro® OB820 (BioGraph) (1088)	2																																																	
WaveLight® Allegro Oculyzer® I & II (1074/1077)	2																																																	
WaveLight® Allegro Topolyzer® (1075/1076)	2																																																	
WaveLight® Allegro Topolyzer Vario® (1029)	2																																																	
WaveLight® EX500® (1016)	2																																																	
WaveLight® FS200® (1025)	2																																																	
Verion™	1**																																																	

Status:	Effective	Supersedes:	SOP-0354
---------	-----------	-------------	----------