

# **Bordetella pertussis/toxin**

**IgM – ELISA**

Enzyme immunoassay for the qualitative determination of IgM-class  
antibodies against *Bordetella pertussis*  
in human serum  
Only for in-vitro diagnostic use



---

Product Number: BOPM0030 (96 determinations)

---

# **CONTENTS**

<b>1. INTRODUCTION</b>	<b>3</b>
<b>2. INTENDED USE</b>	<b>3</b>
<b>3. PRINCIPLE OF THE ASSAY</b>	<b>3</b>
<b>4. MATERIALS</b>	<b>4</b>
4.1. REAGENTS SUPPLIED	4
4.2. MATERIALS SUPPLIED	4
4.3. MATERIALS AND EQUIPMENT NEEDED	4
<b>5. STABILITY AND STORAGE</b>	<b>4</b>
<b>6. REAGENT PREPARATION</b>	<b>4</b>
6.1. COATED SNAP-OFF STRIPS	4
6.2. BORDETELLA PERTUSSIS ANTI-IGM CONJUGATE	4
6.3. CONTROLS	4
6.4. IGM SAMPLE DILUENT	4
6.5. WASHING SOLUTION (20XCONC.)	5
6.6. TMB SUBSTRATE SOLUTION	5
6.7. STOP SOLUTION	5
<b>7. SPECIMEN COLLECTION AND PREPARATION</b>	<b>5</b>
7.1. SAMPLE DILUTION	5
<b>8. ASSAY PROCEDURE</b>	<b>5</b>
8.1. TEST PREPARATION	5
8.2. MEASUREMENT	6
<b>9. RESULTS</b>	<b>6</b>
9.1. RUN VALIDATION CRITERIA	6
9.2. CALCULATION OF RESULTS	6
9.3. INTERPRETATION OF RESULTS	6
9.3.1. RESULTS IN NOVATEC UNITS	6
<b>10. SPECIFIC PERFORMANCE CHARACTERISTICS</b>	<b>7</b>
10.1. PRECISION	7
10.2. DIAGNOSTIC SPECIFICITY	7
10.3. DIAGNOSTIC SENSITIVITY	7
<b>11. LIMITATIONS OF THE PROCEDURE</b>	<b>7</b>
<b>12. PRECAUTIONS AND WARNINGS</b>	<b>7</b>
<b>13. LITERATURE</b>	<b>8</b>
<b>14. ORDERING INFORMATION</b>	<b>8</b>

## 1. INTRODUCTION

*Bordetella* species are non-spore-forming encapsulated bipolar, coccoid (pale-staining) Gram-negative bacilli (about 0.3-0.5 µm thick and 1 µm long). The genus consists of the human parasites *B. pertussis* and *B. parapertussis*, and *B. bronchiseptica* which causes enzootic infections in various wild and domestic animal species.

*Bordetella pertussis* produces a single disease syndrome in man known as pertussis or whooping cough. It is a highly contagious childhood disease (appr. 80% of cases occur before the age of 5 years) which is transmitted by respiratory contact and is associated with a high mortality rate (about 1-2% in the first year of life, later on about 1‰).

In the absence of immunization, essentially no one escapes pertussis. Clinical pertussis is followed by natural acquired immunity which is long-lasting but not permanent. The distribution of the disease is worldwide, though clearly modified by immunization and other poorly defined social, economic, and nutritional factors. In most countries an active vaccination is recommended. Usually the immunization preparation is combined with diphtheria and tetanus toxoids.

Species	Disease	Symptoms	Mechanism of infection
<i>Bordetella pertussis</i>	Pertussis; whooping cough	Catearrrhal stage: illness with non-specific symptoms including increasing cough, nasal symptoms, slight fever Paroxysmal stage: severe, paroxysmal coughing	Droplet infection: attachment of the organism to tracheal and bronchial cilia with loss of function and ultimate destruction incubation period: 7-14 days

The presence of bacterium resp. infection may be identified by

- Microscopy: identification on cultures, IF
- Serology: Detection of antibody production by ELISA

Precise diagnosis is needed for effective treatment of patients, for isolation of unvaccinated infants at risk, and for differentiation of pertussis from atypical diseases and chronic infections.

## 2. INTENDED USE

The NovaTec *Bordetella pertussis* IgM-ELISA is intended for the qualitative determination of IgM class antibodies against *Bordetella pertussis* and *Bordetella pertussis* **toxin** in human serum

## 3. PRINCIPLE OF THE ASSAY

The qualitative immunoenzymatic determination of IgM-class antibodies against *Bordetella pertussis* is based on the ELISA (Enzyme-linked Immunosorbent Assay) technique.

Microtiter strip wells are precoated with *Bordetella pertussis*/toxin antigens to bind corresponding antibodies of the specimen. After washing the wells to remove all unbound sample material horseradish peroxidase (HRP) labelled anti-human IgM conjugate is added. This conjugate binds to the captured *Bordetella pertussis*/toxin specific antibodies. The immune complex formed by the bound conjugate is visualized by adding tetramethylbenzidine (TMB) substrate which gives a blue reaction product. The intensity of this product is proportional to the amount of *Bordetella pertussis*/toxin specific IgM antibodies in the specimen. Sulfuric acid is added to stop the reaction. This produces a yellow endpoint color. Absorbance at 450nm is read using an ELISA microwell plate reader.

## 4. MATERIALS

---

### 4.1. Reagents supplied

- **Bordetella pertussis Coated Wells (IgM):** 12 breakapart 8-well snap-off strips coated with Bordetella pertussis antigen; vacuum sealed, in resealable aluminium foil.
- **IgM Sample Diluent \*\*\*:** 1 bottle containing 100 ml of ready to use buffer for sample dilution; pH  $7.2 \pm 0.2$ ; colored green; white cap.
- **Stop Solution:** 1 bottle containing 15 ml. Ready to use sulfuric acid, 0.2 mol/l; red cap.
- **Washing Solution (20x conc.):** 1 bottle each containing 50 ml of a 20-fold concentrated buffer (pH  $7.2 \pm 0.2$ ) for washing the wells; white cap.
- **Bordetella pertussis anti-IgM Conjugate\*\*:** 1 bottle containing 20 ml of peroxidase labelled rabbit antibody to human IgM; colored red, Ready to use; black cap.
- **TMB Substrate Solution:** 1 bottle containing 15 ml 3,3',5,5'-tetramethylbenzidine (TMB); ready to use; yellow cap.
- **Bordetella pertussis IgM Positive Control\*\*\*:** 1 bottle containing 2 ml; colored yellow; ready to use; red cap.
- **Bordetella pertussis IgM Negative Control\*\*\*:** 1 bottle containing 2 ml; colored yellow; ready to use; blue cap.

\* contains 0.01 % Thimerosal after dilution

\*\* contains 0.2 % Bronidox L

\*\*\* contains 0.1 % Kathon

### 4.2. Materials supplied

- 1 Strip holder
- 2 Cover foils
- 1 Test protocol
- 1 distribution and identification plan

### 4.3. Materials and Equipment needed

- ELISA microwell plate reader, equipped for the measurement of absorbance at 450/620nm
- Incubator 37°C
- Manual or automatic equipment for rinsing wells
- Pipettes to deliver volumes between 10 and 1000 µl
- Vortex tube mixer
- Deionised or (freshly) distilled water
- Disposable tubes
- Timer

## 5. STABILITY AND STORAGE

---

The reagents are stable up to the expiry date stated on the label when stored at 2...8 °C.

## 6. REAGENT PREPARATION

---

*It is very important to bring all reagents, samples and controls to room temperature (20...25°C) before starting the test run!*

### 6.1. Coated snap-off strips

The ready to use breakapart snap-off strips are coated with Bordetella pertussis antigen. Store at 2...8°C. The strips are vacuum sealed. *Immediately after removal of strips, the remaining strips should be resealed in the aluminium foil along with the dessiccant supplied and stored at 2...8 °C; stability until expiry date.*

### 6.2. Bordetella pertussis anti-IgM Conjugate

The bottle contains 20ml of a solution with anti-human-IgM horseradish peroxidase, buffer, stabilizers, preservatives and an inert red dye. The solution is ready to use. Store at 2...8°C. *After first opening stability expiry date when stored at 2...8°C.*

### 6.3. Controls

The bottles labelled with Positive and Negative Control contain a ready to use control solution. It contains 0.1% Kathon and has to be stored at 2...8°C. *After first opening stability expiry date when stored at 2...8°C.*

### 6.4. IgM Sample Diluent

The bottle contains 100ml phosphate buffer, anti human-IgG, stabilizers, preservatives and an inert green dye. It is used for the dilution of the patient specimen. The solution contains antihuman IgG class antibodies to eliminate competitive inhibition from specific IgG class antibody to remove rheumatoid factor. This ready to use solution has to be stored at 2...8°C. *After first opening stability expiry date when stored at 2...8°C.*

## 6.5. Washing Solution (20xconc.)

The bottle contains 50ml of a concentrated buffer, detergents, stabilizers and preservatives. Dilute washing solution 1+19; e.g. 10 ml washing solution + 190 ml fresh and germ free redistilled water. The diluted buffer will keep for at least four weeks if stored at 2...8°C. *Crystals in the solution disappear by warming up to 37 °C in a water bath.*

## 6.6. TMB Substrate Solution

The bottle contains 15ml of a tetramethylbenzidine/hydrogen peroxide system. The reagent is ready to use and has to be stored at 2...8°C, away from the light. *The solution should be colourless or have a slight blue tinge. If the substrate turns into blue, it may have become contaminated and should be discharged. After first opening stability expiry date when stored at 2...8°C.*

## 6.7. Stop Solution

The bottle contains 15ml 0.2 M sulphuric acid solution (R 36/38, S 26). This ready to use solution has to be stored at 2...8°C. *After first opening stability until expiry date..*

# 7. SPECIMEN COLLECTION AND PREPARATION

---

Either human serum or plasma may be used. The anticoagulants sodium citrate and EDTA have been tested and may be used with this assay. If the assay is performed within 24 hours after sample collection, the specimen should be kept at 2...8°C; otherwise they should be aliquoted and stored deep-frozen (-20°C). If samples are stored frozen, mix thawed samples well before testing. *Avoid repeated freezing and thawing.*

## 7.1. Sample Dilution

Before assaying, all samples should be diluted 1+100 with IgM Sample Diluent. Dispense 10µl sample and 1 ml IgM Sample Diluent into tubes to obtain a 1+100 dilution and thoroughly mix with a Vortex. *Positive and negative controls are ready to use and must not be diluted.*

# 8. ASSAY PROCEDURE

---

## 8.1. Test Preparation

Please read the test protocol carefully **before** performing the assay. Result reliability depends on strict adherence to the test protocol as described. Prior to commencing the assay, the distribution and identification plan for all specimens and controls should be carefully established on the result sheet supplied in the kit. Select the required number of microtiter strips or wells and insert them into the holder.

Please allocate at least:

1 well	(e.g. A1)	for the substrate blank,
2 wells	(e.g. B1+C1)	for the negative control and
1 well	(e.g. D1)	for the positive control.

*It is left to the user to determine controls and patient samples in duplicate, if necessary.*

Perform all assay steps in the order given and without any appreciable delays between the steps.

A clean, disposable tip should be used for dispensing each control and sample.

Adjust the incubator to 37° ± 1°C.

1. Dispense 100µl controls and diluted samples into their respective wells. Leave well A1 for substrate blank.
2. Cover wells with the foil supplied in the kit.
3. **Incubate for 1 hour ± 5 min at 37±1°C.**
4. When incubation has been completed, remove the foil, aspirate the content of the wells and wash each well three times with 300µl of washing solution. Avoid overflows from the reaction wells. The soak time between each wash cycle should be >5sec. At the end carefully remove remaining fluid by tapping strips on tissue paper prior to the next step!  
*Note: Washing is critical! Insufficient washing results in poor precision and falsely elevated absorbance values.*
5. Dispense 100µl Bordetella pertussis anti-IgM Conjugate into all wells except for the blank well (e.g. A1). Cover with foil.
6. **Incubate for 30 min at room temperature. Do not expose to direct sunlight.**
7. Repeat step 4.

8. Dispense 100µl TMB Substrate Solution into all wells
9. **Incubate for exactly 15 min at room temperature in the dark.**
10. Dispense 100µl Stop Solution into all wells in the same order and at the same rate as for the TMB Substrate Solution.  
*Any blue color developed during the incubation turns into yellow.*

*Note: Highly positive patient samples can cause dark precipitates of the chromogen! These precipitates have an influence when reading the optical density. Predilution of the sample with physiological sodium chloride solution, for example 1+1, is recommended. Then dilute the sample 1+100 with dilution buffer and multiply the results in NTU by 2.*

11. Measure the absorbance of the specimen at 450/620nm within 30 min after addition of the Stop Solution.

## 8.2. Measurement

Adjust the ELISA Microwell Plate Reader **to zero** using the **substrate blank in well A1**.

*If - due to technical reasons - the ELISA reader cannot be adjusted to zero using the substrate blank in well A1, subtract the absorbance value of well A1 from all other absorbance values measured in order to obtain reliable results!*

**Measure the absorbance** of all wells at **450 nm** and record the absorbance values for each control and patient sample in the distribution and identification plan.

*Dual wavelength reading using 620 nm as reference wavelength is recommended.*

Where applicable calculate the **mean absorbance values** of all duplicates.

## 9. RESULTS

---

### 9.1. Run Validation Criteria

In order for an assay to be considered valid, the following criteria must be met:

- **Substrate blank** in A1: Absorbance value **lower than 0.100**.
- **Negative control** in B1 and C1: Absorbance value **lower than 0.300**.
- **Positive control** in D1: Absorbance value equal to or greater than the cut-off value.

### 9.2. Calculation of Results

The cut-off is calculated by addition of 0.25 absorbance units to the measured absorption of the mean value of the two negative control determinations.

*Example: 0.12 OD neg. control + 0.14 OD neg. control = 0.26 ÷ 2 = 0.13*

*Cut-off = absorbance mean value of the negative control + 0.25*

*Cut-off = 0.13 + 0.25 = 0.38*

### 9.3. Interpretation of Results

Samples are considered **POSITIVE** if the absorbance value is higher than 10% over the cut-off.

Samples with an absorbance value of 10% above or below the cut-off should not be considered as clearly positive or negative

→ **grey zone**

It is recommended to repeat the test again 2 - 4 weeks later with a fresh sample. If results in the second test are again in the grey zone the sample has to be considered **NEGATIVE**.

Samples are considered **NEGATIVE** if the absorbance value is lower than 10% below the cut-off.

#### 9.3.1. Results in NovaTec Units

$$\frac{\text{Patient (mean) absorbance value} \times 10}{\text{Cut-off}} = [\text{NovaTec-Units} = \text{NTU}]$$

*Example:  $\frac{1.216 \times 10}{0.38} = 32 \text{ NTU (NovaTec Units)}$*

Cut-off :	10	NTU
Grey zone:	9-11	NTU
Negative:	<9	NTU
Positive:	>11	NTU

## 10. SPECIFIC PERFORMANCE CHARACTERISTICS

---

### 10.1. Precision

<u>Interassay</u>	<u>n</u>	<u>Mean</u>	<u>Cv (%)</u>
Pos. Serum	13	0.97	7.6
<u>Intraassay</u>	<u>n</u>	<u>Mean</u>	<u>Cv (%)</u>
Pos. Serum	8	0.95	3.5

### 10.2. Diagnostic Specificity

The diagnostic specificity is defined as the probability of the assay of scoring negative in the absence of the specific analyte.

### 10.3. Diagnostic Sensitivity

The diagnostic sensitivity is defined as the probability of the assay of scoring positive in the presence of the specific analyte.

## 11. LIMITATIONS OF THE PROCEDURE

---

Bacterial contamination or repeated freeze-thaw cycles of the specimen may affect the absorbance values. Diagnosis of an infectious disease should not be established on the basis of a single test result. A precise diagnosis should take into consideration clinical history, symptomatology as well as serological data.

In immunosuppressed patients and newborns serological data only have restricted value.

## 12. PRECAUTIONS AND WARNINGS

---

- Only for in-vitro diagnostic use.
- All components of human origin used for the production of these reagents have been tested for anti-HIV antibodies, anti-HCV antibodies and HBsAg and have been found to be non-reactive. Nevertheless, all materials should still be regarded and handled as potentially infectious.
- Do not interchange reagents or strips of different production lots.
- No reagents of other manufacturers should be used along with reagents of this test kit.
- Do not use reagents after expiry date stated on the label.
- Use only clean pipette tips, dispensers, and lab ware.
- Do not interchange screw caps of reagent vials to avoid cross-contamination.
- Close reagent vials tightly immediately after use to avoid evaporation and microbial contamination.
- After first opening and subsequent storage check conjugate and control vials for microbial contamination prior to further use.
- To avoid cross-contamination and falsely elevated results pipette patient samples and dispense conjugate without splashing accurately to the bottom of wells.

**WARNING:** Thimerosal is toxic! Do not swallow. Avoid contact with skin and mucous membranes!

**WARNING:** In the used concentration Bronidox L has hardly any toxicological risk upon contact with skin and mucous membranes!

**WARNING:** Sulfuric acid irritates eyes and skin. Keep out of the reach of children. Upon contact with the eyes, rinse thoroughly with water and consult a doctor!

### 13. LITERATURE

---

Wong,K.H.,and S.K.Skelton.1989.Preparation of filamentous hemagglutinin from *Bordetella pertussis* and assay for serum antibodies to filamentous hemagglutinin and pertussis toxin for clinical and public health lab.J.Clin.Microbiol.27:2805-2810

Zackrisson,G.,I.,Krantz,T.Legergard,P.Larsson,R.Sekura,N.Sigurs,J.Taranger,and B.Trollfors.1988.Humoral antibody response to pertussis toxin in patients with clinical pertussis

Zachrisson,G.,I.Krantz,T.Lagergard,P.Larsson,R.Sekura,NSigurs,S.Taranger,and B.Trollfors.1988.Humoral antibody response to pertussis toxin in patient with clinical pertussismeasured by an enzyme-linked immunosorbent assay.Eur.J.Clin.Microbiol.7:149-154

Granstrom,M.,G.Granstrom,A.Lindtors 1982. Serologic diagnosis of whooping cough by an enzyme-linked immunorbent assay using fimbrial hemagglutinin as antigen.J.Infect.Dis.146:741-745

### 14. ORDERING INFORMATION

---

Prod. No.: BOPM0030      *Bordetella pertussis* IgM-ELISA (96 Determinations)





# SCHEME OF THE ASSAY

Bordetella pertussis IgM-ELISA

## Assay preparation

Prepare reagents and samples as described.  
Establish the distribution and identification plan for all specimens and controls on the result sheet supplied in the kit.  
Select the required number of microtiter strips or wells and insert them into the holder.

## Assay procedure

	Substrate blank (e.g. A1)	Negative control	Positive control	Sample (diluted 1+100)
Negative control	-	100µl	-	-
Positive control	-	-	100µl	-
Sample (diluted 1+100)	-	-	-	100µl
Cover wells with foil supplied in the kit <b>Incubate for 1 h at 37°C</b> Wash each well three times with 300µl of washing solution				
Conjugate	-	100µl	100µl	100µl
Cover wells with foil supplied in the kit <b>Incubate for 30 min at room temperature</b> Wash each well three times with 300µl of washing solution				
TMB Substrate	100µl	100µl	100µl	100µl
<b>Incubate for 15 min at room temperature in the dark</b>				
Stop Solution	100µl	100µl	100µl	100µl
Photometric measurement at 450 nm (reference wavelength: 620 nm)				

## NovaTec Immundiagnostica GmbH

### Technologie & Waldpark

Waldstr. 23 A6  
D-63128 Dietzenbach, Germany

Tel.: +49 (0) 6074-48760 Fax: +49 (0) 6074-487629

Email : info@NovaTec-ID.com

Internet: www.NovaTec-ID.com

E-0030M-09/2002-BMT