

Infusomat® compact^{plus}

Service Manual EN, Version 1.0



For internal use, only!

STANDARD DEVICE ALARMS

Appearance

Header with short information (e.g., "Device alarm") and 4-digit error code (e.g., „0325"). Footer with on/off-icon and short text (e.g., "Switch off device").

NOTICE

In case a device alarm appears, please switch the device off and on. Further action only needs to be taken if the alarm persists.

Error codes of the function microprocessor (FuP)

Code	Definition	Possible cause	Fault rectification
300	Pressure values delayed	Software failure If error appears frequently: Hardware defect	
325	Key panel activated	Key panel failure	Change upper part of housing
326	Key panel not confirmed by KuP	Hardware defect	Change upper part of housing
350	Pressure values delayed	Software failure	

Table 2 – 1 Error codes of the FuP

Error codes of the control microprocessor (KuP)

Code	Definition	Possible cause	Fault rectification
700	Pressure values invalid	Calibration failure Hardware defect	Re-calibrate pump
704	Disposable table damaged	Software failure	
705	Disposable data not plausible	Software failure	
706	Disposable table damaged	Software failure	
707–709	Plausibility of sensor data concerning status of the pump invalid	Software failure	
718	Polarity test not completed during self check (checked on start of infusion)	Software failure	
719	Change of pressure value, user confirmation not detected by KuP	Software failure	
735	Self check not ready on start of infusion	Software failure	

Table 2 – 2 Error codes of the KuP