CERTIFICAT

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Femoral implants for THA and THR indications

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Femoral Heads Têtes Fémorales					
Description	Size Taille	Cat. N°	Description	Size Taille	Cat. N°
Co-Cr / Chrome-Cobalt	ø28 -7mm	H10 1279	Composite Ceramic Composite	ø28 -3.5mm	H14 C1280
Co-Cr / Chrome-Cobalt	ø28 -3.5mm	H10 1280	Composite Ceramic Composite	ø28 +0mm	H14 C1281
Co-Cr / Chrome-Cobalt	ø28 +0mm	H10 1281	Composite Ceramic Composite	ø28 +3.5mm	H14 C1282
Co-Cr / Chrome-Cobalt	ø28 +3.5mm	H10 1282	Composite Ceramic Composite	ø32 -4mm	H14 C1320
Co-Cr / Chrome-Cobalt	ø28 +7mm	H10 1283	Composite Ceramic Composite	ø32 +0mm	H14 C1321
*** Co-Cr / Chrome-Cobalt	ø28 +10.5mm	H10 1284	Composite Ceramic Composite	ø32 +4mm	H14 C1322
Co-Cr / Chrome-Cobalt	ø32 -4mm	H10 1320	Composite Ceramic Composite	ø36 -4mm	H14 C1360
Co-Cr / Chrome-Cobalt	ø32 +0mm	H10 1321	Composite Ceramic Composite	ø36 +0mm	H14 C1361
Co-Cr / Chrome-Cobalt	ø32 +4mm	H10 1322	Composite Ceramic Composite	ø36 +4mm	H14 C1362
Co-Cr / Chrome-Cobalt	ø32 +8mm	H10 1323	Composite Ceramic Composite	ø40 -4mm	H14 C1400
Co-Cr / Chrome-Cobalt	ø36 -4mm	H10 1360	Composite Ceramic Composite	ø40 +0mm	H14 C1401
Co-Cr / Chrome-Cobalt	ø36 +0mm	H10 1361	Composite Ceramic Composite	ø40 +4mm	H14 C1402
Co-Cr / Chrome-Cobalt	ø36 +4mm	H10 1362			
Co-Cr / Chrome-Cobalt	ø36 +8mm	H10 1363			

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Cemented stems: **High nitrogen content stainless steel according ISO5832-9**

Femoral heads: **Co-Cr according ISO5832-12 or Pure alumina according ISO6474-1 or Composite Ceramic according ISO6474-2**

Vacuum packaging. **Gamma ray sterilised**

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Designed and
Manufactured in
France

www.evolutisfrance.com

Evolutis
C R E A T E U R F A B R I C A N T

Stemsys® MI

*Short femoral implant for
less invasive THA*

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Stemsys® MI

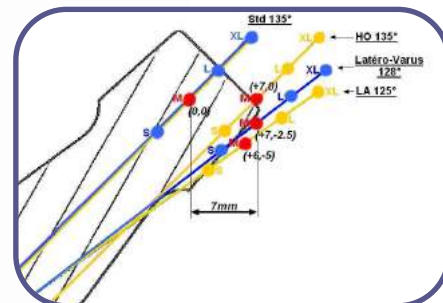
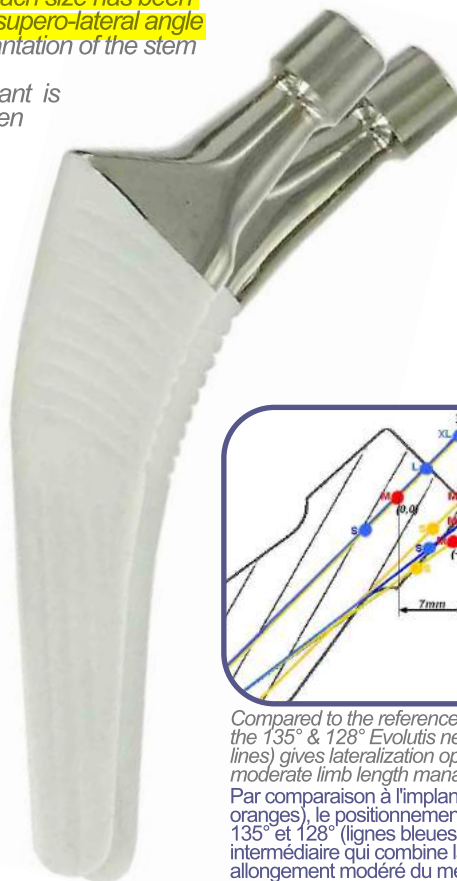
design rationale

The STEMSYS MI femoral implant's design is based on the geometry of the conventional STEMSYS. The length of each size has been reduced by about 25 % and the volume at the supero-lateral angle was milled down to facilitate the preparation and the implantation of the stem beneath the greater trochanter. Because of this volume reduction, the STEMSYS MI implant is indicated in the hemi and total arthroplasties of the hip when associated with a reduced incision surgery.

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Compared to the reference implant (orange lines), the 135° & 128° Evolutis neck positioning (blue lines) gives lateralization options combined with moderate limb length management.

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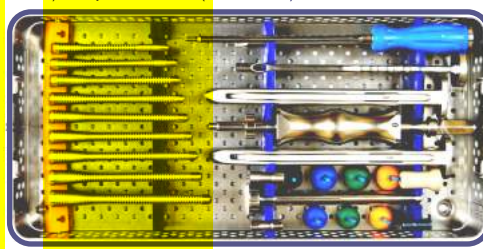
*: Items available only on special request. Not supplied on standard.

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Designed and
Manufactured in
France

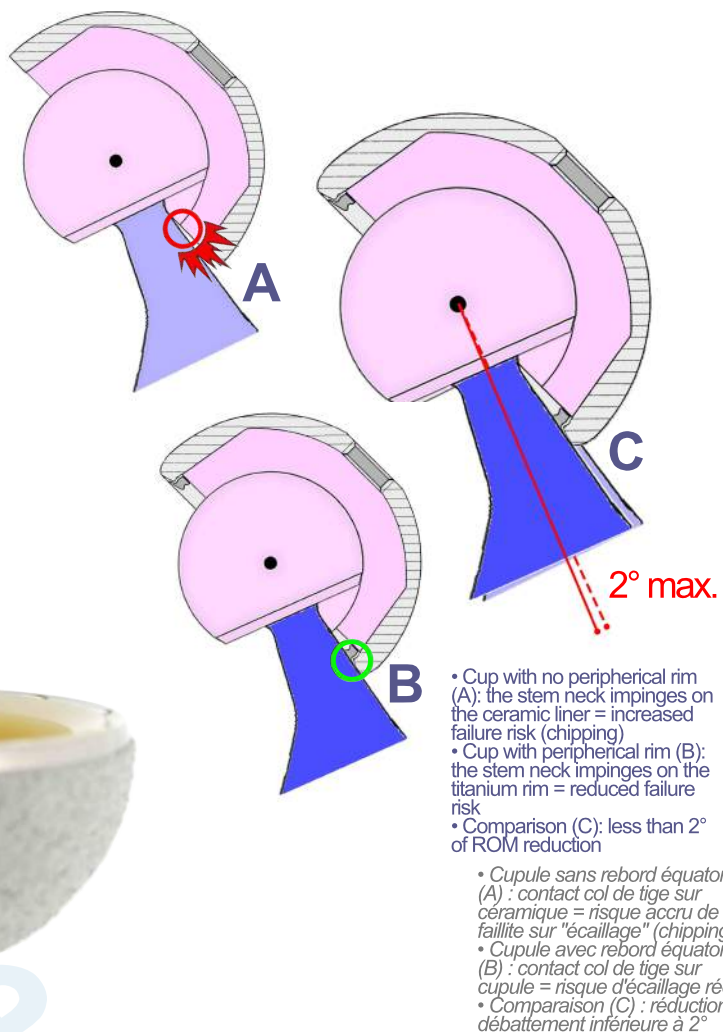
www.evolutisfrance.com

The Captiv FREELINER® cup includes a 2.6mm peripheral rim meant for:

- preventing the stem neck from impinging on the inner edge of the ceramic liner,
- securing the fixation of the PEXEL® and PEXEL®-E liners while preserving the integrity of the conical seating.

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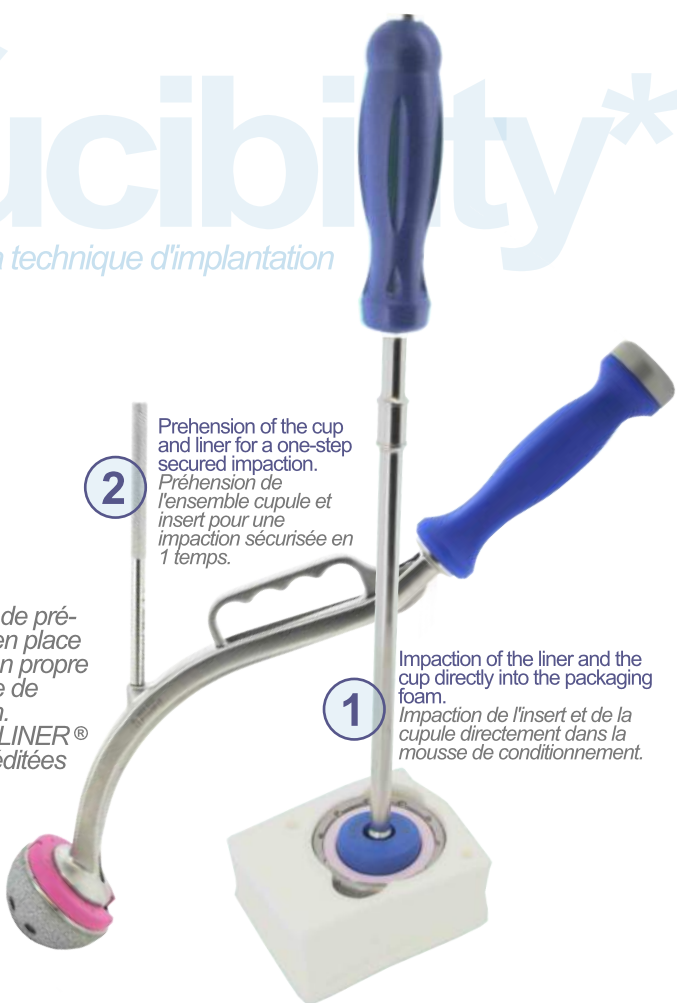
The instrumentation of the Captiv FREELINER® cup allows the operator to pre-impact the liner on the table BEFORE introducing it into the patient's acetabulum. This possibility helps to control the axis of introduction of the liner into the cup, to make sure the fixation taper is clean and dry, and to easily calibrate the impaction force. This surgical protocol which is specific to the Captiv FREELINER® cup has been developed in accordance with the Safety Reminder of the ceramic material manufacturer [1] which recommends to:

- make sure that the acetabular cup and the ceramic insert are clean dry and not damaged,
- carefully assemble the components,
- confirm proper assembly and then impact,

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A colour code (red, yellow, blue, orange, green, pink) facilitates the cup and liner size match. Example for a 58mm cup, the colour code is "green": once the material and the inner diameter are selected, choose the corresponding liner along the green line.

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Evolutis
MOTION INSIDE

Material / Matériau

Cup: TA6V titanium alloy according ISO 5832-3. Porous titanium and Calcium hydroxyapatite coating

Screw: TA6V titanium alloy according ISO 5832-3

Liner: UHMWPE according ISO 5834-1 and 2, or Composite Ceramic according ISO 6474-2

Packaging: vacuum packed and gamma ray sterilized

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www.evolutisfrance.com

E v o l u t i s
C R E A T E U R F A B R I C A N T

Acetabular cup for Ceramic or Highly-Crosslinked PEXEL® -E Polyethylene liner

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[1] Safety Reminder / Biolog inserts. Ceramtec GmbH, Medical Products Division, D-73207 Plochingen, Germany, August 2014. http://www.ceramtec.com/files/mt_safety_reminder_inserts_en.pdf

The **Captiv FREELINER®** acetabular cup for Total Hip Arthroplasty is designed for cementless fixation to the bone, and enable 3 choices of friction bearing:

- moderately crosslinked PEXEL® polyethylene against a metal or a ceramic head,
- highly-crosslinked polyethylene with vitamin-E (PEXEL®-E) against a metal or a ceramic head,
- ceramic composite against a ceramic composite head.

The **Captiv FREELINER®** cup has been designed to ensure maximum safety during implantation and security for the patient when used with either polyethylene or ceramic liners. The cup is uncompromisingly adapted to the different characteristics of these 2 bearings:

- the inner surface of the taper is smooth and free of asperities to facilitate and secure the insertion of the ceramic insert,
- the stability of the polyethylene insert is secured by 10 anti-rotational spurs and a locking ring situated on the equator.

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**Captiv
Freeliner®**

- Choice of 3 different friction bearings
- Primary stability enhanced by the high rugosity of the coating
- Osseo-integration on highly porous substrate (40 to 70%)
- Safety and reproducibility of the surgical technique

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Moderately crosslinked PEXEL® polyethylene

- Moderately crosslinked at 40kGy max.
- VacUpac vacuum packaging
- Anti-dislocation 6mm Posterior wall
- Anterior chamfer of 32°
- Inner Ø28mm

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Highly-crosslinked PEXEL®-E polyethylene with vitamin-E

- Sequentially crosslinked UHMWPE at 100kGy max.
- VacUpac vacuum packaging
- Oxydation inhibited by content of 0.1%wt of α -tocopherol (vitamin-E) blended in the material before compression molding
- with circumferential chamfer of 20°
- Inner Ø32 and 36mm

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Composite of Alumina and Zirconia ceramic

- Composite of alumina and zirconia ceramic
- New SLWZero design with constant radius
- Center of rotation aligned with the equator of the liner
- Liner protected from "chipping" by the peripheral border on the cup
- Inner Ø32, 36, and 40mm

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
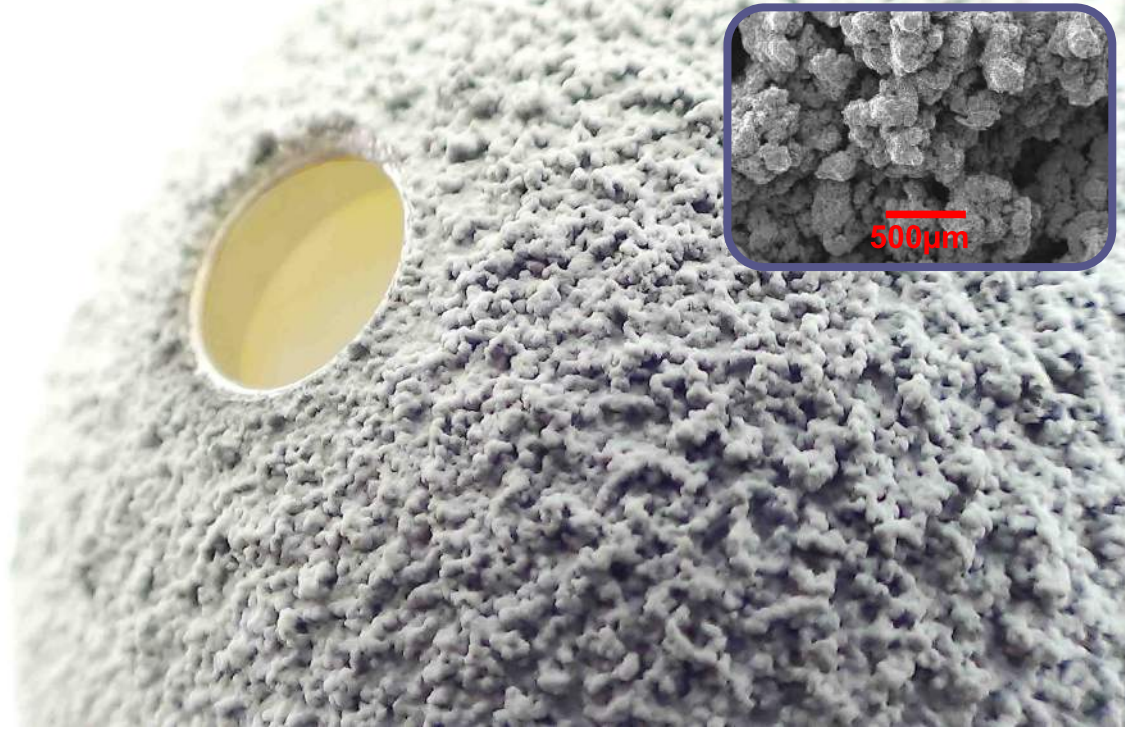
Hemispheric cup
Primary stability through a plasma spray high rough grade porous coating
Dimensional press-fit from 1.26 to 1.66mm according to the diameter of the cup
Optional complementary fixation with screws (Ø6.0 cancellous self-tapping):
- 4 holes in the superior part of the cup on a constant bow of 120°
- angular clearance of 20° for the screws

Cementless osseo-integration coating:
- 80µm of calcium hydroxyapatite
- 400 to 600µm of porous titanium

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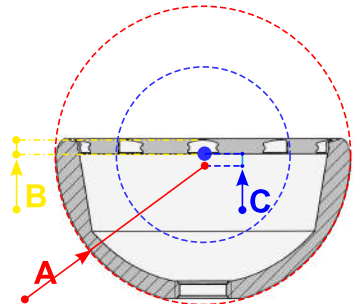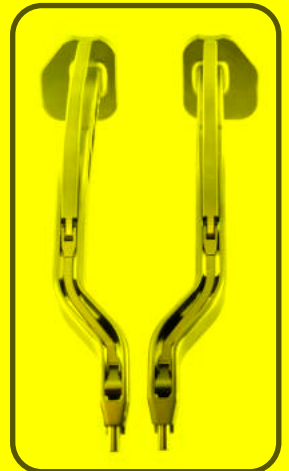
Curved Handle for Standard Cup
*Manche Courbe pour Cupule
Standard*

H76 001

Curved Handle for
Pre-Set Impaction
*Manche Courbe
Impaction sur
Table*

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