www.thermofisher.com

**CampyGen™ Compact Sachet,
CampyGen™ 2.5L Sachet, and
CampyGen™ 3.5L Sachet**

EN

REF CN0020C	Σ 20
REF CN0025A	Σ 10
REF CN0035A.....	Σ 10

Intended Use

CampyGen sachets (CN0020C, CN0025A and CN0035A) are accessories intended to be used with the *in vitro* diagnostic devices part of the AnaeroGen™ product range, for the generation of optimal atmospheric conditions for the growth of microaerophilic microorganisms.

These accessories are for professional use only, are not automated and nor are they companion diagnostics.

Summary and Explanation

Clinically relevant microaerophilic organisms include species of the *Campylobacter* genus, with *Campylobacter jejuni* and *Campylobacter coli* to be associated with the vast majorities of human gastroenteritis worldwide. They have a strict respiratory metabolism, and they may require strictly anaerobic or microaerophilic environment to grow¹.

Having atmosphere generating systems that offer effective performance in creating the optimal atmospheric conditions for the growth of microaerophilic organisms is therefore essential, given the clinical significance of several species that can pose great risks to health when affecting immunocompromised patients. Therefore, it is extremely beneficial to have a simple and easy-to-use system to generate microaerophilic conditions in order to facilitate the growth of microaerophilic organisms, assisting the clinicians' workflow, reducing costs and, ultimately, giving results more efficiently, thus allowing early diagnosis and prompt treatments for patients.

Principle of Method

The CampyGen (CN0020C, CN0025A and CN0035A) are gas generating sachets suitable for the generation of optimal atmospheric conditions for microaerophilic microorganisms when used with the related plastic pouches, boxes or jars, that are supplied separately.

The active ingredient of the sachet is ascorbic acid that reacts when in contact with air, rapidly adsorbing oxygen while simultaneously producing carbon dioxide. Because the reaction starts immediately, it is important to seal the sachet within the appropriate pouch, jar, or box within one minute, to preserve the integrity of the accessory. Although the reaction is exothermic, none of the sachets for CampyGen (CN0020C, CN0025A and CN0035A) will ever exceed 65°C.

Description of Components

CampyGen Compact Sachet (CN0020C) includes 20 sachets intended to be used with Zip Seal pouches (with integral seals) or plastic pouches with a capacity of 4 x 90 mm plates, part of the AnaeroGen products range, but supplied separately.

CampyGen 2.5L Sachet (CN0025A) and CampyGen 3.5L Sachet (CN0035A) are sold in packages containing 10 sachets each, with sachets sizes of 2.5L and 3.5L respectively, intended to be used with the corresponding size 2.5L jar, 2.5L box and 3.5L box respectively, both sold separately.

Materials Required but Not Supplied

- Inoculated media
- Incubators
- Quality control organisms
- AnaeroGen Compact Sealing Clip (AN0005C) A pack of 5 sealing clips intended to be used with the CampyGen Compact Sachet (CN0020C) and the AnaeroGen Compact Plastic Pouch (AG0020C) or AnaeroGen W-Zip Seal Pouches (AG0060C).
- AnaeroGen Compact Plastic Pouch (AG0020C) A pack of 20 plastic pouches (without clips).
- AnaeroGen W-Zip Seal Pouches (AG0060C) A pack of 20 W-Zip seal pouches (with integral seal).
- AnaeroJar™ 2.5L Jar (AG0025A) A 2.5L jar supplied with 2.5L capacity polycarbonate jar base and lid secured to base by four self-venting clips, intended to be used with AnaeroGen 2.5L Sachet (AN0025A) and CampyGen 2.5L Sachet (CN0025A).
- AnaeroBox™ 2.5L Box with Lid and Seal (AB0025A) A 2.5L box intended to be used with AnaeroGen 2.5L Sachet (AN0025A) and CampyGen 2.5L Sachet (CN0025A).
- AnaeroBox 3.5L Box with Lid and Seal (AB0035A) A 3.5L box intended to be used with AnaeroGen 3.5L Sachet (AN0035A) and CampyGen 3.5L Sachet (CN0035A).

Storage

- Store product at 2 - 25°C.
- The product may be used until the expiry date stated on the label.

Warnings and Precautions

- For *in vitro* diagnostic use only.
- For professional use only.
- Inspect the product packaging before first use.
- Do not use the product if there is any visible damage to the packaging or plates.
- Do not use the product beyond the stated expiry date.
- Do not use the device if signs of contamination are present.
- Do not use the device if the colour has changed or there are other signs of deterioration.
- It is the responsibility of each laboratory to manage waste produced according to their nature and degree of hazard and to have them treated or disposed of in accordance with any federal, state and local applicable regulations. Directions should be read and followed carefully. This includes the disposal of used or unused reagents as well as any other contaminated disposable material following procedures for infectious or potentially infectious products.
- As soon as the CampyGen sachet is exposed to the air, the reaction will start. It is therefore essential that the paper sachet is placed in the pouch, box, or jar and sealed within one minute.
- The reaction of the ascorbic acid with oxygen is exothermic. However, the temperature of the CampyGen sachet will not exceed 65°C.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Procedure

1. For the CampyGen Compact Sachet CN0020C place 2 inoculated media plates in the plastic pouches (AG0020 or AG0060).
NOTE: If only one plate is to be inoculated, an uninoculated plate should also be placed in the plastic pouch.
OR
For the CampyGen 2.5L Sachet (CN0025A) and CampyGen 3.5L Sachet (CN0035A) place up to 12 inoculated media plates onto the plate carrier and placed in either the AnaeroGen 2.5L jar (AG0025A), AnaeroBox 2.5L (AB0025A), or Anaerobox 3.5L (AB0035A) respective of size sachet.
Disposable plastic Petri dishes should be of the vented variety to aid gas transfer between the interior and exterior of the plates.
2. Tear open a CampyGen foil sachet at the tear-nick indicated and remove the sachet from within.
3. Immediately place the CampyGen sachet in the plastic pouch, jar, or box.
NOTE: The CampyGen sachet will become warm to touch on exposure to air.
4. Expel excess air from the plastic pouch. Seal the plastic pouch immediately with the AnaeroGen Compact Clip (AN0005).
OR
Close the jar or box lid immediately.
NOTE: The time taken between opening the foil sachet and sealing the plastic pouch should not exceed one minute. Extended exposure will result in loss of reactivity, and microaerophilic conditions may not be achieved in the pouch, jar, or box.
5. Incubate appropriately.
6. After the incubation period remove the plates and examine for the presence of colonies. If the plates require re-incubation, then a fresh CampyGen Compact sachet must be used following steps 2-5 described above.
NOTE: The plates may be initially inspected through the transparent plastic pouch. If the pouch is opened, a fresh CampyGen Compact sachet is required for re- incubation.
7. After incubation, the exhausted CampyGen Compact sachet and plastic pouch should be sterilised and discarded with the non-hazardous laboratory waste.

Quality Control

It is the responsibility of the user to perform Quality Control testing taking into account the intended use of the product, and in accordance with any local applicable regulations (frequency, number of strains, incubation temperature etc.).

The user should periodically check their system for its ability to provide adequate conditions for the growth of appropriate bacteria.

Performance Characteristics

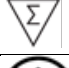
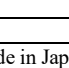
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
Evidence of the repeatability and reproducibility of the CampyGen (CN0020C, CN0025A and CN0035A) are demonstrated by the performance data generated from In- Process QC testing performed during the manufacturing of the CampyGen sachets and by the historical performance achieved through routine QC assessed by the manufacturer.

Bibliography

1. SMI ID 23: Identification of Campylobacter Species - GOV.UK." n.d. Accessed July 19, 2022. <https://www.gov.uk/government/publications/smi-id-23-identification-of-campylobacter-species>.

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code

	Temperature limit
	Use-by date
	Do not re-use
	Consult instructions for use
	Contains sufficient for <n> tests
	Do not use if packaging damaged and consult instructions for use
	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
	Importer - To indicate the entity importing the medical device into the locale. Applicable to the European Union
	Made in Japan

©2022 Thermo Fisher Scientific Inc. All rights reserved.
All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Oxoid Limited, Wade Road, Basingstoke, RG24 8PW, United Kingdom

For technical assistance please contact your local distributor.

Revision Information

Version	Date of modifications introduced
2.0	2023-12-19

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