



A Combination That's **Better Together**

Gemini AS Slide Stainer
Slide Basket Transfer System
ClearVue Coverslipper




The combined staining and coverslipping solution

Knock out unnecessary tasks and increase your laboratory's productivity

The Eprelia Slide Basket Transfer System eliminates the need to manually transfer slide baskets from the Eprelia Gemini AS™ slide stainer to the Eprelia ClearVue™ coverslipper.

Eprelia Slide Basket Transfer System

- Free up your team's time with automatic transfer of slide baskets from the Gemini AS slide stainer to the ClearVue coverslipper.
- Simply place slides in the Gemini AS slide stainer and carry on with other tasks without distraction.
- The system will alert you when the slides have finished coverslipping in the ClearVue coverslipper.
- By eliminating transfer between the Gemini AS slide stainer and ClearVue coverslipper, reduce your team's exposure to harmful fumes.
- Compatible with Gemini AS slide stainers and ClearVue coverslippers made after February 2018 only.

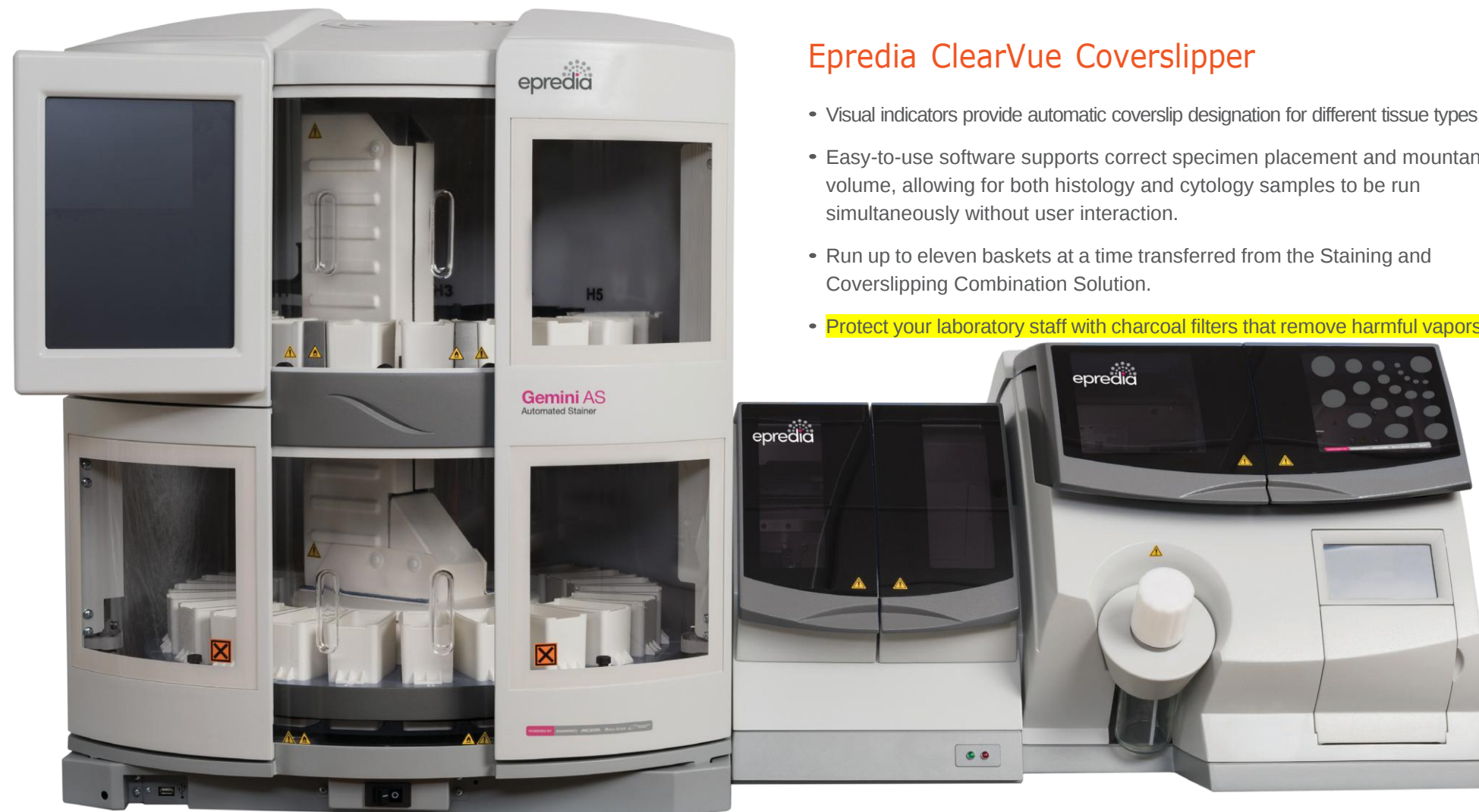

Better Together

Paired with the Staining and Coverslipping Combination Solution, the Gemini AS slide stainer and the ClearVue coverslipper have the power to reduce manual steps in your laboratory workflow.

Eprelia Gemini AS Automatic Slide Stainer

- Improve throughput with 26 reagent stations – enough to run several protocols at once.
- Intelligent software automatically optimizes reagent layouts and allows for multiple racks to be run for maximum output.
- Urgent start gives user the opportunity to prioritize “urgent” baskets without compromising those already in process
- LED lighting and easy-to-access reagent stations provide a clear, accessible view of the staining area.
- Five optional on-board heaters cater to slide drying in high-volume situations.
- USB connectivity allows protocols, reagent layouts, and event logs to be saved for future use, saving time later.

Eprelia ClearVue Coverslipper

- Visual indicators provide automatic coverslip designation for different tissue types.
- Easy-to-use software supports correct specimen placement and mountant volume, allowing for both histology and cytology samples to be run simultaneously without user interaction.
- Run up to eleven baskets at a time transferred from the Staining and Coverslipping Combination Solution.
- Protect your laboratory staff with charcoal filters that remove harmful vapors.





Your complete solution

For complete workflow synergy, use Epredia Hematoxylin, Eosin, mountants, microscope slides, and coverslip holders to enhance the use of your Epredia staining and coverslipping combined solution.

With you when you need us

Epredia offers a variety of service and support options designed to help you minimize downtime and maximize workflow. To learn more, contact your local Epredia sales representative.

Specifications

Category	Specifications
Slide Basket Transfer System with Gemini AS slide stainer and ClearVue coverslipper	

26 reagent stations with 320 ml capacity each. Load up to 15 baskets of 20 slides continuously. Optional accessory for barcode scanning of slide and LIS connectivity on the ClearVue coverslipper to signal when slides are coverslipped. Basket orientation sensors indicate when slide baskets are loaded in correct orientation to go through the Slide Basket Transfer System.

Slide Basket Transfer System

Width (Gemini AS slide stainer + SCombo Solution + ClearVue coverslipper) 157 cm (61.81 in.)

Depth (Including Gemini AS slide stainer touchscreen housing) 79 cm (31.0 in.)

Height (Gemini AS slide stainer) 890 mm (35.04 in.)

Weight with no reagents 165 kg (365 lb)

Weight with typical reagents 179 kg (395 lb)

Electrical Specifications

Power supply voltages 110-1240 VAC (-)
Maximum supply voltage fluctuations not to exceed +/- 10% of nominal voltage

Frequency 50/60 Hz

Power 600 VA

Fuses 2 x 110A, 250V

Ordering Information

Description	Cat. No.
-------------	----------

Slide Basket Transfer System

For use with Gemini AS stainers and ClearVue coverslippers manufactured after February 2018.

A84500001

Gemini AS Slide Stainer

Gemini AS, unheated A81500002

Gemini AS, with five heated stations A81500001

ClearVue Coverslipper

ClearVue coverslipper A79200001

Accessories

Transfer baskets (5) black A84510032

Transfer baskets (5) white A84510041

Operator guide A84510100

Documentation CD A84510100-CD

Xylene tray A84530101

Manual unload tool A84530110

Wash bottle AP18730

4481 Campus Drive
Kalamazoo, MI 49008
United States
+1 (800) 522-7270



Tudor Road, Manor Park
Runcorn, WA7 1TA
UNITED KINGDOM
+44 (0) 800 018 9396
+44 (0) 1928 534 000

Find out more at epredia.com/staining-and-coverslipping-solutions