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BLAYCO® plates are designed and dimensioned to offer full guarantees of security during use. However, it is required to follow some rules to guarantee an appropriate plate functioning.

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WHAT TO DO?

1. **CHECK** all possible cables for breakages or deterioration.
2. **TEST** the alarm of generator as per instructions supplied by the manufacturer.
3. Before placing the plate on the patient, **SHAVE** (if necessary), **CLEAN**, and **DRY** the application site.
4. It is also advisable, to **SMOOTH** the skin of the patient (in case of elderly) in the application site.
5. **APPLY** the plate over a well vascularized muscle area.
6. **PLACE** the electrosurgical plate before covering the patient with surgical drapes.
7. **CHECK** once more cables and connections if during the intervention power settings higher than usual are required.
8. **CHECK** again the plate contact with the patient, if patient is repositioned during the intervention.
9. **KEEP** the electrosurgical pencil clean of tissue residues.
10. **PLACE** the E.C.G. electrodes as far as possible and equidistant from the surgery area.
11. **REMOVE** the return electrode by slowly lifting off to avoid an excessive skin trauma.

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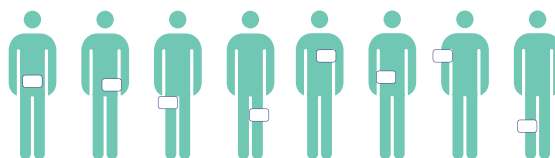
WHAT TO AVOID?

1. **PREVENT** liquids from pooling over the plate area.
2. **DO NOT PLACE** the plate over bony prominences, scars or replacements.
3. **NEVER CUT OR MODIFY** the plate.
4. **DO NOT ALLOW** the electrosurgical pencil to contact objects with ground connection, not even for test operation.
5. **DO NOT ACTIVATE** the electrosurgical pencil until it is in contact with the cutting zone.
6. **DO NOT COIL** plate or electrosurgical pencil cables over metallic objects.
7. **DO NOT ALLOW** the patient to make accidental mass.
8. **DO NOT REUSE** the plate. Reuse of this product may produce failures in the electrosurgical procedure and/or cross-contamination to the patient.
9. **DO NOT RELOCATE** plate after initial placement.
10. **DO NOT USE** power setting higher than 50W with neonatal plates.

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The present list correlating connectors with electrosurgical units model does not suppose any commitment by Telic, S.A.U., who assume no responsibility in that regard as there is not any legal binding with the equipment manufacturer. It is responsibility of the client to check which connector model is compatible with the equipment.

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You are kindly requested to specify the suitable connection when ordering. (REM is a Registered Trademark of Valleylab Inc.)

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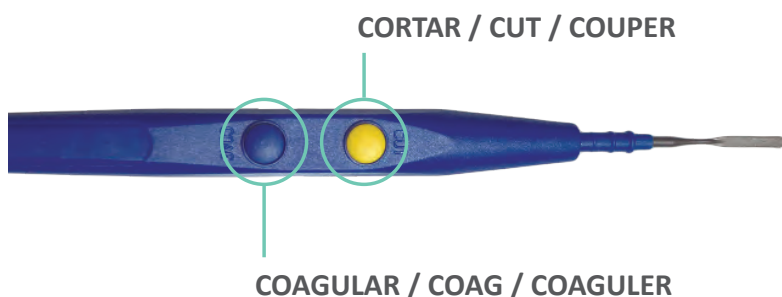
CHARACTERISTICS

- Easy handling
- Light
- Comfortable anatomical shape
- 3-pin standard connector

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Single use hand control electrosurgical pencil, with removable AB-80 accessory.

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AL-40

Dispositivo limpiador de electrodo, autoadhesivo.

Adhesive electrode cleaning device.

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Reusable electrosurgical pencil, with removable AB-80 accessory.

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Mango de bisturí electroquirúrgico reutilizable, con accesorio AB-80 desmontable.
Manche de bistouri électrochirurgical réutilisable, avec accessoire AB-80 amovible.