









YOUR PARTNER IN BREAST CARE








BREAST CANCER

GLOBAL FACTS & FIGURES

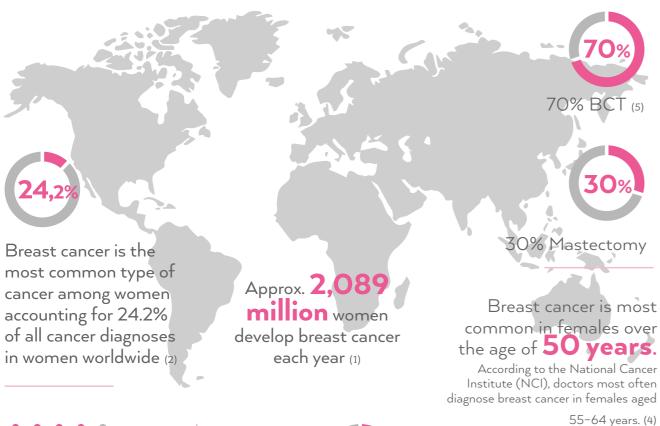



10-40%

are not satisfied with the result of their BCT (6)



Worldwide the overall 5-year relative **survival** rate for breast cancer is 90% (3)

Approx. **20-30%** develop lymphoedema in the arms or chest after breast surgery (8)

Approx. **40-60%**

of women who undergo BCT go on to have further surgery (7)

- https://gco.iarc.fr/today/data/factsheets/cancers/20-Breast-fact-sheet.pdf
 https://gco.iarc.fr/today/
 https://gco.iarc.fr/today/
 https://www.webmd.com/breast-cancer/breast-cancer-survival-rates
 https://www.medicalnewstoday.com/articles/325654#risks-at-different-ages
 https://www.nebi.nlm.nih.gov/pmc/articles/PMC4504664/
 https://pubmed.ncbi.nlm.nih.gov/29605247/
 https://pubmed.ncbi.nlm.nih.gov/18259820/
 https://pubmed.ncbi.nlm.nih.gov/la259820/

- /www.oatext.com/lymphe/dema-following-breast-cancer-the-importance-of-surgical-methods-and-obesity.php#gsc.tab=0

TABLE OF CONTENTS

ABOUT AMOENA	4
Your partner in breast care	4
Patient journey	(
Amoena Solutions	8
Our new products 2021	10
AMOCURA - RECOVERY CARE	1.
Solutions during recovery.	
CuraSupport	
CuraSupport Compression	
CuraSupport Comfort	
CuraSupport accessories.	
CuraScar	
CuraScar Garments.	
CuraScar Silicones	
BREAST FORMS AND SHAPERS	7
Breast forms.	
Fitting after mastectomy	
Breast form portfolio	
Amoena Adapt Air	
Amoena Contact	
Amoena Energy.	
Amoena Natura	
Amoena Essential	
Additional breast forms	
Breast shapers	
Breast shaper portfolio	
Amoena Balance Adapt Air	
Amoena Balance Contact & Amoena Balance Natura	
Amoena Balance Essential	
Breast form & shaper accessories	62
BREAST CARE APPAREL	6
Lingerie	70
Amoena Lace bras	
Amoena Classic bras.	
Amoena Seamless bras	
Amoena High coverage bras	
Active	
Wellness	
Swimwear	
StyleClassic.	
Active	
Breast Care Apparel accessories	
RESOURCES	
4 Tips to increase your sell through	
Merchandising and Point-of-Sale support	
Amoena online support	
Feature overview breast forms & shapers.	
Sizing charts	
Recovery Care - Amocura index	
Breast Care Apparel index	
· · ·	













YOUR PARTNER IN BREAST CARE

At Amoena, we are all in when it comes to supporting breast-operated women to live with confidence. We test more, we invest more in R&D, we listen more closely, and we collaborate with the most admired healthcare professionals and opinion leaders. Being all in means never compromising on details, because every detail supports confidence.

THE AMOENA SOLUTION


We know no detail is too small when it comes to helping breast operated women live confidently. Patents for materials and manufacturing methods are proof of our progress but we'll never stop endeavouring to meet every need with the highest quality products. More than 45 years on from developing the first breast form, we've now sold 17 million, as well as 30 million pieces of breast care apparel. Today, we're proud to support women throughout their journey – from the moment they're diagnosed, to living life to the fullest once more.

We call it The Amoena Solution.




PATIENT JOURNEY

At Amoena we know exactly what women need along their breast cancer journey and how we can make the fitters' daily lives easier. And it's why we now offer products for every stage of a woman's journey.

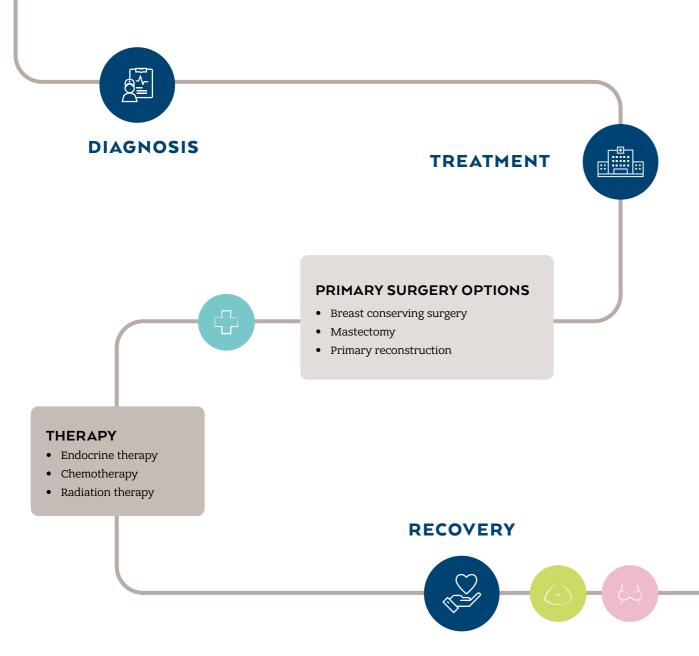

WHERE EVERYDAY LIFE CONTINUES

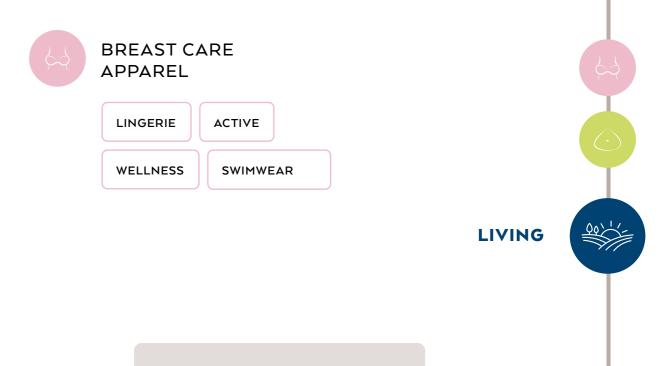

SECONDARY SURGERY OPTIONS

- Reconstruction with implants
- Reconstruction with autologous transplant

WHERE CONFIDENCE GROWS



BREAST FORMS & SHAPERS



WHERE HEALING BEGINS



CURASUPPORT COMPRESSION

CURASCAR GARMENTS

CURASUPPORT COMFORT

CURASCAR SILICONES

AMOENA SOLUTIONS

A perfect match for comprehensive breast care

Our products are designed to work perfectly together. For each stage of the patient journey, a package of different products can be put together as a recommendation for the patient. We have compiled these packages based on our own 45 years of experience and in collaboration with doctors, healthcare professionals and fitters.

FOR BCT WITHOUT RECONSTRUCTION



CURASUPPORT COMPRESSION BRA

ABOUT AMOENA

e.g. Leyla ZIP-ST-LOW



CURASCAR SILICONES

e.g. Anchor

Amoena post-surgery compression bras are designed for stabilisation and fixation of the surgical area. Combined with CuraScar Silicone Patches they ensure to be the optimal scar therapy.

FOR BCT/MASTECTOMY WITH RECONSTRUCTION (AUTOLOGOUS RECONSTRUCTION)



CURASUPPORT COMPRESSION BRA

e.g. Sina HE-ST-MEDIUM



CURASCAR SILICONES

e.g. Mammilla Ring



CURASCAR GARMENTS
e.g. Panty 2-ZIP-BO



CURASCAR SILICONES

e.g. Strips

Amoena post-surgery compression bras are designed for stabilisation and fixation of the surgical area. Our scar garments support the tissue and optimize the healing process following reconstructions. Combined with CuraScar Silicone Patches they ensure to be the optimal scar therapy.

FOR BREAST ASYMMETRY



BREAST SHAPER e.g. Balance Adapt Air



BRAS AND SWIMWEAR

e.g. Amanda Soft Bustier

Our Amoena breast shapers are the simple, non-surgical way to balance missing tissue or asymmetries, e.g. after BCT, and to achieve a natural silhouette. Our specially designed bras hold the shaper securely in place.

FOR LYMPHOEDEMA



WEIGHT REDUCED BREAST FORM
e.g. Cosmetic, Light, Xtra Light options; Adapt
Air



BRAS WITH WIDER OR PADDED STRAPS

e.g. Clara Soft Bra Front Closure

Weight reduced breast forms and bras with wider straps relieve pressure on the shoulders.

FOR SWIMMING



SWIM BREAST FORM e.g. Aqua Wave



SWIMWEAR
e.g. Alabama Half Bodice

High performance products that are a perfect match for a natural silhouette, even during water sports.


income Patches they ensure to be the optimal scal therapy.

Leyla bra, p. 18 / Anchor silicone scar patch, p. 31 / Mammilla Circle silicone scar patch p. 31

ABOUT AMOENA





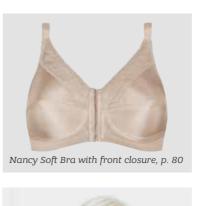







Curascar silicone scar patches, p. 29



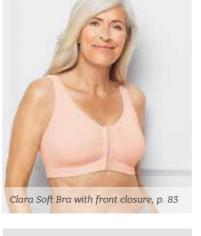







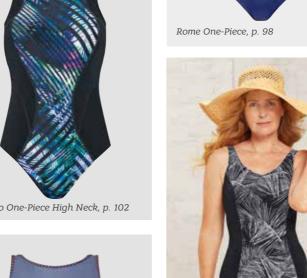



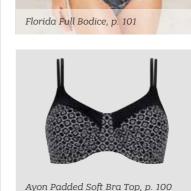




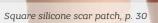







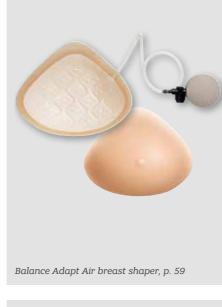





10

PRICE SEGMENTATION

Dividing the products into Basic, Advanced and Premium provides a better overview of the portfolio, allowing you to find just the right

product for your customers.

These products combine the best that Amoena has to offer in terms of materials, workmanship, function and design. For women with the highest standards who want to benefit from the most innovative technologies and

AMOENA PREMIUM

elaborate product development.

AMOENA ADVANCED

AMOENA BASIC

attractive prices.

The comfort and functionality of these products

are of a superior standard. For discerning women who value outstanding quality and innovative manufacturing technologies.

These products meet all the essential requirements with the quality expected from Amoena. For women who do not want


to compromise on materials, workmanship, function or design – and all this at highly

AMOCURA - RECOVERY CARE

The post-surgical phase of a woman's journey is the most fragile. Because it's crucial that the result of the surgery is not compromised, the focus for all our Recovery Care solutions is healing, closely followed by comfort. Our growing range of solutions reflects the increase in the number of surgical procedures, and the fact that every woman has individual needs after surgery. All our products are designed to be combined to create the optimal healing conditions, no matter the woman, and no matter the surgery. Our newly expanded range of products is based on our long experience. Our goal is to give every woman the right combination for her needs after surgery. Our recovery care products can be combined in many ways so they can meet the specific needs of women as they recover from any kinds of breast surgery.

AMOCURA PRODUCTS

Being diagnosed with breast cancer is a life-changing moment for many women. Not only are they worried about the disease, but also about the physical effects they may experience. Amoena is here to support women from the moment they are diagnosed, throughout their treatment and on the road to recovery. As part of the Recovery Care line, we have developed products that support these women through the difficult times right after surgery and during the healing process. 70% of women with breast cancer undergo breast-conserving surgery. Breast-conserving surgery (BCS) removes the cancer while keeping as much of the breast tissue as possible. Usually some surrounding healthy tissue and lymph nodes and vessels are removed too. This means that the number of mastectomies without reconstruction is decreasing and the number of partial surgeries and reconstructions (after a mastectomy) is increasing. Our post-surgical products in the Recovery Care line are ideal for the post-surgery phase. For those who have undergone breast-conserving therapy, which is usually associated with small scars or minor losses, the products can provide optimal support. Even after reconstructive surgery where the breast is restored using autologous tissue (e.g. from the stomach), our post-surgical products are extremely beneficial. Reconstruction results in extensive scarring and our Scar Management products can support them while they heal.

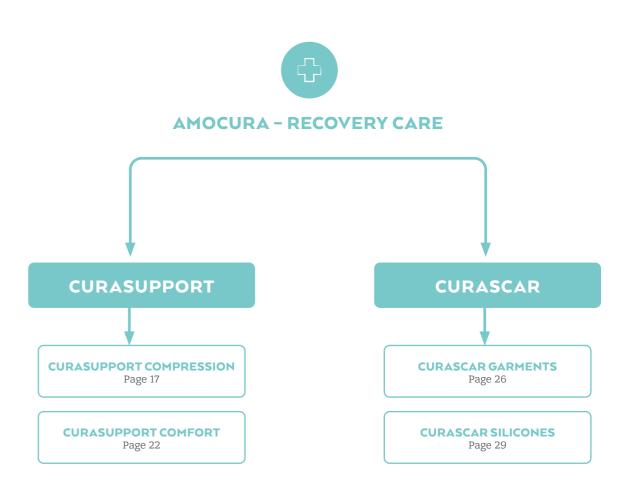


SOLUTIONS DURING RECOVERY

SCAR THERAPY WITH COMPRESSION & SILICONE

Silicone and compression provide the ideal scar therapy combination. Applied early in the treatment phase, combination therapy can act as a prophylactic to minimise the risk of excessive scarring.

HOW IT WORKS

A therapeutically effective pressure is exerted on the closed wound area. Excessive connective tissue remodelling can be counteracted. The combination of constant surface pressure from the compression garment, along with silicone, has a positive influence on scar healing. The silicone creates a moist environment on the scar, which prevents the scar from drying out. It keeps the scar soft and has a positive effect on scar tissue. For optimal scar healing, we recommend the use of our compression textiles in combination with our silicone patches.

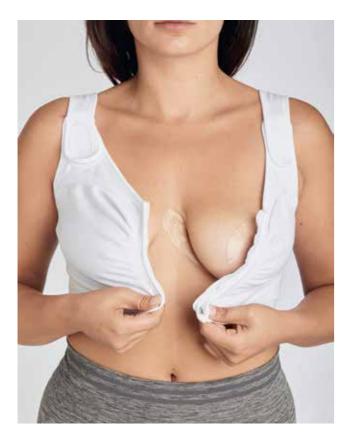

Combination Mammilla Circle and Anchor silicone patches with Leyla seamless compression bra after reconstruction



Combination Square silicone patch with Panty 2-ZIP-BO compression garment after reconstruction with autologous tissue

PRESSURE LEVELS



LIGHT PRESSURE LEVEL

Low pressure level – light pressure for fixation and stabilization – no pressure on the cup area



MEDIUM PRESSURE LEVEL

Medium pressure level – constant and medium pressure for fixation and stabilization with an even pressure also on the breast area



HIGH PRESSURE

High pressure level – high pressure for fixation and stabilization with an even pressure also on the breast

THE AMOENA SOLUTION

For breast conserving surgery and reconstruction

CURASUPPORT COMPRESSION



LEYLA ZIP-ST-LOW

Art. no. 44604 Leyla ZIP-ST-LOW (Rose Nude) 44605 Leyla ZIP-ST-LOW (White)

44606 Leyla ZIP-ST-LOW (Black)

Cups Universal

FEATURES & BENEFITS

- Leyla can support surgical results for various indications
- Light compression level assures sufficient immobilization and stabilization of the operated breast tissue and optimal healing
- Accurate fitting thanks to seamless technology and visual sizing
- Prepared to direct clinical use thanks to pre-washing process
- Seamless and gentle fabric on surgical site reduces skin irritation, discomfort and unwanted pressure



CURASCAR SILICONES



MAMMILLA CIRCLE

Art. no. Dimensions 012 (Clear) Ø outside 75 mm Ø inside 20 mm

Breast

Apply on



ANCHOR

Art. no.
Dimensions
Apply on

013 (Clear) 230 x 145 mm Breast

THE AMOENA SOLUTION

Reconstruction with autologous tissue

CURASCAR GARMENTS







Art. no. 45000 Panty 2-ZIP-BO NU (Nude) 45001 Panty 2-ZIP-BO BK (Black)

BERMUDA

Art. no. 45002 Bermuda 2-ZIP-BO NU (Nude) 45003 Bermuda 2-ZIP-BO BK (Black)

FEATURES & BENEFITS

- Bonded waistband for no pressure points
- Ultrasonic finishing at leg ending for flat ending
- Two zips for easy dressing
- Belly & leg (only Bermuda) compression
- Breathable, OEKO-TEX®-certified fabric
- $\bullet \;\;$ Fabric stays in place if you need to cut it for drainage tubes
- Designed to be combined with Amoena silicone patches



BELLY BANDAGE

Art. no. 45004 Belly bandage 3-VE NU (Nude) 45005 Belly bandage 3-VE BK (Black)

FEATURES

- Compression belt
- Velcro fasteners for easier closing, opening and flexible fit
- Can be combined with Amoena silicone patches

CURASCAR SILICONES



STRIPS

 Art. no.
 010 (C

 Dimensions
 260 x

 Apply on
 Belly/

010 (Clear) 260 x 35 mm Belly/thigh



SQUARES

Art. no.
Dimensions
Apply on

011 (Clear) 260 x 100 mm Belly/thigh



The Leyla garment is hygienically prepared through a prewashing process. It is packaged in a sealed bag to ensure it stays clean for direct clinical use.

ZIP

Zipper in front

во

Bonded

ΗE

VΕ

Hook & eye in Velcro in front

AS

Adjustable

IN

Easy step-in

CURASUPPORT COMPRESSION



LEVEL

LEYLA ZIP-ST-LOW

Post-surgical bra with light pressure level

Art. no. 44605 White

> 44604 Rose Nude 44606 Black

70/75; 80/85; 90/95; 100/105; 110/115; 120/125 Sizes


Cups: Universal

Material: 49% polyamide, 43% cotton, 8% elastane Indication: after various breast surgeries


• Precise fit and comfortable to wear thanks to the seamless circular

- knitting technology and high cotton percentage
- Suitable for direct clinical use due to a pre-washing process
- Immobilisation and stabilisation of the operated breast
- Special knitted zones and targeted light pressure support lymphatic drainage and the healing process;
- Seamless knitted fixation ring around the pre-formed cup (stabilisation of the breast with no compression)
- Universal cup (measuring the underbust is sufficient to find the correct
- $\bullet\,$ Adjustable, wide straps with Velcro fasteners for easy dressing and undressing also provide exceptional comfort
- Hooks and eyes make it easier to do the zip up on top at the front
- Soft fleece on the back makes it possible to securely position the compression belt. This is sold separately,
- In combination with CuraScar Silicones for supportive scar treatment

Please find our colour-coded sizing chart to help you easily identify the right size from your stock on page 112

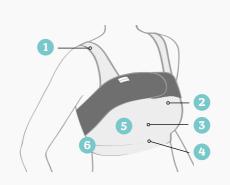

- 1. Adjustable wide lined straps with hook and loop fasteners for easy fitting and wearing comfort
- 2. Seamless design using strong, stretchable material, providing low compression
- ${\bf 3.}~{\bf Zip}~{\bf front}~{\bf closure}~{\bf for}~{\bf easy}~{\bf access}~{\bf to}~{\bf the}~{\bf surgical}~{\bf site}$ with an interior hook and eye placket for added security
- 4. Soft fabric flap at bottom of zipper protects sensitive skin from chafing
- 5. Seamless cup pockets can hold a post-surgical form securely in place

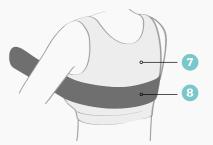

- 6. Targeted stabilization zones assist with optimal lymphatic flow around the torso
- 7. High quality, gentle fabric feels soft, light, and comfortable on sensitive skin
- 8. Belt accessory (sold separately, pg. 21) additionally stabilizes the breast area, attaching to the hook and loop fastener flap in the back of the garment to help keep it in

IMPORTANT NOTE! Due to the seamless circular knitting technology it is not necessary to open the Leyla ZIP-ST-LOW, Sina HE-ST-MEDIUM and Pamela ZIP-ST-HIGH and try on if the size really fits. Thanks to this circular knitting technology, all of them adapts perfectly to the contours of the individual upper body. They are prepared for direct clinical use (OP-table) - individually securely packed in a peel bag. Opening destroys the protective atmosphere, therefore no return possible (or similar).

FEATURES

No pockets

Hight cotton

Seamless

technology

Bilateral

pockets

SARAH SOFT BRA

 \triangle

Post-surgical bra with light pressure level

Indication: after various breast surgeries

0778 White 0779 Black 75-105 A, B, C, D, E Sizes 75 A Hooks 7 rows:

80-85 A; 75-80 B; 75 C 8 rows: 9 rows: 90-95 A; 85-90 B; 80-85 C; 75-80 D; 75 E 100-105 A; 95-100 B; 90-95 C; 85-90 D; 80-85 E 10 rows:

105 B; 100-105 C; 95-100 D; 90-95 E 11 rows: 12 rows: 105 D; 100-105 E 62% cotton, 24% polyamide, 14% elastane

- · Double layer at the front with a high cotton percentage
- Moulded cups and flat seams throughout reduce skin irritation
- · Soft, elasticated underbust band that does not roll up
- Wide, adjustable bra straps with narrow Velcro fasteners at the front
- The Amoena compression belt can be securely attached to the bra using the fleece band on the back. Fleece band can be folded up if not required. Compression belt sold separately, see page 21
- In combination with CuraScar Silicones for supportive scar treatment





HIGH PRESSURE LEVEL

PAMELA ZIP-ST-HIGH

Style

Post-surgical bra with high pressure level

White 45010 45009 Rose Nude Black 45011 70/75; 80/85; 90/95; 100/105; 110/115; 120/125

Sizes: Cups: Universal

Material:

98% polyamide, 2% elastane Indication: after various breast surgeries

- · Precise fit and comfortable to wear thanks to the seamless circular
- Suitable for direct clinical use due to a pre-washing process
- Immobilisation and stabilisation of the operated breast (special knitted zones on the underside of the cup)
- Special knitted zones and a high pressure level support lymphatic drainage and the healing process;
- Pre-formed and flexible seamless cups ensure optimal stabilisation of the operated breast with even pressure, even within the cup (no fixation ring around the cup)
- Adjustable, wide straps with Velcro fasteners for easy dressing and undressing also provide exceptional comfort
- Hooks and eyes make it easier to do the zip up on top at the front
- Soft fleece on the back makes it possible to securely position the compression belt. This is sold separately,
- · In combination with CuraScar Silicones for supportive scar treatment (see p. 29)
- Successor product for Patricia

MEDIUM

PRESSURE LEVEL

SINA HE-ST-MEDIUM

Post-surgical bra with medium pressure level

White 45007 Style 45006 Rose Nude Black 45008

70/75; 80/85; 90/95; 100/105; 110/115; 120/125 Sizes Universal

Cups:

Material: 98% polyamide, 2% elastane Indication: after various breast surgeries

- Precise fit and comfortable to wear thanks to the seamless circular knitting technology
- Suitable for direct clinical use due to a pre-washing process
- · Immobilisation and stabilisation of the operated breast (special knitted zones on the underside of the cup)
- Special knitted zones and a targeted moderate pressure level support lymphatic drainage and the healing process;
- Pre-formed and flexible seamless cups ensure optimal stabilisation of the operated breast with even pressure, even within the cup (no fixation ring around the cup)
- · Adjustable, wide straps with Velcro fasteners for easy dressing and undressing also provide exceptional comfort
- · Hooks and eyes at the front make closure easier
- Soft fleece on the back makes it possible to securely position the compression belt. This is sold separately (see page 21)
- In combination with CuraScar Silicones for supportive scar treatment (see p. 29)

Sizes Underbust band

	in cm
EU 65	63-67
EU 70	68-72
EU 75	73-77
EU 80	78-82
EU 85	83-87
EU 90	88-92
EU 95	93-97
EU 100	98-102
EU 105	103-107
EU 110	108-112
EU 115	113-117
EU 120	118-122
EU 125	123-127

COMPRESSION BELT

To be used with CuraSupport Compression

White 0776N Style Black 0789N Sizes 65-125 72% polyamide, 14% polyester, 8% cotton, 6% elastane

Indication: breast reconstruction with implant.

augmentation with implant

- Developed to help achieve optimal surgical results by keeping the operated breast in place and preventing implants from moving during the healing process
- · Anatomically shaped from firm, stretchy material to fit the upper breast area perfectly
- · Skin-friendly materials reduce irritation and the soft Velcro fastener at the front enables individual fitting
- Soft fleece at the centre back enables it to be attached securely to compression bras from the CuraSupport Compression product line. This allows the belt to be firmly attached to the bra. The fleece band can be folded up when not in use or when wearing the belt with another compression bra
- Can be used with any bra or bandage
- In combination with CuraSupport Compression and CuraScar Silicones for supportive scar treatment (see p. 17-21, 29-31)



The Amoena compression belt is positioned at the back so that it sits at the same height as a bra.



The belt sits over the cleavage at the front.



The Amoena compression belt is properly in place if the Amoena logo can be clearly seen on the upper right edge.

FEATURES

Bilateral pockets

No pockets

Hight cotton

ST Seamless technology

ZIP Zipper in front

BO Bonded

ΗE

VΕ Hook & eye in Velcro in front

Adjustable

AS

IN Easy step-in

CURASUPPORT COMFORT The CuraSupport Comfort bras and camisoles meet specific needs after the wound has healed, e.g. for high sensitivity or radiation burns. They are all very comfortable to wear and are exceptionally user-friendly.

ZIP

Zipper in front

Seamless

во

Bonded

VΕ

Hook & eye in Velcro in front

AS

Adjustable

IN

Easy step-in

FEATURES

No pockets

Hight cotton

Bilateral

CURASUPPORT COMFORT



EMILIA ZIP-ST-LOW

45013 45012



45014 Black 70/75; 80/85; 90/95; 100/105; 110/115; 120/125

Cups

Material 69% cotton, 25% polyamide, 6% elastane

White

Rose Nude

Indication after various breast surgeries

Post-surgical bra with light pressure level

- Precise fit and comfortable to wear thanks to the seamless circular knitting technology
- Adjustable straps behind the shoulders
- Integrated mesh area on the sides and in the underbust region for good ventilation and comfort
- Hooks and eyes make it easier to do the high-quality zip up on top at
- Soft pockets hold a Priform or Leisure Form securely in place



HANNAH SOFT BRA

Front closure post-surgical bra with drain management

Style 2160 White

XS(60/65); S(70/75); M(80/85); L(90/95); XL(100/105); Sizes

A/B; C/D

Cups

4 rows: XS(A/B); S(A/B); M(A/B); XS(C/D) 5 rows: L(A/B); XL(A/B); S(C/D); M(C/D); L(C/D); XL(C/D)

Material 82% cotton, 18% elastane

Indication: after various breast surgeries

- Soft cotton fabric offers soothing comfort to sensitive skin
- Removable drain pouch can attach to bottom band. Drain pouch sold separately, see pg. 25
- Cut and sewn cups with cotton pockets
- Padded, stretch fabric straps with front fasteners for easy access
- Lace on neckline assures a more feminine look
- Soft pockets keep a pri form or leisure form securely in place



MICHELLE CAMISOLE

 \triangle

With drain management

2105 White Style

XS(32/34); S(36/38); M(40/42); L(44/46); XL(48/50); 2XL(52/54)

82% cotton, 18% elastane

Indication: after various breast surgeries

• Ultra-soft cotton fabric offers a snug but nonrestrictive fit; smooth interior side seams offer comfort

- Neckline stretches wide for easy step-in entry
- Very soft piping on neckline and arms are absolutely gentle to the
- Inside band holds removable drain pouch securely in place. Drain pouch sold separately, see pg. 25
- Cotton pockets hold a Priform or Leisure Form securely in place
- * refers to clothing size

CURASUPPORT COMFORT



ESTER SOFT BRA

Sporty front closure post-surgical bra

Style 42576 White 42577 Black/Grey

70-100 A, B, C, D; 80-95 E Sizes 7 rows: 70-85 A; 70-80 B; 70-75 C; 70 D

8 rows: 90-100 A; 85-95 B; 80-90 C; 70-85 D; 75-80 E 10 rows: 100 B; 95-100 C; 90-100 D; 85-95 E

83% polyamide, 17% elastane

Indication: after various breast surgeries

- · Modern design with an athletic look
- Breathable synthetic fabrics; mesh on side and back helps with moisture evaporation
- · Moulded cups are smooth and seamless under clothing
- Breathable pocket fabric
- · Wide padded and adjustable straps



FRANCES SOFT BRA

Post surgical bra with soft cotton mix material

Art.-Nr. 2128 White 2128 Nude Black 2128

More colours are available see page 79. XS(60/65); S(70/75); M(80/85); L(90/95); Sizes

XL(100/105); 2XL(110/115) A/B; C/D; E/F Cups

4 rows: XS (A/B); S (A/B); M (A/B); XS (C/D)

5 rows: L (A/B); XL (A/B); 2XL (A/B); S (C/D); M (C/D); L (C/D); XL (C/D); 2XL (C/D); XS (E/F); S (E/F); M (E/F) 6 rows: L (E/F); XL (E/F)

7 rows: 2XL (E/F) Material 90% cotton, 10% elastane Indication: after various breast surgeries

- Front closure leisure bra
- · Ultra-soft cotton fabric offers soothing comfort to sensitive skin
- For the best support, we recommend wearing an Amoena Leisure Form with this bra



SANDRA SOFT BRA

Front closure post-surgical bra 0622 White Style

S(70 A); M(75 A/B); L(80 B); XL(85 B/C); 2XL(90 B/C); 3XL(95 C); 4XL(100 C/D) Sizes

Hooks 5 rows: S; M 6 rows: L 7 rows: XL; 2XL 8 rows: 3XL; 4XL

Material 50 % polyamide, 45 % cotton, 5 % elastane

Indication: after various breast surgeries

- All over feminine elastic lace
- · Adjustable hook and eye front closure offers a custom fit
- · Cotton pockets designed to hold fibrefill only; fibrefill included
- Adjustable straps



THERAPORT SOFT BRA

Front closure radiation therapy garment

White 2161 Style S: 65-75A; 65-70B; 65C Sizes

M: 80-90A; 75-85B; 70-80C; 65-75 D;

65-70E; 65F

L: 95–105A; 90–100B; 85–95C; 80–90D; 75–85E; 70–80F; 65–75G–H XL: 110-120A; 105-115B; 100-110C; 95-105D;

90-100E; 85-95F; 80-90G; 80-85H 2XL:125-130A; 120-130B; 115-125C; 110-120D; 105-115E; 100-110F; 95-105G; 90-100H

Material 81% cotton, 12% polyamide, 7% elastane Indication: after various breast surgeries

especially brachytherapy • Specially designed for wear immediately after any type of breast surgery or while undergoing radiation treatments, including partial

- breast irradiation (seed radiation and brachytherapy) • Seamless soft cotton garment features hook and loop fastener front
- closure and straps allowing for easy access to treatment areas • The bra shows grey knitted zones to cut it in and accommodate the
- Seamless wide bottom band fits gently around catheters and irradiation site and securely holds the Amoena Drain Pouch (sold separately) if needed
- Soft pockets hold a Priform or Leisure Form securely in place

CURASUPPORT ACCESSORIES



REMOVABLE DRAIN POUCH

Style

S (ca. 15x18 cm); L (ca. 20x20 cm) Sizes Material 95% cotton, 5% elastane Indication: after various breast surgeries

- Attaches to garment securely with the hook and loop fastener tab
- Standard shape and sizes for use with Jackson-Pratt®, Hemovac® and other drain management systems



Bilateral pockets

No pockets

Hight cotton technology

ST Seamless

ZIP Zipper in front

во Bonded

ΗE Hook & eye in Velcro in front

VΕ

AS Adjustable

IN

Easy step-in



CURASCAR GARMENTS



PANTY 2-ZIP-BO

Scar compression panty with approx. 20 - 23 mmHg

Art. no. 45000 Nude

45001 Black

Sizes: XS; S; M; L; XL; 2XL Material: 56% polyamide, 42% elastane, 2% cotton

Indication: - damage to the skin and to the healing function of the skin, with impaired scar formation

- after reconstructive surgery
 to improve scar/wound healing to the point of complete scar
- to stabilise the results of surgery
- Post-surgical scar compression garment
- High-waist cut
- Wide elastic and bonded waistband prevents wound from rubbing or fabric from rolling up
- Breathable material combination
- Material can be cut for drainage tubes
- Side zips for easy dressing and undressing
- Flat leg ending thanks to innovative ultrasonic finishing
- Oeko-Tex certified materials
- Free from latex, silicone and nickel
- In combination with CuraScar Silicones for supportive scar treatment (see p. 29)

BERMUDA 2-ZIP-BO

Long scar compression panty with approx. 20-23 mmHg Art. no. 45002 Nude

45003 Black

XS; S; M; L; XL; 2XL Sizes: Material: 56% polyamide, 42% elastane, 2% cotton

Indication: - damage to the skin and to the healing function of the skin, with impaired scar formation

- after reconstructive surgery - to improve scar/wound healing to the point

of complete scar healing

- to stabilise the results of surgery

• Post-surgical scar compression garment

High-waist cut

• Wide elastic and bonded waistband prevents wound from rubbing or fabric from rolling up

• 3/4 leg length supports lymphatic drainage

Breathable material combination

• Material can be cut for drainage tubes

· Side zips for easy dressing and undressing

• Flat leg ending thanks to innovative ultrasonic finishing

• Oeko-Tex certified materials

• Free from latex, silicone and nickel

• In combination with CuraScar Silicones for supportive scar treatment (see p. 29)

Tips on how to put on Panty
-ZIP-BO and Bermuda 2-ZIP-BO
pen both the zips and the hooks
and eyes underneath. Pull the scar compression garment over the legs. Close all hooks and eyes first to make it easier to next pul up the two zips.

Fitter



technology

pockets

26

ΗE VΕ Bonded Hook & eye in Velcro in front

Adjustable

AS

IN Easy step-in

CURASCAR GARMENTS



BELLY BANDAGE 3-VE

Belly bandage with approx. 20-23 mmHg

Art. no. 45004 Nude

Sizes:

45005 Black

Material:

S/M; L/X; 2XL/3XL Nude: 59% polyamide, 31% elastane, 8% cotton, 2% polyester Black: 33% polyamide, 31% elastane, 29% polyester,

7% cotton

Indication: - damage to the skin and to the healing function of the skin, with impaired scar formation
- after reconstructive surgery
- to improve scar/wound healing to the point of complete

scar healing
- to stabilise the results of surgery

• Post-surgical scar compression garment

• Flexible fit through adjustable Velcro fastener

 Belly bandage divided into three parts allowing for individual size adjustment

• Height of the three Velcro fastener parts: 24 cm

• Specially selected material combination guarantees a perfect fit

Oeko-Tex certified materials

• Free from latex, silicone and nickel

• In combination with CuraScar Silicones for supportive scar treatment (see p. 29)

Fitter

Tips on how to put on
Belly bandage 3-VE
Start by closing the middle
Velcro strip of the 3-part Velcro
fastener. Secure each Velcro strip
correctly in the soft Velcro area.
Depending on the size, this can be done
as follows:

as follows:
Size S at the far ridge –
size M at the centre ridge
Size L at the far ridge –
size XL at the centre ridge
Size 2XL at the far ridge –
size 3XL at the centre ridge
All 3-part Velcro fasteners should
be adjusted so that the level of
compression is comfortable
to wear.



CURASCAR SILICONES

Silicone is an important component when treating scars from injuries or surgery. It ensures a moist environment so the scar does not dry out, and thereby has a positive effect on the healing process.

FEATURES

Bilateral pockets

 \triangle No pockets Hight cotton

ST Seamless technology

ZIP

Zipper in front

во Bonded

VΕ Hook & eye in Velcro in front

Adjustable

IN

AS Easy step-in

CURASCAR SILICONES



STRIPS

Silicone scar patch for scar treatment

010 Clear Art. no. Dimensions: 260 x 35 mm

Material Polydimethylsiloxane, polyurethane film, polyethylene protective film, polyethylene

cover strips

Indication:

Prevention and treatment of impaired scars

Treatment of red and raised scars

• After surgical procedures (e.g. after reconstructive or aesthetic surgery)

Pack size: 4 pcs

Breathable PU film

Pulling aid

UV stable

· Very high elasticity

• Easy to clean, e.g. with Amoena Soft Cleanser

(sold separately see p. 62)

Additional cover strips for better storage after cleaning

Reusable

• Individual form and size adjustment possible (can be cut)

• In combination with CuraScar Garments and Curasupport Compression for supportive scar treatment (see p. 17-21, 26-28)



MAMMILLA CIRCLES

Silicone scar patch for scar treatment

012 Clear Art. no.

Outer diameter 75 mm, inner diameter 20 mm Dimensions:

Material: Polydimethylsiloxane, polyurethane film, polyethylene

protective film, polyethylene cover strips

Indication:

• Prevention and treatment of impaired scars

Treatment of red and raised scars

• After surgical procedures (e.g. after reconstructive or aesthetic

surgery)

Pack size: 2 pcs

Breathable PU film

 Pulling aid UV stable

Very high elasticity

• Easy to clean, e.g. with Amoena Soft Cleanser

(sold separately see p. 62)

• Additional cover strips for better storage after cleaning

Reusable

• Individual form and size adjustment possible (can be cut)

• In combination with CuraScar Garments and Curasupport Compression for supportive scar treatment (see p. 17-21, 26-28)



SQUARES

Silicone scar patch for scar treatment

011 Clear Art. no. Dimensions: 260 x 100 mm

Material: Polydimethylsiloxane, polyurethane film,

polyethylene protective film, polyethylene cover

Indication:

• Prevention and treatment of impaired scars

• Treatment of red and raised scars

• After surgical procedures (e.g. after reconstructive or aesthetic surgery)

Pack size: 2 pcs

• Breathable PU film

Pulling aid

UV stable

Very high elasticity

• Easy to clean, e.g. with Amoena Soft Cleanser

(sold separately see p. 62)

• Additional cover strips for better storage after cleaning

Reusable

• Individual form and size adjustment possible (can be cut)

• In combination with CuraScar Garments and Curasupport Compression for supportive scar treatment (see p. 17-21, 26-28)



ANCHORS

Silicone scar patch for scar treatment

013 Clear Art. no. Dimensions: 230 x 145 mm

Material: Polydimethylsiloxane, polyurethane film, polyethylene

protective film, polyethylene cover strips

Indication:

Prevention and treatment of impaired scars

Treatment of red and raised scars

• After surgical procedures (e.g. after reconstructive or aesthetic surgery)

Pack size: 2 pcs

Breathable PU film

• Pulling aid

UV stable

Very high elasticity

• Easy to clean, e.g. with Amoena Soft Cleanser (sold separately see p. 62)

• Additional cover strips for better storage after cleaning

Reusable

• Individual form and size adjustment possible (can be cut)

• In combination with CuraScar Garments and Curasupport Compression for supportive scar treatment (see p. 17-21, 26-28)



CURASCAR SILICONES SEE HOW EASY IT IS TO USE Watch application videos about CuraScar Silicones. Simply scan the QR code!



CuraScar Silicones in four different styles can be perfectly combined with CuraSupport Compression or CuraScar Garments to optimally treat a scar after breast surgery. You can find details on page 17 & 26

BREAST FORMS & BREAST SHAPERS

Our comprehensive range of breast forms and shapers offer shapes to fit current surgical trends, combined with innovative product features, providing **the best solution for all your customers' needs.**





BREAST FORMS

For optimal care after a mastectomy

In approximately 20-30% of breast cancer cases, the whole breast has to be completely removed, and not all women decide to have reconstruction after a mastectomy. Our full forms are an ideal alternative for these women; together with our special textiles, they create an even silhouette and physical symmetry. Equipped with innovative technologies, they can help to alleviate symptoms of secondary diseases such as lymphoedema and return to the used lifestyle.

Amoena breast forms are ideal following:

- a simple mastectomy
- a radical mastectomy (Halsted mastectomy)
- a modified radical mastectomy

BREAST SHAPERS

For the perfect balance in case of breast asymmetry

Today, 70-80% of all breast cancer cases are treated with a breast-conserving therapy. The tumour is removed in such a way as to preserve as much natural breast tissue as possible. The asymmetry resulting from this can be visually balanced with our partial shapers – they restore a balanced silhouette and ensure the bra fits correctly.

Amoena beast shapers are ideal following:

- breast-conserving therapy (BCT)
- multiple biopsies
- a reconstruction (partial recovery, unfavourable results)
- · a reconstruction with previous expander therapy
- natural breast asymmetry, breast malformation or degeneration



THE AMOENA SOLUTION

Our Breast Care Apparel (starting on page 65) together with our Breast Forms and Shapers create the perfect solution for comprehensive care after breast surgery.

THE AMOENA QUALITY PROMISE





MORE THAN 45 YEARS OF EXPERIENCE IN BREAST CARE

Amoena, the developer of the film-coated silicone breast forms, offers full forms and partial shapers for a natural aesthetic and a secure fit. With more than 45 years of experience and the most stringent quality standards, we guarantee the highest product quality according to DIN ISO 9001, 14001 and 13485 for medical devices. All films and silicone gels have been tested for skin compatibility and correspond to the current state of "biocompatibility" according to DIN ISO 10993.

- Made in Germany
- Highest quality from the technology leader in the field of breast forms
- Over 17 million forms sold worldwide since 1975
- Over 15,000 retailers worldwide trust our products
- Certified according to DIN ISO 9001, 14001 and 13485 for medical devices




Headquarters | Raubling, Germany

TECHNOLOGY LEADER

Constant innovation and setting new standards are driving forces at Amoena. In close cooperation with doctors, medical retailers and women who have had breast surgery, we continuously develop new technologies to optimally respond to patient needs after breast surgery.

OUR TECHNOLOGIES

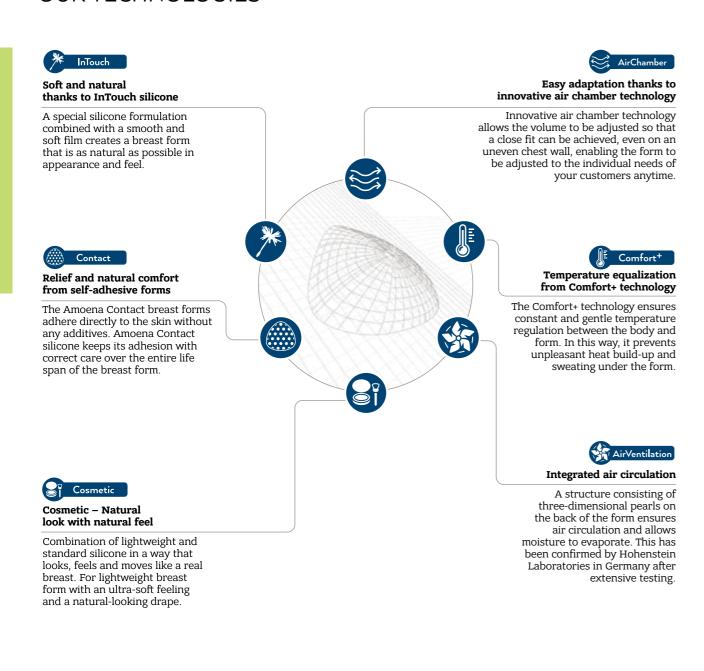



reddot winner 2020

Our most innovative air chamber technology was awarded with the Red Dot Award 2020. The Adapt Air product line won in the category Product Design. The concept, which is designed to provide the highest possible wearing comfort and a natural appearance, was especially emphasized. Discover our adjustable breast forms on page 40 and breast shapers on page 55.



Our Adadpt Air breast forms have been awarded with the German Design Award 2021. With the »Gold« distinction for Excellent Product Design in the category "Medical, Rehabilitation and Health Care", the jury lauded that "The Adapt Air breast prostheses can be perfectly adjusted and simulate the female breast with astonishing realism. A solution that helps the women affected to live a normal everyday life, free of stigmatisation."

BREAST FORMS

Our breast forms not only provide visual and weight balance but also meet needs resulting from treatment and medications such as lymphoedema, uneven or sensitive scar areas. Special product features enable the woman to continue her accustomed lifestyle. Thanks to more than 45 years of experience, we know exactly what women need after breast surgery and how we can make the fitters' daily lives easier.

FITTING AFTER MASTECTOMY

Especially suitable for women with:	• PRODUCT LINE	Full weight balance*	Weight-reduced breast forms**
 Uneven scar area Weight fluctuations Swelling due to lymphoedema 	ADAPT Adjustable to her unique silhouette		Adapt Air Light -25% Adapt Air Xtra Light -40%
Tension in the shoulder/neckBig breastsLymphoedema	CONTACT Adheres directly to the body	Contact	Contact Light -25%
 Hot flushes due to menopause or medication Heavy sweating due to climate or sporting activity 	Optimal temperature equalization for maximum comfort	Energy	Energy Cosmetic -20% Energy Light -25%
 Sensitive scar area (feeling of tension, radiation damage) 	NATURA The most natural wearing comfort	Natura	Natura Cosmetic -20% Natura Light -25% Natura Xtra Light -40%
 Simple optical weight compensation after mastectomy 	ESSENTIAL For a balanced and secure fit	Essential	Essential Light -25%

 $^{{\}rm *Weight\ corresponds\ to\ approx.\ 80\text{--}90\%\ of\ the\ natural\ breast,\ comes\ closest\ to\ the\ natural\ movement\ behaviour\ of\ the\ breast}$

^{**} Weight reduction between 20% - 40% especially suitable for: Lymphoedema, osteoporosis, tension in shoulder/neck, large chest, limited arm mobility, sporting activity

A portfolio of choices for the most personalised fit

How to find the right breast form

Before selecting a breast form, first measure the woman for well-fitting bra, then select the breast form with the appropriate features that best fit her specific needs. Consider her lifestyle, activity level and any physical aspects that will help you determine the best breast form for her. To determine the appropriate bra and breast form size please refer to the breast form sizing chart in the back of this catalog.


		Shallow Cup Form	>	2 Average Cup Form			Full Cup Form		
	Product Line	Weight	Symmetrical Shape 1S 1S ^N	Symmetrical Shape 2S 2S ^N	Asymmetrical Shape 2A	Universal Shape	Symmetrical Shape 3S	Asymmetrical Shape 3A	Extra Shape 3E
		Xtra Light - 40%	Adapt Air Xtra Light 1S ^N 328 Sizes 1-14	Adapt Air Xtra Light 2S ^N 326 Sizes 1-14					
PREMIUM	ADAPT	Light - 25%	Adapt Air Light 1S ^N 329 Sizes 1-14	Adapt Air Light 2S ^N 327 Sizes 1-14				Adapt Light 3A 376 Sizes 4-12 L/R	
CARE Comfort+		Light - 25%		Contact Light 2S 380 Sizes 1-14			Contact Light 3S 385 Sizes 0-10		
	CONTACT	Standard	Contact 1S 384 Sizes 1-14	Contact 2S 381 Sizes 1-12	Contact 2A 383 Sizes 0-12 L/R		Contact 3S 382 Sizes 0-10 Tawny 1-10		Contact 3E 386 Sizes 1-14 L/R
	ENERGY	Light - 25%		Energy Light 2S 342 Sizes 1-14 Tawny 1-14		Energy Light 2U 341 Sizes 1-14			
		Reduced - 20%		Energy Cosmetic 2S 310 Sizes 1-14			Energy Cosmetic 3S 311 Sizes 1-12		
		Standard	Energy 1S 349 Sizes 1-12						
		Xtra Light - 40%	Natura Xtra Light 1S ^N 401 Sizes 1-14	Natura Xtra Light 2S ^N 400 Sizes 1-17 Tawny 1-17					
ADVANCED CARE Comfort*		Light - 25%	Natura Light 1S ^N 402 Sizes 1-14 Natura Light 1S 664 Sizes 1-14	Natura Light 2S 390 Sizes 1-17 Tawny 1-17	Natura Light 2A 392 Sizes 0-14 L/R		Natura Light 3S 391 Sizes 0-12	Natura Light 3A 373 Sizes 4-12 L/R Tawny 4-12 L/R	
		Reduced - 20%		Natura Cosmetic 2S ^N 323 Sizes 1-14 Natura Cosmetic 2S 320 Sizes 1-14			Natura Cosmetic 3S 321 Sizes 1-12		Natura Cosmetic 3E 322 Sizes 1-14 L/R
		Standard	Natura 1S 396 Sizes 1-14 Tawny 4-12	Natura 2S 398 Sizes 1-10	Natura 2A 393 Sizes 0-12 L/R		Natura 3S 395 Sizes 0-12		Natura 3E 397 Sizes 1-14 L/R
BASIC CARE	ESSENTIAL	Light - 20%	Essential Light 1S ^N 314 Sizes 1-14 Tawny 1-14	Essential Light 2S 442 Sizes 1-17 Tawny 1-17	Essential Light 2A 356 Sizes 0-14 L/R		Essential Light 3S 367 Sizes 0-12		Essential Light 3E 556 Sizes 1-14 L/R Tawny 4-14 L/R
BASIC CARE	ESSENTIAL	Standard	Essential 1S 630 Sizes 1-12	Essential 2S 440 Sizes 1-12	Essential 2A 353 Sizes 0-12 L/R		Essential 3S 363 Sizes 0-12		


CUP FORMS


In order to find the right breas form for every woman, Amoen

- 1 = shallow
- 2 = averag
- 3 = ful

A shallow cup form does not automatically mean a small breast. In profile, the top of the breasts can be shallow, but the volume of the cup is in the lower third. This can also occur, for example, with increasing age.

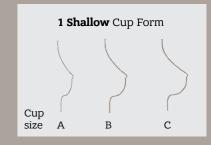

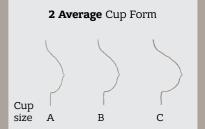

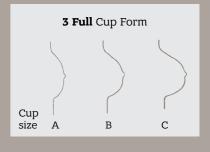


Cup forms are exemplarily shown for sizes A-C but are also available in sizes D-G

Fitter **TIP**

The best way
to judge the cup form is
in a well-fitting bra.
Women with a small underbust
and to have a full cup form, while
women with a large underbust
usually have a shallow cup form

Adapt Air 3-layer breast forms

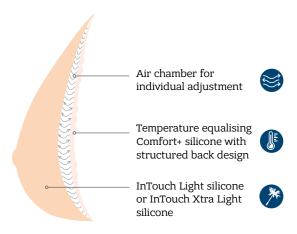

FEATURES & BENEFITS

• Individually adjustable

Integrated air chamber technology allows the breast form size to be adjusted by simply adding or releasing air anytime on the go or at home.

. Close to the body

Adapt Air is comfortably customisable to every chest wall for a close fit and perfect skin contact.

Visual symmetry

Adapt Air can be adapted to every woman's unique shape or bra style for a symmetrical silhouette that's perfect for a full and active life.

• Lightweight

InTouch light silicone makes the breast forms 25% to 40% lighter.* $\,$

• Pleasant skin sensation

The temperature equalising Comfort+ technology offers allday comfort to maintain a woman's ideal body temperature.


Especially suitable for women with:

- Uneven scar area
- Weight fluctuations
- · Swelling due to lymphoedema

Reduced weight options

As wearing reduced weight breast forms relieves pressure on her shoulders they are particularly suitable for women with lymphoedema, osteoporosis, tension in the shoulder and neck areas, bigger breasts, active lifestyle

- 40% - 25%

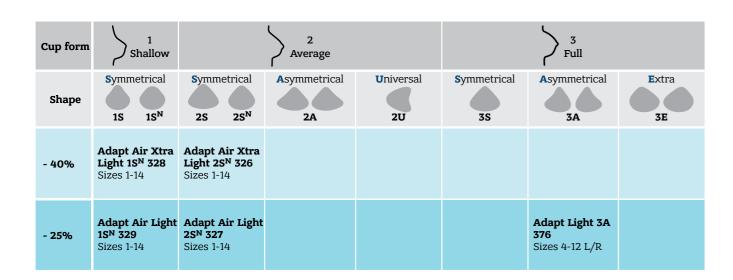



FITTERS FIRST CHOICE

"It is such a simple thing with such a great effect. I can offer even more bras to my customer and simply fine-tune the fit."

Fitter response - Norway



BEST CUSTOMER OPTION

"It provides a close and secure fit.

If needed I can even adjust the volume at home very easliy. Really amazing."

Susann, satisfied Adapt Air user



reddot winner 2020

The Adapt Air product line was awarded the Red Dot Award 2020 in the category Product Design. The concept, which is designed to provide the highest possible wearing comfort and a natural appearance, was especially emphasized.



Our Adadpt Air breast forms have been awarded with the German Design Award 2021. With the "Gold" distinction for Excellent Product Design in the category "Medical, Rehabilitation and Health Care", the jury lauded that "The Adapt Air breast prostheses can be perfectly adjusted and simulate the female breast with astonishing realism. A solution that helps the women affected to live a normal everyday life, free of stigmatisation."

AMOENA ADAPT PORTFOLIO



Adapt Air Xtra Light 1SN 328*

Size	1-14
Cup form	Shallow
Shape	Symmetrical
Layers	3 layers
Weight	-40%

Special features:

 Modern 1S^N shape with rounded footprint and most volume in the lower part for a natural drape. Shorter height and thinner design provides a smooth transition from breast form to chest wall.

Adapt Air Xtra Light 2SN 326*

Size	1-14
Cup form	Average
Shape	Symmetrical
Layers	3 layers
Weight	-40%

Special features:

 Modern 2S^N shape with rounded, narrow footprint that fits more comfortable into the bra pocket and avoids overlapping with the natural breast.

Adapt Air Light 1SN 329*

Size	1-14
Cup form	Shallov
Shape	Symmetrica
Layers	3 layers
Weight	-25%

Special features:

 Modern 1S^N shape with rounded footprint and most volume in the lower part for a natural drape. Shorter height and thinner design provides a smooth transition from breast form to chest wall.





Adapt Air Light 2SN 327*

Size	1-14
Cup form	Average
Shape	Symmetrical
Layers	3 layers
Weight	-25%

Special features:

 Modern 2S^N shape with rounded, narrow footprint that fits more comfortable into the bra pocket and avoids overlapping with the natural breast.

Adapt Light 3A 376**

Sizes	4-12 L/R
Cup form	Full
Shape	Asymmetrical
Layers	3 layers
Weight	-25%

Special features:

- Asymmetrical shape equalizes missing tissue under the arm (especially recommended after radical surgery like Rotter-Halsted)
- Moldable layer on the back adapts to uneven chest wall


Adapt Air Pump 070

**Without integrated air chamber.


Replacement Pump
Please note: A pump is included with every
Adapt Air breast form.
Pump is also sold separately.

HOW TO USE THE ADAPT AIR BREAST FORM


Once the air is adjusted, it will remain locked-in until you need to increase or reduce air as needed.

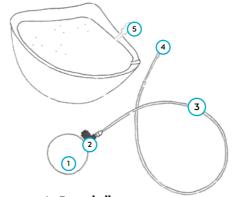


- 1. Pump ball
- Drain valve
- 3. **Hose**
- 4. Cannula
- 5. Valve opening

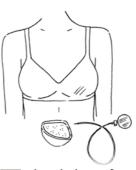

Place the breast form and the pumping device in front of you. Breast form and pump is separated.

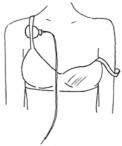


Place the pump under the bra strap of the opposite side, so that both hands are free. Remove the bra strap of the operated side.

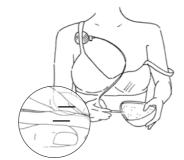


Connect the pump by putting the cannula into the marked opening on the breast form. Open the valve inlet by gently squeezing the thumb and index finger together. Make sure you insert the cannula **completely** into the valve.



Insert the breast form into the bra cup. Make sure that the tube is properly connected with the breast form.

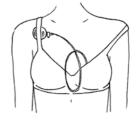

Bring the bra strap back in the correct position.

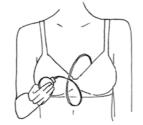

Adjust the breast form by squeezing and releasing the pump ball. For releasing air press the button of the black drain valve and gently press against the breast form.



Once the adjustment is done remove the cannula carefully from the valve. The air will stay in the air chamber.



Now the breast form is individually adjusted.


Take it out of the cup and put it into the pocket of the bra.


Please note:

If you want to add or remove air, the cannula must be fully inserted into the valve.

Adheres directly to the body

Contact 3-layer breast forms

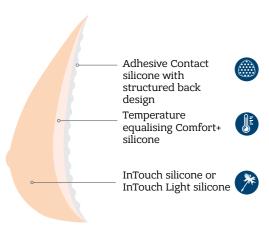

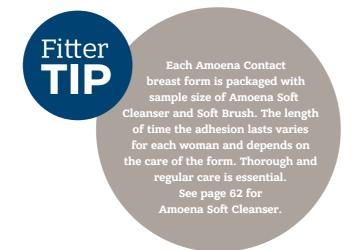

FEATURES & BENEFITS

· Adheres directly to the skin and moves with her body, staying in place with every movement

helpful if she has lymphoedema or a larger bust

- Relieves pressure from the shoulder Wearing the breast form attached relieves pressure on her shoulders and provides a sense of lighter weight - especially
- Easy wear and care

Daily cleaning and care is easy and ensures best adhesion. Provides more freedom in clothing choices like those with lower necklines

· Feels like a part of the body

Because the form is attached, it can feel like a part of her and she may feel more secure and self-confident in her appearance

Especially suitable for women with*:

- Tension in the shoulder/neck
- Big breasts
- Lymphoedema

Reduced weight options

As wearing reduced weight breast forms relieves pressure on her shoulders they are particularly suitable for women with lymphoedema, osteoporosis, tension in the shoulder and neck areas, bigger breasts, active lifestyle

- 25%

Cup form) 1 Shallow		2 Average			> 3 Full	
Shape	Symmetrical 1S 1S ^N	Symmetrical 2S 2S ^N	Asymmetrical 2A	Universal 2U	Symmetrical 3S	Asymmetrical 3A	Extra 3E
- 25%		Contact Light 2S 380 Sizes 1-14			Contact Light 3S 385 Sizes 0-10		
	Contact 1S 384 Sizes 1-14	Contact 2S 381 Sizes 1-12	Contact 2A 383 Sizes 0-12 L/R		Contact 3S 382 Sizes 0-10 Tawny 1-10		Contact 3E 386 Sizes 1-14 L/R

AMOENA CONTACT PORTFOLIO





Contact Light 2S 380*

Sizes 1-14 Cup form Average Shape Symmetrical Layers 3 layers Weight -25%

Contact Light 3S 385*

Sizes 0-10 Full Cup form Shape Symmetrical Layers 3 layers -25% Weight





Contact 1S 384

Sizes 1-14 Cup form Shallow Shape Symmetrical 3 layers Layers Weight Standard

Contact 2S 381

Sizes 1-12 Average Cup form Shape Symmetrical 3 layers Layers Weight Standard







Contact 2A 383

0-12 L/R Sizes Average Cup form Shape Asymmetrical Layers 3 layers Weight Standard

Special features:

- · asymmetrical shape equalizes missing tissue under the arm (especially recommended after radical surgery like Rotter-Halsted)
- also recommended for women with a big underbust
- Size 0 available for small bust size

Contact 3S 382

Sizes	0-10
Tawny sizes	1-10
Cup form	Full
Shape	Symmetrical
Layers	3 layers
Weight	Standard

Special features:

• Size 0 in Ivory available for small bust size

C	on	ta	ct	3	E	3	8	6

Sizes	1-14 L/R
Cup form	Full
Shape	Extra
Layers	3 layers
Weight	Standard

Special features:

- Extension towards the sternum allows a lower neckline
- Recommended especially for women with a full cup and small underbust


HOW TO USE THE AMOENA CONTACT

APPLICATION

Cleanse your chest wall thoroughly. Afterwards, make sure the area is completely dry. The cleaner and drier your skin, the better the breast form will adhere. Please do not touch the adhesive surface of the breast form as this may affect



Slide your bra strap down over vour shoulder and slip your arm out. Lean forward.

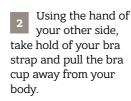

Take the breast Take the preasure form in the palm of your hand (same side as breast form) and, still leaning forward, place it in the cup of your bra. Make sure that the breast form sits perfectly in the cup of your bra.



Place your free hand on the outside of the bra cup. Slowly stand upright, pressing the breast form in the cup attached. carefully into place on

Pull your bra strap up over your shoulder and press the breast form once more to make sure it is firmly

Check the fit. If the position of the breast form needs to be adjusted, simply remove the breast form and reposition.

REMOVAL



Grasp the breast form with one hand and lean forward until you feel the weight of the breast form in your hand.

The fingertips

as close as possible to

the edge of the breast


form.

should be placed

your body.

furthest from your scar area, pull the breast form carefully away from your body.

Please be careful with longer fingernails – they might damage the film.

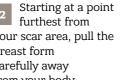

If the adhesive surface accidentally sticks together during removal, it can be easily separated by holding under running water.

CLEANING



Put three drops of the Soft Cleanser on the dry adhesive surface and then moisten the Soft Brush with lukewarm water.



Now, scrub the adhesive surface with the Soft Brush firmly for some time in circular movements using pressure.



Then rinse with plenty of lukewarm water




Finally allow to air-dry.


ADVANCED CARE

Optimal temperature equalization for maximum comfort

Energy 3-layer breast forms

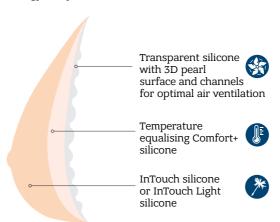

Energy Cosmetic 3-layer breast forms

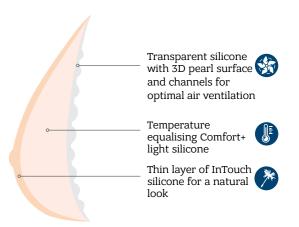

FEATURES & BENEFITS

· Optimal air flow

Unique three-dimensional pearl surface on the back of the breast form allows air to flow freely behind the form increasing air ventilation and moisture evaporation

· Maximum of temperature equalization 30% more of temperature equalising comfort+ silicone reduces perspiration for ultimate comfort all day

Especially suitable for women with:

- · Heavy perspiration (hot flushes) due to menopause or medication
- · Heavy sweating due to climate or sporting activity

Reduced weight options

As wearing reduced weight breast forms relieves pressure on her shoulders they are particularly suitable for women with lymphoedema, osteoporosis, tension in the shoulder and neck areas, bigger breasts, active lifestyle

- 25% - 20%

3 2 Cup form Shallow Full Average **S**ymmetrical **S**ymmetrical **A**symmetrical **U**niversal **S**ymmetrical **A**symmetrical Extra Shape 1SN 2U 1S **2S** 2S^N 2A 3E Energy Light 2U 341 **Energy Light** 2S 342 - 25% Sizes 1-14 Sizes 1-14 Tawny 1-14 Energy Cosmetic 2S Cosmetic 3S - 20% 310 311 Sizes 1-14 Sizes 1-12 Energy 1S Sizes 1-12

Transparent soft

skin comfort

silicone for gentle

The most natural wearing comfort



Energy Light 2S 342*

1-14 Sizes 1-14 Tawny sizes Average Cup form Symmetrical Shape Layers 3 layers Weight -25%

Energy Light 2U 341*

Special features:

Sizes	1-14
Cup form	Average
Shape	Universa
Layers	3 layers
Weight	-25%

• Alternative to the asymmetrical style with more flexible wearing options and shorter extension towards the axillar. Offers possibility to equalize missing tissue even in the upper part of the cup.

Energy Cosmetic 2S 310

Sizes	1-14
Cup form	Average
Shape	Symmetrical
Layers	3 layers
Weight	-20%

Special features:

• The most natural looking lightweight breast form with air ventilation - soft and almost invisible in the bra cup.





Energy Cosmetic 3S 311


Sizes	1-12
Cup form	Full
Shape	Symmetrical
Layers	3 layers
Weight	-20%

Energy 1S 349

Sizes	1-12
Cup form	Shallow
Shape	Symmetrical
Layers	3 layers
Weight	Standard

Special features:

 The most natural looking lightweight breast form with air ventilation - soft and almost invisible in the bra cup.

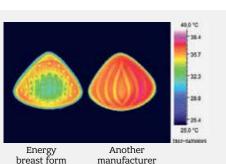

After 30 minutes of physical activity, Amoena Energy optimally equalizes body temperature

The thermal imaging camera shows that the Amoena Energy prosthesis stays cooler due to the higher percentage of comfort+ silicone and the unique three-dimensional pearl surface. The special arrangement of the pearls has been proven to transport 25 times more heat away from the body than other breast forms.

Natura 2-layer breast forms Natura 3-layer breast forms Natura Cosmetic 3-layer breast forms Temperature Temperature Temperature equalising Comfort+ Light equalising equalising Comfort+ Comfort+ silicone silicone silicone Thin layer of InTouch Light or InTouch InTouch Xtra InTouch Light or InTouch silicone for a natural look silicone silicone

Transparent soft silicone for gentle

FEATURES & BENEFITS

· Gentle to sensitive chest wall

Most styles feature a pressure relieving back layer. The soft and clear silicone gel hugs the body and is extra gentle to sensitive scar areas

· Natural look and movement

Selected styles have a very soft cup side so that the form drapes like a natural breast, moves naturally with the body and flattens when lying down

· Pleasant skin sensation

The temperature equalising Comfort+ technology offers all day comfort to maintain a woman's ideal body temperature

Especially suitable for women with:

- · Sensitive scar areas
- Very soft breast tissue

Reduced weight options

As wearing reduced weight breast forms relieves pressure on her shoulders they are particularly suitable for women with lymphoedema, osteoporosis, tension in the shoulder and neck areas, bigger breasts, active lifestyle

- 40% - 25%

- 20%

Cup form) 1 Shallow	(2 Average			3 Full	
Shape	Symmetrical 1S 1S ^N	Symmetrical 2S 2S ^N	Asymmetrical 2A	U niversal 2U	Symmetrical 3S	Asymmetrical 3A	Extra 3E
- 40%	Natura Xtra Light 1S ^N 401 Sizes 1-14	Natura Xtra Light 25 ^N 400 Sizes 1-17 Tawny 1-17					
- 25%	Natura Light 15 ^N 402 Sizes 1-14 Natura Light 15 664 Sizes 1-14		Natura Light 2A 392 Sizes 0-14 L/R		Natura Light 3S 391 Sizes 0-12	Natura Light 3A 373 Sizes 4-12 L/R Tawny 4-12 L/R	
- 20%		Natura Cosmetic 2S ^N 323 Sizes 1-14 Natura Cosmetic 2S 320 Sizes 1-14			Natura Cosmetic 3S 321 Sizes 1-12		Natura Cosmetic 3E 322 Sizes 1-14 L/R
	Natura 1S 396 Sizes 1-14 Tawny 4-12	Natura 2S 398 Sizes 1-10	Natura 2A 393 Sizes 0-12 L/R		Natura 3S 395 Sizes 0-12		Natura 3E 397 Sizes 1-14 L/R

*Light and Xtra Light breast forms are more muted in colour.

Available in Tawny colour



Sizes	1-14
Cup form	Shallow
Shape	Symmetrical
Layers	2 layers
Weight	- 40%

Special features:

- Modern 1S^N shape with rounded footprint and most volume in the lower part for a natural drape.
- Shorter height and thinner design provides a smooth transition from breast form to chest wall.
- Due to a stronger cavity and narrow channels on the back, the prosthesis fits closely to the chest wall



Natura Xtra Light 2SN 400*

Sizes	1-17
Tawny sizes	1-17
Cup form	Average
Shape	Symmetrical
Layers	2 layers
Weight	- 40%

Special features:

- Modern 2S^N shape with rounded, narrow footprint that fits more comfortable into the bra pocket and avoids overlapping with the natural breast.
- Due to a stronger cavity and narrow channels on the back, the prosthesis fits closely to the chest wall
- Up to size 17 for bigger bust size



Natura Light 1SN 402*

Sizes	1-14
Cup form	Shallow
Shape	Symmetrica
Layers	3 layers
Weight	- 25%

Special features:

- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas.
- Modern 1S^N shape with rounded footprint and most volume in the lower part for a natural drape.
- Shorter height and thinner design provides a smooth transition from breast form to chest wall.



Natura Light 3S 391*

Sizes	0-1
Cup form	Ful
Shape	Symmetrica
Layers	3 layer
Weight	- 25%

Special features:

- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas
- Size 0 available for small bust size



Natura Light 3A 373*

Sizes	4-12 L/R
Tawny sizes	4-12 L/R
Cup form	Full
Shape	Asymmetrical
Layers	3 layers
Weight	- 25%

Special features:

- asymmetrical shape equalizes missing tissue under the arm (especially recommended after radical surgery like Rotter-Halsted)
- also recommended for women with a big underbust
- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas



Natura Cosmetic 2S^N 323

Sizes	1-14
Cup form	Average
Shape	Symmetrical
Layers	3 layers
Weight	- 20%

Special features:

- The most natural looking lightweight breast form - soft and almost invisible in the bra cup.
- Modern 2S^N shape with rounded, narrow footprint that fits more comfortable into the bra pocket and avoids overlapping with the natural breast.
- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas



Natura Light 1S 664*

1-14
Shallow
Symmetrical
3 layers
- 25%

Special features:

 Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas



Natura Light 2S 390*

Sizes	1-17
Tawny sizes	1-17
Cup form	Average
Shape	Symmetrical
Layers	3 layers
Weight	- 25%

Special features:

- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas
- Up to size 17 for bigger bust size



Natura Light 2A 392*

Sizes	0-14 L/R
Cup form	Average
Shape	Asymmetrical
Layers	3 layers
Weight	- 25%

Special features:

- Asymmetrical shape equalizes missing tissue under the arm (especially recommended after radical surgery like Rotter-Halsted)
- Also recommended for women with a big underbust
- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas
- Size 0 available for small bust size



Natura Cosmetic 2S 320

Sizes	1-14
Cup form	Average
Shape	Symmetrical
Layers	3 layers
Weight	- 20%

Special features:

- The most natural looking lightweight breast form - soft and almost invisible in the bra cup.
- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas



Natura Cosmetic 3S 321

Sizes	1-12
Cup form	Full
Shape	Symmetrical
Layers	3 layers
Weight	- 20%

Special features:

- The most natural looking lightweight breast form - soft and almost invisible in the bra cup.
- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas



Natura Cosmetic 3E 322

Sizes	1-14 L/R
Cup form	Full
Shape	Extra
Layers	3 layers
Weight	- 20%

Special features:

- Extension towards the sternum allows a lower neckline
- Recommended especially for women with a full cup and small underbust
- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar areas

*Light and Xtra Light breast forms are more muted in colour.

Available in Tawny colour

*Light and Xtra Light breast forms are more muted in colour.

Available in Tawny colour

1-14

4-12

Shallow

2 layers

Standard

Symmetrical

• 2 layer breast form with soft cup side for

• especially suitable for women with very

Natura 1S 396

Special features:

a very natural drape

soft breast tissue

Sizes Tawny sizes

Cup form

Shape

Layers Weight

Natura 2S 398

Sizes	1-10
Cup form	Average
Shape	Symmetrical
Layers	3 layers
Weight	Standard

Special features:

• Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar



Natura 2A 393

Sizes	0-12 L/1
Cup form	Averag
Shape	Asymmetrica
Layers	3 layer
Weight	Standard

Special features:

- asymmetrical shape equalizes missing tissue under the arm (especially recommended after radical surgery like Rotter-Halsted)
- also recommended for women with a big underbust
- · Soft, clear silicone back layer hugs the body and is extra gentle to sensitive scar
- Size 0 available for small bust size





Natura 3S 395

Sizes	0-12
Cup form	Full
Shape	Symmetrical
Layers	3 layers
Weight	Standard

Special features:

- Soft, clear silicone back layer hugs the body and is extra gentle to sensitive
- Size O available for small bust size



Natura 3E 397

L/R
Full
extra
yers
dard


Special features:

- Extension towards the sternum allows a lower neckline
- Recommended especially for women with a full cup and small underbust
- · Suitable for women with very soft breast

AMOENA ESSENTIAL

For a balanced and secure fit

Essential 1-layer breast forms

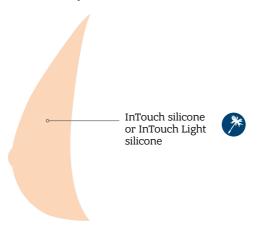

FEATURES & BENEFITS

- · For essential everyday needs
- Designed to meet her essential everyday needs following breast surgery with a high quality breast form
- Symmetrical appearance
 - Gives her the comfort she needs for a natural symmetrical appearance

Especially suitable for women with:


• Simple weight compensation for symmetrical appearance after a mastectomy

Reduced weight options

As wearing reduced weight breast forms relieves pressure on her shoulders they are particularly suitable for women with lymphoedema, osteoporosis, tension in the shoulder and neck areas, bigger breasts, active lifestyle

Cup form) 1 Shallow	2 Average		> 3 Full			
Shape	Symmetrical 1S 1S ^N	Symmetrical 2S 2S ^N	Asymmetrical 2A	Universal 2U	Symmetrical 3S	Asymmetrical 3A	Extra 3E
- 25%	Essential Light 15 ^N 314 Sizes 1-14 Tawny 1-14	Essential Light 2S 442 Sizes 1-17 Tawny 1-17	Essential Light 2A 356 Sizes O-14 L/R		Essential Light 3S 367 Sizes 0-12		Essential Light 3E 556 Sizes 1-14 L/R Tawny 4-14 L/R
	Essential 1S 630 Sizes 1-12	Essential 2S 440 Sizes 1-12	Essential 2A 353 Sizes 0-12 L/R		Essential 3S 363 Sizes 0-12		

*Light and Xtra Light breast forms are more muted in colour.

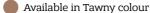



Essential Light 1SN 314*

Modern 1S^N shape with rounded

footprint and most volume in the lower

part for a natural drape. Shorter height and thinner design provides a smooth transition from breast form to chest

Sizes

Tawny sizes

Special features:

Cup form

Shape

Layers Weight

wall.

1-14

1-14 Shallow

1 layer

-25%

Symmetrical

		10

|--|

2556116161 216116 25	
Sizes	1-17
Tawny sizes	1-17
Cup form	Average
Shape	Symmetrical
Layers	1 layer
Weight	-25%



• Up to size 17 for bigger bust size



Essential Light 2A 356*

Sizes	0-14 L/R
Cup Shape	Average
Shape	Asymmetrical
Layers	1 layer
Weight	-25%

Special features:

- Asymmetrical shape equalizes missing tissue under the arm (especially recommended after radical surgery like Rotter-Halsted)
- Also recommended for women with a big underbust
- Size 0 available for small bust size





Essential 2S 440

Sizes	1-12
Cup form	Average
Shape	Symmetrical
Layers	1 layer
Weight	Standard

Essential 2A 353

Sizes	0-12 L/R
Cup form	Average
Shape	Asymmetrical
Layers	1 layer
Weight	Standard

Special features:

- Asymmetrical shape equalizes missing tissue under the arm (especially recommended after radical surgery like Rotter-Halsted)
- Also recommended for women with a big underbust
- Size 0 available for small bust size

Essential 3S 363

Sizes	0-12
Cup form	Full
Shape	Symmetrical
Layers	1 layer
Weight	Standard

Special features:

Size 0 available for small bust size



Essential Light 3S 367*

Sizes	0-12
Cup form	Full
Shape	Symmetrical
Layers	1 layer
Weight	-25%

Special features:

• Size 0 available for small bust size



Essential Light 3E 556*

Sizes	1-14 L/R
Tawny sizes	4-14 L/R
Cup form	Full
Shape	Extra
Layers	1 layer
Weight	-25%

Special features:

- Extension towards the sternum allows a lower neckline
- Recommended especially for women with a full cup and small underbust



Essential 1S 630

Sizes	1-12
Cup form	Shallow
Shape	Symmetrical
Layers	1 layer
Weight	Standard

*Light and Xtra Light breast forms are more muted in colour.

Available in Tawny colour



Premium Priform

Art. no.. 216

Sizes 1/2, 3/4, 5/6, 7/8, 9/10,

11/12, 13/14 Material

Front: 100% polyester Back: 100% cotton

- Temporary breast form designed for use immediately after breast surgery and during times of leisure
- Moulded cup design offers a soft, seamless shape in the pocket of an Amoena post–surgical bra or camisole
- Filled with lightweight fibrefill which can be added or removed to adjust volume to desired size
- Breathable cotton back layer is gentle to the skin
- Ideal solution to restore silhouette during healing and before she is ready for a permanent breast form
- Comes with additional fibrefill

Leisure Form

Art. no. 132N Sizes 3-14 Material

Weighted Foam:

oam: 30% polyurethane, 70% barium sulfate (7727-43-7) 82% polyamide,

18% elastane

Cover:

- Polyurethane foam form is slightly weighted with Barium Sulfate (7727-43-7), a naturally occurring mineral in powder form
- Ultra-soft removable, washable fabric cover helps maintain a clean environment
- · Ideal to wear while sleeping

This form is not a substitute for a weighted silicone breast form, which is designed for everyday use and provides your body with proper balance.

Aqua Wave Swimform

Art. no. 149 Sizes 1-14

Material PU-film, silicone

- A transparent silicone form for water activities, with a unique natural shape
- Symmetrical form features raised waves on back that allow air and water to flow through, reducing the amount of suction
- Offers a shallow to average fit
- Packaged with a fashionable waterproof drawstring bag that is large enough to hold a wet swimsuit





Standard Priform

Art. no. 214
Sizes 1/2; 3/4; 5/6; 7/8; 9/10; 11/12; 13/14
Material Front 100% polyester
Back 100% cotton


- Temporary breast form designed for use immediately after breast surgery and during times of leisure
- Filled with lightweight fibrefill which can be added or removed to adjust volume to desired size
- Breathable cotton back layer is gentle to the skin
- Ideal solution to restore silhouette during healing and before she is ready for a permanent breast form

BREAST SHAPERS - RESTORING SYMMETRY

Today, 70-80%* of all breast cancer cases are treated with a breast-conserving therapy. The tumour is removed in such a way as to preserve as much natural breast tissue as possible.

Women often experience breast asymmetry after completing their breast cancer treatment and recovery process. It is uncomfortable – physically and emotionally. A silicone breast shaper can be a great solution for restoring symmetry and rebuilding confidence.

Our high-quality silicone breast shapers comes in various shapes and thicknesses to find the perfect fit for every woman's needs. And with our new Balance Adapt Air shapers we now make shaper fitting even easier for you – with an integrated air chamber to simply individually adjust the volume as needed. Find out more on page 59

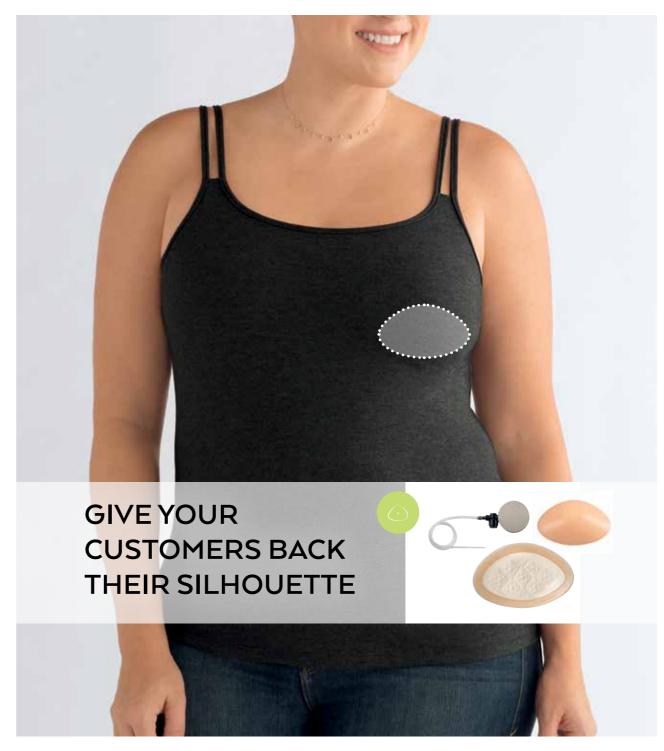

^{*} Sources: https://www.ncbi.nlm.nih.gov/pmc/articles/PMC4504664/

BREAST SHAPERS

Amoena's breast shapers are a solution for women who experience breast asymmetry after a lumpectomy, reconstruction, multiple biopsies and even naturally.

- Our new portfolio has more shapes, styles, weight options and thicknesses
- Most styles feature a back layer with integrated Comfort+ temperature equalising material to reduce heat build-up behind the shaper
- Designed with tapered edges to match curves
- The shapers fill the bra cup where there is missing or uneven tissue, providing a smooth silhouette
- A special fitting method (CCD) has been developed to help you find the best thickness for your costumers' needs. You will find it on pg. 114

MULTIPLE THICKNESS VARIANTS

A wide portfolio of full shapers available in three thickness variants.

HUGS THE BREAST

The delicate lines on the full shapers allow a flexible, snug fit adapted to the natural curves of the breast.

NATURAL FEEL

High-quality silicone provides an exceptionally natural fit and feeling.



COMFORT+ TECHNOLOGY

Our patented Comfort+ technology absorbs, stores and releases body heat to help regulate and balance fluctuations in body temperature for ultimate comfort all day.

A SHAPE FOR EVERY BREAST

A wide variety of shapes allows every woman to benefit from a true fit.

SMOOTH TRANSITION

Thin, natural-looking edges ensure a smooth transition to the chest.

BREAST SHAPER PORTFOLIO

The ultimate range of breast shapers

	Thickness	Th	in	Medium	Volume	Spe	ecial
	Cup difference	< 1	cm	1 – 2 cm	≥ 2 cm		
	SHAPE	Oval	Delta	Delta	Delta	Ellipse	Varia
PREMIUM	BALANCE ADAPT Comfort*			Balance Adapt Air 233 Sizes 2–12	r MD		Balance Adapt Air SV 234 Sizes S-XL
PREI	BALANCE CONTACT Comfort*			Balance Contact MD 229 Sizes 2–12	Balance Contact VD 230 Sizes 3/4-9/10		Balance Contact SV 286 Sizes S-XL
ADVANCED	BALANCE				Balance Natura Light VD 221 Sizes 3/4–11/12		
ADVA	NATURA Comfort [†]	Balance Natura TO 227 Sizes S-XL	Balance Natura TD 217 Sizes 1–10	Balance Natura MD 220 Sizes 2–12	Balance Natura VD 222 Sizes 3/4-9/10	Balance Natura SE 231 Sizes S-XL	Balance Natura SV 285 Sizes S-XL
BASIC	BALANCE ESSENTIAL Balance Essential T 228 Sizes S-XL				Balance Essential Light VD 224 Sizes 3/4–11/12 Tawny All Sizes		
		Essential TO 228	Balance Essential TD 218 Sizes 1–10	Balance Essential MD 223 Sizes 2–12 Tawny 4–10	Balance Essential VD 225 Sizes 3/4-9/10	Balance Essential SE 232 Sizes S-XL	

4 DIFFERENT PRODUCT LINES FOR DIFFERENT NEEDS

Balance Adapt

Individually adjustable thanks to an integrated air chamber technology for a very close fit to the chest wall; with our temperature equalising Comfort+ technology

Especially suitable for: uneven scar area, weight fluctuations, swelling due to lymphoedema

• Balance Contact

Adheres directly to the skin and features Comfort+, our patented temperature equalising technology Especially suitable for: lymphoedema

• Balance Natura

For the most natural wearing comfort; with our temperature equalising Comfort+ technology **Especially suitable for:** sensitive scar area

• Balance Essential

A simple way to restore symmetry

Especially suitable for: simple compensation of missing breast tissue

4 DIFFERENT SHAPES FOR EVERY TYPE OF BREAST ASYMMETRY

• Delta & Oval

Full shapers with rounded anatomical design in three thicknesses

• Ellipse & Varia

Partial shapers designed to cover or fill in a section of the breast

A portfolio of choices to help you solve any and all fitting challenges:

FITTING CHALLENGE One breast is slightly smaller. One breast is Missing tissue on part Missing or uneven tissue significantly smaller. of the lower breast on part of the upper or side breast area. FITTING SOLUTION OVAL DELTA **ELLIPSE** Use Oval to accommodate Use Delta as the go-to Use Ellipse at the lower Use Varia at the top or side of part of the breast to the breast - a versatile shape small differences in cup shape to compensate for volume – a double-purpose that can be positioned for missing volume across replace volume and push shape that can be positioned the entire breast area. up the entire breast. many different fitting options. either horizontally or vertically. Available in Thin thickness Available in three Available in four sizes Available in four sizes - for a delicate fit. thicknesses (Thin. - for just the right - for a natural silhouette Medium, Volume) amount of push-up and across the entire breast. and four sizes lift to create a natural (corresponding to bra silhouette. sizes) - to solve cupdifference challenges.



Our Balance Premium Fit Kit (see also on page 61) with 15 shapers in different sizes and thicknesses makes it easy to find the right shaper and helps to reduce your inventory.

A special fitting method (CCD) has been developed to help you find the best thickness for your costumer's needs. You will find it on page 114

AMOENA BALANCE ADAPT AIR

Individually adjustable with air



FEATURES & BENEFITS

• Individually adjustable

Integrated air chamber technology allows to adjust the breast shapers volume by simply adding or releasing air anytime

· Close to the body

with a flexible rear side Balance Adapt Air is comfortably customisable to every chest wall for a close fit and perfect skin contact

Lightweight

due to the air chamber for a perfect wearing comfort

Visual symmetry


Balance Adapt Air can be adapted to every woman's unique shape or bra style for a symmetrical silhouette that's perfect for a full and active life

Comfortable skin sensation

The temperature-equalising Comfort+ technology in combination with a structured back layer offers optimal ventilation and helps to maintain a woman's ideal body temperature

Especially suitable for women with:

- Uneven scar area
- Weight fluctuations
- Swelling due to lymphoedema

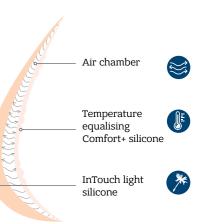



Balance Adapt Air MD 233

Sizes	2-12
Thickness	Medium
Shape	Delta
Layers	3 layers

Special features:

- Basic thickness (without air) equals a Medium Delta (1-2 cm) but thanks to Adapt Air technology volume can be increased to equal a Volume Delta shape, multiple volume options
- Optimal solution also during breast reconstruction surgery using a tissue expander thanks to air chamber technology the volume of the breast shaper can be adjusted as needed during therapy



Balance Adapt Air SV 234

Sizes	S-XL
Thickness	Special
Shape	Varia
Layers	3 layers

Special features:

- Designed to cover or fill a section of the breast
- Can be rotated to fill in where it's needed most
- · Volume can be adjusted as needed



reddot winner 2020

The Adapt Air product line was awarded the Red Dot Award 2020 in the category Product Design. The concept, which is designed to provide the highest possible wearing comfort and a natural appearance, was especially emphasized.



Our Adadpt Air product line has been awarded with the German Design Award 2021. With the "Gold" distinction for Excellent Product Design in the category "Medical, Rehabilitation and Health Care", the jury lauded that "The Adapt Air breast prostheses can be perfectly adjusted and simulate the female breast with astonishing realism. A solution that helps the women affected to live a normal everyday life, free of stigmatisation."

 $^*\mbox{Light}$ and Xtra Light breast forms are more muted in colour.

Balance Contact MD 229

Sizes 2-12 Thickness Medium Delta Layers 2 layers

Balance Contact VD 230

3/4-9/10 Sizes Thickness Volume Shape Delta 2 layers Layers

Balance Contact SV 286

S-XL Sizes Special Thickness Shape Varia 2 layers Layers







Balance Natura Light VD 221

Sizes 3/4-11/12 Thickness Shape Delta Layers 2 layers

Balance Natura TO 227

S-XL Sizes Thickness Thin Shape Oval Layers 2 layers

Balance Natura TD 217

1-10 Sizes Thickness Thin Delta Shape Layers 2 layers







Balance Natura MD 220

Sizes 2-12 Thickness Medium Delta Shape Layers 2 layers

Balance Natura SE 231

S-XL Sizes Thickness Special Ellipse Shape 2 layers Layers

Balance Natura SV 285

S-XL Sizes Thickness Special Varia Shape 2 layers Layers

Balance Natura VD 222

3/4-9/10 Sizes Thickness Volume Shape Delta 2 layers

AMOENA BALANCE ESSENTIAL







Balance Essential Light VD 224*

3/4-11/12 Sizes Tawny Sizes 3/4-11/12 Thickness Volume Shape Delta Layers 1 layer Weight -2.5%

Balance Essential TO 228

Sizes S-XL Thickness Thin Shape Oval 1 layer Layers Weight -25%

Balance Essential TD 218

Sizes Thickness Thin Shape Delta Layers 1 layer







Balance Essential MD 223

Sizes 2-12 Tawny Sizes 4-10 Thickness Medium Shape Delta 1 layer

Balance Essential VD 225

Sizes 3/4-9/10 Thickness Volume Shape Delta Layers 1 layer



Balance Essential SE 232

Sizes S-XL Thickness Special Shape Ellipse 1 layer Layers





BALANCE PREMIUM FIT KIT

Our Premium Fit Kit with 15 shapers in different sizes and thicknesses makes it easy to find the right shaper and helps to reduce your inventory.

PREMIUM FIT KIT 245

15 partials in different sizes and a sizing chart

Balance Natura TO - 227 Size: M, L, Size: 3, 5 Balance Natura TD - 217 Balance Natura MD - 220 Size: 4, 8 Balance Natura SE - 231 Size: M, L Balance Natura SV - 285 Size: S Balance Natura Light VD 221 Size: 5/6, 9/10 Balance Contact VD - 230 Size: 7/8 Balance Contact SV - 286 Size: L Balance Adapt Air MD - 233 Size: 6


Balance Adapt Air SV - 234 Size: M

Available in Tawny colour

BREAST FORM & SHAPER ACCESSORIES





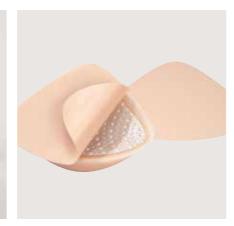

Amoena Soft Cleanser & Soft Brush

Art. No 087 Size 150 ml

- Amoena Soft Cleanser is specifically formulated for the Contact adhesive surface (Contact breast form and shaper products as well as the adhesive nipples) and contains no oil-based substances
- It is also suitable for cleaning nonadhesive breast forms
- For the best adhesion, use the Amoena Soft Cleanser with the Amoena Soft Brush.

Art. No 089 / 1 piece

Amoena Soft Brush is specially designed for Amoena Contact products and has very soft rounded bristles which will not harm the adhesive surface.

Adhesive Nipples

Package 2 per case

Art. no. 136 Ivory
137 Almond (shown)
138 Bronze
139 Tawny
134 Demo Kit
Sizes Small (Ø 3,4 cm)
Medium (Ø 3,9 cm)
Large (Ø 4,9 cm)

- Adheres securely to a breast form, partial breast forms, natural breast or reconstructed breast with a silicone adherite.
- Packaged with one 30 ml bottle of Amoena Soft Cleanser for easy cleaning
- Demo Kit displays one of each colour in each size for use in effectively determining size and colour needed (for retailers only)

Contact Back Pad and Foils

Art. no. 172 for styles 380, 381 (2S) 171 L/R for style 386 (3E) 170 L/R for style 383 (2A) 167 for style 384 (1S) 168 for styles 382, 385 (3S) 286 for style 286 (SV) 173 for styles 229 (BMD), 230 (BVD)

Package 1 Back Pad 2 Foils per bag

- Back Pad allows Contact breast form to be worn unattached and inside a bra pocket
- Foils protect the adhesive back during storage
- Available in breast forms sizes according to shape; see list in style information





Adapt Air Pump

Art. no. 070

• Replacement Pump for Adapt Air breast form and shapers

Breast form Cover

Art. no. 160 Sizes 1-17

Package 1 breast form cover

- Breast form cover protects the breast form
- Gentle fabric with no seams in the front
- Opening on the rear side
- Can be washed, 30°C handwash
- Suitable for breast form shape 2S and 3S



BREAST CARE APPAREL

Our special textiles suit every lifestyle and are the perfect complement to our breast forms. Innovative materials and skilled craftsmanship combine function and style with a perfect fit. In addition to maximum comfort, all products have integrated invisible pockets on both sides. As well as lingerie, for everyday wear or achieving a feminine look, the Amoena range also includes colourful swimwear for every figure, active clothing for sport and exercise, and comfortable, .feel-good styles for lounging and relaxation









SPECIAL FEATURES OF AMOENA BRAS

Our special bras are adapted to the individual needs of women who have undergone breast surgery and combine a feminine look with functionality and highest quality standards. All textiles are designed at the headquarters in Raubling/Germany and the fit is developed for the individual needs of breast-operated women. Our prosthetic bras go through a special manufacturing process and take on average 8-9 minutes longer to produce than bras without pockets. The Amoena Special Bras are perfectly fit to the Amoena Breast Prostheses and Shapers, offering breast-operated women a visually perfect result with maximum comfort.



Soft, comfortable, adjustable straps

- Support the prosthesis and relieve the shoulders and
- Are individually adaptable and become wider with increasing size
- Some styles have wider and padded straps for extra comfort and support

Pockets invisible on both sides

- For right and/or left care
- Pockets fit perfectly with Amoena breast prostheses/ shapers and keep them securely in place
- · High-quality pocket materials with various functions are pleasant on the skin and ensure maximum wearing comfort

Perfectly fitting cups

- Fit perfectly with our Amoena breast prostheses/ shapers and hide unevenness
- Elastic trim at the décolleté provides extra security
- Additional fabric layers for reinforcement offer best fit and support for increasing sizes

Supporting anatomically shaped under bust band

- · Offers stability and increases wearing comfort
- Most styles have a wider under bust band for a secure fit of the breast prosthesis/shapers
- Are characterized by high softness

Higher cut side panels

- Guarantee the support of our Amoena breast prostheses/shapers
- Have a soft finish and ensure optimum arm mobility
- · Possible scars and unevenness are well covered

Soft adjustable closure

- Different closure heights depending on bra size, at least 2 hooks
- Lightly padded hook and eye fasteners increase the wearing comfort
- For movement restrictions in the shoulder and arm area, we offer styles with front closure to allow easy dressing and undressing

YARNS AND MATERIALS

When designing our textile collection, we particularly focus on using natural yarns and materials. For example, organic cotton production, reduction in water consumption and the recycling of fabrics like nylon, are all topics that our design teams look at as part of their daily work. The protection of our environment is a goal that not only serves our planet, but also results in textile collections of the highest performance and quality standards.



ORGANIC COTTON

No agrochemical products such as artificial fertilisers, pesticides, herbicides or fungicides are used in the cultivation of organic cotton. Despite the reduced crop yield that usually results from this, there are clear benefits, such as environmental protection and reduced health risks for plantation workers.



Worldwide, around 50% of all textile fibres for clothing are



Organic cotton from ecological



A reduction of 91% in water consumption



Controlled organic cultivation without the use of chemical or synthetic fertilisers or plant protection products





TENCEL[™] is produced from wood sourced from controlled,



It is produced using an innovative closed-loop process

TENCEL



Lyocell is actually a man-made fibre. However, unlike synthetic fibres (such as polyester), lyocell is a cellulose fibre. These fibres are derived from plant cellulose, released from the cell walls by means of a physical process. Lyocell comes mainly from eucalyptus wood and the Austrian company Lenzign AG markets in under the brand name TENCEL™ Lyocell.







TENCEL™ fibres are particularly soft



TENCEL[™] fibres contribute to fabric breathability





Bamboo is an extremely rapidly renewable raw material



Environmentally friendly material due to lower water consumption, rapid renewal and cultivation without pesticides

BAMBOO

Bamboo is a rapidly renewable raw material that, as well as being a fibre source, is used in the production of food, building materials, consumer goods or musical instruments. It is usually cultivated on plantations without the use of any pesticides. Bamboo fibres come from the stem of the plant. Their extraction is a time-consuming process, as the outer layer is very hard (so hard that the stalk is even used for sharpening knives). In textile fibre manufacturing, bamboo is used to produce viscose, for example, just like Tencel.



Environmental protection: Bamboo is rapidly renewable and has a multitude of uses



Cultivation without pesticides

OEKOTEX



Standard 100 by Oeko-Tex is one of the world's best-known labels for textiles tested for harmful substances. The $\mbox{\rm Oeko-Tex}$ testing institutes use it for the globally standardised certification of the safety of textile products in terms of human health. This involves subjecting not only the textiles themselves, but also upstream products and all the components of the product to thorough testing. Products that pass this test are permitted to bear the label "Standard 100 by Oeko-Tex - Tested for harmful substances". The certification is valid for one year. We only use materials certified by Oeko-Tex in our production process.

DARWIN polyester Material



Darwin by Carvico: P.B.T. fabric made of 100% polyester, and therefore totally chlorine proof, it ensures a perfect fit over time and boasts unprecedented techno-performances. It is UV protective (UPF 50+), breathable and fast drying. Darwin is ideal for competition swimwear and for water sports, even the most extreme ones.



The material has an excellent **UV** protection



Two-way elastic for a perfect fit and a supersoft touch



Breathable for a more comfortable wearing



Chlorine-resistant for exceptional durability

SYNTHETIC YARNS AND MATERIALS



ECONYL® Regenerated Nylon



ECONYL® fibre is a true innovation: Fishing nets from aquaculture industries and ghost-nets are among the elements recycled during its production. During a comprehensive regeneration process, this nylon waste is returned to its original condition. It is identical to standard nylon yarn, but can be recycled and reconfigured infinitely for the resource-friendly production of new products.



ECONYL® regenerated nvlon helps closing the loop



Waste from the world's oceans, textiles and floor coverings are recycled during the production of ECONYL® regenerated nylon



Environmentally friendly material, as new yarn is produced from waste



ECONYL® can be recycled infinitely

VITA





Breathable fabric for your well-being experience



Quick dry COOLMAX® technology is permanent and transports moisture away from the body

COOLMAX® Specialty polyester



Fabrics made with COOLMAX® fibre transport moisture away from the body to keep the wearer cool and dry, helping to optimize performance. Due to the special cross section of the fibre, the moisture management and air permeability benefits are built in (are permanent) and don't wash off. It is an easy-care fabric with excellent wash - and colourfastness. www.COOLMAX.com



Synthetic fabric high consistancy of fabric performance



Permanent performance




The material has an excellent UV protection up to 50+



Two-way elastic for a perfect fit and a supersoft touch

VITA Fabric made from ECONYL® Polyamid and LYCRA® XTRA LIFE™ elastane

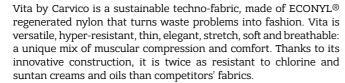



Produced with ECONYL® regenerated nylon that can be recycled infinitely





Chlorine-resistant for a more



Excellent resistance to damaging effects of sunscreen lotion / oil and heat

LYCRA® XTRA LIFE™ Elastic Fibre

One of the brand names for elastic fibres is LYCRA®, a registered trademark. Invented



in 1958 it revolutionized the apparel world with impressive properties no natural fibre boasts. It is distinctive for its quality and exceptional elasticity and recovery benefits. In addition to the above, The LYCRA® XTRA LIFE™ subbrand stands for long lasting fit and helps make your swimsuit look like new for longer. It resists degradation from chlorine, heat and suntan lotions up to 10 x longer than unprotected elastane.

www.LYCRA.com



Lightweight synthetic fibre. High consistency of performance



Long lasting performance, looks like new for longer



Cups

AMOENA LACE BRAS



AMANDA SOFT BRA BUSTIER

Art. no. 44537 White Sizes 75-95 A, B, C, D

Hooks 2 rows 75-85 A, 75-80 B, 75 C 3 rows 90-95 A, 85-95 B, 80-95 C, 75-95 D

Material Cup A, B, C: 78% polyamide, 19% elastane,

3% polyester

Cup D: 78% polyamide, 20% elastane, 2% polyester

- Modern design with a deeper neckline specially designed for women who wear partial breast shapers or full breast form in 3E shape
- Pockets made of breathable, temperature balancing microfibre fabric
- Comfortable, adjustable elastic straps
- Soft padded back closure
- Side bones offering additional side support



AMANDA WIRE BRA

 $\triangle = \bigoplus 12,4$

White Art. no. 44538 Sizes 75-110 A, B, C, D, E, F; 80-105 G Hooks

2 rows: 75-85 A; 75-80 B; 75 C 3 rows: 90-110 A; 85-110 B; 80-105 C;

75-100 D; 75-95 E; 75-90 F; 80-85 G

4 rows: 110 C; 105-110 D; 100-110 E; 95-110 F; 90-105 G Material Cup A, B, C: 73% polyamide, 20% elastane, 7% polyester

Cup D, E, F, G: 74% polyamide, 22% elastane, 4% polyester

- Big size range
- Features half-moon support on cup sides specially designed to support larger sizes
- Pockets made of breathable, temperature balancing microfibre fabric
- Comfortable, padded front straps & adjustable, elastic back straps
- Side bones offering additional side support



AMANDA PANTY

Art. no. White 44539

36-54 Sizes

Material 75% polyamide, 11% elastane, 8% cotton, 6% polyester

• Elegant Jacquard fabric in combination with lace inserts

• Leg line workmanship without elastics offers invisible wearing

POCKET PATTERN / FABRICS



2 Cut & Sewn

Microfibre

Cotton

6 COOLMAX®

Other

Tulle

71

Pockets



AURELIE SOFT BRA



Art. no. 44153 Off-White Sizes 70-100 A, B, C, D, E; 75-95 F Hooks 2 rows: 70-85 A; 70-80 B;

> 70-75 C; 70 D 3 rows: 90-100 A; 85-100 B;

> > 80-100 C; 75-100 D; 70-100 E; 75-95 F

Material Cup A, B, C: 67% polyamide, 25% elastane, 8% polyester Cup D, E, F: 68% polyamide, 26% elastane, 6% polyester

• Big size range

• Pockets made of breathable, temperature balancing microfibre fabric

Comfortable, adjustable elastic straps

Soft padded back closure

• Side bones offering additional side support



AURELIE WIRE BRA

Art. no.

▲ ∪ ⊕ 2,4

44154 Off-White

70-100 A, B, C, D, E; 75-95 F Sizes Hooks 2 rows: 70-85 A; 70-80 B; 70-75 C; 70 D

> 3 rows: 90-100 A; 85-100 B; 80-100 C; 75-100 D; 70-100 E; 75-95 F

Material Cup A, B, C: 67% polyamide, 24% elastane, 9% polyester

Cup D, E, F: 67% polyamide, 25% elastane, 8% polyester

• Big size range

• Pockets made of breathable, temperature balancing microfibre fabric

• Adjustable, elastic straps

Soft padded back closure



AURELIE PANTY

Art. no. 44156

Off-White

36-50 Sizes

Material 55% polyamide, 26% elastane,

11% cotton, 8% polyester

• Rich lace inserts at front and back mirror the lace in the Aurelie bras

Looks and feels luxurious

• Small crystal pendant at the center bow is trimmedin gold for special



FEATURES

Bilateral

























Moulded

2 Cut & Sewn

Tulle

Microfibre

5 Cotton

6 COOLMAX® Fabric



AMOENA LACE BRAS



CELINE SOFT BRA

Light Rose/Grey Art. no. 44509

75-95 A, B, C, D Sizes 2 rows: 75-95 A; 75-90 B; 75-85 C; Hooks

75-80 D

3 rows: 95 B; 90-95 C; 85-95 D

Material: 80% polyamide, 14% elastane, 6% polyester

- Center front is adjustable and offers two different wearing options: higher coverage to hide radiation burns or scars or lower center front for a deeper décolletté
- Pockets made of breathable, temperature balancing microfibre fabric
 Adjustable, elastic straps
- Soft padded back closure
- Side bones offering additional side support

CELINE PANTY

44512 Light Rose/Grey Art. no.

Sizes 36-46

Material 65% polyamide, 24% elastane, 11% cotton

Coordinates perfectly with the Celine braDelicate light lace with subtle two-tone effect

FEATURES

Bilateral



















Cut & Sewn

POCKET PATTERN / FABRICS

COOLMAX® Fabric



KAROLINA PADDED SOFT BRA

▲ ● () 2,4

White/Light Nude Art. no. 44506

Sizes 70-95 AA, A, B, C, D, E 2 rows: 70-95 AA, A; 70-90 B; 70-85 C; Hooks 70-80 D; 70-75 E

3 rows: 95 B; 90-95 C; 85-95 D; 80-95 E

Material 55% polyamide, 27% polyester, 14% elastane, 4% Viscose

- Pockets made of breathable, temperature balancing microfibre fabric
- Padded molded cups to hide uneveness, for example scars
- Comfortable, adjustable elastic straps
- Soft padded back closure



▲∪⊕ **|** 2,4

Art. no. 44507

White/Light Nude

Sizes 70-95 A, B, C, D, E

2 rows: 70-95 A; 70-90 B; 70-85 C; Hooks

70-80 D; 70-75 E

3 rows: 95 B; 90-95 C; 85-95 D; 80-95 E Material Cup A, B, C: 77% polyamide, 20% elastane, 3% Viscose

Cup D, E: 77% polyamide, 21% elastane, 2% Viscose

- Pockets made of breathable, temperature balancing microfibre fabric
- Comfortable, adjustable elastic straps
- Soft padded back closure
- Side bones offering additional side support



KAROLINA PANTY

Art. no. 44508

White/Light Nude 36-46

Material

Sizes

70% polyamide, 18% elastane, 10% cotton, 2% Viscose

• Panty completes the seductive look of Karolina

- Luxurious fan lace in combination with a shiny hide-and-seek stripe
- Gold pendant center detail matches Karolina bras



FEATURES

Bilateral



















Cut & Sewn

Tulle

POCKET PATTERN / FABRICS

Moulded

Microfibre

5 Cotton

6 COOLMAX® Fabric

Other



Cups

Cups

AMOENA CLASSIC BRAS







A/B; C/D; E/F 4 rows: XS(A/B, C/D); S(A/B); M(A/B) 5 rows: XS(E/F); S(C/D, E/F), M(C/D, E/F);

Nude

Blue/White

L(A/B, C/D); XL(A/B, C/D); 2XL(A/B, C/D)L(E/F); XL(E/F) 7 rows: 2XL(E/F)

Material 90% cotton, 10% elastane

Front closure bra

- Ultra soft cotton fabric is ideal for sensitive skin
- Holds a leisure form, breast form, or shaper securely in place
- Higher coverage of side and back part
- Wide underbust band
- For the best support, we recommend wearing an Amoena Leisure Form with this bra (see pg. 54)



ISADORA SOFT BRA 0947

0948

44114

▲ ● ①()2,5,7

▲ ● () () **2,5**

Sizes 75-115 A; 70-110 B, C, D; 70-100 E, F, G Hooks 2 rows: 75-90 A; 70-90 B, C; 70-85 D

White

Nude Black

95-115 A; 95-110 B; 95-105 C; 90-100 D;

70-95 E; 70-90 F; 70-85 G

4 rows: 110 C; 105-110 D; 100 E; 95-100 F; 90-100 G 44% polyamide, 29% polyester, 14% elastane, 13% cotton Material

Art. no.

Art. no.

- Features half-moon support on cup sides specially designed to support larger sizes
- Decorative embroidery on mesh at the top cup adds a light touch
- Ultra soft breathable cotton pockets are ideal for sensitive skin
- Wider padded front straps to relieve the shoulders
- Adjustable, elastic back straps
- Soft padded back closure



JASMIN SOFT BRA 0900N

▲ ● ① ② 2,5,7

44233 Black 70-90 AA, A, B; 75-90 C Sizes

2 rows: 70-90 AA, A, B; 75-90 C Hooks 68% cotton, 18% polyamide, 7% elastane, 7% polyester

White

- Made from a stretchy lace cotton which makes it light and comfortable
- Ultra soft breathable cotton pockets are ideal for sensitive skin
- Wider padded front straps to relieve the shoulders
- Adjustable, elastic back straps
- Soft padded back closure

POCKET PATTERN / FABRICS

Moulded

2

Cut & Sewn

Tulle

Microfibre

Cotton


6 COOLMAX®

Pockets

AMOENA CLASSIC BRAS





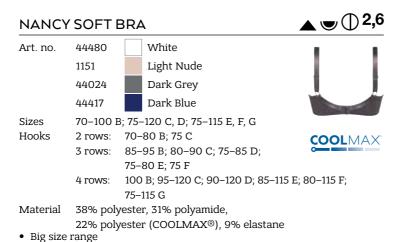



• Pockets are molded and made of breathable, temperature balancing

• Easy-fitting bra with floral lace along the neckline

COOLMAX® fabric




- Big size range
- Based on bestseller style Nancy with additional feature of front closure
- Easy-fitting bra with floral lace along the neckline
- Pockets are molded and made of breathable, temperature balancing COOLMAX® fabric

22% polyester (COOLMAX®), 9% elastane

- Adjustable, elastic straps
- · Additional soft padded back closure



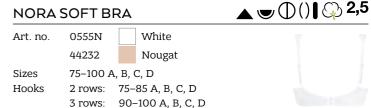


Delicate floral Jacquard weave with high cotton content

Material 48% cotton, 44% polyamide, 8% elastane

- Features half-moon support on cup sides specially designed to support larger sizes
- Ultra soft breathable cotton pockets are ideal for sensitive skin
- Wider padded front straps to relieve the shoulders
- Adjustable, elastic back straps
- Soft padded back closure
- Side bones offering additional side support



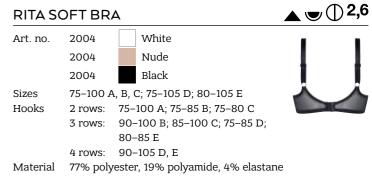

- Fabric has a light sheen and the stretch lace neckline hugs the body
- Pockets are molded and made of breathable, temperature balancing
- Adjustable, elastic straps
- Soft padded back closure



RUTH SOFT BRA

▲ ● ① ② 2,5 White 0873 Art. no. 0872 Nude 70-105 AA, A, B; 70-100 C; Sizes 70-95 D, E; 70-90 F Hooks 2 rows: 70-105 AA; 70-100 A; 70-95 B; 70-90 C; 70-85 D 3 rows: 105 A; 100-105 B; 95-100 C; 90-95 D; 70-95 E; 70-90 F Material 100% cotton

- Very high cotton Content of 100% (exclusive of straps and elastics) is ideal for sensitive skin
- Pockets are made of ultra soft breathable cotton which is ideal for sensitive skin
- · Adjustable, elastic straps
- Side bones offering additional side support

6

FEATURES

Bilateral















Moulded







High Cotton



Moulded

2 Cut & Sewn

Tulle

Microfibre

5 Cotton COOLMAX®

Other



AMY SOFT BRA

▲ ● 1,4

NEW

CLARA SOFT BRA FC ()1,7

Art. no. 44310 Rose Nude Black 44311

S, M, L Sizes

Material 72% polyamide, 28% elastane

- Bonding technology without any seams or closures for ultimate wearing comfort
- Whole bra made of breathable, temperature balancing microfibre fabric
- Higher coverage of side and back part
- Extra wide straps and underbust band
- Suitable for volume fluctations due to stretchy material
- Ideal to wear while sleeping



BARBARA PADDED WIRE BRA

Art. no. 0457

Sizes

Hooks

Nude

70-90 A, B, C; 70-85 D

3 rows: 70-90 A, B, C; 70-85 D Material 72% polyamide, 14% elastane,

14% polyester

- Strapless bra designed in a smooth bonded fabric for a flawless look
- Padded molded cups to hide uneveness, for example scars
- Versatile adjustable strap options: strapless, straight, crisscross or halterneck; packaged with additional clear straps for an invisible strap option
- Higher coverage of side and back part
- Pockets are molded and out of tulle material
- Soft padded back closure
- Side bones offering additional side support





BECKY SOFT BRA

▲ • • ○ ○ 1,5

44676 Art. no. 44677

Rose Nude

Off-White

S, M, L, XL, 2XL Sizes

Material 60% Organic cotton, 32% polyamide,

8% elastane

- High organic cotton content is ideal for sensitive skin
- Seamless knitted technology without any seams or closures for ultimate
- Higher coverage of side and back part
- Wide straps relieve the shoulders
- Wider underbust band
- Suitable for volume fluctations due to stretchy material
- Ideal to wear while sleeping

Rose Nude Art. no. 44743 Sizes S, M, L, XL Material 78% Bamboo. 17% polyamide, 5% elastane • Front closure bra • Bamboo fabric is ideal for sensitive skin due to comfortable handfeel, good breathability, moisture absorption • Ultra soft brushed fabric on the inside • Seamless knitted technology without closures for ultimate comfort • Higher coverage of side and back part Removable padded foam cups • Wider padded straps to relieve the shoulders Wide underbust band • Suitable for volume fluctations due to stretchy material • Ideal to wear while sleeping POCKET PATTERN / FABRICS

FEATURES

Bilateral























High Cotton

Moulded Cut & Sewn Tulle

Microfibre

Cotton

6 COOLMAX® Fabric

NEW ▲ ● ⑥ ⑤ 1,5 **KITTY TOP** Art. no. 44736 Rose Nude Sizes S, M, L, XL, 2XL Material 69% cotton, 25% polyamide, 6% elastane • Based on Kitty SB with adjustable elastic back • High cotton content is ideal for sensitive skin Seamless knitted technology without closures for ultimate comfort • Wider straps relieve the shoulders Wide underbust band • Suitable for volume fluctations due to stretchy material • Light cup padding can be removed if desired • Ideal to wear while sleeping **FEATURES**

Cut & Sewn Cups

Cut & Sewn Foam Cups

Moulded

Comfort

AMOENA SEAMLESS BRAS



KITTY SOFT BRA

Material



Rose Nude Art. no. 44607 44674 Magenta

Sizes S, M, L, XL, 2XL

69% cotton, 25% polyamide,

- 6% elastane • High cotton content is ideal for sensitive skin
- Seamless knitted technology without closures for ultimate comfort
- Higher coverage of side and back part
- Wide straps relieve the shoulders
- Wide underbust band
- Suitable for volume fluctations due to stretchy material
- Light cup padding can be removed if desired
- Ideal to wear while sleeping



LARA COTTON PADDED SOFT BRA A O 1,5

Art. no. 44673 Nude

Sizes 75-90 AA; 70-100 A, B, C, D, E 2 rows: 75-90 AA; 70-100 A, B, C, D, E Hooks Material 91% Organic cotton, 9% elastane

• Based on bestseller bra Lara SB

• High Organic cotton Content is ideal for sensitive skin

- Padded molded cups to hide uneveness, for example scars
- Optimal T-Shirt bra to wear invisible under clothing
- Pockets are molded and made of ultra soft breathable cotton which is ideal for sensitive skin
- Adjustable elastic straps
- Soft padded back closure
- Side bones offering additional side support



LARA PADDED SOFT BRA



Off-White Art. no. 0752 0674 Nude Black 0675 75-90 AA; 70-100 A, B, C, D, E

Sizes 2 rows: 75-90 AA; 70-100 A, B, C, D, E Material 84% polyamide, 16% elastane

• Padded molded cups to hide uneveness, for example scars

- Optimal T-Shirt bra to wear invisible under clothing
- Pockets are molded and made of breathable, temperature balancing microfibre fabric
- Adjustable elastic straps
- Soft padded back closure
- Side bones offering additional side support

POCKET PATTERN / FABRICS

Moulded

High Cotton

2 Cut & Sewn

Tulle

Microfibre

Cotton

COOLMAX®

Other

Bilateral

Wire Bra

Soft Bra

LARA SATIN PADDED SOFT BRA

▲ ● 11,4

Art. no. Nude 44214 44212 Black

70-100 AA, A, B, C, D, E Sizes 2 rows: 70-100 AA, A, B, C, D, E Hooks 74% polyamide, 15% elastane, 11% polyester



- Padded molded cups to hide uneveness, for example scars
- Optimal T-Shirt bra to wear invisible under clothing with satin trim and center rhinestone detail
- Pockets are molded and made of breathable, temperature balancing microfibre fabric
- Adjustable elastic straps
- Soft padded back closure
- Side bones offering additional side support



LEE SOFT BRA



44540 White 75-95 A, B, C, D; 75-90 E Sizes 2 rows: 75-90 A; 75-85 B; Hooks 75-80 C; 75 D

> 3 rows: 95 A; 90-95 B; 85-95 C; 80-95 D; 75-90 E

Material 90% cotton, 7% elastane, 2% polyurethane, 1% polyamide

- High cotton content is ideal for sensitive skin
- Wide straps relieve the shoulders
- Molded cups and pockets are made of ultra soft breathable cotton which is ideal for sensitive skin
- Symbolic pink ribbon in center front



MAGDALENA SOFT BRA



Nude Art. no. 0463 75-110 B, C, D, E, F Sizes

4 rows: 75-95 B; 75-90 C; Hooks 75-85 D; 75-80 E; 75 F

5 rows: 100-110 B; 95-110 C; 90-110 D; 85-110 E; 80-110 F

54% polyamide, 24% elastane, Material

22% polyester

Bonding technology offers ultra wearing comfort

- High back fabric smooths the side and back areas for a flawless look
- Cups and pockets are molded and made of breathable, temperature balancing microfibre fabric
- Adjustable elastic straps
- Soft padded back closure



MAGDALENA PADDED SOFT BRA

5 rows: 95 C; 90-95 D

▲ ● 1,7 44533 Nude 70-95 AA, A, B, C, D 4 rows: 70-95 AA, A, B; 70-90 C; 70-85 D

• Bonding technology offers this T-Shirt bra ultra wearing comfort

- High back fabric smooths the side and back areas for a flawless look
- Padded molded cups to hide uneveness, for example scars

Material 41% polyester, 36% polyamide, 23% elastane

- Pockets are molded and made of breathable, temperature balancing microfibre fabric
- Adjustable elastic straps

Art. no.

Sizes

Hooks

Art. no.

· Soft padded back closure

44534

1150

44416

44023



MARA PADDED SOFT BRA



70-100 AA, A, B, C, D, E Sizes Hooks 2 rows: 70-90 AA; 70-85 A; 70-80 B; 70-75 C; 70 D

3 rows: 95-100 AA; 90-100 A; 85-100 B; 80-100 C; 75-100 D; 70-100 E

Material 83% polyamide, 11% elastane, 6% polyester (COOLMAX®)

Off-White

Light Nude

Dark Blue

Dark Grey

- Padded molded cups to hide uneveness, for example scars
- Pockets are molded and made of breathable, temperature balancing COOLMAX® fabric
- Adjustable elastic straps
- · Soft padded back closure



























POCKET PATTERN / FABRICS

Moulded

2 Cut & Sewn

Tulle

Microfibre

5 Cotton

6 COOLMAX®

Other



Foam Cups

Cups

Cups

Foam Cups

AMOENA SEAMLESS BRAS



MIRA SOFT BRA

▲ ● ()1,4

44610 Light Taupe Art. no. 70-95 AA, A, B; 75-90 C, D; 75-85 E Sizes

2 rows: 70-90 AA, A; 70-85 B; Hooks 75-80 C, D, E

> 3 rows: 95 AA, A; 90-95 B; 85-90 C, D; 85 E

Material 79% polyamide, 21% elastane • Molded seamless cups with textured jacquard fabric in modern

- geometric dessin • Pockets are molded and made of breathable, temperature balancing
- microfibre fabric
- Wider padded front straps relieve the shoulders
- Adjustable, elastic back straps
- Soft padded back closure
- · Side part support



MONA SOFT BRA

0606

0568

0614 0591

Art. no.

White Champagne Cognac Black

70-95 AA, A, B; 75-90 C, D; 75-85 E Sizes 2 rows: 70-90 AA, A; 70-85 B; 75-80 C, D, E Hooks 3 rows: 95 AA, A; 90-95 B; 85-90 C, D; 85 E

Material 79% polyamide, 21% elastane

- Molded seamless cups with textured jacquard fabric in modern dessin
- Pockets are molded and made of breathable, temperature balancing
- Wider padded front straps relieve the shoulders
- Adjustable, elastic back straps
- Soft padded back closure
- Side bones offering additional side support



TANYA SOFT BRA

 $\blacktriangle \bigcirc \bigcirc ()$ 1,4

44678 Art. no.

44679 Black 70-95 A, B, C, D

Sizes 2 rows: 70-90 A; 70-85 B; 70-80 C; 70-75 D

3 rows: 95A; 90-95 B; 85-95 C; 80-95 D

Material 69% cotton, 18% polyamide, 13% elastane

Nude

- High cotton content is ideal for sensitive skin
- Molded cups are smooth under clothing
- Molded pockets are made of ultra soft breathable cotton which is ideal for sensitive skin
- Wide straps for added comfort
- Adjustable elastic straps

POCKET PATTERN / FABRICS

Moulded

2 Cut & Sewn

Tulle

Microfibre

5 Cotton

6 COOLMAX®

Other

Pockets

AMOENA HIGH-COVERAGE BRAS



AMBER PADDED SOFT BRA

1,4


Black Art. no. 44259 Sizes 70-95 AA, A, B, C, D, E

Hooks

2 rows: 70-95 AA, A; 70-90 B; 70-85 C; 70-80 D; 70-75 E

3 rows: 95 B; 90-95 C; 85-95 D; 80-95 E

Material 79% polyamide, 21% elastane

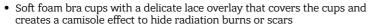

- Padded molded cups to hide uneveness, for example scars
- Pockets are molded and made of breathable, temperature balancing
- Comfortable, adjustable elastic straps
- Soft padded back closure



AMBER TOP

Art. no. 44268 White 44261

Black

75-95 A/B, C/D, E/F Sizes 76% polyamide, 24% elastane Material

Package 1 per box

this lacy accessory

• Make almost any bra a camisole bra with

• Offers a smart solution for lower necklines to hide scars or radiation burns

• Fits over the body and hooks to straps with secure fasteners





▲ ● ()1,4



DANA SOFT BRA

Art. no.

0862 White 0750

Black

70-90 AA; 70-100 A, B, C; Sizes 75-95 D, E

Hooks 2 rows: 70-90 AA; 70-85 A; 70-80 B, C; 75 D

3 rows: 90-100 A; 85-100 B, C; 80-95 D; 75-95 E

Material 87% polyamide, 13% elastane

• Higher coverage with delicate light lace insert to hide e.g. radiation

- Pockets are molded and made of breathable, temperature balancing microfibre fabric
- Jacquard fabric with feminine dot dessin
- Side cups support for a secure fit for larger sizes
- Decorative adjustable, elastic straps
- Soft padded back closure



FEATURES

Bilateral























High Cotton

2 Cut & Sewn

Moulded

Tulle

Microfibre

5 Cotton

6 COOLMAX®



AMOENA ACTIVE BRAS



ESTER SOFT BRA

42576 42577

Art. no.

Art. no.

▲ ● ● 1,4

White

Black/Grey

70-100 A, B, C, D; 80-95 E Sizes 7 rows: 70-85 A; 70-80 B; 70-75 C; 70 D 8 rows: 90-100 A; 85-95 B; 80-90 C;

75-85 D; 75-80 E

10 rows: 100 B; 95-100 C; 90-100 D; 85-95 E

Material 83% polyamide, 17% elastane

- Front closure feature for easy dressing • Features half-moon support on seamless molded cup
- Breathable fabrics; mesh on side and back help with
- moisture evaporation
- Pockets made of breathable, temperature balancing microfibre fabric
- Wider padded front straps to relieve the shoulders
- Adjustable, elastic back straps
- Higher coverage of side and back part
- Wider underbust band

0654

0658



PERFORMANCE SPORTS BRA

White

Black

70-95 AA, A, B, C, D, E Sizes 3 rows: 70-95 AA, A, B, C, D, E

Material 85% polyamide, 15% elastane • Seamless, molded cups

- Breathable mesh panels at center front, sides, and backs for better moisture evaporation
- Wider padded front straps to relieve the shoulders
- Wider underbust band
- Adjustable, elastic back straps
- · Soft padded back closure



▲ ● ● 1,4

LIGHT SUPPORT

4 9 1,6




POWER SPORTS BRA

Black/Grey Art. no.

70-95 AA, A, B, C, D, E Sizes 3 rows: 70-95 AA, A, B, C, D, E Hooks

84% polyamide, 16% elastane Material

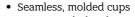

- Sporty mesh detailing at the neckline; mesh panels in the back helps with moisture evaporation
- Features half-moon support on seamless molded
- Pockets are molded and made of breathable, temperature balancing fabric
- Wider padded front straps to relieve the shoulders
- Wider underbust band
- Adjustable, elastic back straps
- Soft padded back closure



Bilateral



















High Cotton

POCKET PATTERN / FABRICS

Moulded

2

Cut & Sewn

Tulle

Microfibre

Cotton

6 COOLMAX®







Sizes

• Round neckline in front and back

• Adjustable double spaghetti straps





VALLETTA TOP

70230 White Art. no. 44075 Nude

> Charcoal Melange 44076 70229

White, Black: 36-54 Nude, Charcoal Melange: 36-50


Material 90% Modal, 10% elastane

• Ultra soft Modal (Tencel) fabric for ultra comfort day

Supportive built-in pocketed shelf bra with soft padded cups can hold a breast form or shaper securely in place, new: Amoena Wave Seam

• Bilateral pockets

- Ideal for women who are extra sensitive to traditional bras due to treatment effects
- Round neckline in front and back
- Adjustable double spaghetti straps

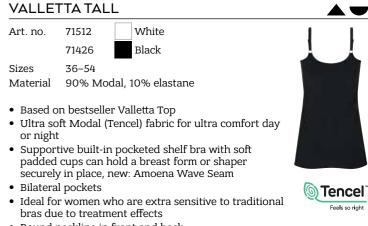



SPECIAL FEATURES OF AMOENA SWIMWEAR

Amoena Special Swimwear combines attractive, colourful styles for every figure and makes you feel like summer and sun. All styles are especially developed for the individual needs of breast-operated women. The secure breast form fit, especially when swimming, as well as the optimal wearing comfort invites you to feel good. Our collection is designed at our headquarters in Raubling/Germany and the fit is developed exclusively for women who have undergone breast surgery. Our swimwear are elaborately manufactured from the highest quality materials. The special workmanship – the Amoena Wave Seam at the integrated bra ensures the optimal fit of the Amoena breast forms, breast shapers and the Amoena Aqua Wave, which was specially developed for swimming.



Soft and comfortable straps

- Support the prosthesis and relieve the shoulders
- Most styles have individually adjustable straps
- Become wider with increasing size
- Some styles have multifunctional strap solutions: removable, wearable as neckholder, crossed or straight

Various neckline forms

- Variably adjustable necklines allow individual wearing options
- Higher necklines cover possible scars and burns

Special integrated construction

- Keeps the Amoena breast forms/breast shapers/Aqua Wave close to the body
- Integrated lightly padded cups shells hide unevenness
- The Amoena Wave Seam is a special workmanship on the under bust band of the inner bustier to provide additional support at the side for the Amoena breast forms/breast shapers/Aqua Wave

Invisible pockets on both sides

- For right and/or left care
- Are accessible from the side for easy insertion, a small opening on the top of the bag helps to place the breast forms/breast shapers optimally
- The side pocket opening prevents from falling out while swimming

Amoena Wave Seam

 The special curved lateral seam on the underbust band provides additional support



Shape of leg lines


• The moderate flattering shapes are suitable for all age groups and figure types

Back necklines

- Variations from high to low cut back neckline solutions
- Removable, horizontal and adjustable straps provide additional support for low-cut back necklines
- Additional back material on the closure to adjust the under bust width on the bikini tops (TP Tops)

High quality, functional materials:

- Elastic comfort lining for optimal view protection
- Selection of recycled materials to support sustainability
- Selection of technically equipped materials such as Xtra Life Lycra, UV protection, chlorine resistant, latex free

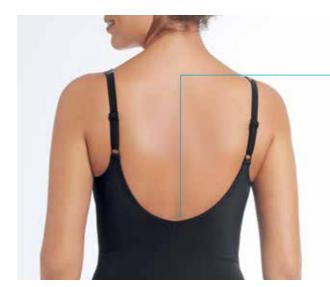

xtra life™



ROME ONE-PIECE

Art. no.


71379 Navy/Turquoise

36-46 B, 38-46 C Sizes Material 78% polyamide,

22% elastane (LYCRA® XTRA LIFE™)

• Outstanding placement print in vibrant colours in front and back

- Classic and elegant styling
- Adjustable back straps
- Recycled fabric VITA features excellent UV protection, ultra chlorine and pilling resistent and suncream and
- Xtra Life LYCRA® for long lasting fit and shape retention, with high chlorine resistance • Supportive built-in pocketed shelf bra with soft padded
- New: integrated innovative Amoena Wave Seam

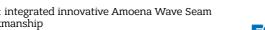



▲○ ∅







ALABAMA HALF BODICE

71514 Navy/Rust 38-54 B, C; 40-54 D, E Sizes Material 80% polyamide,

20% elastane (LYCRA® XTRA LIFE™)

- Essential patterns for every figure
- Mesh lining in front part for medium shaping effects
- Easy polka dot print for a vintage look
- Center front is adjustable and offers two different wearing options: higher coverage to hide radiation burns or scars or lower center front for a deeper
- Solid straps are elastic and adjustable to relieve the
- Supportive built-in pocketed shelf bra with soft padded
- New: integrated innovative Amoena Wave Seam workmanship



ALABAMA TANKINI

Navy/Rust 71513 Art. no.

Sizes 38-50 B, C Material 80% polyamide,

20% elastane (LYCRA® XTRA LIFE™)

- A-line pattern flatters the figure
- · Easy polka dot print for a vintage look
- Pattern based on Ayon Tankini
- Solid straps are elastic and adjustable to relieve the shoulders
- Xtra Life LYCRA® for long lasting fit and shape retention, with high chlorine resistance • Supportive built-in pocketed shelf bra with soft
- padded cups
- New: integrated innovative Amoena Wave Seam workmanship





ALABAMA HIGH WAIST BRIEF

71332 Navy/Rust Art. no.

Sizes 38-50 Material 80% polyamide,

20% elastane (LYCRA® XTRA LIFE™)

- Brief can be worn as normal Panty or High Waist Brief
- Xtra Life LYCRA® for long lasting fit and shape retention, with high chlorine resistance





FEATURES































Navy/Rust 71326 38-54 B,C; 38-52 D Sizes Material 80% polyamide,

20% elastane (LYCRA® XTRA LIFE™)

• Essential patterns for every figure

• Mesh lining in front part for medium shaping effects

• Easy polka dot print for a vintage look

• High neckline with mesh insert hides scars or

• Adjustable elastic back straps

Supportive built-in pocketed shelf bra with soft padded

• New: integrated innovative Amoena Wave Seam workmanship





AMOENA SWIMWEAR - CLASSIC



AYON SARONG

Black/White 71113 Art. no.


Sizes 40-56 B, C Material 80% polyamide,

20% elastane (LYCRA® XTRA LIFE™)

- Geometric black and white all over print in front and back
- Short skirt with rushing in front flatters the figure
- Mesh lining in front part for medium shaping effects
- Solid black insert frames the neckline
- Black Panty underneath gives extra coverage
- Xtra Life LYCRA® for long lasting fit and shape retention, with high chlorine resistance
- Solid black straps are elastic and adjustable to relieve the shoulders
- Supportive built-in pocketed shelf bra with soft padded cups





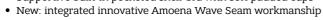




AYON PADDED SOFT BRATOP

Black/White 71117 Art. no.

36-46 B, C, D Sizes 80% polyamide, Material

20% elastane (LYCRA® XTRA LIFE™)

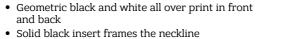

Black double spaghetti straps can be changed to a -fancy straight / neckholder combination

• Xtra Life LYCRA® for long lasting fit and shape retention, with high chlorine resistance Solid black straps are elastic and adjustable in the back





AYON HIGH WAIST BRIEF

Art. no. Black/White

Sizes 80% polyamide, Material

20% elastane (LYCRA® XTRA LIFE™)

- Brief can be worn as normal panty or high waist brief
- Xtra Life LYCRA® for long lasting fit and shape retention, with high chlorine resistance







Pockets





















TORONTO ONE-PIECE HIGH NECK

▲○ ∅

Black/Purple/Mint

36-48 B, 38-48 C Sizes Material 100% polyester

- Sportive palm leaf print
- High Neckline to hide scars and radiation burns
- Black inserts at sides flatter the figure
- Stylish criss cross optic at back
- Wider straps to relieve the shoulders
- DARWIN® fabric is chlorine proof with perfect shape
- Supportive built-in pocketed shelf bra with soft padded cups
- New: integrated innovative Amoena Wave Seam workmanship





SILICONE SHOULDER PADS

Art. no. 050

BREAST CARE APPAREL ACCESSORIES

052 Display

Sizes One size Package 2 per box

- Provides comfort from straps that tend to dig into the shoulder
- Soft, clear silicone pads slip easily over standard size bra strap
- May assist with weight distribution for women with certain medical conditions
- Display holds 12 sets of shoulder pads



BRA EXTENDERS

Art. no. 122 White 123 Champagne Black 121 Sizes 2, 3, 4 Haken Höhen

- Perfect addition to our bras
- Ideal for volume fluctations



AQUA WAVE SWIMFORM

Art. no. 149 1-14 Sizes

- $\bullet\,$ A transparent silicone form for water activities, with a unique natural shape
- Symmetrical form features raised waves on back that allow air and water to flow through, reducing the amount of suction
- Offers a shallow to average fit
- Packaged with a fashionable waterproof drawstring bag that is large enough to hold a wet swimsuit






RHODES ONE-PIECE

Dark Blue 70842

36-52 B, C Material 100% polyester

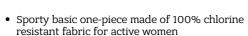

- The placement of the double seams in front is slimming and comfortable
- Tighter fit for additional support and shape
- DARWIN® fabric is chlorine proof with perfect shape
- retention
- Wider straps to relieve the shoulders
- Supportive built-in pocketed shelf bra with soft padded cups
- New: integrated innovative Amoena Wave Seam workmanship
- Round neckline in front and back
- Adjustable double spaghetti straps

FEATURES



























 $\triangle \bigcirc \bigcirc$



RESSOURCES

MERCHANDISING TIPS & SIZING CHARTS

Having the right resources at your fingertips only enhances your expertise. We've gathered sizing and measuring charts for breast forms and lingerie, plus tips and available point-of-sale support. So that your entire staff and store reflects your desire to give customers every possibility for comfort and confidence.





4 TIPS TO INCREASE YOUR SELL-THROUGH


Direct merchandising displays the product in such a way to generate curiosity and motivate consumers to make a purchase. It's all about the way we show products, and when we do it right, we see how quickly items can move out of the door. These four merchandising tips are backed by research and used globally by the leading retailers. It is essential to practice proper merchandising; everything in your store sends a message to your customer and influences their buying decisions. Keep up the excellent work!

1

PLAN AHEAD

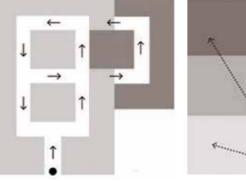

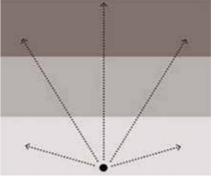

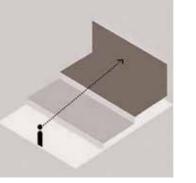

Begin With Your Floor Plan

Help guide your customers through your store. Customers shop counter-clockwise and have a natural tendency to move towards the right of the store upon entering. Identify the best flow of traffic in your store and ensure there is enough space for customers to move around comfortably, and merchandise is in reaching distance.

Plan Your Product Placement

Think about where to place new and old merchandise; plan where to place the older product within your space to make room for new arrivals. Highlight new items at the front of the store and place older items further back. Always maintain clear sightlines.

Create Zones Based on Collections

Select an area of the store for your key pieces, such as the front wall, feature table, or t-stand. Your newer collection will be at the front, and your older group of merchandise will be closer to the back. Merchandising the product left to right will help the customer's eye flow naturally.

Establish a Welcoming and Inviting Atmosphere

Offer seating for customers to spend more time in your store.



GROUP YOUR MERCHANDISE



Group by COLOUR

Merchandising by colour attracts attention and encourages multiple purchases. Colours trigger the pleasure centers in our brains. Shoppers will touch and explore more when they see groupings of colour. Feel free to colour block on a wall, front table or a floor fixture.

Group by THEME

Grouping by theme can be profitable because it influences customer decision-making. Visually, grouping merchandise by theme suggests items to customers they may not have thought of initially. Be sure to use props and include add-ons that relate to your theme.

Group by COMMODITY

Commodity merchandising is grouping items or styles that are alike. It is useful if you have a lot of one style, such as Vallettas, recovery bras, or breast forms. This type of grouping draws focus and attention on the product to help move items out the door.

Group Using THE RULE OF THREE

Customers' eyes focus longer on the product or a display when it appears balanced. Shoppers are more likely to explore the items and can lead to purchasing. This rule applies to mannequins, fixtures, merchandise, or props. Empty Space is Negative Space. Try to avoid negative space in merchandising. Negative space will not create a balance to capture the customer's attention.



SWITCH IT UP —



Moving your merchandise from its old location to a new one can increase sales on average by 25%. Refresh the product in your window every 3-4 weeks and twice a season for instore merchandise. This is especially important when you don't have new items. A new product placement makes an old item NEW in the eyes of a shopper.

MAINTAIN YOUR SPACE -



Garment care is the easiest way to make your space more appealing and is crucial for maintaining your area visually.

- · Use correct hangers
- · Size all products
- · Tuck in tags

Keep your space clean and tidy to enhance the customer shopping experience!

MERCHANDISING AND POINT-OF-SALE SUPPORT

Amoena offers several tools to assist you in merchandising and displaying products in your store and fitting room.




Breast Form Display

- Holds any shape of breast
- Integrated holder for A4 sized brochures
- Size: 28 x 24 x 7 cm
- Back slot holds an A6, product-specific graphic
- Package: 1
- Must be ordered separately. Ask your account manager for details.



Styrofoam Head

- · For scarves and hats
- · Size: 29 cm high • Package: 1
- Must be ordered separately. Ask your account manager

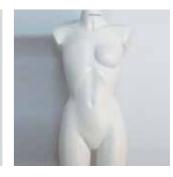

Mastectomy Torso

Full Body

- White with Amoena logo
- Torso size: Bra / 75B; Dress / 38
- Size: 45 x 80 x 35 cm
- Package: 1
- Must be ordered separately. Ask your account manager for



Breast Form Holders


- Clear with Amoena Logo
- Shapes: 1S, 2S/3S, 2U, 3A Right, 3U, 3E Left, 3E Right, Contact, Balance Varia
- Package: 1



1. Garment Hanger

2. Swimwear Hanger

- Clear with Amoena logo
- Package: 10

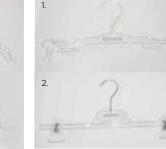

1. Pants Hanger

2. Lingerie Hanger

- Clear with Amoena logo
- Package: 10



Acrylic Branding Block

- Clear with Amoena logo
- Size: 20 x 12 x 3 cm
- Package: 1



Self-adhesive Logo Sticker

- Clear with black, right reading Amoena logo in sizes 300 x 60 mm and 500 x 90 mm
- Clear with a frosted, backward reading Amoena logo for use inside glass in size 275 x 40
- Package: 3, one of each size and style



A4 Literature Holder


- · Clear with Amoena Logo
- Size: 230 x 30 mm
- Package: 1



Full Mannequin

- Fixed Pose • White with Amoena logo
 - Torso size: Bra / 75B; Dress / 38
 - Size: Approximately 1,80 cm high, including base
 - Package: 1
 - Must be ordered separately. Ask your account manager for

A quick and easy reference point to view all that Amoena offers, including the latest products as well as relevant information about the industry and ways to better serve your customer. Find out more at **www.amoena.com**

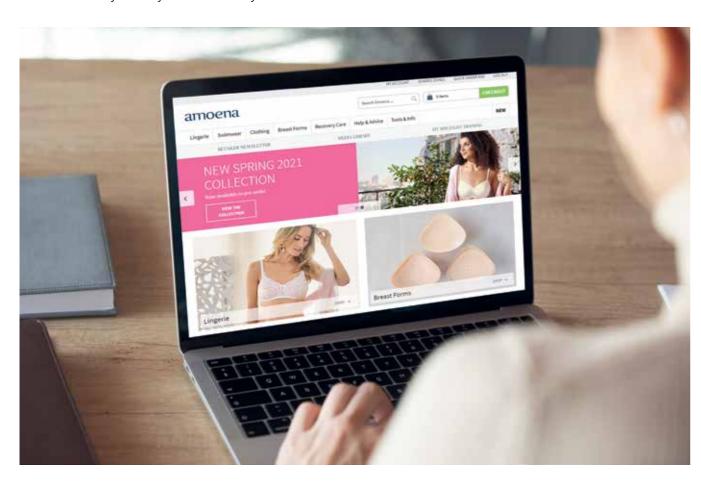



AMOENA MEDIA LIBRARY

A comprehensive resource where you can download product photos, get advertising assistance, look at merchandising ideas, and more.

www.amoenalibrary.com



SOCIAL MEDIA

Check out our social media channels on Facebook, Instagram and YouTube to find more information about our products and company. You can also use them as insparation for your channels.

FITTER TIPS

Here are some tips on how to provide the best service to your customers, that will also help make your daily work easier. You can find the right bra size and cup form for a perfect silhouette – and show confidence doing it – once you know these simple tips!





How to find the right bra size

- Determine the underbust measurement in centimeters to get the band size.
- 2. Then determine the **half overbust measurement** in centimeters and *multiply by two* to get the *cup size*.
- 3. Find your measurements in the chart below to determine the bra size.

These charts are only a guide. For a perfect fit, size up or down as needed.



Underbust measurement:

While she is wearing a bra, measure around her torso where the bra band sits.



Half overbust measurement:

Measure from the middle of her chest (sternum) over the fullest part of a remaining breast to the center of her back at her spine.

These instructions should only be used as guidance. You may have to choose a size smaller or larger for the perfect fit.





Tips for fitting: Underbust band:

The underbust band should not dig in, but should be a tight fit. The back of the bra should sit horizontally under the shoulder blades and should not ride up so that it supports the breast perfectly. If part of the breast can be seen underneath the band, try changing sizes. Try one band size smaller and one cup size larger or one band size larger and one cup size smaller.

If the cup presses/squeezes the breast at the front, under the arm or at the underbust band, try a larger cup size. The cup should be perfectly in line with the breast contour and cover it as much as possible.



Finding the right Cup Form: Women with a small underbust tend towards full cup forms, women with a large underbust tend towards shallow cup forms. The pocket of the bra should harmonise with the footprint of the breast form. If the pocket of the bra is too small, the breast form will be squeezed and can affect the life of the breast form.

6

View in the mirror: Once the bra and breast form are adjusted, encourage the customer to judge her new shape by looking into the mirror, not down at her chest. She'll be seen by other people from the front, not from



Perfect silhouette: Look at the customer from the side to be sure that the size and shape of the breast form is correct.



Showing confidence: If you are confident, it will show and the fitting and products won't be questioned as often. If the first bra you try on fits great... don't try anything else! Trust yourself.



Secret on bra size: There's no need to tell the customer which bra size you've chosen until she is happy with the look. Sometimes knowing the size can cause negative feelings, despite how good it looks on her.

FEATURE OVERVIEW BREAST FORMS

		ı														
				rd	Reduced (-20%)	-25%)	ght	Regular Ivory	vory	Regular Tawny	awny	able e	Self Adhesive	Temperature Regulation	ıtion	Soft Rear Side
				Standard	educe	Light (-25%)	Xtra Light (-40%)	tegula	Light Ivory	tegula	Light Tawny	Adjustable Volume	elf Ad	empe	Ventilation	oft Re
Product line	Product	Style	Sizing	os .		ight	×-	н		lour	H	4,5		Benefit		<i>S</i> 3
	Adapt Air Xtra Light 1S ^N	328	1-14						_/			_/		_/		
_ +	Adapt Air Xtra Light 2S ^N	326	1-14				_/		1/			_/		_/		
F lab	Adapt Air Light 1S ^N	329	1-14			./	V					./		·/		
ADAPT Comfort*	Adapt Air Light 2S ^N	329	1-14			V /			V /			V		V /		
						V			V			٧		V /		
	Adapt Light 3A	376	4-12 L/R			V			V					V		
	Contact Light 2S	380 385	1-14 0-10			V			V				V	V		
– –	Contact Light 3S Contact 1S	384	1-14			V			V				V	V		
V t	Contact 1S Contact 2S	381	1-14	V				V					V	V		
F Sugar	Contact 2A	383	0-12 L/R													
CONTAC	Contact 3S	382	0-12 L/K					./								
	Contact 3S TW	382T	1-10	<u>v</u>						_/			<u>v</u>	\ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \ \		
	Contact 3E	386	1-14 L/R	<u>v</u>									<u></u>	V		
	Energy Light 2S	342	1-14	•		/		_	/				_	/	/	
→ □	Energy Light 2S TW	342T	1-14			/					/			/	<u> </u>	
ENERGY Comfort*	Energy Light 2U	341	1-14			/			/					-	<u> </u>	
in in its second and	Energy Cosmetic 2S	310	1-14		_			V	ľ					V	V	
	Energy Cosmetic 3S	311	1-12		V			√						V	√	
	Energy 1S	349	1-12	√				√						√	√	
	Natura Xtra Light 1S ^N	401	1-14				\checkmark		√					√		
	Natura Xtra Light 2S ^N	400	1-17				✓		\checkmark					√		
	Natura Xtra Light 2S ^N TW	400T	1-17				\checkmark				√			√		
	Natura Light 1S ^N	402	1-14			\checkmark			√					√		\checkmark
	Natura Light 1S	664	1-14			V			V					V		V
	Natura Light 2S	390	1-17			V			√					V		V
	Natura Light 2S TW	390T	1-17			√					√			√		√
	Natura Light 2A	392	0-14 L/R			√			V					√		√
4	Natura Light 3S	391	0-12			√			V					√		√
URA nfort ⁺	Natura Light 3A	373	4-12 L/R			√			√					√		√
\ATC	Natura Light 3A TW	373T	4-12 L/R			√					✓			√		V
NAT	Natura Cosmetic 2S	320	1-14		V			V						V		V
	Natura Cosmetic 2S ^N Natura Cosmetic 3S	323	1-14		V			√						V		V
	Natura Cosmetic 35 Natura Cosmetic 3E	321 322	1-12 1-14 L/R		V /			√						V /		V /
	Natura 1S	396	1-14 L/R 1-14	√	V											V
	Natura 1S TW	396T	4-12	<u>v</u>				√		/				·/		
	Natura 2S	398	1-10	<u></u>				_/						V		/
	Natura 2A	393	0-12 L/R	<u></u>				<u> </u>						/		
	Natura 3S	395	0-12	<u></u>				√						1		<u> </u>
	Natura 3E	397	1-14 L/R	V				—						-		
	Essential Light 1S ^N	314	1-14			√			V							
	Essential Light 1S ^N TW	314T	1-14			1					√					
	Essential Light 2S	442	1-17			√			√							
	Essential Light 2S TW	442T	1-17			√					√					
₫	Essential Light 2A	356	0-14 L/R			\checkmark			\checkmark							
ESSENTIAL	Essential Light 3S	367	0-12			V			√							
Ш	Essential Light 3E	556	1-14 L/R			V			V							
Si	Essential Light 3E TW	556T	4-14 L/R			√					√					
	Essential 1S	630	1-12	V				√								
	Essential 2S	440	1-12	√				√								
	Essential 2A	353	0-12 L/R	√				√								
	Essential 3S	363	0-12	V				\checkmark								

FEATURE OVERVIEW BREAST SHAPERS

				Standard	Reduced (-20%)	Light (-25%)	Xtra Light (-40%)	Regular Ivory	Light Ivory	Regular Tawny	Light Tawny	Adjustable Volume	Self Adhesive	Temperature Regulation	Ventilation	Soft Rear Side	
Product line	Product	Style	Sizing	St		ធ្វើ ight	ΧĽ	Re		our	ï	Ϋ́		enefit		Sc	L
₩	Balance Adapt Air MD	233	2-12			✓		✓				✓		✓			
BALANCE ADAPT AIR	Balance Adapt Air SV	234	S-XL			✓		√				√		√			
出片 🕟	Balance Contact MD	229	2-12	√				√					✓	√			
BALANCE CONTACT	Balance Contact VD	230	3/4-9/10	√				√					√	√			
W C C	Balance Contact SV	286	S-XL	√				√					✓	✓			
	Balance Natura Light VD	221	3/4-11/12			√			√					√			1
	Balance Natura TO	227	S-XL	√				√						√			
교육 [‡]	Balance Natura TD	217	1-10	√				√						√			
BALANCE NATURA	Balance Natura MD	220	2-12	√				√						√			Г
Z Z Z	Balance Natura VD	222	3/4-9/10	√				√						√			
	Balance Natura SE	231	S-XL	√				√						√			
	Balance Natura SV	285	S-XL	√				√						√			
	Balance Essential Light VD	224	3/4-11/12			\checkmark			\checkmark								1
	Balance Essential Light VD TW	224T	3/4-11/12			√					√						
핑절	Balance Essential TO	228	S-XL	√				√									
BALANCE	Balance Essential TD	218	1-10	√				√									
AP SE	Balance Essential MD	223	2-12	√				√									
ш <u>ш</u>	Balance Essential MD TW	223T	4-10	√						√							
	Balance Essential VD	225	3/4-9/10	√				√									
	Balance Essential SE	232	S-XL	√				√									

SIZING CHART FOR AMOCURA - RECOVERY CARE

112

CURASUPPORT SIZING CHART

Leyla ZIP-ST-LOW, Sina ZIP-ST-Medium, Pamela ZIP-ST-HIGH and Emilia ZIP-ST-LOW

The underbust measurement has a corresponding colour and symbol which have a clearly designated bra size. The colour and symbol can be found on the packaging label.

		OLIC	ierbust circuini	erence		
cm	68-77	78-87	88-97	98-107	108-117	118-127
	→	-	-	-	-	-
			Universelles Cup	1p		
EU size	70/75	80/85	90/95	100/105	110/115	120/125
Symbol	0		٥	•		•



CURASCAR SIZING CHART

Belly bandage 3-VE


Waist circumference Measure the narrowest part of your waist.

Panty 2-ZIP-BO



Stand with your feet together and measure the widest part of your hips. Waist circumference Measure the narrowest part of your waist. Hip circumference

S/M	67 - 74 m / 74 - 76 m	4 75 – 30 inch
L/XL	77 – 94 cm / 30.5 – 37 inch	5 – 37 inch
2XL/3XL	95 – 112 cm / 37.25 – 44 inch	7.25 – 44 inch
	Height compression belly bandage: 24 cm / 9.25 inch	: 24 cm / 9.25 inch
1	[cm]/[inch]	jnch]
Size	cT	CH
XS	61 – 66 / 24 – 26	86 – 91 / 34 – 36
S	66 - 71 / 26 - 28	91 – 97 / 36 – 38
M	71 – 76 / 28 – 30	97 – 102 / 38 – 40
Г	76 – 81 / 30 – 32	102 – 107 / 40 – 42
χ	81 – 89 / 32 – 35	107 – 114 / 42 – 45

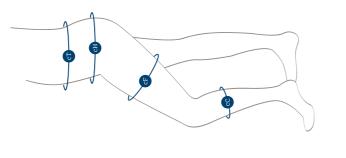

114 - 124 / 45 - 49

89 - 99 / 35 - 39

2XI

Bermuda 2-ZIP-BO

Hip circumference
Stand with your feet together and measure the widest part of your hips.

Thigh circumference
Measure the circumference halfway down your thigh. Waist circumference Measure the narrowest part of your waist. Calf circumference Measure the thickest area of your calf.

		[cm]	[cm]/[inch]	
Size	cT	CH	CF.	ည
xs	61 – 66 / 24 – 26	86 - 91 / 34 - 36	48 – 51 / 18.75 – 20	26 - 29 / 10.25 - 11.5
S	66 - 71 / 26 - 28	91 – 97 / 36 – 38	51 – 54 / 20 – 21.25	29 – 32 / 11.5 – 12.75
ĭ	71 – 76 / 28 – 30	97 – 102 / 38 – 40	54 - 57 / 21.25 - 22.5	32 - 35 / 12.75 - 13.75
1	76 – 81 / 30 – 32	102 – 107 / 40 – 42	57 – 60 / 22.5 – 23.75	35 – 38 / 13.75 – 15
Х	81 – 89 / 32 – 35	107 – 114 / 42 – 45	60 – 65 / 23.75 – 22.5	38 - 41 / 15 - 16.25
2XI.	89 – 99 / 35 – 39	114 – 124 / 45 – 49	65 – 70 / 25.5 – 27.75	41 – 44 / 16.25 – 17.5

SIZING CHART FOR BREAST FORMS

First, measure the band size and corresponding cup size as shown on the front page of this chart. Then, determine the right Amoena breast form size by following the column down. This size chart is meant as a guide to assist you during the breast form finding process. However, please also use your expertise and feeling.


							120	115
						120	115	110
					120	115	110	105
				120	115	110	105	100
			120	115	110	105	100	92
		120	115	110	105	100	92	06
	120	115	110	105	100	92	90	82
	115	110	105	100	92	06	85	80
Band size	110	105	100	98	06	85	80	75
Band	105	100	92	06	85	80	75	
	100	92	06	85	80	75	70	
	92	06	85	80	75	70	65	
	90	85	80	75	70	65		
	85	80	75	70	65			
	80	75	70	65				
	75	70	65					
	70	65						
	65	60						
Cup size	AA	Ą	В	ט	Д	Щ	ц	ט

moena Nati	l and Amoer		as Amoena Essentia	available	`Only														
17*	16*	15*	14	13	12	11	10	6	8	2	9	2	4	3	2	1	0		
								size	form s	Breast									

Amoena breast forms

Additional breast forms

CROSS-CUP DIFFERENCE (CCD) METHOD FOR BREAST SHAPER FITTING SIZING CHART FOR BREAST SHAPERS

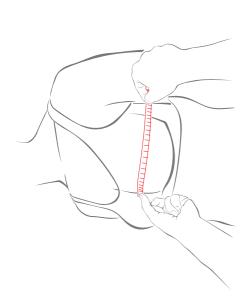

This fitting method was designed to help you find the best thickness (Thin, Medium or Volume) for your customers' needs. Which shape is right for her is still based on visual observation during a personal fitting.

The cross-cup is the measured width between the front center of the bra (at her sternum) to the side seam.

1. Measure cross-cup non-surgery side
2. Measure cross-cup surgery side
3. Substract the width of step 2 from the width of step 1
4. Total is the Cross-Cup Difference (CCD)

TRY	THIN Shaper	MEDIUM Shaper	VOLUME Shaper
CCD MEASUREMENT	Less than 1 cm or 3/8"	1 to 2 cm or 3/8" to 3/4"	More than or equal to 2 cm or $3/4$ "

		120	115	110	105	100	95	06	12
	120	115	110	105	100	95	06	82	11
	115	110	105	100	95	06	85	80	10
	110	105	100	95	06	85	80	75	6
	105	100	95	06	85	80	75		&
Band size	100	95	06	82	80	75	70		7
Band	92	06	85	80	75	70	65		9
	06	85	80	75	70	65			72
	85	80	75	70	65				7
	80	75	70	65					ю
	75	70	65						2
	70	65							1
Cup size	AA	А	В	υ	D	ш	Ш	U	Shaper Size

			12	11/12
			11	11/
	XL	10	10	9/10
	×	6	0	/6
	L	8	8	2/8
Shaper sizes	I	7	7	/_
Shape	M	9	9	5/6
	4	r.	5	2/
	S	4	4	77
	0)	М	23	3/4
		7	7	
		1		
Form	Oval	Delta	Delta	Delta
Cross-cup difference	CD < 1 cm	→ Thin Shaper	CD 1 − 2 cm → Medium Shaper	$CD \ge 2 cm$ $\Rightarrow Volume Shaper$

SIZING CHART FOR LINGERIE AND SWIMWEAR

Measurement based on body height: 165-172cm

EU Underbust measurement in cm	63-67	68-72	73-77	78-82	83-87	88-92	93-97	98-102	103-107	108-112	113-117	118-122
EU band size	65	70	75	80	85	06	95	100	105	110	115	120
EU Overbust measurement in cm results in the corresponding cup size	results in the	correspondi	ng cup size									
AA	75-77	80-82	85-87	90-92	95-97	100-102	105-107	110-112	115-117	120-122	125-127	130-132
A	61-77	82-84	87-89	92-94	66-26	102-104	107-109	112-114	117-119	122-124	127-129	132-134
М	79-81	84-86	89-91	96-46	99-101	104-106	109-111	114-116	119-121	124-126	129-131	134-136
υ	81-83	88-98	91-93	86-96	101-103	106-108	111-113	116-118	121-123	126-128	131-133	136-138
Q	83-85	06-88	93-95	98-100	103-105	108-110	113-115	118-120	123-125	128-130	133-135	138-140
ш	85-87	90-92	95-97	100-102	105-107	110-112	115-117	120-122	125-127	130-132	135-137	140-142
ц	87-89	92-94	66-26	102-104	107-109	112-114	117-119	122-124	127-129	132-134	137-139	142-144

56	54	52	50	87	46	77	42	07	23	36	34	EU panty, Swimwear one-piece
144-146	139-141	134-136	129-131	124-126	119-121	114-116	109-111	104-106	99-101	94-96	89-91	ซ
142-144	137-139	132-134	127-129	122-124	117-119	112-114	107-109	102-104	97-99	92-94	87-89	Ц
140-142	135-137	130-132	125-127	120-122	115-117	110-112	105-107	100-102	95-97	90-92	85-87	ш
138-140	133-135	128-130	123-125	118-120	113-115	108-110	103-105	98-100	93-95	88-90	83-85	О
136-138	131-133	126-128	121-123	116-118	111-113	106-108	101-103	86-96	91-93	88-98	81-83	บ
134-136	129-131	124-126	119-121	114-116	109-111	104-106	99-101	96-46	89-91	84-86	79-81	В
132-134	127-129	122-124	117-119	112-114	107-109	102-104	66-26	92-94	87-89	82-84	62-22	А

52 54	48 50	7 97	7 7	77	40	38	36	34	EU panty, Swimwear one-piece and two-piece size
-------	-------	------	----------------	----	----	----	----	----	--

EU Hip measurement in cm	88-92	95-96	96-100	100-104	104-108	108-112	112-116	116-120	120-124	124-128	128-132	132-136

113-117

103-107 108-112

93-97 98-102

88-92

83-87

78-82

73-77

63-67 68-72

58-62

EU Waist measurement in cm

3XL

2XL

X

ᆸ

∑

XS

EU International dress size

EII Inte

			ZXL 3XL	
			2XL	
			XL	
·			XL	
-			П	
			M	
			M	
	S	S	S	Σ
D		S	S	S
•		XS	XS	S
1-0	A	В	บ	Q

RESOURCES

AMOCURA - RECOVERY CARE INDEX - CURASUPPORT

					Ме	dical	l nee	ds/F	eatu	res								
Product name	Picture	Product line	Basic/ Advanced/ Premium	Colour / Art. No.	Easy handling for restricted movement	Fixation and support	Post-op oedema	Front Closure	Wider straps to relieve the shoulders	Seamless knitted technology	Prepared for direct clinical use	Bonding technology/flat seams	Supports scar healing	No pockets	Bilateral pockets	High cotton content for sensitive skin	Size	Page
Compression Belt	-	CuraSupport Compression	premium	0776N 0789N		✓		✓					√				65 - 125	21
Drain Pouch	66	CuraSupport Comfort	basic	2107												✓	S, L	25
Emilia ZIP- ST-LOW		CuraSupport Comfort	basic	45013 45012 45014	√	√		√	√	√					✓		70/75 - 120/125 Universal cup	23
Ester SB	24	CuraSupport Comfort	basic	42576 42577	√	√		✓							✓		70 - 100 A-D 80 - 95 E	24
Frances SB		CuraSupport Comfort	basic	2128	√			✓	√						✓	√	XS, S, M, L, XL, 2XL Cup: A/B, C/D, E/F	24
Hannah SB		CuraSupport Comfort	basic	2160	√			✓							√	√	XS, S, M, L, XL, 2XL, 3XL, 4XL Cup: A/B, C/D	23
Leyla ZIP-ST- LOW	M	CuraSupport Compression	advanced	44605 44604 44606	√	√	√	√	√	√	√		√		✓		70/75 - 120/125 Universal cup	15
Michelle Camisole	X	CuraSupport Comfort	basic	2105 2105 2105	✓				√						✓	✓	XS, S, M, L, XL, 2XL, 3XL, 4XL	23
Pamela ZIP- ST-HIGH		CuraSupport Compression	premium	45010 45009 45011	✓	√	√	√	√	✓	√		√		✓		70/75 - 120/125 Universal cup	21
Sandra SB		CuraSupport Comfort	basic	0622	√			✓							√		S, M, L, XL, 2XL, 3XL, 4XL	24
Sarah SB		CuraSupport Compression	premium	0778	√	✓	√	✓	√				√	✓		√	75 - 105 A, B, C, D, E	20
Sina HE-ST- MEDIUM		CuraSupport Compression	advanced	45007 45007 45008	✓	√	✓	√	√	√	✓		√		✓		70/75 - 120/125 Universal cup	20
Theraport SB		CuraSupport Comfort	basic	2161	√			✓	√	✓					√	√	XS, S, M, L, XL, 2XL	25

AMOCURA - RECOVERY CARE INDEX - CURASCAR

					Ме	dica	l nee	ds/F	eatu	res								
Product name	Picture	Product line	Basic/ Advanced/ Premium	Colour / Art. No.	Easy handling for restricted movement	Fixation and support	Post-op oedema	Front Closure	Wider straps to relieve the shoulders	Seamless knitted technology	Prepared for direct clinical use	Bonding technology/flat seams	Supports scar healing	No pockets	Bilateral pockets	High cotton content for sensitive skin	Size	Page
Anchors		CuraScar Silicones	advanced	013									✓				230 x 145 mm 9.06 x 5.71 inch	15
Belly bandage 3-VE		CuraScar Garments	advanced	45004 45005	√	√	√	✓				√	√				S/M, L/XL, 2XL/3XL	16
Bermuda 2-ZIP-BO		CuraScar Garments	advanced	45002 45003	✓	✓	✓	✓				✓	✓				XS, S, M, L, XL, 2XL	16
Mammilla Circles		CuraScar Silicones	advanced	012									✓				Ø outside 75 mm; Ø inside 20 mm Ø outside 2.95 inch Ø inside 0.79 inch	31
Panty 2-ZIP BO		CuraScar Garments	advanced	45000 45001	✓	√	√	√				√	√				XS, S, M, L, XL, 2XL	27
Squares		CuraScar Silicones	advanced	011									√				260 x 100 mm 10.24 x 3.94 inch	16
Strips		CuraScar Silicones	advanced	010									√				260 x 35 mm 10.24 x 1.38 inch	16

BREAST CARE APPAREL INDEX - LINGERIE

					Ме	dical	l nee	eds/F	eatu	res				
Product name	Picture	Product line	Basic/ Advanced/ Premium	Colour / Art. No.	Padded cups to hide uneven scars	Higher coverage to hide radiation burns or scars	Wider straps to relieve the shoulders	Breathable fabric for optimized air circulation	Breathable pocket fabric	High cotton content for sensitive skin	Stretchy material for volume fluctations	Front Closure	Size	Page
Amanda Soft Bra	4	lace	premium	44536					√				75-110 A, B, C, D, E, F; 80-105 G	71
Amanda Soft Bra Bustier	64	lace	premium	44537					✓				75-95 A, B, C, D	70
Amanda Wire Bra		lace	premium	44538					√				75-110 A, B, C, D, E, F; 80-105 G	71
Amanda Panty		lace	premium	44539									36-54	71
Amber Padded Soft Bra		higher coverage	premium	44259	✓	✓			✓				70-95 AA, A, B, C, D, E	90
Amber Top	包多	higher coverage	basic	44268 44261		✓							75-95 A/B, C/D, E/F	90
Amy Soft Bra		seamless	basic	44310 44311		✓	✓	✓	✓		✓		S, M, L	82
Aurelie Soft Bra		lace	premium	44153					✓				70-100 A, B, C, D, E 75-95 F	72
Aurelie Wire Bra		lace	premium	44154					✓				70-100 A, B, C, D, E 75-95 F	73
Aurelie Padded Soft Bra		lace	premium	44155	✓				✓				70-100 AA, A, B, C, D, E	72
Aurelie Panty		lace	premium	44156									36-50	72
Barbara Padded Wire Bra		seamless	advanced	0457	✓				√				70-90 A, B, C 70-85 D	82
Becky Soft Bra		seamless	basic	44676 44677			√			✓	✓		S, M, L, XL, 2XL	82
Celine Soft Bra		lace	premium	44509		✓			✓				S, M, L, XL	75
Celine Panty		lace	premium	44512									75-95 A, B, C, D	75
Clara Soft Bra Front Closure		seamless	basic	44743			✓	✓			✓	√	36-46	83
Dana Soft Bra		higher coverage	advanced	0862 0750		✓	✓		✓				70-90 AA 70-100 A, B, C 75-95 D, E	90

BREAST CARE APPAREL INDEX - LINGERIE

					Me	dical	nee	ds/F	eatu	res				
Product name	Picture	Product line	Basic/ Advanced/ Premium	Colour / Art. No.	Padded cups to hide uneven scars	Higher coverage to hide radiation burns or scars	Wider straps to relieve the shoulders	Breathable fabric for optimized air circulation	Breathable pocket fabric	High cotton content for sensitive skin	Stretchy material for volume fluctations	Front Closure	Size	Page
Ester Soft Bra	4	Active	premium	42576 42577		✓	✓	✓	√			√	70-100 A, B, C, D; 80-95 E	93
Fleur Soft Bra		classic	advanced	44672 44671 44695			✓	✓	√	√		√	XS, S, M, L, XL, 2XL Cup: A/B, C/D, E/F	78
Frances Soft Bra		classic	basic	2128			✓	✓	✓	✓		✓	XS, S, M, L, XL, 2XL Cups: A/B; C/D; E/F	7 9
Isabel Soft Bra		higher coverage	basic	2118 2118		✓			√	√			70-85 AA, 70-100 A 70-105 B, C 70-95 D; 75-95 E	91
Isadora Soft Bra		classic	advanced	0947 0948 44114		✓	✓		√				75-115 A 70-110 B, C, D 70-100 E, F, G	79
Jasmin Soft Bra		classic	premium	0900N 44233			✓		√	✓			70-90 AA, A, B 75-90 C	79
Karolina Soft Bra		lace	premium	44505					✓				70-95 A, B, C, D, E	77
Karolina Padded Soft Bra		lace	premium	44506	√				✓				70-95 AA, A, B, C, D, E	76
Karolina Wire Bra		lace	premium	44507					√				70-95 A, B, C, D, E	76
Karolina Panty		lace	premium	44508									36-46	76
Kitty Soft Bra		seamless	basic	44607 44674	√	√	√			√	√		S, M, L, XL, 2XL	85
Kitty Top		seamless	advanced	44736		✓				√	√		S, M, L, XL,2XL	84
Lara Padded Soft Bra	4	seamless	advanced	0752 0674 0675	√				✓				75-90 AA 70-100 A, B, C, D, E	85
Lara Cotton Soft Bra		seamless	advanced	44673	√			√	✓	√			75-90 AA 70-100 A, B, C, D, E	85
Lara Satin Padded Soft Bra		seamless	advanced	44212 44214	√				√				70-100 AA, A, B, C, D, E	86

BREAST CARE APPAREL INDEX - LINGERIE

					Me	dica	l nee	ed/Fe	atur	es		1		
Product name	Picture	Product line	Basic/ Advanced/ Premium	Colour / Art. No.	Padded cups to hide uneven scars	Higher coverage to hide radiation burns or scars	Wider straps to relieve the shoulders	Breathable fabric for optimized air circulation	Breathable pocket fabric	High cotton content for sensitive skin	Stretchy material for volume fluctations	Front Closure	Size	Pa
Lee Soft Bra	T	seamless	advanced	44540			✓	✓	√	✓			75-95 A, B, C, D 75-90 E	8
Magdalena Soft Bra		seamless	advanced	0463					√				75-110 B, C, D, E, F	8
Magdalena Padded Soft Bra		seamless	premium	44533	√				✓				70-95 AA, A, B, C, D	
Mara Soft Bra		seamless	basic	44534 1150 44023 44416	✓				√				70-100 AA, A, B, C, D, E	
Mara Soft Bra Front- closure		seamless	advanced	44741 44742	√				√			√	70-100 AA, A, B, C, D, E	
Mira Soft Bra		seamless	premium	44610			√		√				70-95 AA, A, B 75-90 C, D; 75-85 E	
Mona Soft Bra		seamless	premium	0606 0568 0614 0591			✓		✓				70-95 AA, A, B 75-90 C, D 75-85 E	
Nancy Soft Bra		classic	basic	44480 1151 44024 44417					√				70-100 B 75-120 C, D 75-115 E, F, G	
Nancy Soft Bra Front- closure		classic	advanced	44739 44740					✓			✓	70-100 B 75-120 C, D 75-115 E, F, G	
Nora Soft Bra	4	classic	premium	0555N 44232			√		✓	✓			75-100 A, B, C, D	
Performance Soft Bra		Active	advanced	0654		✓	√		✓				70-95 AA, A, B, C, D, E	
Power Soft Bra		Active	advanced	1152		✓	√		✓				70-95 AA, A, B, C, D, E	
Rita Soft Bra		classic	basic	2004					√				75-100 A, B, C 75-105 D 80-105 E	

BREAST CARE APPAREL INDEX - LINGERIE

					Me	dical	nee	d/Fe	atur	es				
Product name	Picture	Product line	Basic/ Advanced/ Premium	Colour / Art. No.	Padded cups to hide uneven scars	Higher coverage to hide radiation burns or scars	Wider straps to relieve the shoulders	Breathable fabric for optimized air circulation	Breathable pocket fabric	High cotton content for sensitive skin	Stretchy material for volume fluctations	Front Closure	Size	Page
Ruth Soft Bra		classic	basic	0873 0872				✓	✓	✓			70-105 AA, A, B 70-100 C 70-95 D, E 70-90 F	
Tanya Soft Bra		seamless	basic	44678 44679		✓	√	✓	✓	✓			70-95 A, B, C, D	89
Valletta Top		wellness	advanced	70230 44075 44076 70229	✓	✓		✓					36-54	95
Valletta Tall		wellness	advanced	71512 71426	✓	✓		✓					36-54	95
Zipper Soft Bra		active	advanced	44070		✓	✓		✓			✓	S, M, L, XL	92

BREAST CARE APPAREL INDEX - SWIMWEAR

					Me	dical	nee	ed/Fe	atur	es				
Product name	Picture	Product line	Basic/ Advanced/ Premium	Colour / Art. No.	Padded cups to hide uneven scars	Higher coverage to hide radiation burns or scars	Wider straps to relieve the shoulders	Breathable fabric for optimized air circulation	Breathable pocket fabric	High cotton content for sensitive skin	Stretchy material for volume fluctations	Front Closure	Size	Page
Alabama Half Bodies High Neckline		style	advanced	71326	✓	✓	√						38-54 B,C 38-52 D	99
Alabama Half Bodies		style	advanced	71514	√	✓							38-54 B,C 40-54 D, E	99
Alabama Tankini		style	advanced	71513	✓								38-50 B, C	98
Alabama Highwaist Brief		style	advanced	71332									38-54	98
Ayon Sarong		classic	premium	71113	√	✓	✓						40-56 B, C	100
Ayon Soft Bra Top		classic	basic	71117	✓								36-46 B, C, D	100
Ayon High Waist Brief		classic	basic	71118									36-50	100
Florida Full Bodies		classic	advanced	71343	√	✓	✓						38-56 B, C 40-54 D, E	101
Rhodes One-Piece		active	advanced	70842	√	✓	✓						36-52 B, C	102
Rome One- Piece		style	advanced	71379	✓								36-46 B 38-46 C	98
Toronto One-Piece Highneck		Active	advanced	71390	√	✓	✓						36-48 B 38-48 C	102



Amoena Medizin-Orthopädie-Technik GmbH Kapellenweg 36 · 83064 Raubling · Germany amoena.com

Amoena is an ISO 9001 / 13485 / 14001 certified company.

Amoena breast forms, recovery care garments, pocketed bras and pocketed swimwear are labeled with the CE marked.

