

MEDICATION CABINETS

Mod. AF

INSTRUCTION MANUAL

Index

- 1.0 Intended purpose
- 2.0 Security Information
- 3.0 Technical specification
- 4.0 Before to use
- 5.0 Practice
- 6.0 Maintenance and cleaning
- 7.0 Disposal
- 8.0 Guarantee

Products are subjected to changes without notification



Revision 01-04-2019 / V1

FRANCEHOPITAL SAS

Zona Industriale, 11

I-39011 LANA – BOLZANO

Tel. +39 0473 552611

Fax +39 0473 552699

Email: export@francehopital.com



1.0 INTENDED PURPOSE

The Linea AF cabinets have been designed for the storage of drugs, medicines, narcotics (if present the treasury) and health benefits in a hospital environment.

The cabinets have been designed and built according to criteria of functionality, ease of use, ergonomics and safety in use.

The construction materials are environmentally friendly and do not release harmful substances during use and during final disposal.

The construction of the surfaces allows a radical and definitive cleaning, at any point.

2.0 SECURITY INFORMATION

- Avoid leaving doors and drawers open and protruding: risk of impact.
- The storage of toxic or flammable substances is forbidden.
- Do not allow any cleaning liquids to leak into the inside of the cabinet and into the drawers.

3.0 TECHNICAL SPECIFICATION

- External covering in wooden agglomerate covered in white melamine, thickness mm. 20 low formaldehyde emission class E1, with anti-dust rim.
- Doors in fireproof bilaminate class 1
- Internal transparent plastic compartments
- two PTS doors with 90° opening on steel hinges
- Loading capacity of each door 6 Kg
- Lock with gear box with two expansions. Epoxy powder coated steel frame at the base
- Ø 50 mm, adjustable feet
- Where present: Key lock drug safe compartment with melamine structure and door – not removable. Dim: 40x24x40 H cm
- 5 height adjustable shelves

4.0 BEFORE TO USE

No special preparation for use is required, except for the positioning of the inner trays and the attachment of the handles.

The assembly is ready for use

5.0 PRACTICE

Avoid moving the cabinet once it is in place.

In case of handling use forklifts.

6.0 MAINTENANCE AND CLEANING

The cabinet is maintenance-free, except for cleaning.

Melamine and bilaminate can be easily cleaned and their surfaces remain disinfected and with a very high level of hygiene.

To remove fingerprints from surfaces, all you need is a soft cloth, a sponge soaked in soapy water or a microfibre cloth.

For more resistant dirt such as grease or other stains (such as syrups, tea, etc.), just use a simple cream product.

Avoid using abrasive products that can leave scratches.

The satin surfaces must be cleaned in the sense of satin finish.

Always use cleaning products according to the instructions on the packaging.

7.0 DISPOSAL

At the end of its life cycle, the cabinet can be completely recycled.

The electrical parts (where present) must be entrusted to a service of disposal of special waste.

8.0 GUARANTEE

The cabinet is guaranteed for 24 months from the date of purchase.