

SILICONE LARYNGEAL MASK TUBE

LMT-Standard





SILICONE LARYNGEAL MASK TUBE

Fortune® Silicone Laryngeal Mask Tube (LMT-Standard) Features:

- 100% medical grade silicone for superior biocompatibility.
- Reusable 40 times.
- Durable polysulfone connector withstands repeated autoclave procedure.
- Individual serial number for tracking and recording cycles of uses.
- X-ray opaque line works both for confirmation of intubated tube.
- X-ray opaque line also works as a reference line for insertion angle.
- Ribbed structure throughout the internal lumen to minimize friction.
- Soft and low friction cuff reduces intubation difficulty and provides excellent sealing and patient comfort.
- Non-epiglottis-bar design provides easy and clear access through the lumen.
- Smooth yet firm back plate of the cuff facilitates insertion.
- Unique embedded inflation line merged into the tube, which lessens extra care handling during insertion and avoids accident damage by teeth or on-site instrument.

A vented port

- A vented port with plug design offering a complete autoclaving solution - needless performing cuff deflation before autoclaving. Conventional products that require “complete” cuff deflation were found the problem of removing dead-space air completely; this residual air could cause cuff expansion during autoclaving and reduce product life.

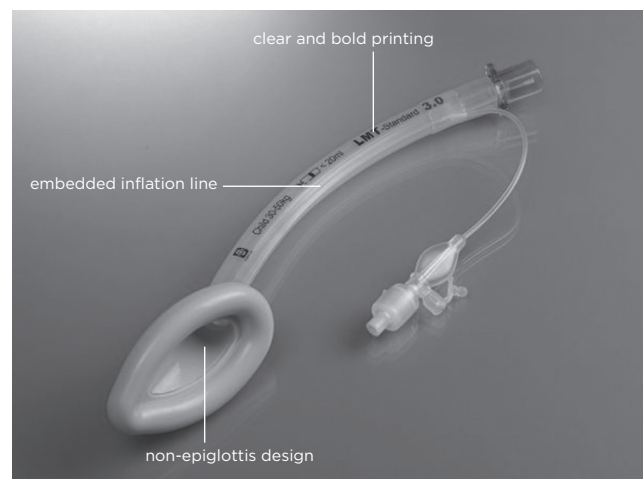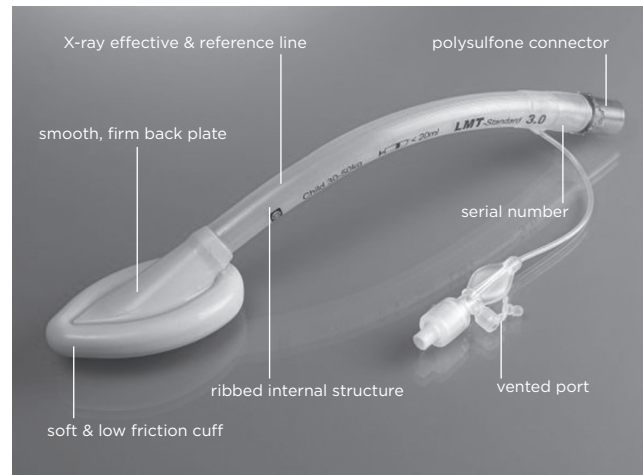


SILICONE LARYNGEAL MASK TUBE

SPECIFICATIONS

Laryngeal Mask Tube, LMT-Standard:

REF. No.	Size	Patient Weight	Maximum Inflation Volume	Description
1580-0010	1.0	Neonate < 5kg	4 ml	
1580-0015	1.5	Infant 5-10 kg	7 ml	
1580-0020	2.0	Infant 10-20 kg	10 ml	Reusable silicone
1580-0025	2.5	Child 20-30 kg	14 ml	Polysulfone connector
1580-0030	3.0	Child 30-50 kg	20 ml	X-ray opaque line
1580-0040	4.0	Adult 50-70 kg	30 ml	Vent port
1580-0050	5.0	Adult 70-100 kg	40 ml	
1580-0060	6.0	Adult >100 kg	50 ml	

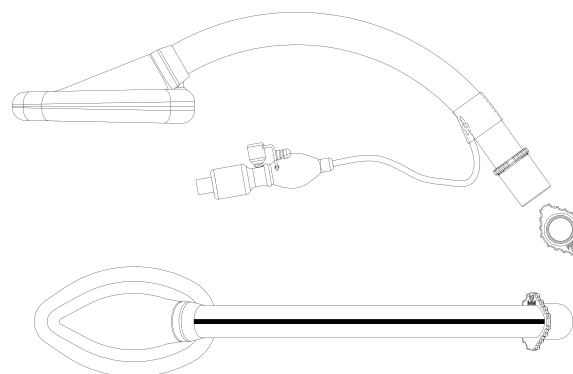

1580-00XX



6 F., No. 29, Sec. 2, Jhongjheng E. Rd.,
 Danshuei Dist, New Taipei City. 251 Taiwan
 Tel.:(886) 2-2624-2233 Fax.:(886) 2-2624-2266
 www.fortunemed.com Mail:info@fortunemed.com

Distribuator autorizat in Moldova



www.ericon.md

Tel:0-22-520108

mob.079410042

FONDAT 1992

15,Zelinski str.
 Chişinău,MOLDOVA

