


1.5.3 Setting the Radiography Parameters Using the APR (Anatomical Programs)


■ Setting the APR

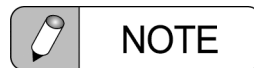
When delivered, initial setting of APR is registered. Change the setting suitably in accordance with the radiography parameters often used by clinical.



-  **Reference**
- For further details on APR Number initial settings,
See "[APR Number](#)" P.Appendix-2.
 - For further details on Anatomical Program parameter settings,
See "[2.1.9 APR Settings \[aPr\]](#)" P.2-11.

■ Selecting an APR for DR Radiography Mode

Selecting radiography mode in the DR system selects the corresponding APR and changes the radiography parameters to those registered for the program.

-  **Reference** For further details on operating the DR system, refer to the CXDI Manual.



When the  (user settings button) is pressed to switch to the user settings mode, and then the  (user settings button) is pressed again to return to the normal mode, the APR changes. From that point, select the radiography mode using the DR system once again.