

### ■ Setting AEC Radiography

You can configure the items for AEC (automatic exposure control) radiography.


#### Turning AEC ON/OFF and Setting Photo Pickup Field

- 1 Press the lower  icon on the touch panel.




The [Radiography Condition Details] screen is displayed.

- 2 Press .

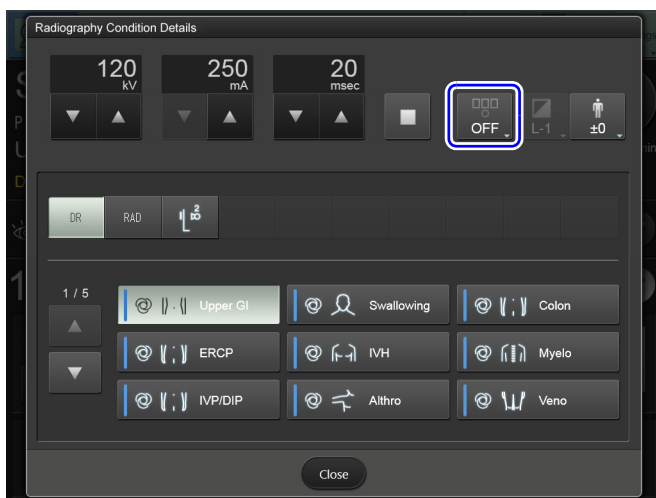

- 3 Select the photo pickup field you want to configure.

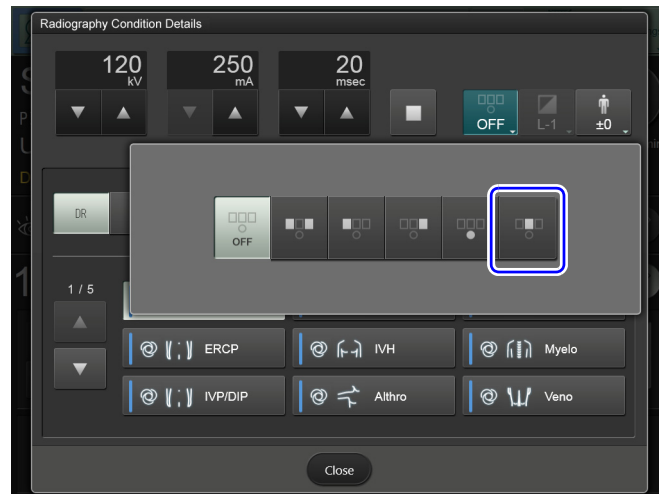

- 4 Press [Close] when the settings are completed.  
The [Radiography Condition Details] screen closes.



### NOTE

Select the center photo pickup field of the phototimer ( ) if the magnification size is 12" or less. X-ray may not be radiated in the upper photo pickup fields ( , , ), stopping the AEC from operating normally. However, select the upper center photo pickup field ( ) when selecting urology mode.



**Reference** "Operation in Urology Mode" P.4-61