iChemVELOCITY CalChek Reagent

REF 800-7703

TABLE OF CONTENTS

English	2
Français (FR)	4
Deutsch (DE)	6
Italiano (IT)	8
Español (ES)	10
Português Portugal (PT-PT)	12
Dansk (DA)	14
Svenska (SV)	16
Norsk (NO)	18
Ελληνικά (EL)	20
日本語 (JP)	22
中文 (ZH-CN)	24
Magyar (HU)	26
Polski (PL)	28
Čeština (CZ)	30
Slovenčina (SK)	32
한국어 (KO)	34
Türkçe (TR)	36
Русский (RU)	38
Hrvatski (HR)	41
Български (BG)	43
中文 (ZH-TW)	45
Română (RO)	47
Србија (SR)	49
Українська (UK)	51
Português Brasil (PT-BR)	53
Tiếng Việt (VI)	55

iChemVELOCITY CalChek Reagent

REF 800-7703

Rx Only in the U.S.A.

INTENDED USE

The iChemVELOCITY CalChek Reagents are a set of buffer-based solutions intended for the *in vitro* monitoring of specific gravity, color, and clarity. The iChemVELOCITY CalChek Reagents are manufactured for use only with the iChemVELOCITY and DxU Urine Chemistry Systems and are intended for use exclusively by healthcare professionals.

For *in vitro* Diagnostic Use with the iChemVELOCITY and DxU Urine Chemistry Systems.

PRINCIPLE

Specific Gravity: Specific gravity is determined by refractometry in the Urine Chemistry System.

Color and Clarity: Color and Clarity are measured directly from transmitted and scattered light in the sample using proprietary algorithms embedded in the Urine Chemistry System.

WARNING AND PRECAUTIONS

- The iChemVELOCITY CalChek Reagents have been determined to be nonhazardous under the guidelines issued by OSHA in 29 CFR 1910. 1200(d).
- Appropriate precautions in the collection, handling, storage and disposal of specimens should be exercised accordingly.
- Wear appropriate personal protective equipment when handling.
- Avoid solution contact with eyes and skin. Refer to the Safety Data Sheet for first aid measures.
- Consult local or state authorities with regard to recommended method of disposal.

GHS HAZARD CLASSIFICATION

iChemVELOCITY CalChek
Reagent #1,#2,#3

EUH208

May produce an allergic reaction.

reaction mass of:
5-chloro-2-methyl-4-isothiazolin
-3-one [EC# 247-500-7] and
2-methyl-4-isothiazolin-3-one [EC#
220-239-6](3:1) < 0.05%

SDS Safety Data Sheet is available at beckmancoulter.com/techdocs

NON-REACTIVE INGREDIENTS

- Water
- Food coloring
- Salts
- Suspended particles
- Preservatives

STORAGE AND STABILITY

- Store at 18° C to 25° C (65° F to 77° F).
- CalChek reagents are stable for twelve months from the date of manufacture.
- Open expiration: Unused reagents may be used for a one-time repeat if necessary within eight hours of opening. Discard unused reagents after this period.
- Do not freeze.
- Protect reagents against light.
- Do not use reagents after the expiration date printed on the tube labels.

MATERIALS PROVIDED

- 10 X 10mL iChemVELOCITY CalChek Reagents

MATERIALS REQUIRED, BUT NOT PROVIDED

- iChemVELOCITY or DxU Urine Chemistry Analyzer

- CalChék Racks
- Personal Protective Equipment

PROCEDURE

Mix by inverting CalChék tubes. Remove stopper. Place in appropriate CalChék racks. Follow the directions in the system's Instructions for Use. The system will automatically read each analyte at a specified time. iChemVELOCITY CalChék reagents come with a barcoded, convenient 10-vial package. The results are displayed on screen or printed as soon as they are available.

CALCHEK VALUES			
CalChék #	Type	Calibrated Values	Report Values
1	Specific Gravity	1.002 +/- 0.003	Pass/Fail
2	Specific Gravity	1.030 +/- 0.005	Pass/Fail
3	Specific Gravity	1.060 +/- 0.005	Pass/Fail
4	Color	Colorless	Pass/Fail
5	Color	Straw	Pass/Fail
6	Color	Normal Yellow	Pass/Fail
7	Color	Amber	Pass/Fail
8	Clarity	Slightly Cloudy	Pass/Fail
9	Clarity	Cloudy	Pass/Fail
10	Clarity	Turbid	Pass/Fail

TECHNICAL SUPPORT

For additional information, or if damaged product is received, call Beckman Coulter Customer Service at 800-854-3633 (USA or Canada) or contact your local Beckman Coulter Representative.

A Glossary of Symbols (PN C27648) is available at beckmancoulter.com/techdocs.

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

May be covered by one or more pat. - see www.beckmancoulter.com/patents

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Updated copyright year
Phiên bản AB, 08/2021
Beckman Coulter Ireland Inc., Lismeehan, O'Callaghan's Mills, Co. Clare, Ireland +(353) (0) 65 683 1100