



# Disposable Feeding Bag



**Suzhou Yaxin Medical Products Co., Ltd.**

Ad:No.12, Zhongta Road, Mudu Town, Suzhou 215000, Jiangsu province, China

Mob:0086- 15250178777

E-mail:jimwu@yx-yiliao.com

Tel: 0086-512-66613547

Fax:0086-512-66264505

Web: yxmedical.en.alibaba.com, www.yx-yiliao.com

## ***Introduction:***

Feeding bag should be considered for malnourished patients, or for those who are at the risk of malnutrition and have functional gastrointestinal tract but are unable to maintain an adequate or safe oral intake.

Usually used together with feeding tube and feeding syringe.

## ***Features:***

There are TWO main styles of feeding bag, **Pump type** and **Gravity type**.


Pump style is designed to be used with feeding pumps.

Gravity style does not require an enteral feeding pump to be used.

We have different volume bag, 500ml, 1000ml, **1200ml**, 1500ml and 2000ml. The most popular size is 1000ml and 1200ml.

## ***Details:***

Disposable Feeding Bag consists of **PVC transparent graduated bag**, **150cm PVC tube**, flow control valves, flow observation window and connector. Robert clamp is available but not necessary.

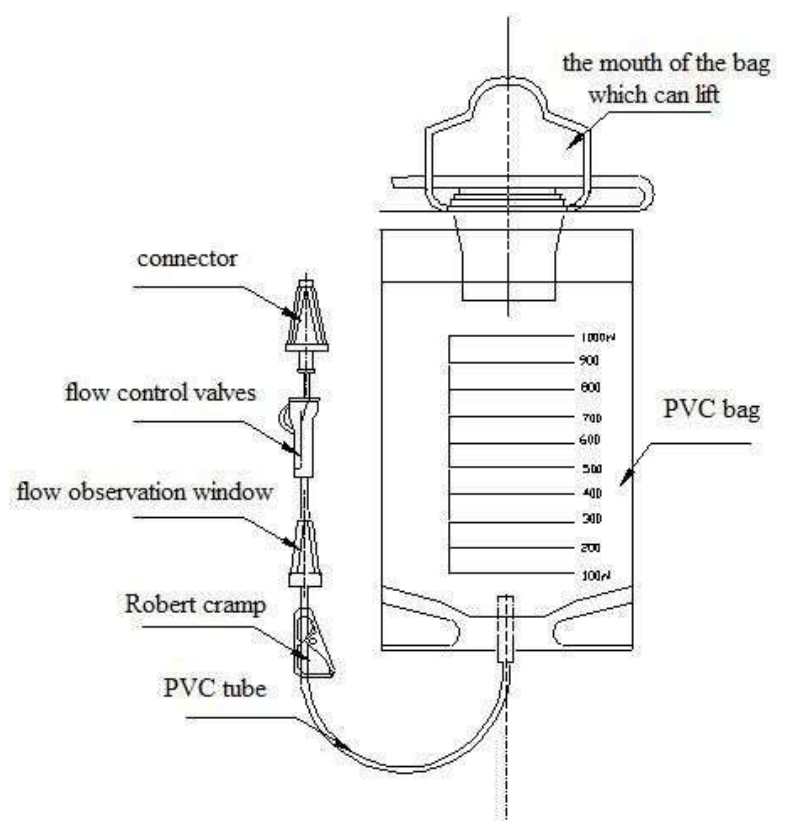

***Directions:***

1. Close control clamp completely
2. Fill bag with desired amount of formula
3. Close top cap
4. Hang bag or refrigerate for later use
5. Open control clamp. Fill entire line with fluid and adjust drops speed using the roller clamp
6. After a few minutes re-adjust control drops speed (If necessary)
7. If desired, you may fill cooling additives to ice pouch

***PRECAUTIONS:***

1. Use within 24 hours after filling.
2. Single patient use only
3. Refrigerate filled bags during storage
4. Not for I.V. use
5. Do not reuse
6. Use of this device is limited to those qualified in its use.

## Feeding Bag Model



Size	Ref	Style
1000ml	04020101	Gravity
1200ml	04020102	Gravity
1000ml	04020103	Pump
1200ml	04020104	Pump

### Caution:

1. Read instruction for use before using the device.
2. This product is sterilized for single use only, discard after single use. Please do not repeat sterilization, do not clean, do not use it once again.
3. Should be used the product immediately after opening the individual package, please destroy when it was used.
4. Do not use it if the inner package has been previously opened or damaged, or package be affected with damp, or the product went moldy.
5. Do not use after expiry date. If over the expiry date, please discard it.
6. Sterile by ETO. Single use only.

### Storage:

Store in a cool, dry and dark place. Avoid direct sunlight.

### ***Actual Photos***



**04020102**

**1200ml, Gravity**

### ***Packing Photos***



**1 pc / Poly Bag, 100 bags / Carton(52\*40\*35cm, 9kg)**