

# **WATO EX-55/65 Anesthesia Machine**



## **Service Manual**

---

# Intellectual Property Statement

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this product and this manual. This manual may refer to information protected by copyrights or patents and does not convey any license under the patent rights of Mindray, nor the rights of others. Mindray does not assume any liability arising out of any infringements of patents or other rights of third parties.

Mindray intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden. Release, amendment, reproduction, distribution, rent, adaption and translation of this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

、, **MINDRAY** and **WATO** are the registered trademarks or trademarks owned by Mindray in China and other countries. All other trademarks that appear in this manual are used only for editorial purposes without the intention of improperly using them. They are the property of their respective owners.

Contents of this manual are subject to changes without prior notice.

## Revision History

This manual has a revision number. This revision number changes whenever the manual is updated due to software or technical specification change. Contents of this manual are subject to change without prior notice. Revision 1.0 is the initial release of the document.


- Revision number: 7.0
- Release time: 2011-08

© Copyright 2008-2011 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

---

## 6.2.21 Remove the Monitor Board

The monitor board is located beside the power box. Disconnect the connection lines and tubes from the monitor board. Then unscrew the four screws as shown below to remove the monitor board.

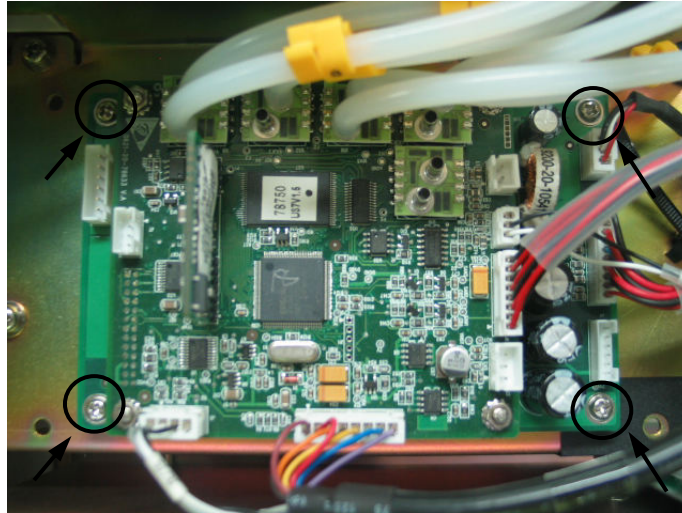

The following picture shows the appearance of monitor board.




---

## 6.3 Disassemble the Breathing System


### 6.3.1 Remove the O<sub>2</sub> Sensor

1. Remove one end of the O<sub>2</sub> sensor cable from the **O<sub>2</sub>%** connector on the anesthesia machine. Unplug the other end of the cable from the O<sub>2</sub> sensor.

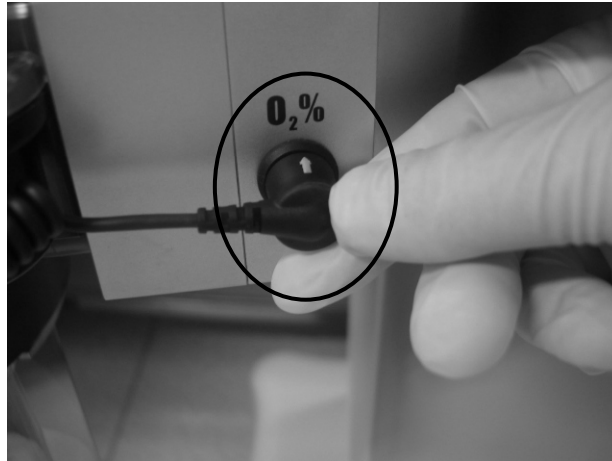

2. Turn the O<sub>2</sub> sensor counterclockwise to take it out.



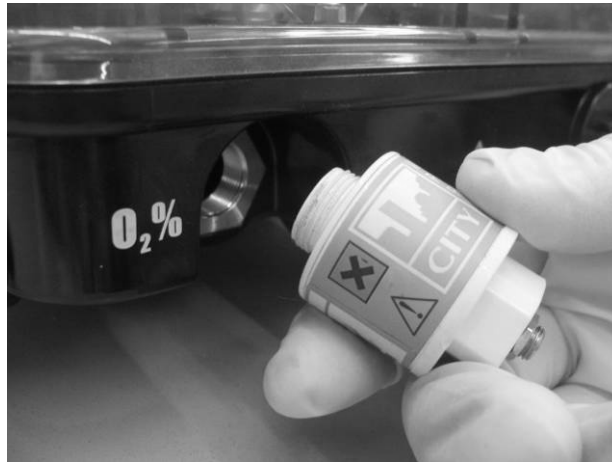

### 6.3.2 Remove the Breathing Tubes

#### NOTE

- 
- When disassembling the breathing tube, hold the tube connectors at both ends of the tube to prevent damage to the tube.
- 

1. Remove the filter from the Y piece.



2. Disconnect the breathing tubes from the inspiration/expiration connectors on the circuit.

