




Hefei C&P Nonwoven Products Co.,Ltd
Add: No.22Park road,Feidong new city development area,Hefei,Anhui,China
TEL:86-551-67707456 Fax:86-551-67709567

E-mail: info@cpnonwoven.com.cn www.medicaldrape.com

TECHNICAL DATA SHEET

Date:2024-08-10

PRODUCT : REINFORCED SURGICAL GOWN STERILE AND DISPOSABLE

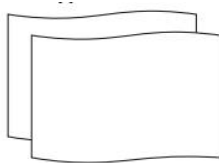

PRODUCT DETAILS:

- **REINFORCED SURGICAL GOWN
THEY ARE ULTRASOUND WEIDING**

MATERIAL:

- BLUE 45GSM SMS(3 layers SPUNBOND + MELTBLOWN + SPUNBOND NONWOVEN) REINFORCEMENT 40GSM (POLYETHYLENE + POLYPROPYLENE)
- WITH KNITTED CUFFS AND AJUSTABLE VELCRO NECK FASTNERS
- WITH 60*60CM WRAP,BLUE(38GSM POLYPROPYLENE)
- WITH 2 PIECES 40*30CM HAND TOWELS,WHITE(WOODENPULP)

CHARACTERISTICS


- PROTECT DOCTORS AND NURSES DURING SURGERY
- RESISTANT TO LIQUID PENETRATION AND MICROBIAL PENETRATION
- ULTRASONIC WELDING



Hefei C&P Nonwoven Products Co.,Ltd
 Add: No.22Park road,Feidong new city development area,Hefei,Anhui,China
 TEL:86-551-67707456 Fax:86-551-67709567

E-mail: info@cponwoven.com.cn www.medicaldrape.com

- **EFFECTIVE MICROBIAL BARRIER IN WET AND DRY STATE**
- **RESISTANT TO BURST, TENSION AND TRACTION**
- **WITHOUT PARTICAL RELEASE**
- **ANTI STATIC TREATMENT**
- **STATIC CAPACITY, LOW NENORY CAPACITY**
- **RETARTED BURNING**
- **ODORLESS**
- **COOL AND BREATHEABLE**

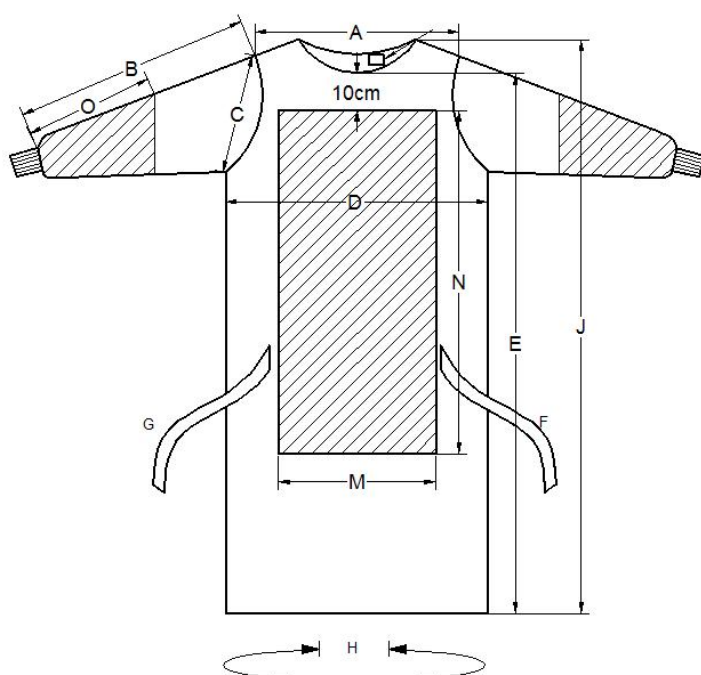

	Measuring Part/cm	S	M	L	XL	XXL	Toierance
COD E		DE-GR000	DE-GR010	DE-GR011	DE-GR012	DE-GR013	
Gow n	Length(J)	117	115	130	140	155	(±2.0cm)
	Cirumference(H)	145	150	157	165	170	(±2.0cm)
	Chest Width(D)	60	70	75	75	76	(±1.5cm)
	Arm Length(B)	50	60	60	65	65	(±1.5cm)



Hefei C&P Nonwoven Products Co.,Ltd
Add: No.22Park road,Feidong new city development area,Hefei,Anhui,China
TEL:86-551-67707456 Fax:86-551-67709567

E-mail: info@cpnonwoven.com.cn www.medicaldrape.com

	Sleeve Width(C)	27	28	30	31	31	(±1.0cm)
	Shoulder(A)	55	57	59	66	73	(±1.5cm)
	Reinforcement Width(M)	50	50	50	50	50	(±1.5cm)
	Reinforcement Length(N)	70	70	75	75	90	(±1.5cm)
	Sleeve Reinforcement Length(O)	40	40	40	40	40	(±1.5cm)

INTENDED USE

Destined to be used by surgical team members to prevent the transference of infectious agents during surgical procedures.

STERILIZATION

Products are sterilized by ethylene oxide according to standart EN ISO 11135. They are sterilized with EO sterilizer in house. Sterilization is documented and monitored through necessary forms, biological indicators, chemical indicators, dosimeters and data loggers.

PACKAGING

All components are packaged with pouch pack,pouch has necessary peeling opening available.

LABELING

Adhesive labels are identifying the product for traceability and registration purposes. And all labels indicate the information below

- Product series name and product name
- Set type(dimensions and type of drape)
- Set contents with quantities
- Manufacturing date
- Date of expiry
- Lot number
- Reference number
- Sterilization indicator
- Method of sterilization
- Warnings (for ex: do not use open packages)
- Manufacturer contact and company information
- CE mark

SHELF LIFE



Hefei C&P Nonwoven Products Co.,Ltd
Add: No.22Park road,Feidong new city development area,Hefei,Anhui,China
TEL:86-551-67707456 Fax:86-551-67709567

E-mail: info@cponwoven.com.cn www.medicaldrape.com

Product have a shelf life of five(5) years according to the test of Package Verification Test of Disposable sterilization pouch Accelerated Aged for 5years.

STORAGE CONDITIONS

Keep the product under normal conditions for healthcare materials in a cool and dry place protected from sunlight. The use of medical grade paper and film guarantees its sterility for the duration of its shelf life under normal transport and storage conditions.

QUALITY CERTIFICATION AND COMPLIENCE

- All process runs in compliance with EN 13795 AND EN ISO 13485

Transport

Gowns are transported in a carton boxes. Basic warnings and information is printed on cartons with manufacturer information and contact details. CE marking and name can be seen on carton

Comments

The product must be used by specialized personel only. Check the package's integrity before use. Do not use if the package is open or damaged.

