

TYVEK REELS USER GUIDE

- *Tyvek reels are 5, 7.5, 10, 15, 20, 25, 30, 35 and 40 cm(+/-1 cm) width and 70m.length (+/-1 mt)
- *Tyvek reels should be cut according to the size of material/device which will be exposed to the sterilisation cycle.
- *The material/device is placed into Tyvek Reel.The open end is sealed between 130-140 degrees with suitable heat sealer.
- *Tyvek Reel will be ready to sterilisation cycle after checking the sealing quality.
- *It is suitable for hydrogen peroxide sterilisation.
- *It has one indicator on it. The indicator is for plasma process.
- *After sterilisation process indicator color change is red to yellow.
- *It is suitable for ethylene and steam sterilisation process as well. But there is no EO and FO indicators on it.
- *If necessary chemical indicator strip can be put into the package.

ATTENTION

- *Please pay attention to the deformation and tear on the packs.
- *Do not overload the packs.
- *Please pay attention to the seal strength of both ends.
- *Do not use damaged products
- *Keep products at room temperature 25°C