

PerOssal® 1x50


Gentamicin / Rifampicin Combined Application

Preparation

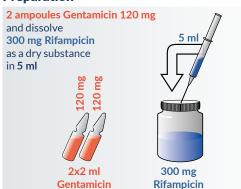



2 ampoules Gentamicin 120 mg and dissolve 600 mg Rifampicin as a dry substance in **10 ml**

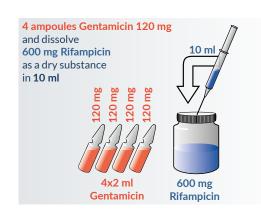


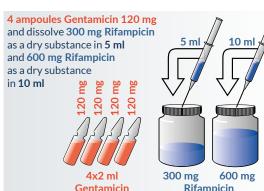


Drenching

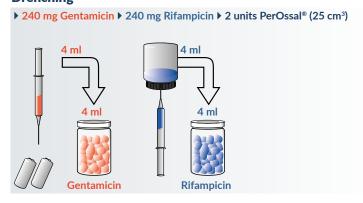

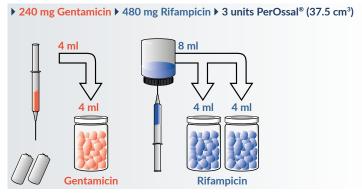

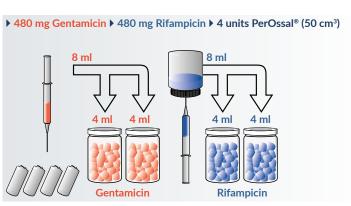

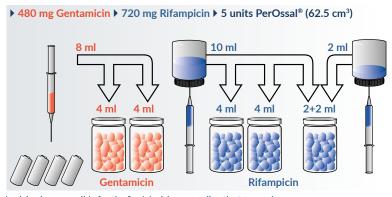


The dosage recommendation relates to the in vitro results. The treating physician is responsible for the final decision regarding the type and quantity of the corresponding antibiotic. The contraindications of the applied antibiotic have to be considered.

PerOssal® 1x6 & 2x6

Gentamicin or Tobramycin Single Application

Preparation 1 ampoule 40 mg **Tobramycin** 1x1.0 ml

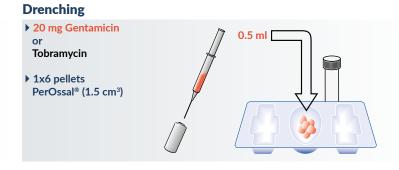




The dosage recommendation relates to the in vitro results, as well as to the results obtained within the scope of clinic testing. The treating physician is responsible for the final decision regarding the type and quantity of the corresponding antibiotic. The contraindications of the applied antibiotic have to be considered.

Vancomycin Single Application

Preparation



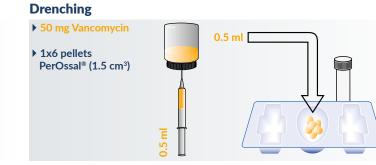



The dosage recommendation relates to the in vitro results, as well as to the results obtained within the scope of clinical testing. The treating physician is responsible for the final decision regarding the type and quantity of the corresponding antibiotic. The contraindications of the applied antibiotic have to be considered.

OSARTIS GmbH

Auf der Beune 101, 64839 Münster, Germany

phone +49 (0) 6071 - 929 0 e-mail info@osartis.de fax +49 (0) 6071 - 929 100 web www.osartis.de

OSARTIS

PerOssal®

Dosage Recommendation






PerOssal® 1x50

1 unit with 50 pellets is sufficient for filling of 12.5 cm

Gentamicin or Tobramycin Single Application

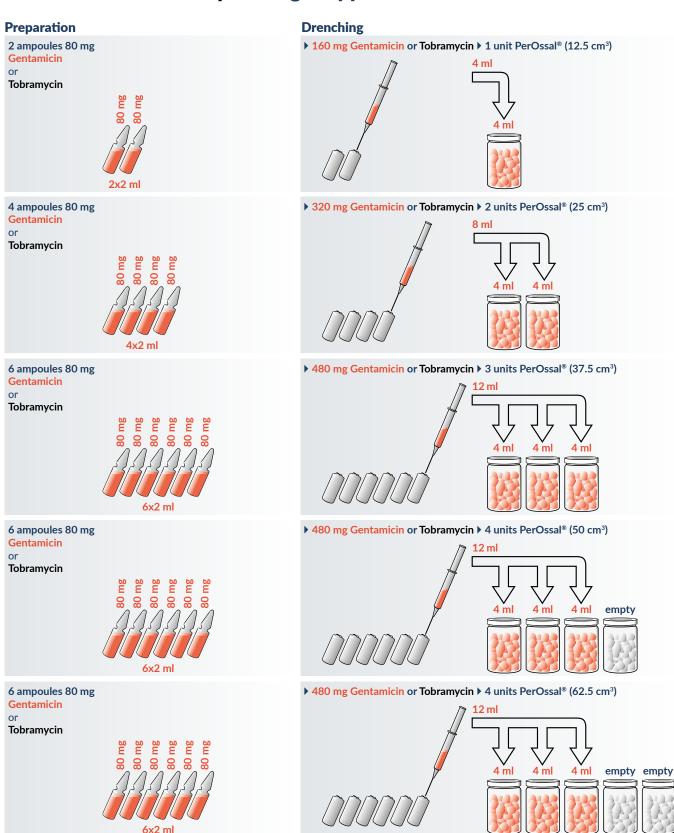


The dosage recommendation relates to the in vitro results, as well as to the results obtained within the scope of clinical testing. The treating physician is responsible for the final decision regarding the type and quantity of the corresponding antibiotic. The contraindications of the applied antibiotic have to be considered.

PerOssal® 1x50

1 unit with 50 pellets is sufficient for filling of 12.5 cm³

Vancomycin Single Application

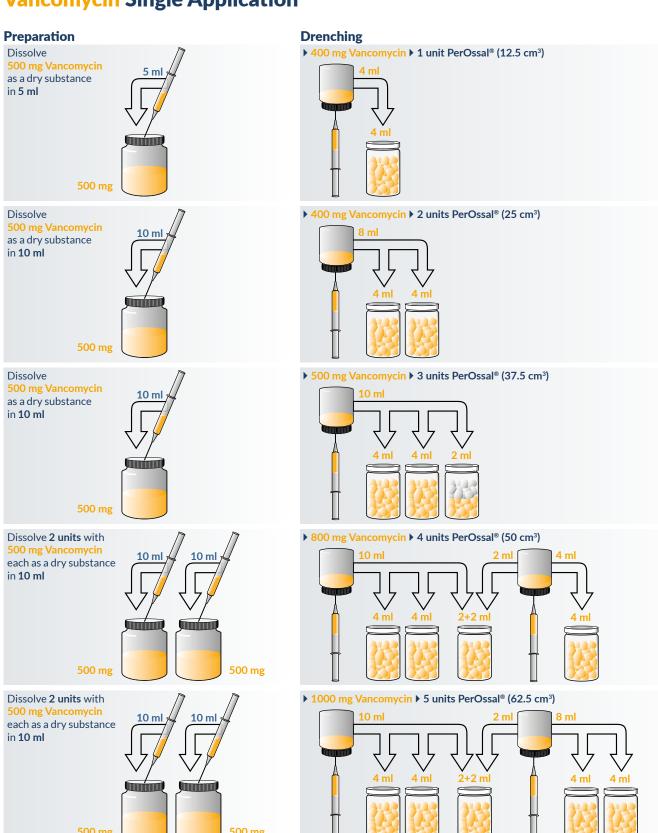

The dosage recommendation relates to the in vitro results, as well as to the results obtained within the scope of clinical testing. The treating physician is responsible for the final decision regarding the type and quantity of the corresponding antibiotic. The contraindications of the applied antibiotic have to be considered.

PerOssal® 1x50


1 unit with 50 pellets is sufficient for filling of 12.5 cm³


Vancomycin / Rifampicin Combined Application


Preparation Dissolve 500 mg Vancomycin as a dry substance in 5 ml and 300 mg Rifampicin as a dry substance in 5 ml Vancomycin Rifampicin







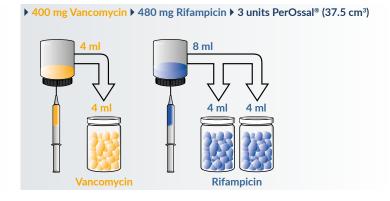

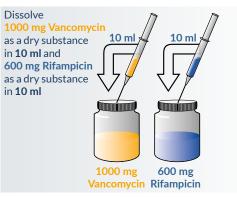

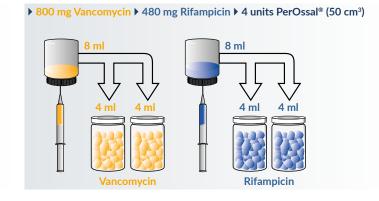

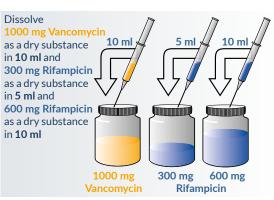

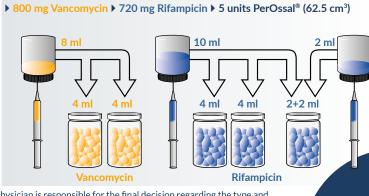

The dosage recommendation relates to the in vitro results. The treating physician is responsible for the final decision regarding the type and quantity of the corresponding antibiotic. The contraindications of the applied antibiotic have to be considered.