



DISINFECTION WIPE FOR SURFACES USER MANUAL

AKASPRAY TUCHER

- **READY TO USE DISINFECTION WIPE**
- **EFFECTIVE AT 30 SECOND**

Product Properties

- Effective at 30 second.
- Included VAH (DGHM) list.
- Aldehyde and Phenol free.

Usage Areas

AKASPRAY TUCHER is aldehyde and phenol free and a fast effective alcohol based disinfection wipe. It is used at medical equipments (patient beds, doors, medical instruments, cabinets etc.) and surfaces existing at surgery rooms, intensive care rooms, dental units.

Usage form

- Take sufficient wipe according to surface size that will be wiped.
- Visible contaminations (blood etc.) existing on the surface are cleaned firstly.
- Then take some wipes again and wipe equipment completely.
- Wait 30 second for effect time.

Uyarı: Appy electronic devices after switch off.

Ingredient

%50 Etanol, %10 2-Propanol, %0,05 Didesilmetiloksietilamonyumpropiyonat

Advantages

- Low effect duration
- Baktericidal (incl. TBC ve MRSA), fungucidal ve virucidal (HIV, HBV, HCV)
- Not Irritant
- Not harmful for surfaces.
- Easy drying and nice odour.

First aid informations:

- In the case of eye contact, wash plenty of water minimum 15 min. Appy to physician.
- In the case of swallowing, not vomit. Drink plenty of water and shake your mouth and throat with water. Immediately appy to physician.

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📠 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr