


## 2.2 Control Panel

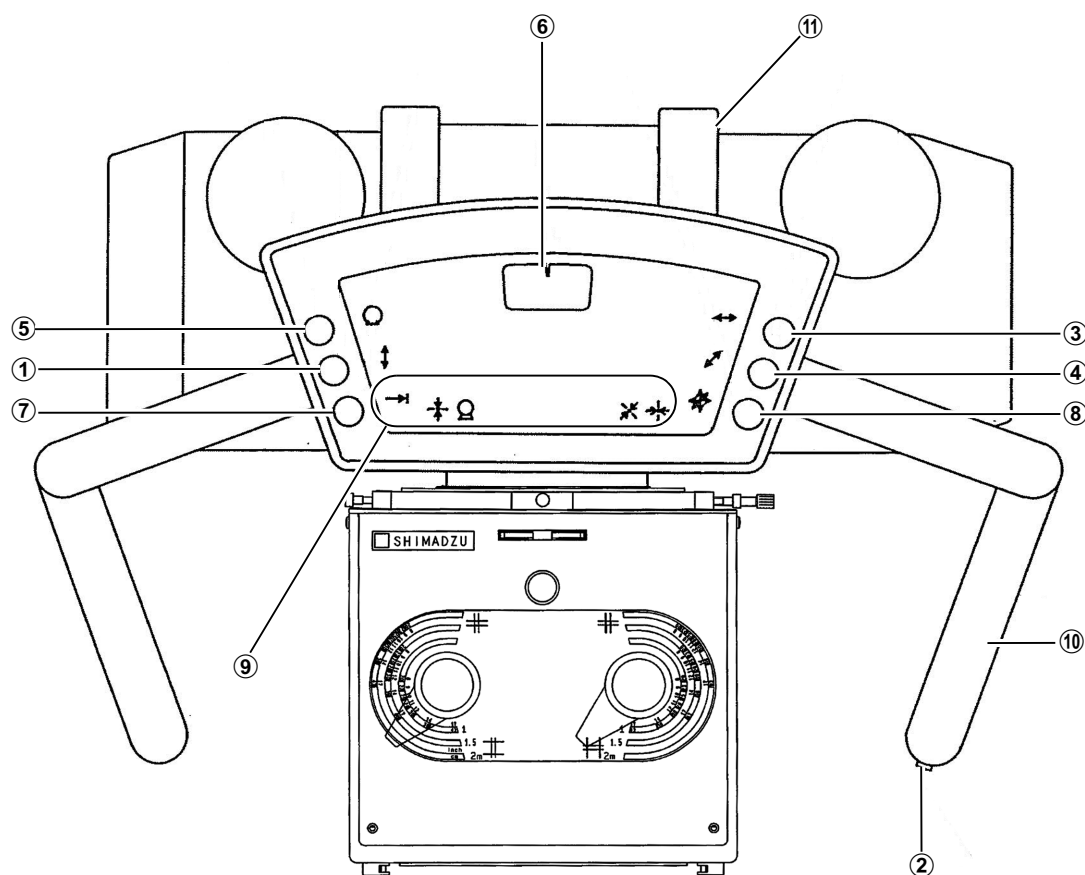

No.	Name	Reference
①	Lock release switch for vertical movement	3.2.1
②	Lock/release switch for vertical movement (for about 5 seconds)	
③	Lock release switch for longitudinal movement	3.2.2
④	Lock release switch for lateral movement	3.2.3
⑤	Lock release switch for tilt	3.2.5
⑥	Tilt indicator of X-ray tube unit	
⑦	Center stop mode switch	3.2.4
⑧	Simultaneous lock/release switch for vertical movement, longitudinal movement, and lateral movement	-
⑨	Indicator lamp	-
⑩	Rotation lock handle of X-ray tube unit	-
⑪	Rotation scale plate of X-ray tube unit (for SHIMADZU X-ray tube unit)	3.2.6