

# BACTRYL™ SPRAY | BACTRYL™ WIPES

Disinfectant and Cleaner



# BACTRYL™ DISINFECTANT

**Powerful disinfectant and surface cleaner, available in both spray and wipes**

**Ready-to-use cleaning and disinfecting agent** with bactericidal, virucidal, fungicidal, yeasticidal and mycobactericidal properties on surfaces and on invasive and non-invasive medical devices. It is a class IIb Medical Device



## Effective cleaning and disinfection

- Tested by two accredited laboratories and approved by independent experts<sup>1,2</sup>
- Double action in one product, cleaning and disinfection, thanks to the combination of quaternary ammonium compounds and chlorhexidine
- Registered on the VAH-List of Disinfectants (The Association for Applied Hygiene, Germany)
- Have demonstrated complete inactivation of enveloped viruses such as MVA, BVDV, HIV, HBV, Coronavirus, and Influenza

## Tested compatibility

- Compatible with a wide range of materials, including HDPE, Teflon, PVC, PET, silicone, neoprene rubber, and metals such as aluminum and stainless steel
- Can be used with invasive and non-invasive devices
- Gentle and safe for surfaces, Neutral pH



**Safe for surfaces in medical, endoscopic, and dental facilities,** including invasive and non-invasive devices, instruments, accessories and equipment.



## Safe and easy to use

- Ready to use—just spray and wait, or spray and wipe for faster action
- Safe for operators, contains no carcinogenic, mutagenic, or toxic ingredients
- No bleach, no aldehydes, no phenols
- Broad application throughout your medical or dental site

## Rapid and lasting action

- Disinfectant in 1 minute on bacteria, yeast and enveloped viruses (in compliance with EN 14885)
- Disinfectant in 30 seconds on bacteria and yeast if rubbed on with a towel/cloth (EN 16615)
- Fungicidal and Mycobactericidal activity between 2-10 minutes according to the stains

# BACTRYL™ Disinfectant meets the relevant requirements of European Directives for a wide spectrum of action.

In compliance with EN 14885 and EN 16615 (surface disinfectant)

## DISINFECTANT ACTIVITY

ACTIVITY	STANDARD
Bactericidal*	EN 13727, EN 13697 <i>Pseudomonas aeruginosa, Staphylococcus aureus, E. coli, Enterococcus hirae, Salmonella enterica typhimurium, Salmonella choleraesuis, Proteus mirabilis.</i>
Fungicidal/Yeasticidal*	EN 13624, EN 13697 <i>Candida albicans, Aspergillus niger, Trichophyton mentagrophytes</i>
Bactericidal/Yeasticidal	EN 16615 <i>Pseudomonas aeruginosa ATCC 15442, Staphylococcus aureus ATCC 6538, Enterococcus hirae ATCC 10541, Candida albicans ATCC 10231</i>
Mycobactericidal/ Tuberculicidal	EN 14348 <i>M. smagmatis, M. avium, M. terrae</i>
Virucidal**	EN14476 (HIV, HBV, HCV) DVV/RKI (enveloped viruses)** BVDV, Vaccinia Virus Strain Ankara HIV, HBV, HCV e influenza viruses.

\* Tested according to requirements and methods for VAH Certification of Chemical Disinfection procedures. VAH Listed.

\*\* Validated in external lab in according to the Guideline of the Robert-Koch-Institute RKI (German Federal Health Authority) and the Deutsche Vereinigung zur Bekämpfung der Viruskrankungen DVV (German Registered Association for Combating Viral Diseases).

## ORDERING INFORMATION

PART NUMBER	DESCRIPTION	UNITS PER BOX
500046	Bactryl Spray, Bottle of 1000 ml	4 bottles
500047	Bactryl Wipes, Tube with 110 wipes	6 tubes

**Bench tested and confirmed with two independent labs.**

**Tests performed in clean and dirty conditions.**

**Recommended for use with Cantel capital equipment.**

Use with confidence on Cantel equipment such as: CLEANASCOPE™ System Carts and Trays, SURESTORE™ Endoscope Storage & Transportation, ENDODRY™ Drying and Storage System, ISA™ Automated Endoscope Reprocessor, ADVANTAGE PLUS™ Automated Endoscope Reprocessor, ADVANTAGE PLUS™ Pass-Thru Automated Endoscope Reprocessor, RAPIDAER™ Endoscope Reprocessor along with a wide range of other health care cleaning and disinfection applications.



## THE COMPLETE CIRCLE OF PROTECTION

As the global vanguard in infection prevention, **only Cantel delivers the Complete Circle of Protection**, a full-value, proactive partnership dedicated to helping you remove risk, streamline operational efficiencies and optimize your success.

### References

1. H.P. Werner. HygCen Germany. (2018, February). Test report SN 24115 of 2017-02-27.
2. F.A. Pitten. HygCen Germany. (2018, June). Test report PL 18-46 180615

BACTRYL™, CLEANASCOPE™, SURESTORE™, ENDODRY™, ISA™, ADVANTAGE PLUS™ and RAPIDAER™ are trademarks of Cantel Medical Corp and its affiliates. Marks not registered in all jurisdictions.

[www.cantelmedical.eu](http://www.cantelmedical.eu)

Global Headquarters | p: +1.973.890.7220

### TO PLACE AN ORDER

Medivators B.V. | p: +31455471471 (customer service) e: [csdistributors@medivators.com](mailto:csdistributors@medivators.com)  
Cantel (Germany) GmbH | p: +4921190980750 e: [CSDE@cantelmedical.de](mailto:CSDE@cantelmedical.de)  
Cantel Medical (Italy) srl | p: +39069145399 e: [info@cantelmedical.it](mailto:info@cantelmedical.it)

