CryoPro[®]

criochirurgie cu azot lichid

CORTEX TECHNOLOGY

CryoPro®

...criochirurgie

Două dimensiuni: 0.35 l & 0.5 l

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CryoPro® Mini, 0.35 l LN₂

GB

As the leading European supplier of liquid nitrogen cryosurgical equipment Cortex Technology offers a complete range of treatment units, accessories and storage tanks.

Combining our extensive experience with the most up-to-date design features we proudly present the CryoPro® series of cryosurgical treatment units providing out-standing performance in terms of safety, ease of operation and reliability.

Further, the CryoPro® offers full compatibility with other popular liquid nitrogen units, i.e. spray apertures and contact probes for other systems may be used with the CryoPro® units as well.

Method

Simplicity, high cure rate and reliability are keywords for liquid nitrogen cryosurgery as a well established and preferred treatment for various skin lesions. The treatment is associated with minimal pain, and none or minimal anaesthesia is needed.

In cryosurgery a number of other coolants such as nitrous oxide, carbon dioxide and dimethyl ether / propane are used. However, these coolants do not reach temperatures below minus 90°C and can only be used for contact freezing.

Further, some are environmentally unfriendly and introduce safety hazards.

Liquid nitrogen provides superior efficacy at minus 196°C and offers both spray and contact freezing. Also, liquid nitrogen is safe, friendly to the environment, and readily available from gas suppliers.

The low temperature, the variety of accessories and the high cure rate combined with excellent cosmetic results and ease of operation makes CryoPro® liquid nitrogen cryosurgery the most versatile cryosurgical treatment available at a very low cost per treatment.

Apparatus

The CryoPro® is available in two sizes: **CryoPro® Maxi** with 500 ml capacity (20 - 24 hrs. static holding time) and **CryoPro® Mini** offering a 350 ml capacity (12 - 14 hrs. static holding time).

Each unit comes with a set of spray applicators for most common lesions (e.g. warts, small tumors etc.). Optionally, contact probes are available in various sizes. These probes are particularly useful to control the lateral spread of the freeze (e.g. near the eye or in the oral cavity). Contact probes are selected to fit the actual lesion, and they offer a deep freeze with minimal lateral spread.

Applications

Cryosurgery using liquid nitrogen is the treatment of choice or an alternate method of treatment for many skin lesions as well as cervical erosions.

More information can be found in published literature of which some are referenced on the reverse page.

Warranty & Approvals

CryoPro® units are covered by a worldwide three year guarantee against material and manufacturing defects.

Cortex Technology has obtained the FDA 510(k) under 21CFR878.4350 and the CE-mark acc. to Council Directive 93/42/EEC.

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RO

Storage containers

Liquid nitrogen is stored in special superinsulated tanks, so-called dewars. Cortex Technology supplies such dewars in sizes from 5 - 60 l capacity and static holding time up to 350 days.

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To transfer liquid from the dewar to the CryoPro[®] unit the use of a **withdrawal device** is highly recommended.

Our withdrawal devices are detachable, which means no excess evaporation from the dewar, when the withdrawal device is not in use. Not only does this provide the most economical and safe extraction of liquid from the storage tank, the withdrawal device also provides a filter to prevent ice crystals in the dewar from being transferred to the CryoPro[®] unit ensuring trouble-free operation.

The withdrawal devices are fully adjustable and fit dewars of 10 l capacity and up.

Transportation of dewars is facilitated by an optional **roller base** available for dewars of 20 - 60 l capacity.

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has been approved in conformity with the requirements of

Annex V, section 3.2 - Production quality assurance

of Council Directive 93/42/EEC concerning medical devices as transposed into Danish law.

The certificate covers the following activities:

Manufacture and final test of cryosurgery equipment for use within dermatology in class IIa

The EC certificate is valid provided that the quality system continues to conform to the above-mentioned scope and provided that the company does not introduce substantial changes to the quality system without the approval of Presafe Denmark A/S. This EC certificate is issued pursuant to the Presafe Denmark A/S terms and conditions for the certification of medical devices and entitles the manufacturer to affix the CE mark. The certificate is based on successful audit of the manufacturer. The manufacturer is subject to periodical audits as per the Directive.

Heidi Jørgensen
Authorized person

For Presafe Denmark A/S

Date of issue: 2017-10-05
Expires: 2022-10-05
Initial date of issue: 1998-08-10
Reference: aur5a1706v260f176

The following product families in class IIa are covered by the certificate:

Dermatological products

Certificate number: DGM – 093
Certificate type: EC Certificate

Date of issue: 2017-10-05
Expires: 2022-10-05
Initial date of issue: 1998-08-10
Reference: aur5a1706v260f176

Quality System Certificate

Certificate No.:
DGM – 610

Reference:
aur5a2108v310f176

Date of issue:
2021-09-30

Valid Until:
2024-09-30

Initial date of issue:
2007-05-25

This is to certify that the quality system of:

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fulfills the requirements in:

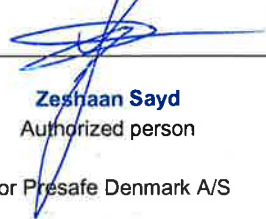
DS/EN ISO 13485:2016

The certificate covers the following activities:

Manufacture, testing and service of cryosurgical units and cryosurgical accessories

The certificate is valid provided that the quality system continues to conform to the above-mentioned scope and provided that the company does not introduce substantial changes to the quality system without the approval of Presafe Denmark A/S. Certifications to DS/EN ISO 13485:2012, EN ISO 13485:2012, ISO 13485:2003, DS/EN ISO 13485:2016, EN ISO 13485:2016, ISO 13485:2016, DS/EN ISO 9001:2015, EN ISO 9001:2015, and ISO 9001:2015 include the requirements of any applicable corrigendum. The quality system certificate is issued pursuant to Presafe Denmark A/S' terms and conditions for the certification of quality systems for medical devices.

Presafe Denmark A/S
Notified Body, Identification No. 0543
Tuborg Parkvej 8, 2900 Hellerup, Denmark

Zeshaan Sayd
Authorized person
For Presafe Denmark A/S

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DECLARATION OF CONFORMITY

Manufacturer: Cortex Technology ApS.
Smedevænget 10
DK-9560 Hadsund
Denmark

hereby declare that the following medical devices in Class 2a:

Product Identification CryoPro® Mini, Mini CryalJet E0200213
CryoPro® Maxi, Maxi CryalJet E0200113

are consistent with the Danish Medicines Agency circulation no. 1263 of 15th December 2008 and Council Directive 93/42/EEC of 14th June 1993 and its implementation in the Member States, where the company's products are marketed, verified by Presafe 0543.

This declaration is based on:

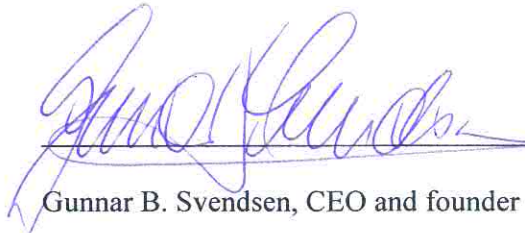
The procedure for conformity in accordance with Annex V and Annex VII of the above notice.

Signature from: ☒ Managing Director

☐ Legally binding person

Date: December 15th 2014

Signature

Gunnar B. Svendsen, CEO and founder